

# suiteHEART® Software

## Addendum to Instructions for Use

**NeoSoft, LLC**



NS-03-015-0006 Rev. 1  
©2016 NeoSoft, LLC  
All rights reserved

# Revision History

Document Revision	Date of Issue	Description
1	7-January-2016	Initial release

## Manufacturer



NeoSoft, LLC  
N27 W23910A Paul Rd.  
Pewaukee, WI 53072 USA

Phone: 262-522-6120  
website: [www.neosoftllc.com](http://www.neosoftllc.com)

Sales: [orders@neosoftmedical.com](mailto:orders@neosoftmedical.com)  
Service: [service@neosoftmedical.com](mailto:service@neosoftmedical.com)

## Medical device directive

This product conforms to the requirements of the council directive 93/42/EEC concerning medical devices when it bear the following CE Mark of Conformity.



European representatives:



EMERGO EUROPE  
Molenstraat 15  
2513 BH, The Hague  
The Netherlands



**CAUTION:** *United States federal law restricts this device to sale by, or on the order of, a physician.*

This page intentionally left blank.

---

# Addendum

This addendum applies to suiteHEART® (software version 3.0.0) Instructions for Use, part number NS-03-008-0001.

suiteHEART® Software now supports remote connectivity capability. The remote connectivity capability allows suiteHEART® Software users remote access from computers not loaded with suiteHEART® Software. The remote user will have the ability to access the suiteHEART® Software system and have full functionality. Remote connectivity is supported for only a single concurrent remote connection.

suiteHEART® remote connectivity needs to be installed/configured/supported by your Information Technology personnel. Remote connectivity has been verified by NeoSoft using the following platforms:

- TeamViewer (version 11)
- Citrix GoToMyPC (version 8.4)
- Windows Remote Desktop (Microsoft Windows 7/8 Professional/Enterprise)

**NOTE:** When using Windows Remote Desktop, screen resolution on the remote computer must be set to 1920x1080 video display resolution.

The performance of the suiteHEART® Software via the remote connection cannot be guaranteed. Performance is dependent upon factors that are outside the scope of the suiteHEART® Software application. These factors include:

- Remote computer performance
- Internet download/upload speeds (Mbps)
- Network bandwidth speed (wired ethernet or wireless connection)

This page intentionally left blank.



