

suiteHEART® Software

cMRI Analysis Software

Instructions for Use

NeoSoft, LLC



NS-03-008-0001 Rev. 13
Copyright 2017 NeoSoft, LLC
All rights reserved

Revision History

Rev	Date	Description of Change	Safety Related Update (Yes/No)
1	26AUG2014	First Release	No
2	19SEPT2014	Updated Screenshots	No
3	20NOV2014	Add Medical Device Directive Information	No
4	07MAY2015	Updated for 3.0.0 product release	No
5	22MAY2015	Updated arrow key descriptions	No
6	16JUNE2015	Updated Screenshots	No
7	28JULY2015	Updated Screenshots	No
8	24AUG2015	Emergo Updates	No
9	25SEPT2015	Added Recommended System Requirements	No
10	03FEB2016	Updated for 3.0.1 project release	No
11	16SEPT2016	Updated for the 4.0.0 product release	No
12	23MAR2017	Updated for the 4.0.2 product release	No
13	15SEPT2017	Updated for the 4.0.4 product release	No



NeoSoft, LLC
N27 W23910A Paul Road
Pewaukee, WI 53072 USA

Phone: 262-522-6120
website: www.neosoftllc.com

Sales: orders@neosoftmedical.com
Service: service@neosoftmedical.com

The Instructions for Use for this device are provided electronically in Portable Document Format, (.pdf). A pdf viewer is required to view the Instructions for Use. A paper copy of the Instructions for Use can be provided upon request, at no cost, within 7 calendar days, by emailing service@neosoftmedical.com.

Instructions for Use may be accessed in the following ways:

1. After launching the application, click "Help" or "About" from the main screen. Select the "Instructions for Use" option. The Instructions for Use will open in a pdf viewer.
2. If the original installation package received from NeoSoft is available, open the zip file and navigate to the "Documentation" folder, then the "Instructions for Use" folder and double click on the Instructions for Use.pdf in your language, denoted by EN - English, FR - French, DE - German.
3. Navigate to the folder where the application is installed. Locate the "Documentation" folder, open the "Instructions for Use" folder and double click on the Instructions for Use.pdf in your language, denoted by EN - English, FR - French, DE - German.
4. Electronic copies of Instructions for Use are also available at www.neosoftllc.com/neosoft/product_manuals/.

This page intentionally left blank.

Medical device directive

This product conforms with the requirements of council directive 93/42/EEC concerning medical devices when it bears the following CE mark of Conformity:



European Representatives:



EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

Canada:

Health Canada device license number: 99490



CAUTION: *United States federal law restricts this device to sale by, or on the order of, a physician.*

This page intentionally left blank.

Table of Contents

Medical device directive i

Safety 1

- Introduction 1
- Indications for Use 1
- Intended Use 2
- Supported DICOM Image Formats 2
- Safety Notices 3
- Equipment Hazards 3

Getting Started 5

- Launching and Exiting the Application 5
 - Launching suiteHEART® Software 5
 - Exiting suiteHEART® Software 6

User Interface Overview 7

- Overview 7
- Analysis Modes 8
 - Series Navigation 8
- Editor Window and Mode View 9
 - File Menu Options 9
 - Tools Menu Options 9
 - Help Menu Options 10
 - Image View Controls 10
- Mode Views 11
 - Cine Mode 11
 - Matrix Mode 11
 - Cross Reference Mode 12
 - Image Manipulation Tools 13
- Quick Keys 14
- Analysis View 16
 - Analysis View Review 17
- Edit Tab Labels 19
- Report View 20
- Browse DB 21
 - Features of the Browse DB 21
 - Browse DB Procedure 22

Image Management Tools 23

- Viewer **23**
 - Viewer Functionality **24**
- Compare Mode **26**
 - Sample Workflow **28**

Defining Preferences 31

- Setting Preferences **31**
 - Global Settings **31**
 - Template Preferences **37**
 - Macro Preferences **40**
 - Import Preferences **42**
 - Export Preferences **42**

Editing Contours 45

- Contour Edit Options **45**
 - Conventional Editing **45**
 - Nudge Tool **46**
 - Delete a Contour **47**

Function Analysis 49

- Ventricles **49**
 - Calculate Index Measurements **50**
 - Auto LV & RV Segmentation **50**
 - Ventricular Function Analysis Results **59**
 - Left Ventricular Regional Analysis **61**
 - Dyssynchrony Analysis **62**
 - Manual LV Function Analysis Procedure **63**
 - Manual RV Function Analysis Procedure **66**
 - Fast LV Function Analysis Procedure **69**
- Atria **70**
 - Manual LA and RA Function Analysis Procedure **70**
 - Fast LA or RA Function Analysis Procedure **71**
 - Atrial Dimensions and Area **71**
- Linear Measurements **72**
 - Linear Measurement Set-up **72**
- Valve Plane Analysis **74**

Flow Analysis 77

- Flow Window Components **78**
- Flow Analysis **79**
 - Generate Flow Results **79**
 - Contour Editing **84**
 - Flow Correction Options **86**
 - Flow Curve Options **88**

User Defined Peak Velocity	90
Display Modes	91
Review Summary Tables	94
Qp/Qs Tab	95
Calculate Qp/Qs	96

Myocardial Evaluation 97

ME Quantitative Analysis Procedure	98
Late Enhancement	98
Polar Plot Display Formats	99
T2 Analysis	101
Combined Analysis	103
Late Enhancement and T2	103
Signal Differential Results	107

T1 Mapping Analysis 109

Launch T1 Mapping Analysis	109
Native T1 Analysis Procedure	110
Post Enhancement T1 Analysis Procedure	110
Display a Color Parametric Map	111
ECV Quantification	111
Create a 16-Segment Polar Map	111
Reset the Analysis	112
Review the T1 Curves	112

T2 Mapping Analysis 113

Launch T2 Mapping Analysis	113
T2 Analysis Procedure	114
Display a Color Parametric Map	114
Create a 16-Segment Polar Map	114
Reset the Analysis	115
Review the T2 Curves	115

Time Course Analysis 117

Launch Time Course	117
Time Course Quantitative Procedure	117
Define Endocardial and Epicardial Contours	118
Review the Image Registration	118
Generate Curve and Color Map Overlay	119
Review Results	119

Patent Foramen Ovale (PFO) Analysis 123

Launch PFO	123
Select Atrial Anatomy	124
Generate Left Atrial (LA) Intensity Curve	124
Generate Right Atrial (RA) Intensity Curve	124

- Review Curve Data and Select Phase Range **125**
- Editing Contours **126**
- Reset Graph and Contours **126**
- Review Final Curve Results **126**

T2Star 127

- Heart Analysis Procedure **128**
 - Create Myocardial Colormap **129**
 - Fitting Parameters **129**
 - Review the T2Star Results **130**
 - Review and Update the T2Star Curve **130**

3D/4D Viewer 133

- 3D/4D Viewer Components **134**
- 3D/4D Viewer Layout and Series Creation Outputs **138**

Structured Reporting 145

- Define Report Content **145**
- Structured Report View **145**
 - History Tab **145**
 - Impression Tab **147**
 - Images Tab **148**
 - Polar Plots Tab **149**
 - Preview the Report **152**
 - Approve the Exam **152**
 - Export Options **154**
 - Review an Approved Exam **155**

Report Database 157

- Report Database Tool Procedure **158**
 - Open the Database Tools Window **158**
 - Select Search Criteria **158**
 - Create a Query **158**
 - Activate the Search **159**
 - View the Results **160**
 - Save a Query **161**
- Delete a Favorite **162**
- Export Search Results to an HTML File **163**
- Export the Database **163**
- Import a Database **164**

Tablet Mode 165

- Enable Tablet Mode **165**
- Image Manipulation Tools **166**

Appendix 167

Technical Reference **167**

Appendix A - Reference Articles **167**

Appendix B - Functional Analysis Scan Plane Example **168**

Appendix C - Supported Manufacturers **170**

Index 171

This page intentionally left blank.

Safety

Introduction

To assure efficient and safe use it is essential to read this safety section and all associated topics before attempting to use the software. It is important for you to read and understand the contents of this manual before attempting to use this product. You should periodically review the procedures and safety precautions.

The software is intended for use by trained and qualified personnel only.



CAUTION: Federal Law restricts this device to sale, distribution and use by or on the order of a physician.

The terms danger, warning, and caution are used throughout this manual to point out hazards and to designate a degree or level of seriousness. Hazard is defined as a source of potential injury to a person. Familiarize yourself with the terminology descriptions listed in the following table:

Table 1: Safety Terminology

Graphic	Definition
 DANGER:	Danger is used to identify conditions or actions for which a specific hazard is known to exist which <u>will</u> cause severe personal injury, death, or substantial property damage if the instructions are ignored.
 WARNING:	Warning is used to identify conditions or actions for which a specific hazard is known to exist.
 CAUTION:	Caution is used to identify conditions or actions for which a potential hazard is known to exist.

Indications for Use

suiteHEART® Software is an analytical software tool, which provides reproducible tools for the review and reporting of medical images. suiteHEART® Software can import medical images from a MR system and display them in a viewing area on the computer screen. The viewing area allows access to multiple studies and series of multi-slice, multi-phase images. Multi-phase sequences of images can be displayed in cine mode to facilitate visualization.

A report input interface is also available. Measurement tools on the report interface make it possible to quickly and reliably fill out a complete clinical report of an imaging exam. Available tools include: point, distance, area, and volume measurement tools such as ejection fraction, cardiac output, end-diastolic volume, end-systolic volume, and volume flow measurements.

Semi-automatic tools are available for left ventricular contour detection, valve plane detection, vessel contour detection for flow analysis, signal intensity analysis for myocardium and infarct sizing measurement, and T2 star analysis.

The results of the measurement tools are interpreted by the physician and can be communicated to referring physicians.

When interpreted by a trained physician these tools may be useful in supporting the determination of a diagnosis.

Intended Use

suiteHEART® Software is intended to assist trained clinical personnel in the qualification and quantification of cardiac function. The software provides the tools to adjust the parameters of the DICOM images and provides presentation states where the user can appreciate various MRI acquired images of the heart and vasculature over time. Additionally, the software provides tools for measuring linear distances, areas, and volumes that can be used to quantify cardiac function. Finally, the software provides the tools for volumetric flow measurements and the ability to calculate flow values.

Supported DICOM Image Formats

suiteHEART® Software supports the following DICOM format; MR and Enhanced MR. Refer to the suiteHEART® Software DICOM Conformance Statement manual for further detail on supported formats.



CAUTION: Data stored as a DICOM image that has been imported by an external PACS may not be compatible viewing for suiteHEART® Software.

Safety Notices



WARNING: The application assists in the analysis of the images only and does not automatically produce quantifiable results. The use and placement of quantitative measurements is at the discretion of the user. Misdiagnosis can occur if measurements are inaccurate. Measurements should only be created by a properly trained and qualified user.



WARNING: Artifacts on an image can be misinterpreted, leading to misdiagnosis. Do not use images containing artifacts for diagnosis. Analysis should only be accomplished by a properly trained and qualified user.



WARNING: Diagnosis for the wrong patient could occur if images do not contain patient name or ID. Do not use images that do not contain patient name and ID for diagnosis. Visually confirm the patient information prior to analysis.



CAUTION: Using images upon which an image filter has been applied could result in altered results. The user must exercise discretion before analyzing pixel intensity corrected images. The software will display a warning message if loading images that have been filtered.

Equipment Hazards



CAUTION: Using equipment that is damaged or has been compromised can put the patient at risk by delaying diagnosis. Make sure that equipment is in proper working order.



CAUTION: Applications may run on equipment that includes one or more hard disk drives, which may hold medical data related to patients. In some countries, such equipment may be subject to regulations concerning the processing of personal data and free circulation of such data. Release of personal data may result in legal action depending on the applicable regulatory body. It is strongly recommended that access to patient files be protected. The user is responsible for understanding the laws regulating patient information.

This page intentionally left blank.

Getting Started

Launching and Exiting the Application

suiteHEART® Software is an application that can be used for analysis, review, and reporting of Cardiac MRI (Magnetic Resonance Imaging) studies. This manual provides a detailed explanation of the suiteHEART® Software user interface and the workflow for performing quantitative analysis on cardiac MR images.

Launching suiteHEART® Software

1. Launch suiteDXT via the desktop shortcut.

NOTE: Both suiteDXT and suiteHEART® Software applications must remain running (simultaneously) to facilitate the necessary file transfer(s) between the applications.

2. On the Main Screen go to the Launch Application drop-down menu and select suiteHEART® Software.

FIGURE 1. Launch Application



3. Select a study from the study list and do one of the following:
 - Select the Launch button.
 - Right mouse click and select "Launch using <selected application>."
 - Double click on the study.
4. Exams containing images with pixel intensity filters applied will be listed in a message box prior to opening the exam.

NOTE: The screen resolution must be set to 1920x1080 or higher, otherwise the software will not launch.




WARNING: Using images with pixel intensity filters applied for analysis may cause inaccurate results.

Exiting suiteHEART® Software

To exit the application select **File > Exit**.

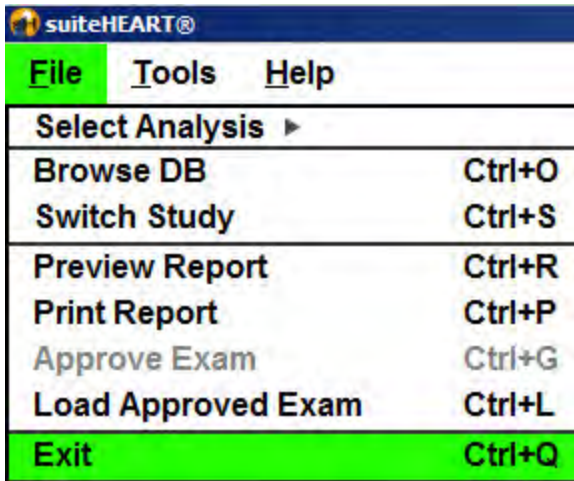FIGURE 2. Close suiteHEART® Software




User Interface Overview

Overview

The suiteHEART® Software interface has three main panels as shown in Figure 1.

- Analysis View: Provides analysis tools for each analysis mode.
- Image View: Provides quick access for image analysis and review functions.
 - Comprised of thumbnail views, editor window, and mode view.
- Report View: Provides the tools used for structured reporting.

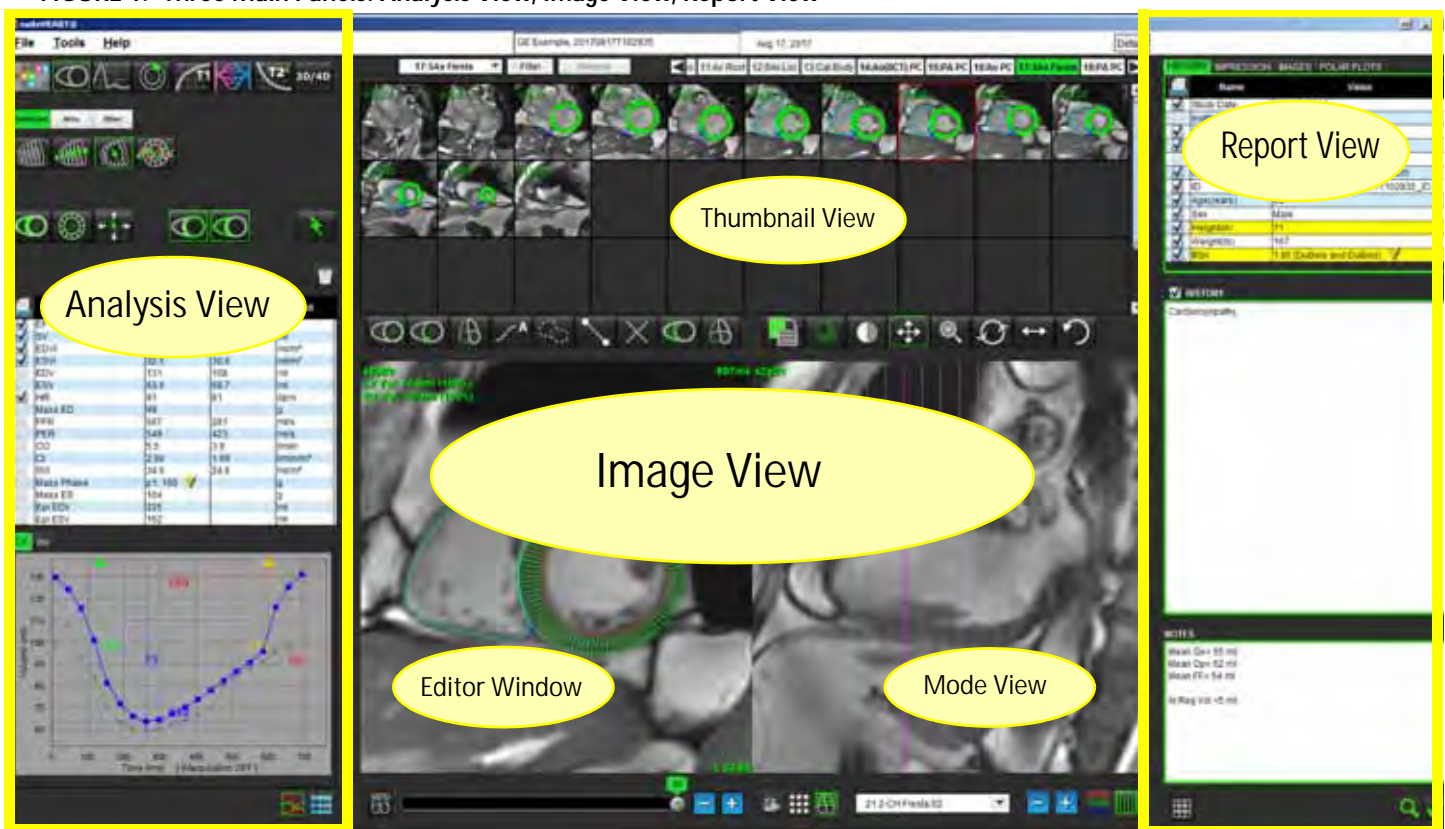FIGURE 1. Three Main Panels: Analysis View, Image View, Report View



Analysis Modes

Table 1: Mode Buttons


Viewer	Function Analysis	Flow Analysis	Myocardial Evaluation	T1 Mapping	T2 Mapping	Time Course Analysis	T2 Star Analysis	3D/4D Viewer

NOTE: Patent Foramen Ovale (PFO) Analysis can be selected from the file pull-down menu or by using Ctrl 5 on the keyboard.

Series Navigation

To view images or change series within the selected study use the left and right arrow buttons at the top of the Image View. The series file pull-down menu, located to the left of the Filter button, can also be used to select the series. Series which have analysis or regions of interest present will be identified in bold text, as shown in Figure 2.



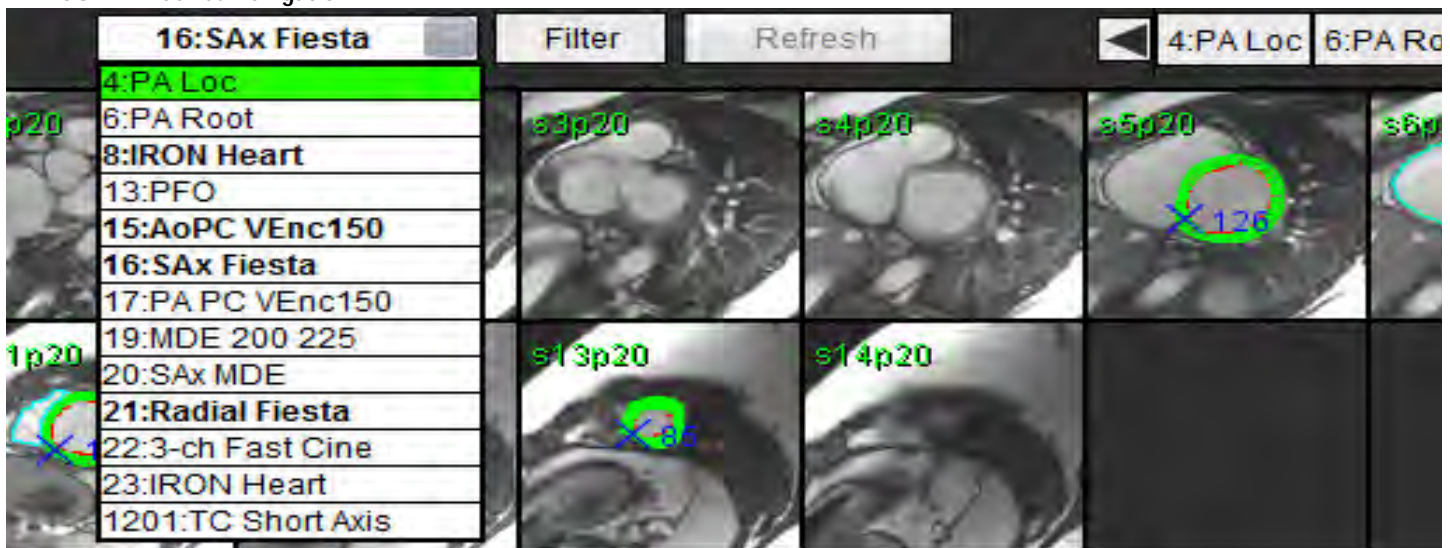





FIGURE 2. Series Navigation



Editor Window and Mode View

Performing a right mouse click on an image in the Image View will activate image manipulation tools.

Table 2: Image Manipulation Tools

	Window/Level
	Pan
	Zoom
	Rotate
	Flip
	Send to Report
	Scan Parameters
	Reset

File Menu Options

- Select Analysis** – Selects the analysis mode (Function, Flow, ME, Time Course, PFO, T2Star, T1 Mapping, T2 Mapping and 3D/4D)
- BrowseDB** – Opens local database
- Switch Study** – Lists available studies for quick access
- Preview Report** – Preview of the formatted report
- Print Report** – Prints the report
- Approve Exam** – Approves and locks a final report with a signature
- Load Approved Exam** – Restores a previously opened report
- Exit** – Closes the application while saving current analysis results to a secondary capture (SCPT) series.

Tools Menu Options

- Toggle Annotation** – Toggles the display of the ROI annotation
- Create Annotation >**
 - Linear** – Provides measurement of a straight line distance
 - Crosshair** – Provides sampling of single pixel data
 - Region of Interest** – Provides area measurements
 - Label** – Provides image annotation
- Preferences >**
 - Edit Preferences** – Opens the preferences editor to set software and template preferences
 - Import Preferences** – Restores user preferences and macros
 - Export Preferences** – Exports all user preferences
- Report Database**
- Export >**
 - Export Report** – Generates a report based on current analysis and saves it as a secondary capture (SCPT) series.
 - Export Report to Excel** – Generates Excel spreadsheet with analysis results.

Export Cine DICOM – Saves a DICOM cine of the currently selected series as a SCPT file.

Export Cine Files – Exports currently selected series images to any of the selected file formats. Available formats are: compressed QuickTime movie, JPEG, TIFF, PNG or uncompressed AVI movie.

Export to Matlab – Exports a Mat-file in binary form.


Help Menu Options

Instructions for Use – suiteHEART® Software Instructions for Use


DICOM Conformance Statement – suiteHEART® Software DICOM Conformance Statement

About suiteHEART® – Version information about the application

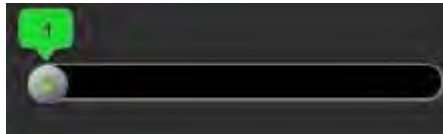Image View Controls



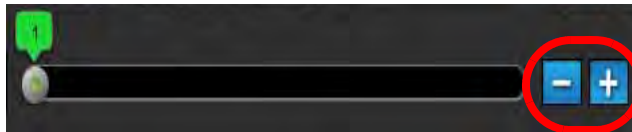The Phase Slider Bar controls the cine phase selection.



The Image Step Icons allow for slice-to-slice navigation when the thumbnail view is in slice or phases.



Slice Classification: This button is only applicable for left ventricular Regional Analysis and Quantitative Myocardial Evaluation Analysis modes. Slice classification is only relevant to short axis images where quantitative polar plots are generated.

On your keyboard, the Left and Right Arrow Keys control navigation between slices and the Up and Down Arrow Keys control navigation between phases.

Mode Views

The mode view has three available formats:

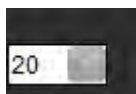
Cine Mode



- Cine: Controls the viewing of a cine image in a movie mode.



- Cine Mode Bar: Defines the start and end frame of the cine movie.



- Frames Per Second (FPS): Click on the arrow or enter a value in the text box to change the cine speed



- Play Icon: Located next to the cine mode bar



- Pause Icon: Located next to the cine mode bar

Matrix Mode




Matrix Mode Icon: Displays a grid of the images in slices/phases

The images selected for analysis are highlighted and the endocardial and epicardial contours are noted. Matrix mode may be used for phase navigation. The ED and ES buttons may be used to select the phases accordingly on the matrix. Matrix mode can also be used for slice selection. Clicking on a matrix entry results in the slice being loaded in the Image Editor.

The matrix mode supports the deletion of contours on a slice location within a row or column which can be performed by selecting the slice number or phase number and performing a right mouse click and selecting delete. Contours from a single phase, group of phases or slices can also be performed by directly selecting the desired phase and slice locations on the matrix and performing a right mouse click and selecting delete.

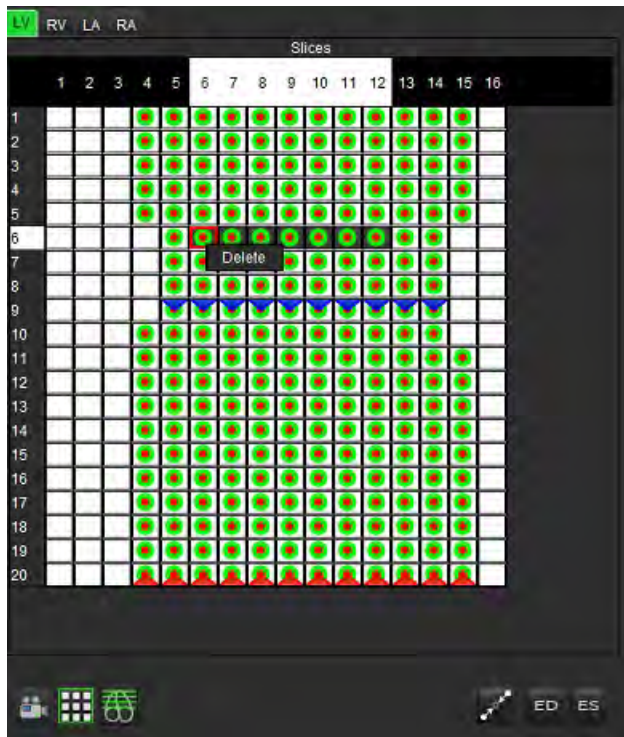FIGURE 3. Matrix Mode



Matrix Mode is used to assign the end-systolic and end-diastolic phases. This should be used when the heart rate changes during acquisition to allow an accurate measurement of end-systolic and end-diastolic volumes. Select the ES button and click on the cells in the matrix mode to set the specified slice / phase to end-systolic. Select the ED button and click on the cells in the matrix mode to set the specified slice / phase to end-diastolic. The volumes are re-calculated automatically as the end-systolic and end-diastolic image are selected.

Cross Reference Mode

Cross Reference mode displays the long axis view of an image when the short axis view is currently displayed in the image editor window. The long axis view is an orthogonal slice within an angle of the displayed image in the editor window. A drop-down menu of all the orthogonal slices available is provided, along with a button to toggle the display of the cross reference slice indicators. Use the minus and plus to navigate between slice locations.

FIGURE 4. Cross Reference Controls

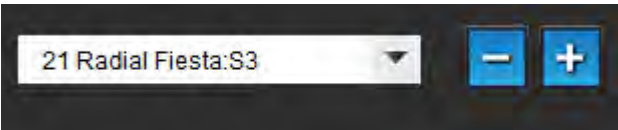


FIGURE 5. Cross Reference Mode Icon



Image Manipulation Tools

Table 3: Icons and Names




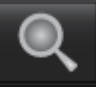



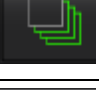
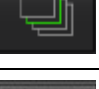










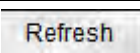
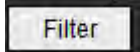
	Slice/Phase Review Toggle
	Window/Level – Select and use middle mouse button to make adjustment
	Pan – Select and use middle mouse button to make adjustment
	Zoom – Select and use middle mouse button to make adjustment
	Rotate – Select and use middle mouse button to make adjustment
	Flip Horizontal – Flips the image horizontally
	Scope All – Applies image manipulation to all slices
	Scope Current to End – Applies image manipulations from the current slice to the end slice
	Scope Current Only – Applies image manipulation to the current slice only
	Viewport Layout - Change viewer layout
	Compare Mode - Change to compare mode
	Review Mode - Change to review mode
	Show Cross Reference lines - Toggles cross reference lines on/off

Table 3: Icons and Names

	Colormap Overlay - Toggles slice classification colormap on/off
	Reset – Resets the W/L, Pan, Zoom and Rotate back to default, based on the scope setting
	Region of Interest – Provides area measurements
	Crosshair – Provides sampling of single pixel data
	Linear – Provides measurement of a straight line distance
	Label – Provides the addition of user annotation in the Editor window
	Refresh – Click button to update the Image View with newly networked images
	Filter – Sorts series by pulse sequence type according to analysis mode. Can be deselected by selecting ALL. Filters can be set under Preferences. The filter button will be green if a filter is in use.

Quick Keys

Table 4: Quick Keys

Function	Action
Image Zoom	Ctrl + Middle Mouse Button
Rotate Image	Ctrl + Shift + Middle Mouse Button
Image Pan	Shift + Middle Mouse Button
Window / Level	Middle Mouse Button
Toggle Annotation	Ctrl-T
Quit Application or Exit	Ctrl-Q
Open Browse DB	Ctrl-O
Open Preview Report	Ctrl-R
Print Report	Ctrl-P


Table 4: Quick Keys

Function	Action
Approve Exam	Ctrl-G
Load Approved Exam	Ctrl-L
Edit Preferences	Ctrl-E
Report Database	Ctrl-D
Switch Study	Ctrl-S
Function	Ctrl-1
Flow	Ctrl-2
Myocardial Evaluation	Ctrl-3
Time Course	Ctrl-4
PFO	Ctrl-5
T2 Star	Ctrl-6
T1 Mapping	Ctrl-7
T2 Mapping	Ctrl-8
3D/4D	Ctrl-9
Navigate between Slices	Left & Right Arrow Keys
Navigate between Phases	Up & Down Arrow Keys

Analysis View

The Analysis View is available for each analysis mode.

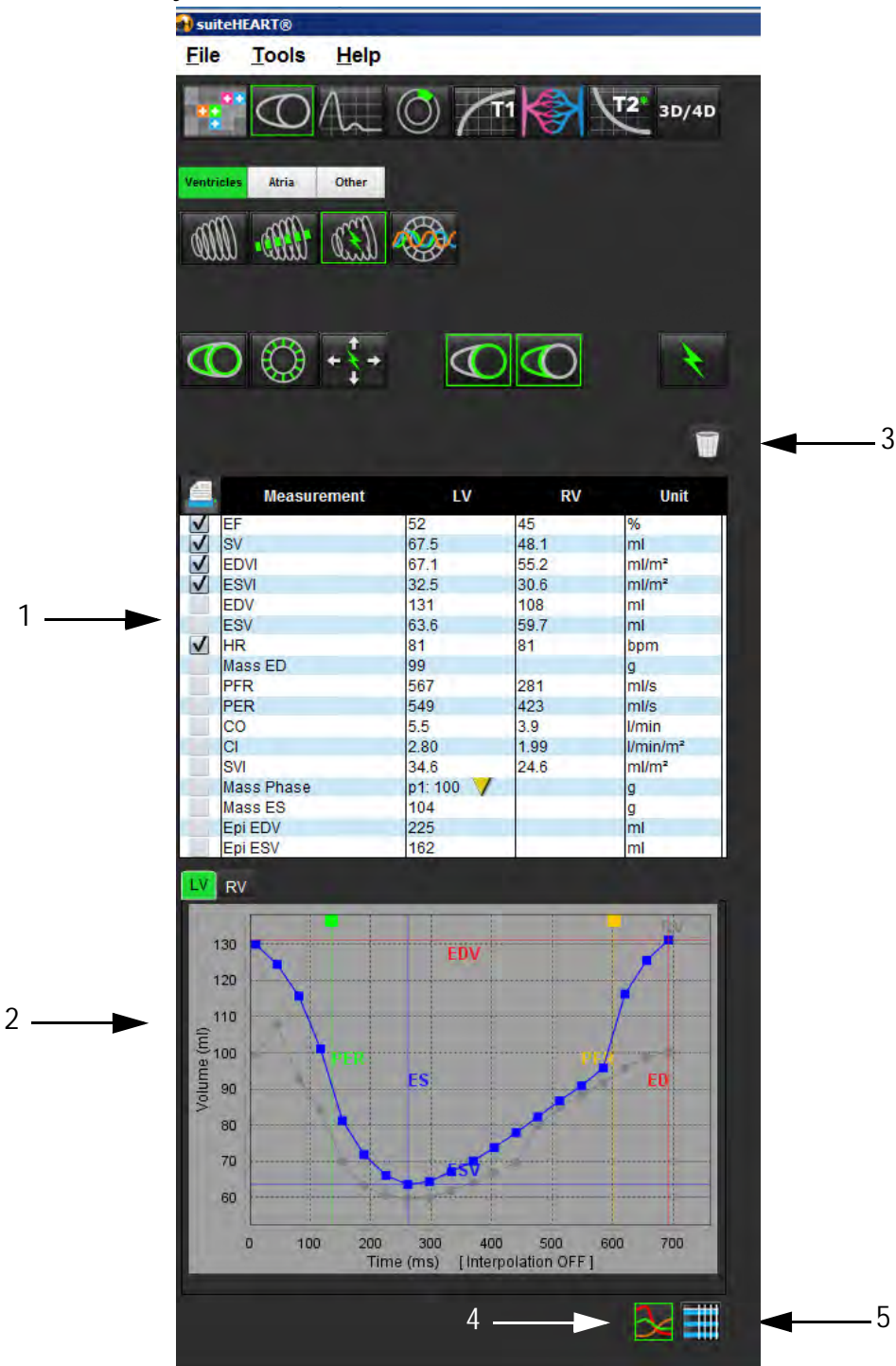FIGURE 6. Analysis View Features





1. Measurement table, 2. Curve results, 3. Reset, 4. Graphs, 5. Tables

Analysis View Review

Measurement Table

FIGURE 7. Results Parameters: Select or deselect from inclusion on the report by clicking the box next to the parameter

	Measurement	LV	RV	Unit
<input checked="" type="checkbox"/>	EF	69	65	%
<input checked="" type="checkbox"/>	SV	91.4	88.8	ml
<input checked="" type="checkbox"/>	EDVI	69.5	71.8	ml/m ²
<input checked="" type="checkbox"/>	ESVI	21.6	25.3	ml/m ²
<input checked="" type="checkbox"/>	EDV	133	137	ml
<input checked="" type="checkbox"/>	ESV	41.3	48.3	ml
<input checked="" type="checkbox"/>	HR	73	73	bpm
<input checked="" type="checkbox"/>	Mass ED	106		g
<input checked="" type="checkbox"/>	PFR	677		ml/s
<input checked="" type="checkbox"/>	PER	497		ml/s
<input checked="" type="checkbox"/>	CO	6.7	6.5	l/min
<input checked="" type="checkbox"/>	CI	3.49	3.39	l/min/m ²
<input checked="" type="checkbox"/>	SVI	47.8	46.5	ml/m ²
<input type="checkbox"/>	Mass Phase	p1: 100 		g
<input type="checkbox"/>	Mass ES	148		g
<input type="checkbox"/>	Epi EDV	234		ml
<input type="checkbox"/>	Epi ESV	182		ml



- Trashcan Icon: Resets measurements performed by the Analysis View












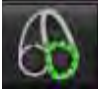




Curve Results

Curve results can be displayed as a graph or in tabular format by clicking the desired icon located in the lower right hand corner of the Analysis View.

FIGURE 8. Graph (left) and Table (right) Icons: Displays curve results



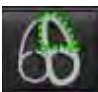
Table 5: Analysis Tools

 Left Ventricular Endocardial ROI	 Left Ventricular Local ROI
 Left Ventricular Epicardial ROI	 Left Ventricular Blood Pool ROI
 Right Ventricular Endocardial ROI	
 Right Ventricular Epicardial ROI	
 Left Ventricular Basal Line	
 Right Ventricular Basal Line	
 Right Ventricular Insertion Point	
 Left Ventricular Papillary Muscle ROI	
 Right Ventricular Papillary Muscle ROI	
 Left Atrial ROI	
 Right Atrial ROI	
 Long Axis LV Endocardial ROI	
 Long Axis LV Epicardial ROI	
 Left Ventricular Septal ROI	

Edit Tab Labels

Labels on tabs are editable if a dot resides on the tab beside the label.

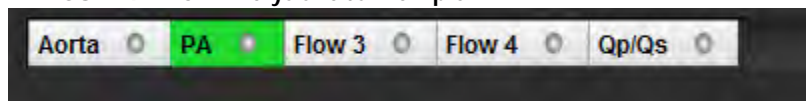
FIGURE 9. Flow Analysis Tabs Example



Edit Tab Procedure:

1. Select the dot on the tab.
2. Enter the new label name.

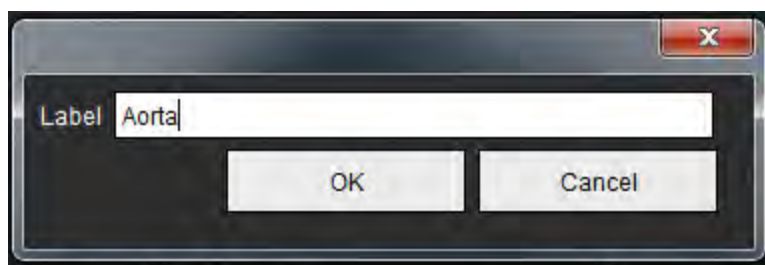FIGURE 10. Edit Tab Label Pop-up Window



Report View




suiteHEART® Software has four report views for structured reporting. Refer to Structured Reporting on page 145 for more information.

FIGURE 11. Reporting Tabs

	Name	Value
<input checked="" type="checkbox"/>	Study Date	Jan 17, 2007
<input type="checkbox"/>	Institution	
<input checked="" type="checkbox"/>	Referred By	
<input checked="" type="checkbox"/>	Copies To	
<input type="checkbox"/>	Description	
<input checked="" type="checkbox"/>	Name	suiteHEART Example Case 01
<input checked="" type="checkbox"/>	ID	AW1903342710.717.1400755457
<input checked="" type="checkbox"/>	Age	38
<input checked="" type="checkbox"/>	Sex	Female
<input checked="" type="checkbox"/>	Height(in)	0
<input checked="" type="checkbox"/>	Weight(lb)	195
<input checked="" type="checkbox"/>	BSA	0.00 [DuBois and DuBois] ▼

☒ HISTORY

NOTES



- Preview Report: Used for previewing a report



- Toggles between the review and analysis screens



- Approve Exam: Used for report signature




- Toggles between the analysis and review screens

NOTE: ROI editing can be performed on the review screen.

Browse DB

The Browse DB window provides a view of the current contents of the local database. It features a view of exams in the local database and the controls that allow you to choose which exams to view or add to the switch study listing.

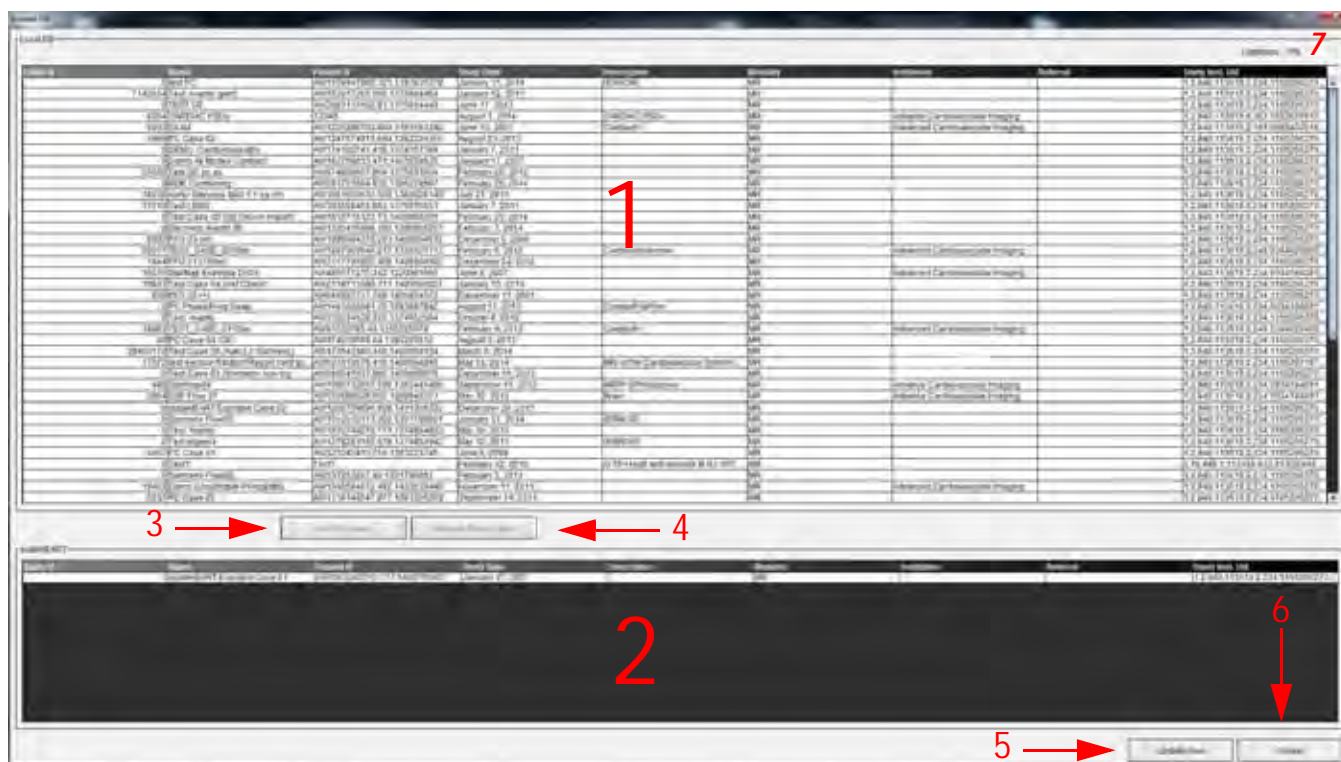FIGURE 12. Browse DB Window



1. Local database listing, 2. suiteHEART® Software database viewer, 3. Add to Viewer button, 4. Remove from Viewer button, 5. Update View button, 6. Cancel button, 7. Database

Features of the Browse DB

The Browse DB always defaults to the local database.

1. **Local database listing** – displays the exams that are stored in the local database.
2. **suiteHEART® Software database viewer** – displays exams that are in the current suiteHEART® Software database.
3. **Add to Viewer** – Adds the selected exam from local database (shown in the top portion of the window) to the suiteHEART® Software database viewing area.
4. **Remove from Viewer** – Removes the exam from suiteHEART® Software database viewing area.
5. **Update View** – Closes the Browse Database window and bring the exams in the viewable listing area to the application viewer. Used to populate the switch studies window.
6. **Cancel** – Closes Browse Database window with no changes to the list.
7. **Database** – Displays available disk space on the Database directory.

Browse DB Procedure

Exams can be viewed by selecting them from the local database, adding them to the suiteHEART® Software database Viewer list and clicking **Update View**.

Add Exams to the suiteHEART® Software Switch Study List

1. Click **File > Browse DB**.
2. Locate the exam in the database viewer and click on the exam to highlight it.
3. Click **Add to Viewer**.
4. Click **Update View**.
5. The exam now appears in the suiteHEART® Software Switch Study List.

Remove Exams from the suiteHEART® Software Switch Study List

1. Click **File > Browse DB**.
2. Locate the exam and then click **Remove from Viewer**.
3. Click **Update Viewer**.



CAUTION: Do not delete the study currently opened in the suiteHEART® Software.

Exams must be loaded into suiteHEART® Software before they can be shown in the Viewer. See “Browse DB Procedure” to learn how to populate the Switch Study List.

Switch Studies within suiteHEART® Software

1. Click **File > Switch Study**.
The Available Studies window is displayed with a list of all the exams that were previously loaded by the Browse DB procedure.
2. Select the exam.
If you choose not to switch studies after opening the Switch Studies window, click anywhere outside of the window to return to the application.

Image Management Tools

Viewer

The viewer allows for the quick review of the study with cross referencing. The viewer interface displays the listing of the series that have been acquired for the selected study with each series displayed in a viewport. New series types can be created for analysis and review within the viewer interface. User defined hanging protocols for routine acquired series can also be created to expedite study review.

NOTE: When launching a study for the first time the viewer will automatically open and all series will be displayed.

NOTE: Exporting functionality can only be performed in the suiteHEART analysis modes.

FIGURE 1. Viewer Overview



1. Study series listing, 2. Series/Slice viewport, 3. Cross Reference, 4. Save series, 5. Hanging Protocols, 6. Switch to Report tabs

Viewer Functionality

Create a New Series

The viewer allows for the creation of series types that can be used for Function, ME and Time Course analysis or for review only (custom). Series that are created will be added to the series listing for that study and are available for viewing and analysis within the suiteHEART® Software application.

NOTE: Each slice of the new series used for analysis modes for Function, ME or Time Course must have the same number of phases, be of the same acquisition prescription and must be parallel.




WARNING: The user is responsible for creating new series for analysis that contain the correct images for analysis. Incorrectly formed series may be analyzed but could produce inaccurate results. The user should be properly trained in cardiac analysis and must be aware of the slice location images copied into the new series. Do not delete original images that have been used for DICOM import.

1. Select the desired series or images from the series listing.
2. Select a group of series by performing a Shift click or a Ctrl click for adding a single series.
3. Clicking and dragging allows for the ordering of the images within the viewports.
4. To delete an image from a viewport, select the viewport and press the Delete key on the keyboard.




5. Select from the save series pane Figure 2.

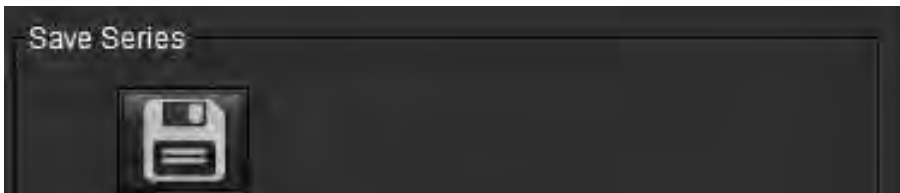FIGURE 2. Save Series Pane



6. Type in a series name for the Series description.
7. Select the appropriate series type from the pull-down menu of either Function, ME or Time Course (Figure 3). If **Custom** is selected, images with different scan planes and sequence types can be saved as a series.

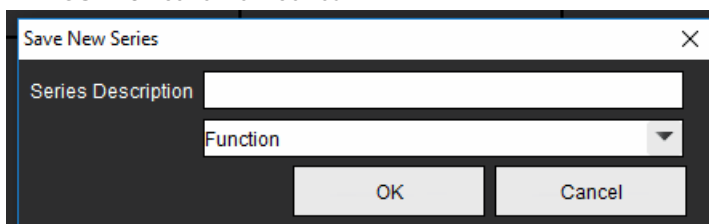FIGURE 3. Save New Series



Create a Hanging Protocol

Expedite the review process by creating a Hanging Protocol which saves a user defined viewport layout based upon the series label.

NOTE: Hanging Protocols require that the series labels are the same for each study. If series labels have been changed the images will not appear in the viewport.



WARNING: The user is responsible for creating Hanging Protocols used for review that contain the correct series types. If the series labels are changed in a study the Hanging Protocol needs to be resaved. Always review the series listing to confirm the correct series types are used for review.

1. Select the desired series or images from the series listing.
2. Select a group of series by performing a Shift click or a Ctrl click for adding a single series.
3. Clicking and dragging allows for the ordering of the images within the viewports.
4. To delete an image from a viewport, select the viewport and press the Delete key on the keyboard.




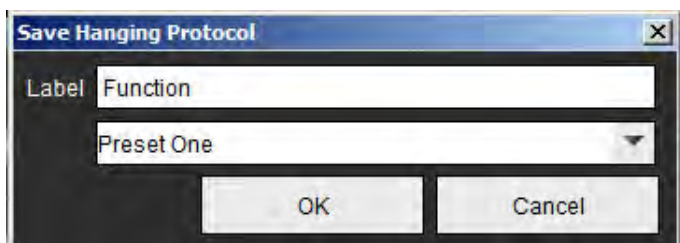
5. Select  from the Hanging Protocol pane.
6. Type in a label name and select a preset number from the pull-down menu (Figure 4).
7. Click OK to save.

FIGURE 4. Save Hanging Protocol



Accessing Reporting View Tabs



To access the reporting view tabs, click



To return back to viewer functionality, click

Compare Mode

Compare mode gives you the ability to review images/series from a current exam, or from a prior exam, simultaneously within the same interface.

NOTE: Editing of analysis types, Export DICOM Cine and Export Cine files is not available in compare mode.

NOTE: Images sent to a report in compare mode will be in bitmap format. No image manipulations will be possible on these images.



WARNING: Prior to the review or comparison of exams or series within an exam, visually confirm all exam patient indicator information for both viewers.

FIGURE 5. Compare Mode Viewer



Viewer	Callout	Description
Viewer 1	1	Series pull-down
	2	Series selector
	3	Currently viewed patient exam indicator line
	4	Image controls
	5	Viewport layout selections
Viewer 2	6	Currently viewed patient exam indicator line
	7	Exam selector
	8	Series selector
	9	Viewport layout selections
Both Viewers	10	Change scope settings
	11	Toggle for Review Mode
	12	Toggle for Analysis Mode
	13	Toggle synchronized cine

Sample Workflow



1. Select  from the lower right side of the Report View Window.
2. Select  to split the interface into two viewers, as shown in Figure 6.


FIGURE 6. View in Compare Mode



3. Change the series in Viewer 1 by using the series selection pull-down menu or right/ left arrows.
 - This upper viewer always displays the current study that has been previously launched.


4. In Viewer 2, use the series pull-down to choose a different series, within the same exam, to compare with that which is shown in Viewer 1.
 - When a viewport is selected in any viewer and if the slice is parallel such as a short axis series, the corresponding slice, based on slice location, will be highlighted.

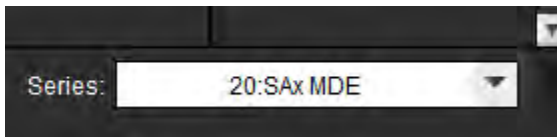FIGURE 7. Series Pull-down, Viewer 2




5. Use the exam selector, to compare a different exam in Viewer 2 to the current exam shown in Viewer 1.

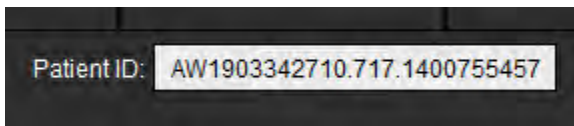FIGURE 8. Exam Selector, Viewer 2




6. Confirm proper exam selection by checking the exam indicator information for both viewers.

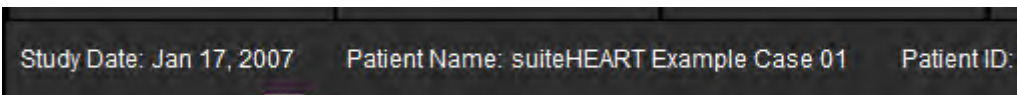FIGURE 9. Exam Indicator Information



7. Performing a right mouse click on either viewer will open the image manipulation tools.
 - The scope selection applies to both viewers.

NOTE: Performing image locate from the Images tab will not be valid if the image is from a different study.

NOTE: If a cine series is selected in both viewers and both series have the same number of phases, click the  to sync the cine views.

This page intentionally left blank.

Defining Preferences

Selecting **Preferences** from the Tools menu on the suiteHEART® Software Interface menu bar displays three options:

- Edit Preferences
- Import Preferences
- Export Preferences

IMPORTANT: It is advisable to set up user preferences prior to analyzing the first case to be reported. Changes made to preferences do not take effect until a new exam is started.

Setting Preferences

The Edit Preferences feature allows for the customization of reporting features. The global settings include:

- Report Preferences
- Authorized Report Approvers
- Series Filter
- Miscellaneous
- Auto-save Preferences
- Export Preferences

User defined result parameter ranges can be created under the Template Preference tab. Macros for structured reporting can be created under the Macro Preferences tab.

Global Settings

Report Preferences

Configures header information that appears on all reports.

FIGURE 1. Report Layout Tab

The screenshot shows the 'Report Layout Tab' with the 'Global Settings' tab active. The 'Report Preferences' section is visible, containing two checked checkboxes: 'Use the field values below in Report' and 'Support even and odd row'. Below these are input fields for 'Report Title', 'Report Sub Title 1', 'Report Sub Title 2', 'Header Line 1', 'Header Line 2', 'Header Line 3', and 'Header Line 4'. To the right of these fields is a 'Logo' section with a 'Browse' button. At the bottom, there are radio button options for 'Paper Size' (A4 and LETTER) and 'Graph Size' (Large and Small).

Report Preferences Procedure

1. From the Image View menu bar, select **Tools > Preferences > Edit Preferences**.
2. Select the **Global Settings** tab.
3. Place the cursor in the desired field of the **Report Preferences** panel and enter information.

The titles, headers, and logo will appear on a report with the specified paper size. To omit this information from the report deselect the "Use the field values below in Report" checkbox. This will take effect for all patient reports that are printed.

Checking "support even and odd row" will highlight the result rows on the report.
4. To insert a site logo into the report, prepare the file in a jpeg, png, or gif format and save to hard drive or CD-ROM. Select **Browse** under the Logo section and locate the file from the system browser window. Select the proper logo file and select **Open**.

The logo should now appear on the report preferences panel.

5. Select **Save and Exit** to store your entries and close Edit Preferences.
 - Select **Cancel** to exit the window without accepting any changes.
 - Select **Reset** to reset all values on the Global Settings tab without exiting the window.

Authorized Report Approvers

The application has a report approval feature that locks the final report. Once locked, the report cannot be modified. The credentials of the approvers are added, modified and deleted as described.

FIGURE 2. Authorized Report Approvers




Manage Report Approvers Procedure

1. From the Image View menu bar, select **Tools > Preferences > Edit Preferences**.
2. Select the **Global Settings** tab and place the cursor in the **Authorized Report Approvers** panel.
3. Select the **Add** tab to add a user name to the authorized approvers list.
 - Enter the user name.
 - Enter the password twice.
 - Select **Add**.
4. Select the **Modify** tab to change the password of a user on the authorized approvers list.
 - Select the user to modify.
 - Enter the old password.
 - Enter the new password twice.
 - Select **Apply**.
5. Select the **Delete** tab to delete a user from the authorized approvers list.
 - Select the user(s) to delete.
 - Select **Delete**.

Series Filter

Based on analysis modes types, a series filter can be applied to expedite selection of the appropriate series for analysis. The filter preferences can also be selected during analysis by clicking the filter button on the main panel, above the thumbnail view.

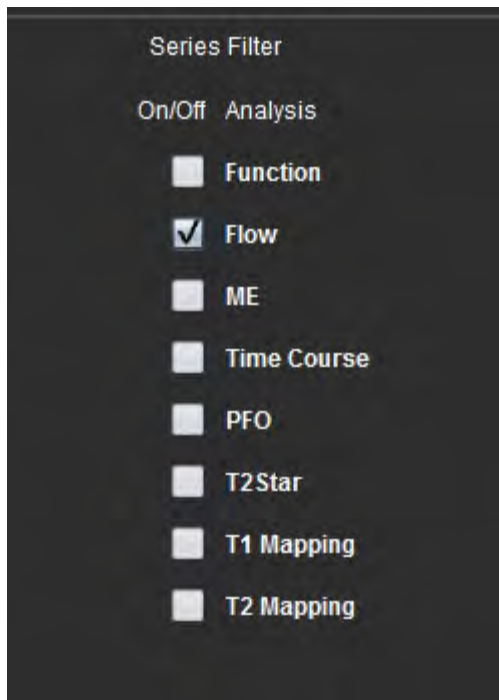FIGURE 3. Filter Preferences




Select Filter Preference

1. From the Image Viewer menu bar, select **Tools > Preferences > Edit Preference**.
2. Select Global Settings tab.
3. Click the appropriate on/off selection for each analysis type.
4. Select **Save and Exit** to store your entries and close Edit Preferences.
 - Select **Cancel** to exit the window without accepting any changes.
 - Select **Reset** to reset all values on the Global Settings tab without exiting the window.

NOTE: If a series filter has been applied and the required series is not present, a message will appear: "There are no series associated with the selected analysis type." Clicking OK will disable the filter and display all of the series in the study.

Miscellaneous

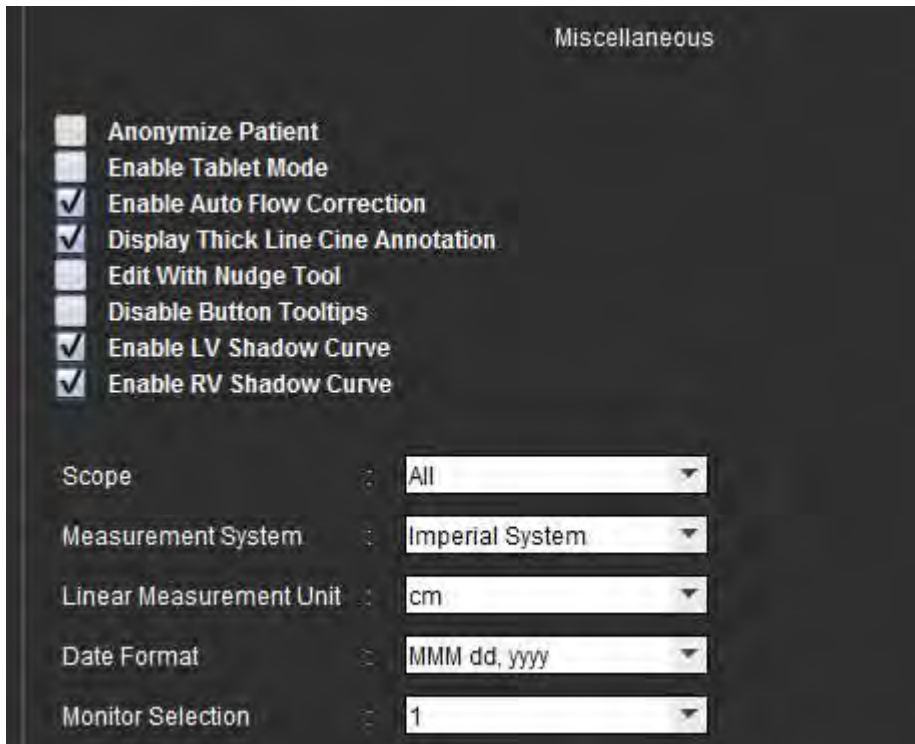FIGURE 4. Miscellaneous Panel




Edit Miscellaneous Parameters Procedure

1. From the Image View menu bar, select **Tools > Preferences > Edit Preferences**.
2. Select the **Global Settings** tab and place the cursor in the **Miscellaneous** panel.
3. Check the **Anonymize Patient** check box to hide the patient name and id from the report.
All patients' names will be displayed as "anonymous" and the ID will be blank. These changes will apply to the report and Image View.
4. Enable Tablet Mode to run the application on a tablet.
5. Check the **Enable Auto Flow Correction** check box to automatically perform auto phase error correction.
6. Check the **Display Thick Line Cine Annotation** check box to show annotation as thick lines.
7. Check the **Edit with Nudge Tool** check box to turn on the editing tool for all analysis sessions.
8. Check the **Disable Button Tooltips** to hide tooltips.
9. Check either **Enable LV** or **RV Shadow Curve** to display both curves.
10. Select a default setting for **Scope**, the **Measurement System**, a **Linear Measurement unit**, the **Date Format** and **Monitor** from the pull-down menus.
11. Select **Annotation Font Size** by clicking on the radial button.
12. Select **Save and Exit** to store your entries and close Edit Preferences.
 - Select **Cancel** to exit the window without accepting any changes.
 - Select **Reset** to reset all values on the Global Settings tab without exiting the window.

Auto Save Preferences

The Auto Save Preferences panel sets the time interval in minutes for the system to automatically generate secondary capture files (SCPT) containing the current analysis. These SCPT files are saved along with the exam. Every time the auto save time interval is reached, a new secondary capture image is added to the series.

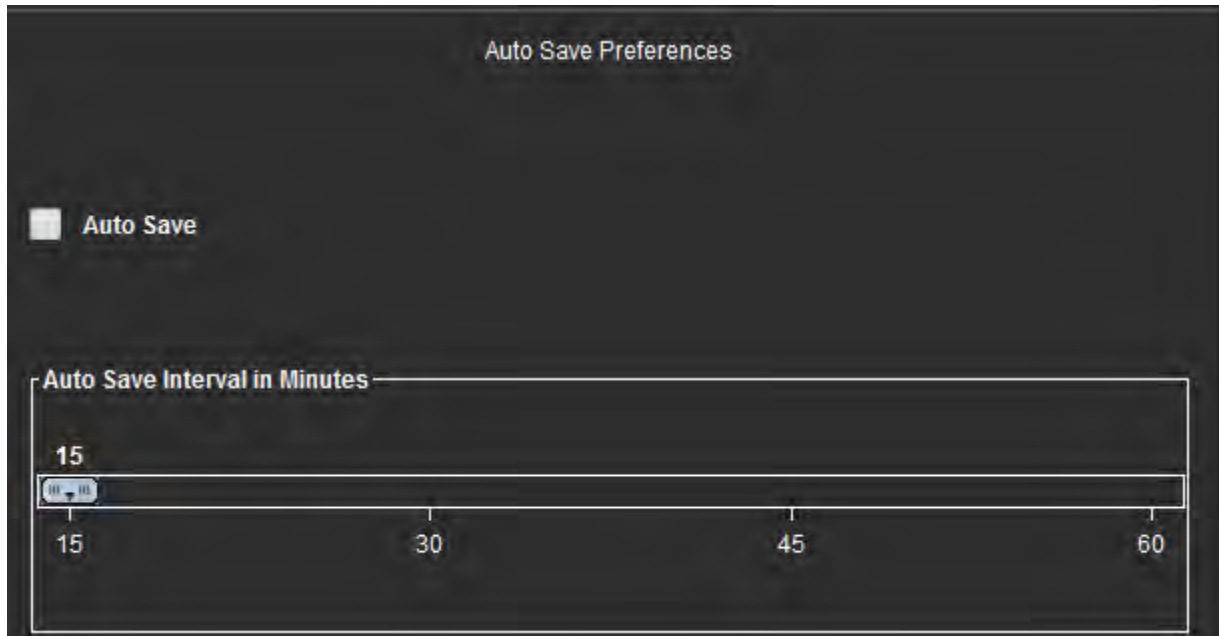FIGURE 5. Auto Save Preferences Window




Edit Auto Save Preferences Procedure

1. From the Image View menu bar, select **Tools > Preferences > Edit Preferences**.
2. Select the **Global Settings** tab and place the cursor in the **Auto Save Preferences** panel.
3. Select the **Auto Save** check box to enable the auto save feature.
4. Drag the auto save interval marker to the desired time in minutes.
5. Select **Save and Exit** to store your entries and close Edit Preferences.
 - Select **Cancel** to exit the window without accepting any changes.
 - Select **Reset** to reset all values on the Global Settings tab without exiting the window.

Export Preferences

The Export Preferences panel allows you to select the image formats for exporting image and video data. The exporting feature allows you to create uncompressed AVI movies, compressed QuickTime movies, and JPEG, TIFF and PNG files of the image data.

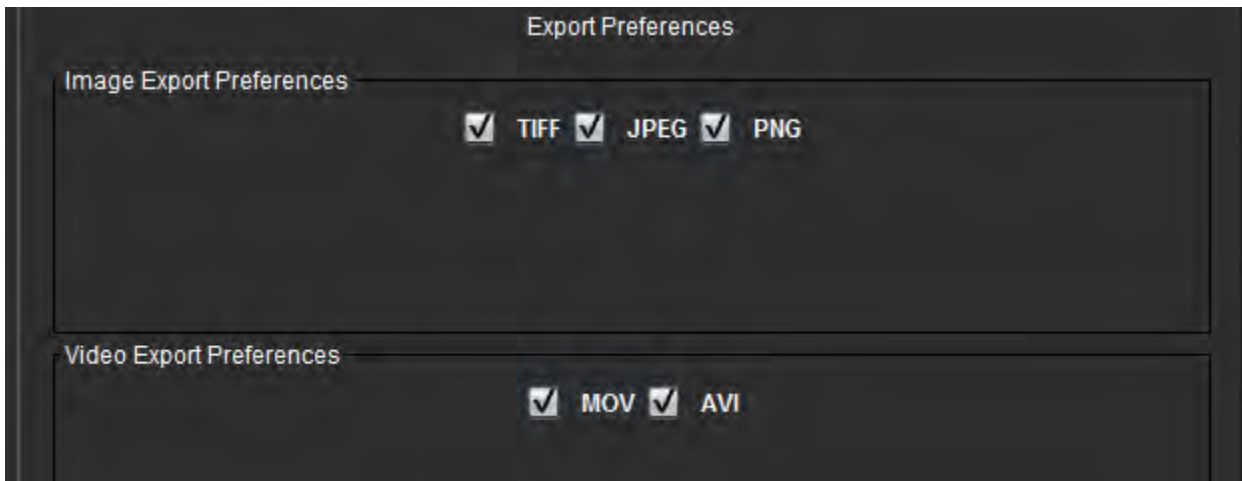FIGURE 6. Export Preferences Window




Export Preferences Procedure

1. From the Image View menu bar, select **Tools > Preferences > Edit Preferences**.
2. Select the **Global Settings** tab and place the cursor in the **Export Preferences** panel.
3. Select the appropriate image data types.
4. Select **Save and Exit** to store your entries and close Edit Preferences.
 - Select **Cancel** to exit the window without accepting any changes.
 - Select **Reset** to reset all values on the Global Settings tab without exiting the window.

Template Preferences

The application provides a tool to create user defined templates based on age, BSA, and weight that provide a structured workflow for measuring and reporting of specific quantitative parameters.

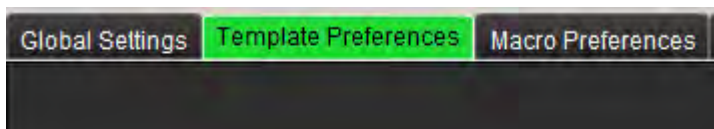FIGURE 7. Template Preferences Tab




Considerations

Before starting analysis, the user defined template must be selected from the main interface. Click on the **Default** button at the upper right and select the template to be used. Changing the template after performing analysis will apply the preference range applied in the template.

NOTE: Imported studies with previous suiteHEART analysis may show the name of the template used for that study. That template may not be available in the current software.

It is recommended that if two systems are being used for analysis, create the template preference file on the first system and then import into the second system. Template preference files imported from a different system will overwrite the template preferences if they have already been created on that system.

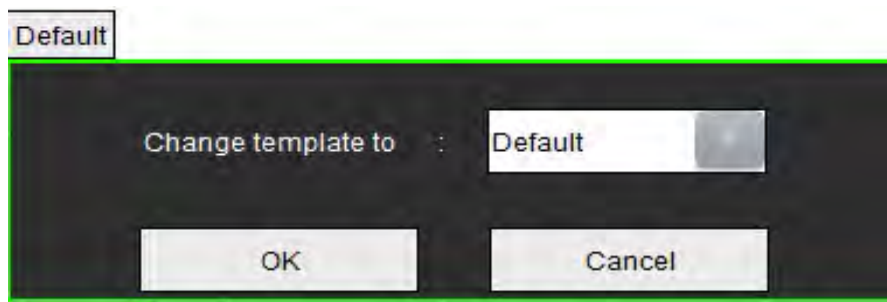FIGURE 8. Change Template



Create Template Procedure


The following is a guide to creating a user defined template. It is up to the clinician's judgement to verify the validity of the parameter range utilized.

Create a Template

All new templates are created initially by duplicating a pre-existing template. The Default template will be used in the example since it is shipped with the product and always available. The default template is not editable. To create a user defined template perform the following:

1. Select **Tools > Preferences > Edit Preferences**.
2. Select the **Template Preferences** tab.
3. Click **Create New** button.
4. Select the preference range of either age, BSA, or weight.

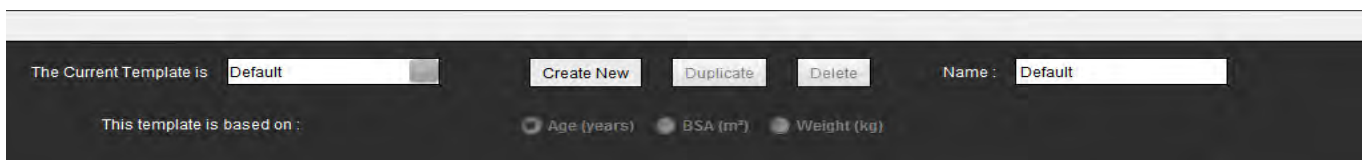FIGURE 9. Change Template Window



5. Type in a new name for the template.
When a new name is entered, **The Current Template is** pull-down menu will update.
6. Enter range preferences for the desired parameters.
7. Select **Save and Exit**.
- Select **Cancel** to exit the window without saving any changes.

Duplicate Template

1. Select **Tools > Preferences > Edit Preferences**.
2. Select the **Template Preferences** tab.
3. Select the template from **The Current Template is** pull-down menu.
4. Click the **Duplicate** button.


Delete a Template

1. Select **Tools > Preferences > Edit Preferences**.
2. Select the **Template Preferences** tab.
3. Select the template from **The Current Template is** pull-down menu.
4. Click the **Delete** button.

Edit the Preference Ranges

1. Select **Tools > Preferences > Edit Preferences**.
2. Select the **Template Preferences** tab.
3. Select a template other than default.

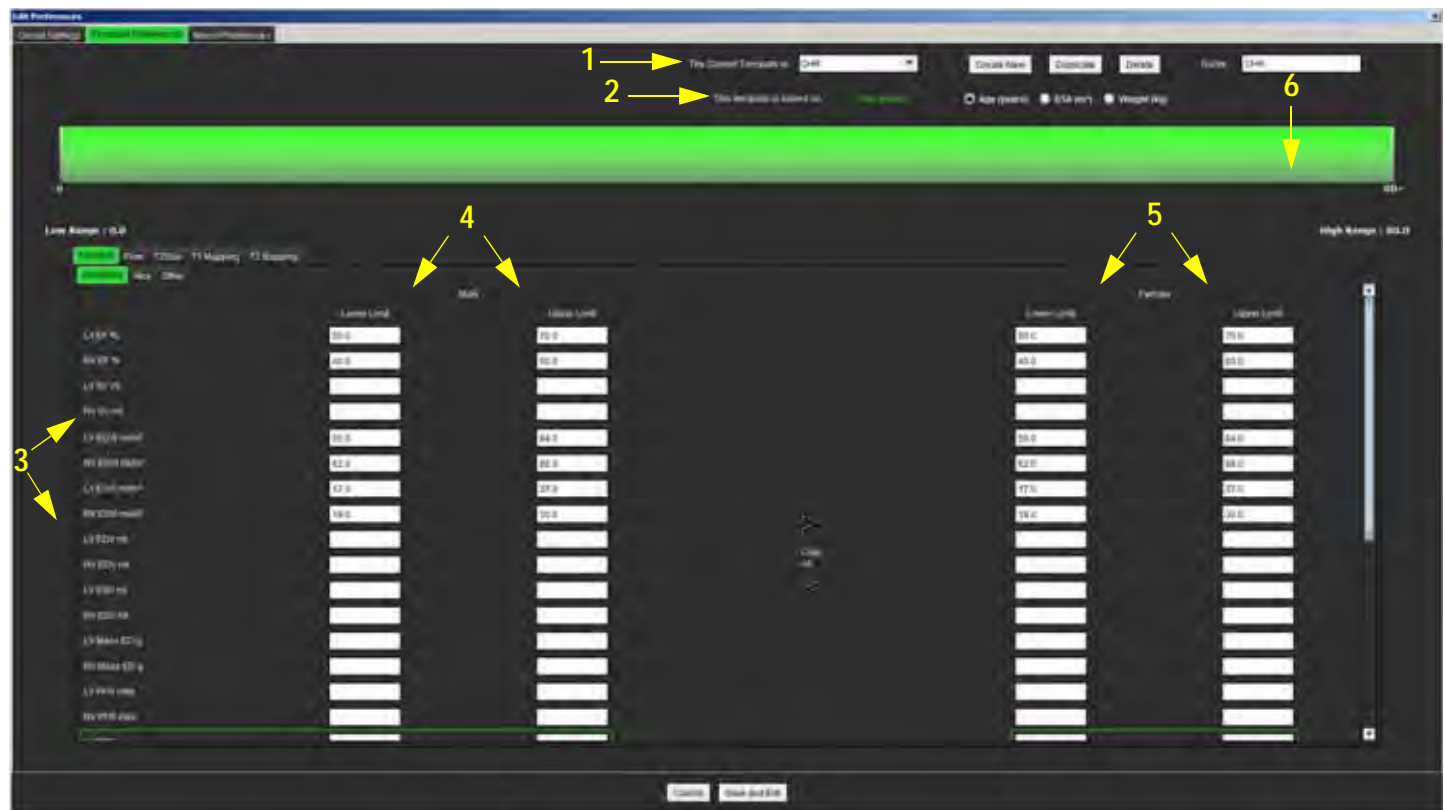FIGURE 10. Template Preferences



1. Current template, 2. Category selection, 3. Parameter measurements per analysis, 4. Male upper and lower limits, 5. Female upper and lower limits 6. Range bar.

4. Select the desired template category. Selections are Age, BSA, and Weight.

NOTE: The selected template will be the one applied to the session.

5. Left-click on the Range bar to activate.
The bar turns green when active.

6. Right-click on the Range bar to create a range divider bar in the center of the Range bar.
 - The range divider bars can be dragged to adjust the location.
 - Multiple range divider bars can be created.
 - Range divider bars can be deleted by placing the cursor close to the bar and selecting **Delete Range** from the right mouse menu.
7. Enter the parameter range values for the appropriate analysis mode. Enter both the lower and upper limits. Differentiate between male and female values where necessary. Use the **Copy All** arrows to copy values between genders. Use the scroll bar to navigate to the measurements for all the analysis types.



WARNING: Values entered for parameter ranges are the sole responsibility of the user. Confirm all parameter ranges prior to analysis. Incorrect parameter values could lead to misdiagnosis.

8. Select **Save and Exit** to store your entries and close Preferences.
 - Select **Cancel** to exit the window without accepting any changes.

NOTE: For a template to be valid, parameter values must be entered as numerical numbers with both upper and lower values entered. If inconsistencies in the values are found, the following message will appear “Invalid normal range selected. Please correct and save again.” The parameter needing correction will be highlighted in red. Saving a blank template is not allowed and will cause the following message “Unable to Save Template(s)” to display.

NOTE: See Appendix A for more information.

Macro Preferences

Macros can significantly reduce the time spent reporting a cardiac MRI case. All macros are independent from templates. The macros’ streamlined user interface automates tasks, including the following:


- Generate predefined clinical impressions and techniques that can be automatically inserted into the report.
- Automatically insert quantitative results from the analysis reporting windows.

Add an Impressions Macro

NOTE: Creating a History or Technique macro follows the same workflow as creating an Impression macro.

1. Select **Tools > Preferences > Edit Preferences**.
2. Select the **Macro Preferences** tab.
3. Select **Add Impressions Macro**. A new button appears in the Impression Macros panel.

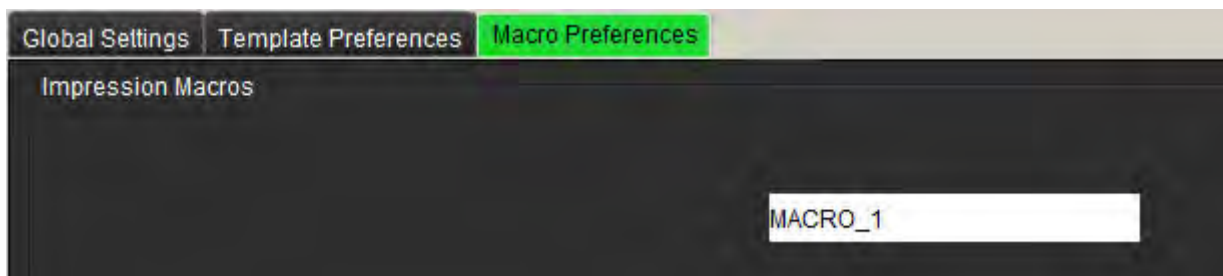FIGURE 11. Impression Macros Window




4. Place the cursor inside the new button field and edit the name as desired.

NOTE: The macro buttons created can be reordered. Click and drag the desired button to a new position within the list.

Enter the Macro Text

1. Place the cursor in the Macro Information text box and enter relevant text.
2. To enter a calculation, select any of the analysis tabs below and select the desired parameter button, which will be automatically entered into the macro information. In this example, the LV Ejection Fraction parameter was selected and entered at the end of the text.

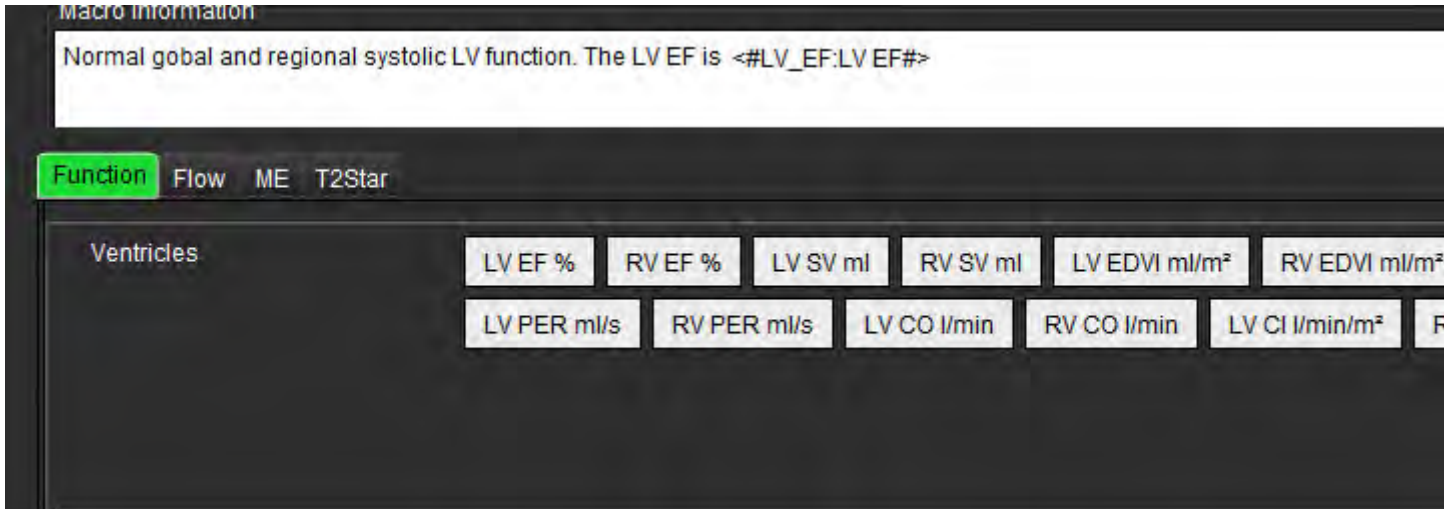FIGURE 12. Macro Information Window



3. Select **Save and Exit** to save your changes to the new macro and exit the Macro Editor.
 - Select **Cancel** to exit the Macro Editor without saving changes.

Execute a Macro


As a prerequisites to macro execution, analysis results must be generated prior to executing macros that involve numerical calculations. Technique and Impression macros can be created to automate report generation.

Delete a Macro

1. Select **Tools > Preferences > Edit Preferences**.
2. Select the **Macro Preferences** tab.
3. Select the macro from the list.

In the example shown, the macro named MACRO_3 is selected for deletion.

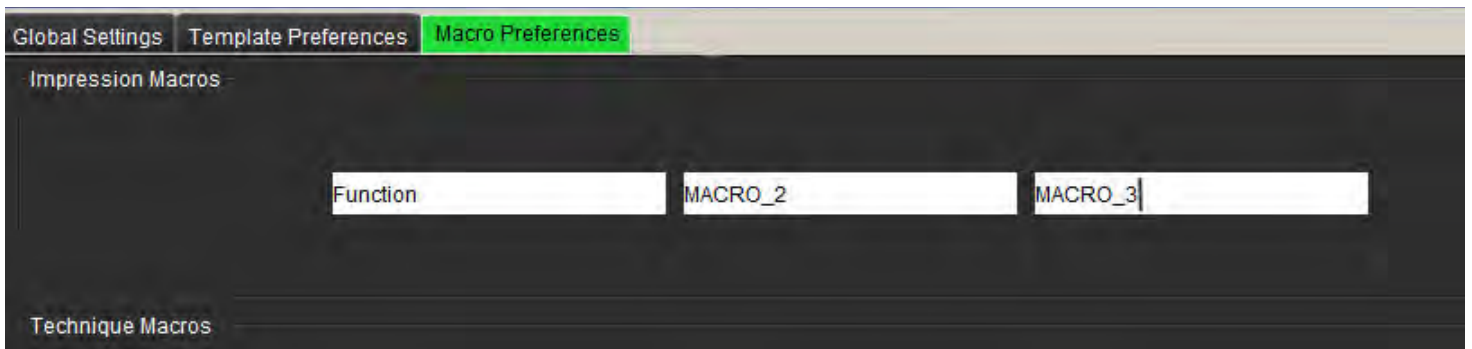FIGURE 13. Macro Selection List



4. Select **Remove Selected Macro(s)**.


Import Preferences

Templates can be imported from the file system.

Import Preferences Procedure

1. Select **Tools > Preferences > Import Preferences**.

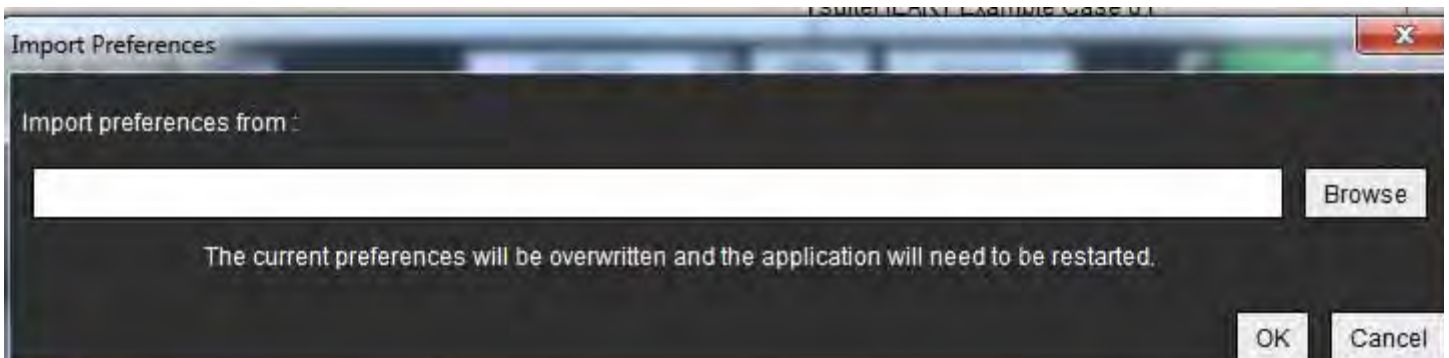FIGURE 14. Import Preferences Window



2. Select the **Browse** button, select the location of the preference file and then select the **Open** button.
3. Select **OK** to perform the import procedure as defined.
 - Select **Cancel** to exit the window without importing the template

NOTE: Importing preferences from prior versions (3.0.1 or below) of suiteHEART® Software is not supported. Please contact NeoSoft Support at service@neosoftmedical.com for help with importing preferences from prior versions.


Export Preferences

Templates can be exported to the file system.

Export Preferences Procedure

1. Select **Tools > Preferences > Export Preferences**.

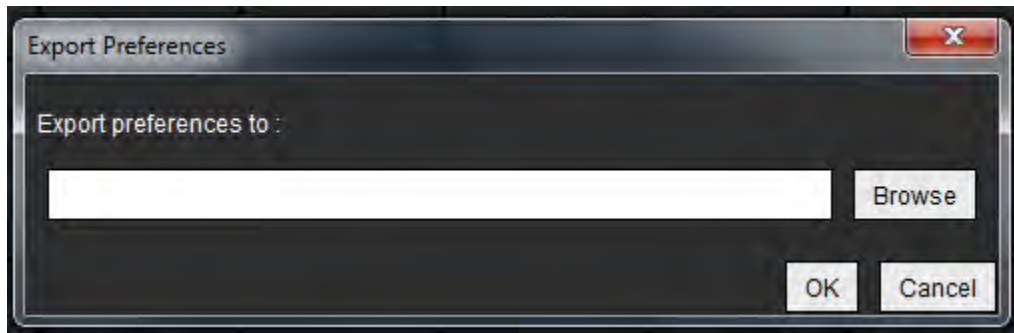FIGURE 15. Export Preferences Window



2. Select **Browse**, select the folder in which to place the preference file and then select **Save**.
3. Select **OK** to perform the export procedure as defined.
 - Select **Cancel** to exit the window without exporting the template.

This page intentionally left blank.

Editing Contours


Editing contours, as described in this section, is available in all analysis modes. This feature is available in both the Editor Window and in Review Mode.

Contour Edit Options

Conventional Editing

1. In the Editor Window, left mouse click on the contour. The contour will turn purple when selected.
2. Left mouse click and drag on the center of the contour to move it, as shown in Figure 1.
 - If the selected contour was created using the point spline method, the points are displayed for editing. Left mouse click and drag any of the points to adjust the contour size and shape as shown in Figure 1.
 - If the selected contour was created using the free-hand trace tool, left mouse click and hold to update the contour.

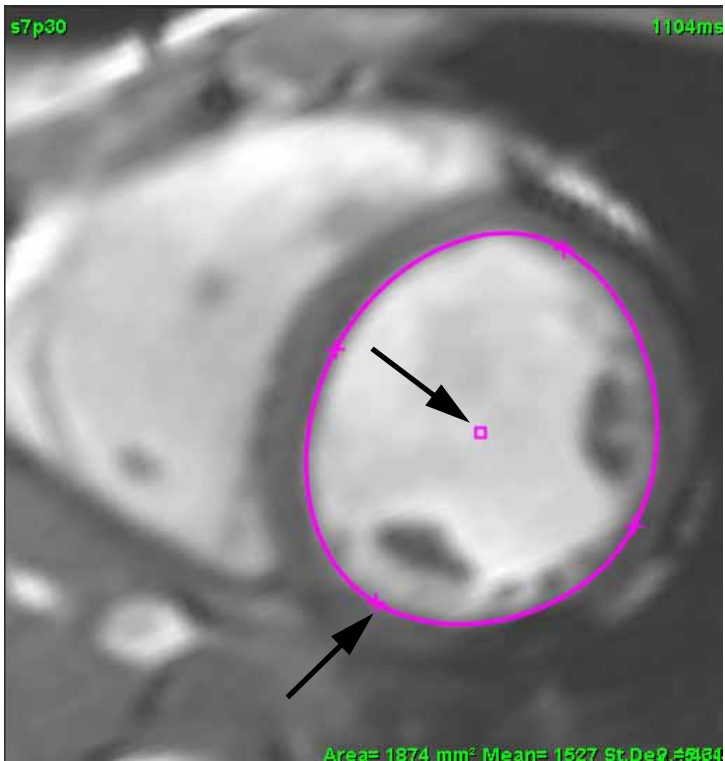
FIGURE 1. Conventional Contour Edit



Nudge Tool

1. To activate the nudge tool, left mouse click on the contour to select it. Then right mouse click and select the nudge tool from the pop-up menu, as shown in Figure 2.
 - When the nudge tool is applied, the selected point spine ROI automatically becomes a free-hand ROI.

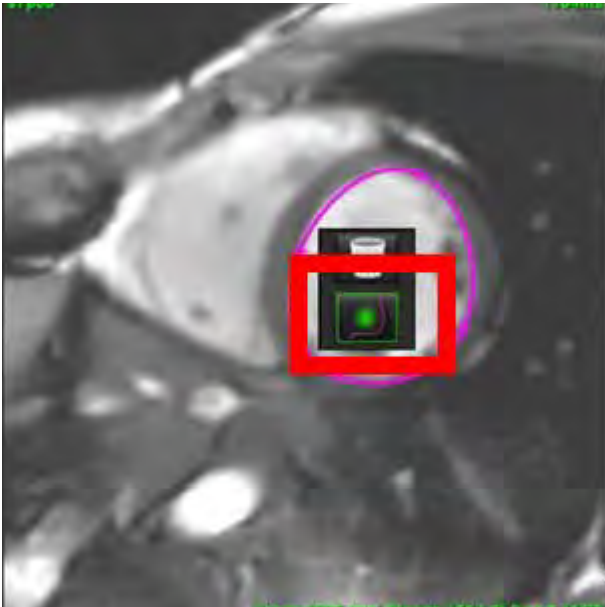
FIGURE 2. Nudge Tool Activation



2. The cursor will appear as a square. Position the cursor away from the ROI and press and hold the left mouse button. The nudge tool will appear, as shown in Figure 3.


NOTE: The size of the nudge circle defaults to the size that is an equal distance from the mouse point to the selected ROI. Reposition the cursor to change the size.

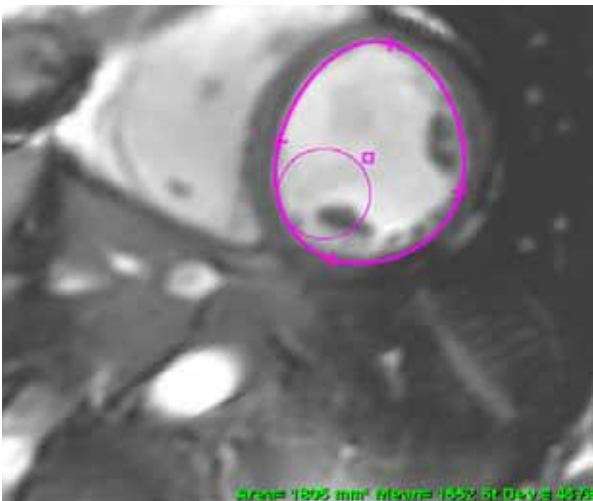FIGURE 3. Nudge Tool



3. To deactivate the nudge tool, left mouse click on the contour, then right mouse click and select the nudge tool from the pop-up menu, as shown in Figure 4.

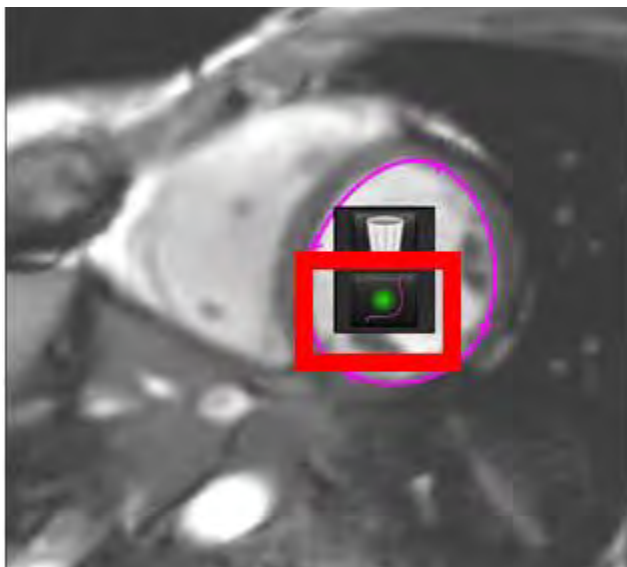FIGURE 4. Nudge Tool Deactivation




NOTE: The default on/off state of the nudge tool can be set in Preferences.

Delete a Contour

1. Left mouse click on the contour to select it and press the delete key on the keyboard.
- or
2. Left mouse click on the contour to select it, then right mouse click and choose the trashcan from the pop-up menu, as shown in Figure 5.

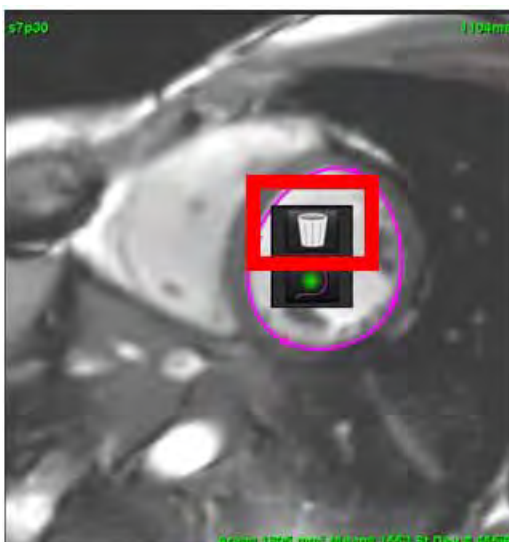FIGURE 5. Contour Deletion



This page intentionally left blank.

Function Analysis

The user is responsible for the accurate and complete placement (and correct assignment) of all regions of interest (ROIs), including those generated or modified by the auto segmentation algorithms. The quantitative values generated by the software depend on the accurate and complete placement (and correct assignment) of these regions of interest.

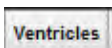
The study Preprocessing feature allows for the preprocessing of function analysis. Refer to the suiteDXT Instructions for Use NS-03-009-0001.

This section details the typical steps used for a cardiac function analysis. The sample workflows provide an overview of the steps used in the application to complete a cardiac function analysis. The procedures describe how to perform the following measurements:



WARNING: Incorrect scan plane may cause inaccurate analysis results. See Appendix B.


There are three categories for analysis:



- Includes volume analysis for the left (LV) and right ventricles (RV)



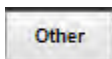
- Includes volume analysis for the left (LA) and right atrium (RA).



- Includes pre-defined linear measurements and user defined measurements can be added.

Ventricles

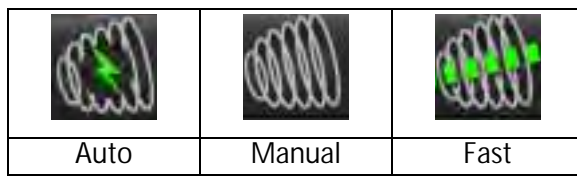Select the analysis type:




NOTE: Only one set of results are saved. If the method is changed, the prior analysis is deleted.

IMPORTANT: It is recommended that you are qualified in performing cardiac analysis, if the analysis results are to be used to reach a diagnosis.



WARNING: The application assists in the analysis of the images only and does not automatically produce quantifiable results. The use and placement of quantitative measurements is at the discretion of the user. Misdiagnosis can occur if measurements are inaccurate. Measurements should only be created by a properly trained and qualified user.

To reset ventricular results, click the  and select

Reset LV ROI
Reset RV ROI
Reset All
Reset Slice
Reset Slice (LV ONLY)
Reset Slice (RV ONLY)

NOTE: Matrix mode can be used for the deletion of contours.

Calculate Index Measurements

1. Select **History Tab**.
2. Enter patient **Height** and **Weight**.


The end-diastolic volume index, end-systolic volume index, cardiac output index and stroke volume index measurements are calculated and added to the Measurement table.


NOTE: The BSA calculation method can be selected under the History tab located on the Report View.

Auto LV & RV Segmentation

The auto segmentation feature produces standard parameters of cardiac function without anatomical input. After segmentations results are generated, ROI types can be selected or deselected for viewing. Segmentation editing can also be performed from user input.

To start LV & RV segmentation perform the following:

1. Select the short axis series and adjust the window/level.
2. Click .

3. Click  for auto segmentation.

4. Make the appropriate selections from the segmentation tool bar, as shown in Figure 1.

5. Select  to start auto segmentation.

FIGURE 1. Segmentation Toolbar

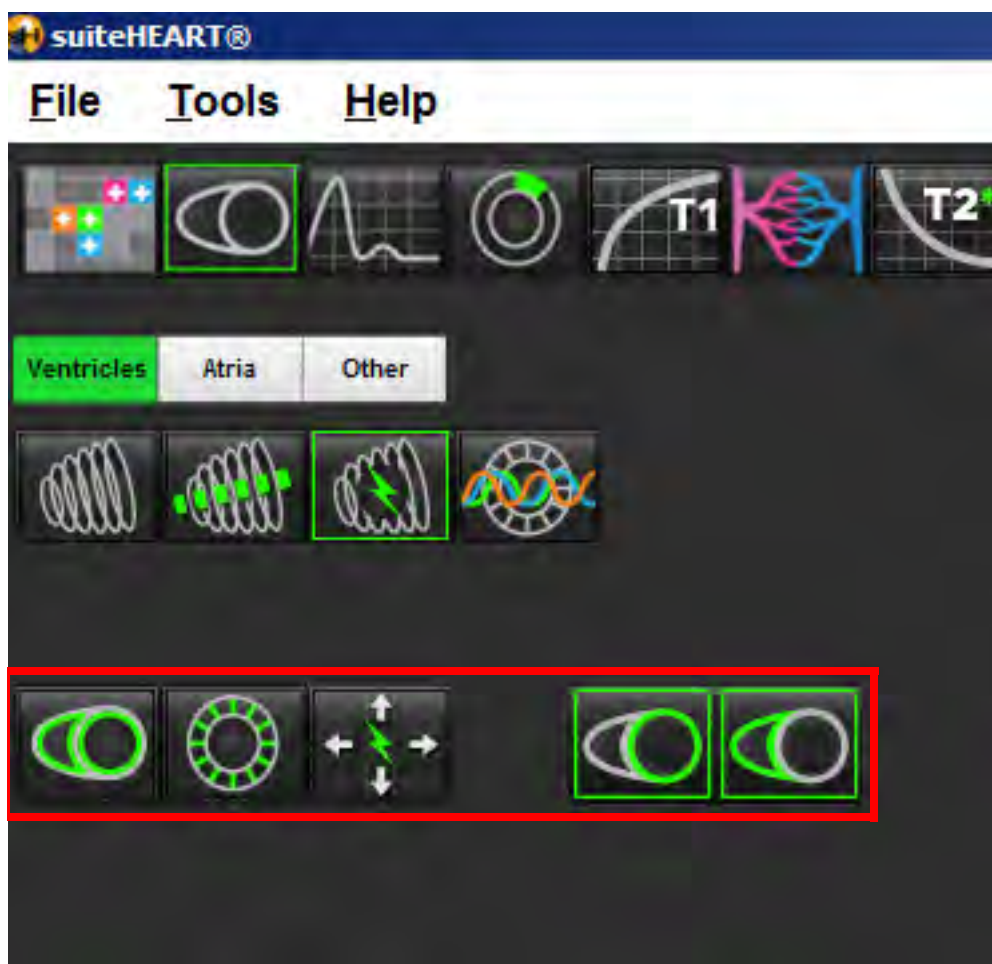


Table 1: Auto Segmentation Contour Types








				
Smooth Mode – Includes the papillary muscles in the ventricular volume.	Papillary Mode – Excludes the papillary muscles from the ventricular volume.	Show endocardial and epicardial contours. (Must be selected initially before the start of auto segmentation.)	Show endocardial contours.	Show chords.

Table 2: Auto Segmentation Propagation Types

			
Segment all phases and all slices	Segment all slices in a single phase	Segment a single slice in all phases	Segment showing contours for ED/ES phases only

Table 3: Ventricular Selection


	
Left Ventricle - Generate segmentation or display	Right Ventricle - Generate segmentation or display



Perform Auto Segmentation for All Phases and All Slices


This option is required for the generation of regional analysis, dyssynchrony and valve plane analysis results.

1. Select the short axis series and adjust the window/level.

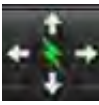
2. Click .

3. Click  for auto segmentation.

4. From the segmentation bar select either smooth  or papillary mode .

5. To generate myocardial mass results, select .

NOTE: If only endocardial traces are initially generated, then selecting both endocardial and epicardial traces or chords for review will issue a warning symbol.

6. Select  for all phases and all slices.

7. Select either  or  or both.

NOTE: For optimal RV segmentation, select both the epicardial and endocardial traces.

8. Select  to start auto segmentation.

Review Segmentation Accuracy/Editing

1. Play the short axis series in cine mode and review the accuracy of the contours.

2. Edit any contours that are inaccurate.

3. Review placement of RV insertion point on each slice. Correct the RV insertion point for the basal line slices.


4. Review matrix mode and confirm ED and ES assignments.



NOTE: Contour editing is supported for smooth mode. Perform a contour edit and select start auto segmentation.


Perform Auto Segmentation for All Slices in a Single Phase

1. Select the short axis series and adjust the window/level.

2. Click .

3. Click .


4. From the segmentation bar select either smooth  or papillary mode .


5. To generate myocardial mass results, select .

6. Review the short axis images and select the end-diastolic phase.

NOTE: If only endocardial traces are initially generated, then selecting both endocardial and epicardial traces or chords for review will issue a warning symbol.

7. Select either  or  or both.

8. Select  for all slices in a single phase.

9. Select  to start auto segmentation.

10. Review the short axis images and select the end-systolic phase.

11. Select  to start auto segmentation.

Review Segmentation Accuracy/Editing

1. Play the short axis series in cine mode and review the accuracy of the contours.

2. Edit any contours that are inaccurate.

3. Review matrix mode and confirm ED and ES assignments.

Basal Line Placement


The base of the ventricle can be defined using long axis cine images. The identification of the basal plane can potentially improve auto segmentation results. It is recommended to perform basal placement after performing auto segmentation.

1. To ensure optimal basal segmentation for the LV, select a 2-Chamber view in the cross-reference mode.




2. Select .
3. Define the basal line, as shown in Figure 2. Review the placement of the basal line on the appropriate end-systolic and end-diastolic phases using the cine controls.

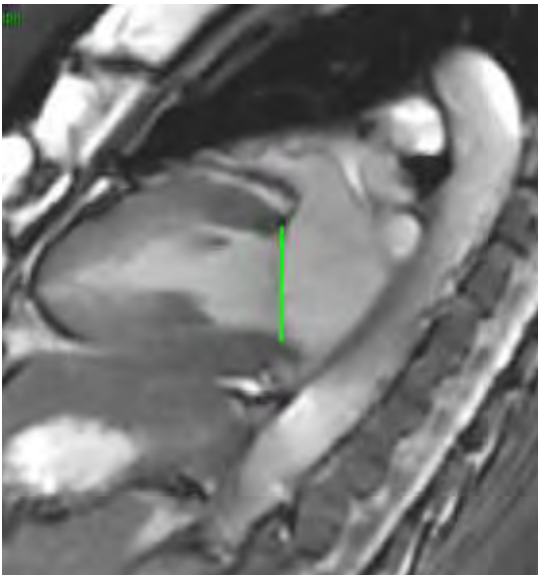FIGURE 2. Basal Line




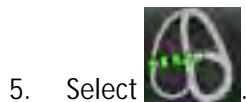4. To ensure optimal basal segmentation for the RV, select a 4-Chamber view in the cross-reference mode.



5. Select .
6. Define the basal line, as shown in Figure 3. Review the placement of the basal line on the appropriate end-systolic and end-diastolic phases using the cine control.

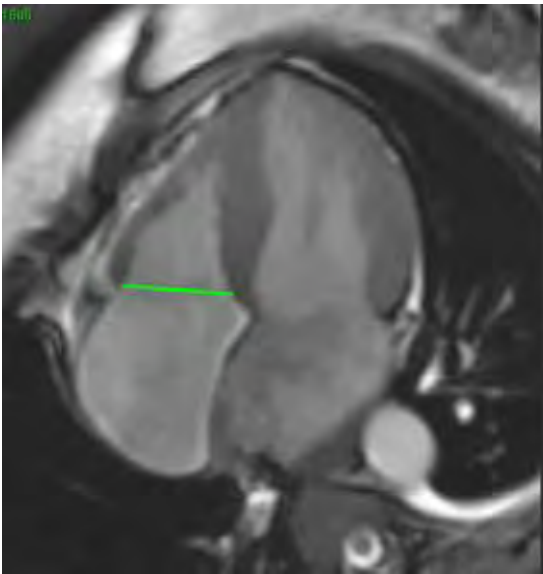FIGURE 3. Basal Line Definition



7. Review the updated calculations by reviewing the cross-reference slices in relationship to the basal line and review the ED and ES assignments in matrix mode. See “Reviewing End-Diastolic (ED) and End-Systolic (ES) Assignments” on page 57 for detailed information.
8. To reset the result back to the original value, right mouse click and hold directly on the basal line to select delete or left mouse click on the basal line and use the delete key on the keyboard.


Interpolation Functionality for Auto

NOTE: When performing auto segmentation function analysis for either the LV or the RV the interpolation functionality is turned “off.”

Interpolation functionality can be applied for the following conditions:

- After performing “All Slices Single Phase” and if a trace is removed from a slice, the interpolation can be enabled by selecting interpolation “On” as shown in Figure 4.
- If the same cardiac phase is assigned across slices for either end-systole or end-diastole and/or a slice has been skipped the basal line interpolation can be applied automatically.
- If “All Slices All Phases” has been performed and if the ED and ES assignments are all the same phase the basal line interpolation can be applied automatically.

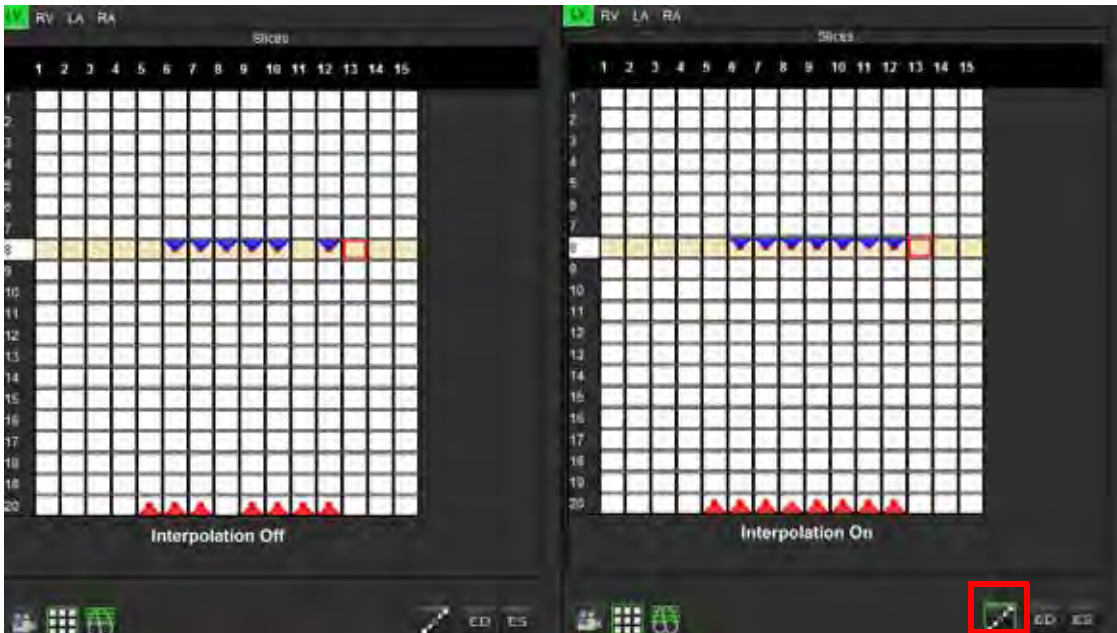FIGURE 4. Interpolation




Reviewing End-Diastolic (ED) and End-Systolic (ES) Assignments

Upon completion of performing segmentation, review the matrix mode and confirm the end-diastolic or end-systolic assignments.

NOTE: For any ventricular volume analysis type if the End-Diastolic (ED) or End-Systolic (ES) traces are performed on a different phase the following message will appear.

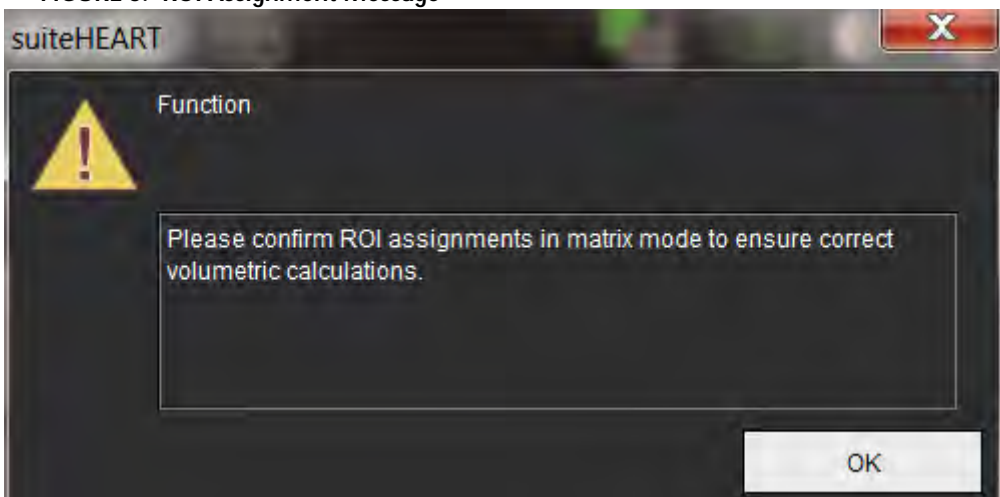FIGURE 5. ROI Assignment Message




1. Select the Matrix mode  button.
2. The view port changes showing a matrix representing all of the slice locations and phases acquired. In Figure 6 all of the LV end-diastolic phases have been assigned as indicated by the red triangles. The blue triangles indicate assigned LV end-systolic phases. The red dots represent phases not yet assigned.

FIGURE 6. Assignment of LV End-Diastolic Phases


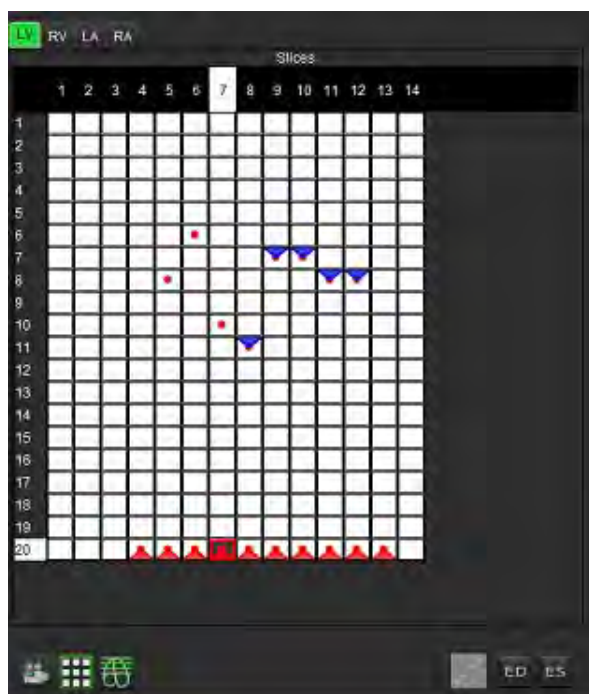


FIGURE 7. Assignment of LV End-Systolic Phases

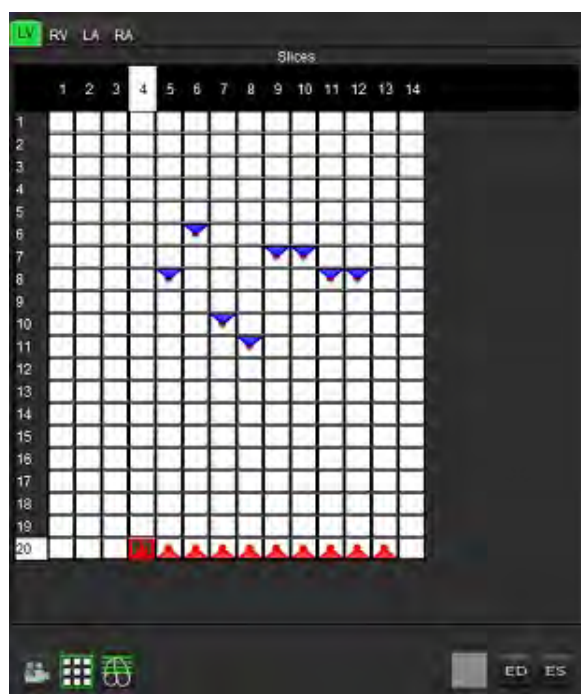


3. In this example, to assign the LV end-systolic phases click on the **ES** button and click on the appropriate phase that has a red dot. A blue triangle will appear after clicking on the matrix box. As shown in Figure 7, all of the end-diastolic and/or end-systolic assignments are now correct.
4. Repeat the above steps as needed for the RV. Click on the RV Tab for the RV.

Ventricular Function Analysis Results

Volume Curve

When auto segmentation is performed for all phases and all slices for either the LV or RV, a ventricular volume versus time curve is generated, as shown in Figure 8. This curve can be included on the report. Markers with drag handles can be adjusted.

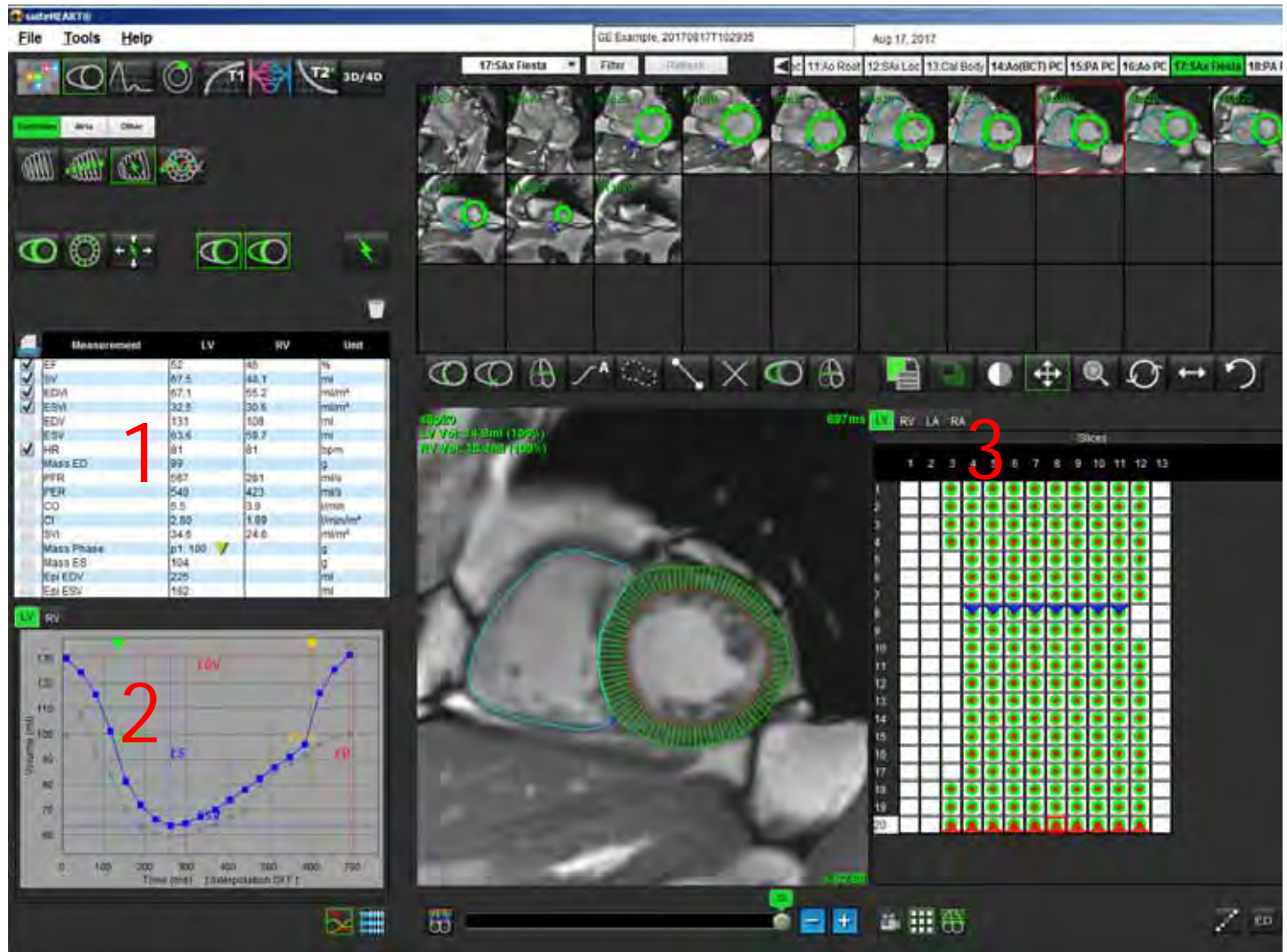FIGURE 8. Ventricular Auto-Segmentation Results



1. Volumetric measurements, 2. Volume curve, 3. Matrix mode

- The red cursor marks end-diastolic volume.
- The blue cursor marks end-systolic volume.
- The green cursor marks Peak Ejection Rate (PER) ml/sec. (Interactive Vertical Cursor).
- The yellow cursor marks Peak Filling Rate (PFR) ml/sec. (Interactive Vertical Cursor).

Volumetric results are displayed on the function analysis tab.

- To review the ventricular mass results, right-click on the inverted yellow triangle for either the LV or RV.
- Only the phase selected from the Measurement table is shown in the report.

FIGURE 9. Mass Phase Pull-down Menu

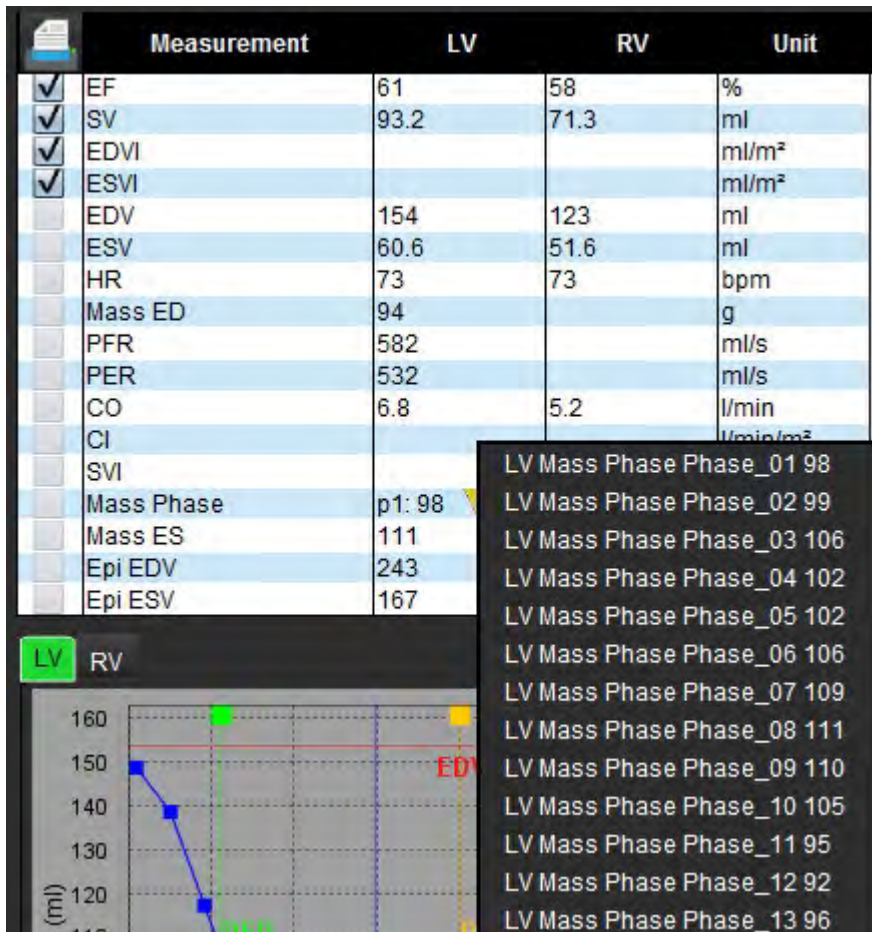


FIGURE 10. Chamber Volume Table

Phase	TDel (ms)	ENDO Volume(ml)	EPI Volume(ml)
1	10	106	242
2	56	94.7	213
3	103	76.6	205
4	150	59.3	196
5	197	47.4	168
6	244	37.5	140
7	291	32.0	136
8	338	30.4	135
9	385	32.1	136
10	431	38.6	139
11	478	58.0	187
12	525	76.6	196
13	572	92.2	227
14	619	96.8	235
15	666	98.3	240
16	713	99.0	242


Chamber Endo Contour Epi Contour


Full LV and RV Volumetric results are displayed in the Chamber Volume table.

Left Ventricular Regional Analysis

LV Regional Analysis allows for the review of wall motion, wall thickness, wall thickening and wall thickness results.

1. Perform Auto LV segmentation for all slices in all phases (refer to page 53).
2. Review the placement of the RV insertion point on each slice. Correct the RV insertion point for the basal line slices.

3. To add an RV insertion point to a slice location, click RV insertion point , select an auto segmented slice and deposit the RV insertion point.

4. Click the slice classification  and confirm basal, mid and apical classification.




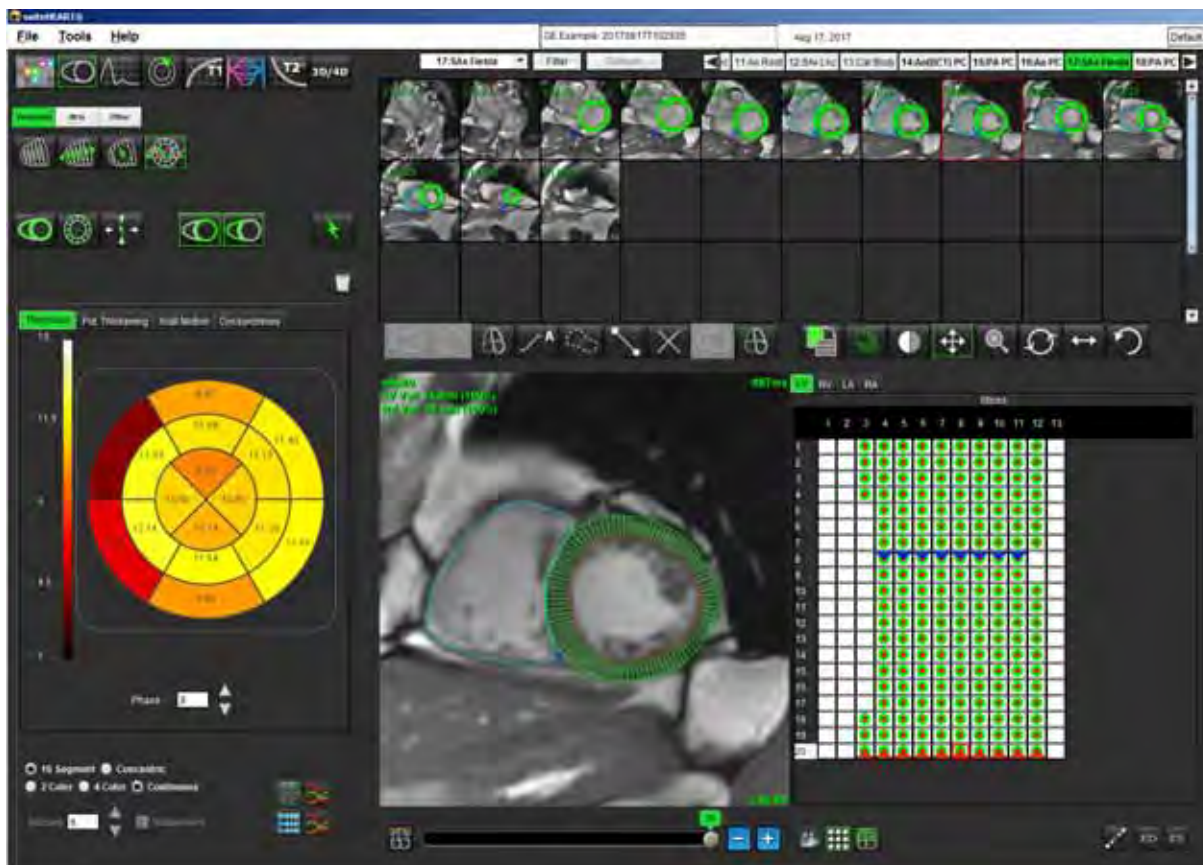
5. Click Regional Analysis . Thickness, Percent Thickening and Wall Motion will display in a plot, graph or table formats.

FIGURE 11. Output of Regional Analysis



Dyssynchrony Analysis

Dyssynchrony is an extension of the Regional Analysis results allowing for the calculation of the Temporal Uniformity of Wall Thickness (TUWT) base upon circumferential information obtained from the Regional Analysis. Literature reference located in Table 4.

Dyssynchrony Analysis Procedure


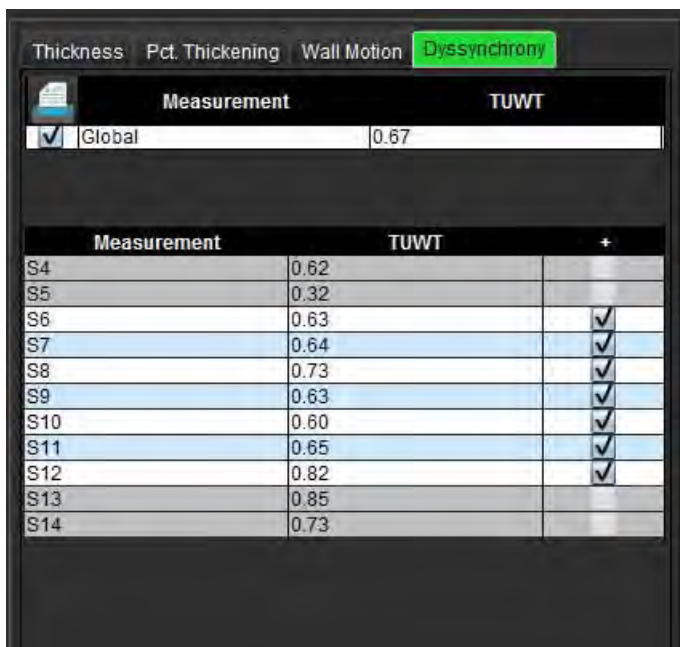
1. Perform LV auto segmentation (see "Perform Auto Segmentation for All Phases and All Slices" on page 53).
2. Select Regional Analysis .
3. Select the Dyssynchrony tab.
4. The measurement table will show the results for each slice and the mean global result.
5. The global result calculation is optimal when only LV mid-ventricular slices are included. To remove a slice result from the global result calculation, click directly on the box with the checkmark in the far right column (Figure 12).

FIGURE 12. Global Result Calculation



Measurement		TUWT
<input checked="" type="checkbox"/>	Global	0.67




Measurement	TUWT	+
S4	0.62	
S5	0.32	
S6	0.63	<input checked="" type="checkbox"/>
S7	0.64	<input checked="" type="checkbox"/>
S8	0.73	<input checked="" type="checkbox"/>
S9	0.63	<input checked="" type="checkbox"/>
S10	0.60	<input checked="" type="checkbox"/>
S11	0.65	<input checked="" type="checkbox"/>
S12	0.82	<input checked="" type="checkbox"/>
S13	0.85	
S14	0.73	

Table 4:


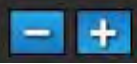
Result	Reference
Temporal Uniformity of Wall Thickness (TUWT)	*Bilchick et al, "Cardiac Magnetic Resonance Assessment of Dyssynchrony and Myocardial Scar Predicts Function Class Improvement Following Cardiac Resynchronization Therapy", JACC, Vol.1:No 5: 2008 p.561-8

Manual LV Function Analysis Procedure

NOTE: It is recommended that the end-diastolic and end-systolic phases are used. Processing should begin on the end-diastolic phase. The analysis workflow is typically performed from the base to the apex.

1. Select .
2. Select the appropriate short axis series from the Image View.
3. Click .
4. Click  button for Volume measurements.
5. Locate the end-diastolic phase.

Define the Endocardium

1. Select .
2. Trace the endocardial contour.
3. Proceed to the next slice using  or use <-- and --> or select the thumbnail.
4. Repeat steps 2 and 3 until the entire left ventricle is segmented.

The Endocardial contour tool will stay selected to expedite the segmenting of multiple slices.

5. Locate the end-systolic phase.
6. Repeat steps 2 and 3 on the end-systolic phase until the entire left ventricle is segmented.

The results in the measurement table continually update after 3 contours have been drawn and as more volume measures are added.

NOTE: The software automatically defines the end-diastolic phase as the phase with the largest volume, and the end-systolic phase as the phase with the smallest volume. The end-diastolic and end-systolic phase assignments are updated during segmentation.

7. Review all results on the measurement table.
8. To ensure optimal basal segmentation, select a 2-chamber long axis view in the x-reference mode.


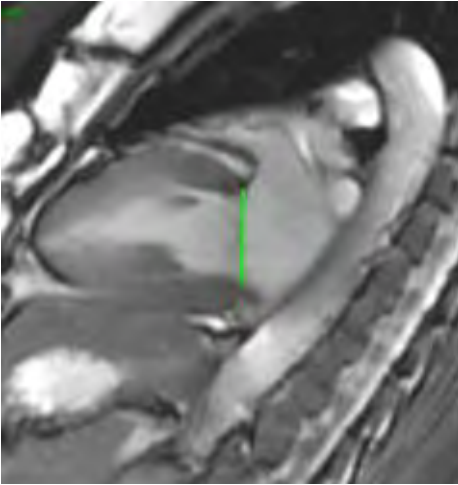
9. Select .
10. Define the basal line, as shown in Figure 13. Review the placement of the basal line on the appropriate end-systolic and end-diastolic phases by using the cine controls.


FIGURE 13. Basal Line



11. Review the updated calculation by reviewing the cross-reference slices in relationship to the basal line.

As shown in Figure 14, the interpolated volume calculation is based upon the relationship of the basal line intersection with the slice (pink line) this volume is now included in the volume results. The actual region of interest will not be shown. Slices that have been interpolated will state the volume amount with the percentage of interpolation in left hand corner of the image, as shown in Figure 14.

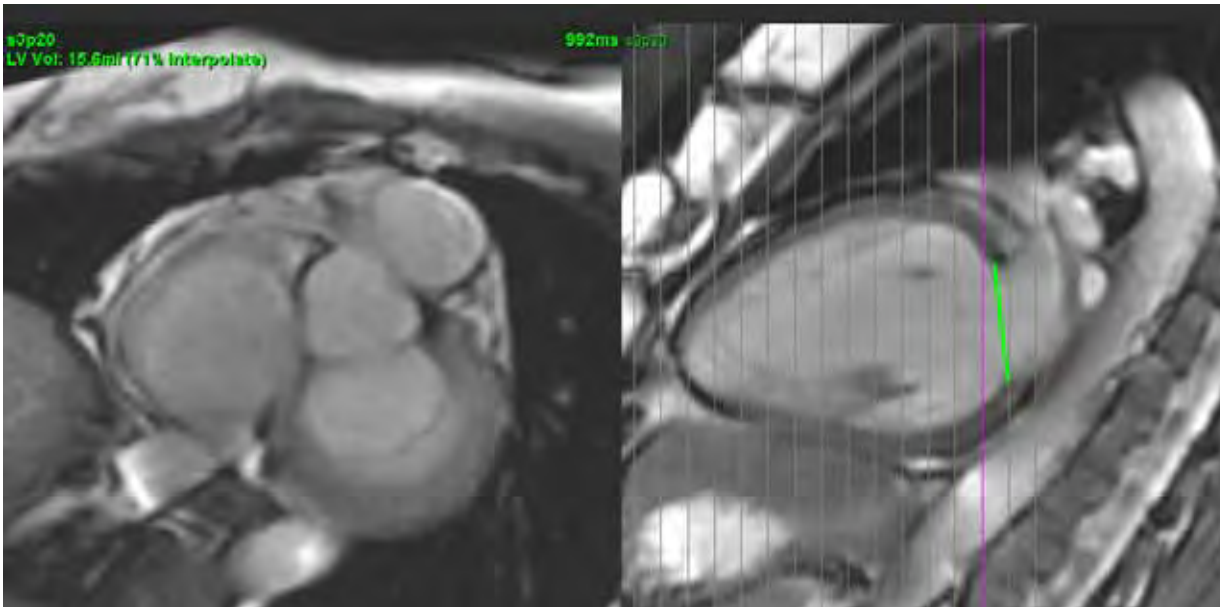FIGURE 14. Volume Calculation



12. To reset the results back to the original manual inputs, perform a right mouse click hold directly on the basal line and select delete or left click and press the Delete key on the keyboard.

IMPORTANT: If the basal line is drawn before the Endocardial ROIs, the Endo/Epi ROIs shall be drawn starting from the LV apical slices towards the basal slices. This is important because the application sums the slice volumes to determine which side of the basal line is the LV. If Endo/Epi ROIs are drawn starting close to the basal slices or on LA slices, the software could misinterpret location of the LV. Check the volume contribution of each slice with an ROI.

Review Accuracy

1. Play the short axis series in cine mode and review the accuracy of all contours.
2. Edit any contours that are inaccurate.



CAUTION: The results are dependent upon the accurate and complete segmentation of the left ventricle. Misdiagnosis may occur if measurements are inaccurate. Measurements should only be created by a properly trained and qualified user.

Reviewing End-Diastolic (ED) and End-Systolic (ES) Assignments

See “Reviewing End-Diastolic (ED) and End-Systolic (ES) Assignments” on page 57.


Interpolation Functionality for Manual

When performing manual function analysis for either the LV or the RV the interpolation functionality is turned “off.”

Interpolation functionality can be applied for the following conditions:


- If the same cardiac phase is traced across slices for either end-systole or end-diastole and a slice has been skipped the interpolation can be enabled by selecting interpolation “On” as shown in Figure 15.
- If the same cardiac phase is traced across slices for either end-systole or end-diastole and/or a slice has been skipped the basal line interpolation can be applied automatically.

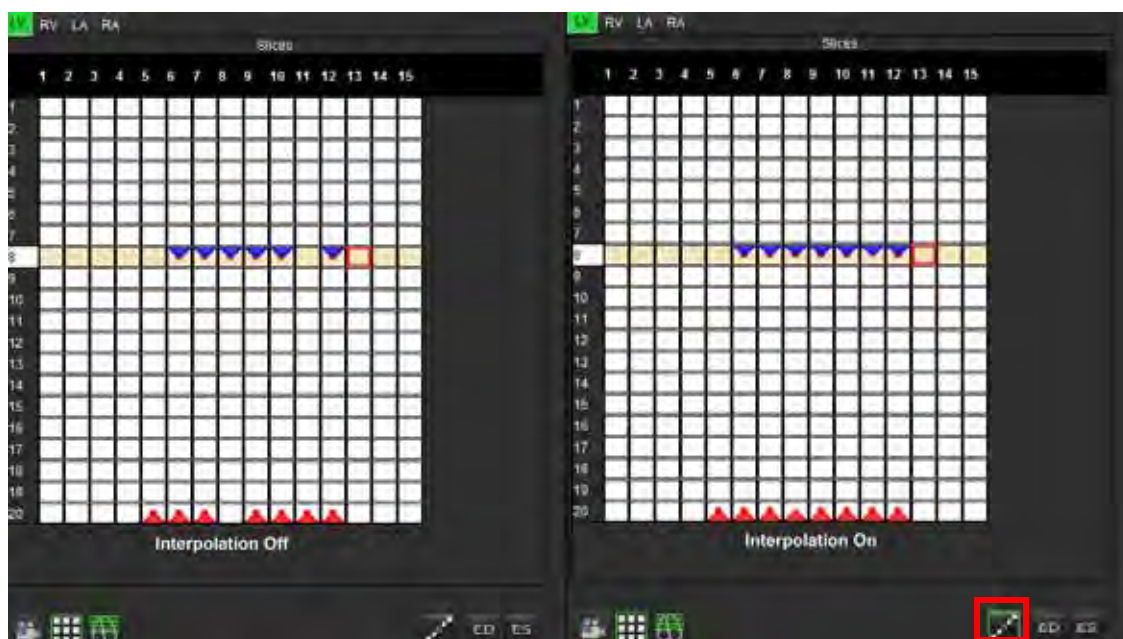FIGURE 15. Interpolation



Manual LV Myocardial Mass Procedure

1. Select the appropriate cardiac phase.

2. Select  for epicardium.

3. Trace the epicardial contour for the LV.

4. Proceed to the next slice using  or use <-- and --> or select the thumbnail.

5. Repeat steps 3 and 4 until the entire left ventricular epicardium is segmented.

The LV Mass result is automatically updated as the epicardial contours are defined.

The results in the measurement table continually update after 3 contours have been drawn.

Manual RV Function Analysis Procedure

NOTE: It is recommended that the end-diastolic and end-systolic phases are used. Processing should begin on the end-diastolic phase. The analysis workflow is typically performed from the base to the apex.

1. Select the appropriate short axis series from the Image View.

2. Click .

3. Locate the end-diastolic phase.

4. Click the  button.

Define the Endocardium

1. Select .

2. Trace the endocardial contour.

3. Proceed to the next slice using  or use <-- and --> or click the thumbnail.

4. Repeat steps 2 and 3 until the entire right ventricle is segmented.

The Endocardial contour tool will stay selected to expedite the segmenting of multiple slices.

5. Locate the end-systolic phase.

6. Repeat steps 2 and 3 on the end-systolic phase until the entire right ventricle is segmented.

The values in the measurement table continually update after 3 contours have been drawn and as more volume measures are added.

NOTE: The software automatically defines the end-diastolic phase as the phase with the largest volume, and the end-systolic phase as the phase with the smallest volume. The end-diastolic and end-systolic phase assignments are updated during segmentation.

7. Review all results on the measurement table.
8. To ensure optimal basal segmentation select a 4-chamber long axis view in the x-reference mode.




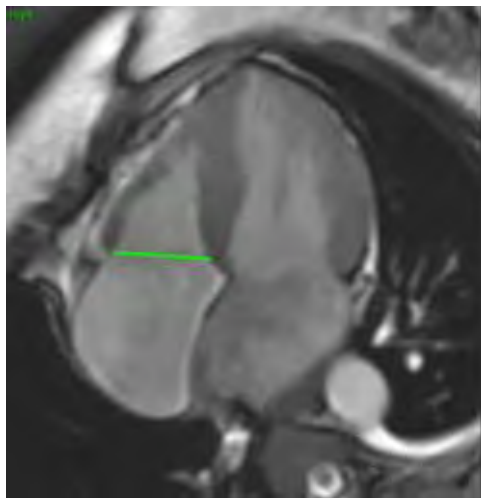
9. Select .
10. Define the basal line as shown in Figure 16. Review the placement of the basal line on the appropriate end-systolic and end-diastolic phases by using the cine controls.


FIGURE 16. Basal Line Definition



11. Review the updated calculation by reviewing the cross-reference slices in relationship to the basal line.

As shown in Figure 17 the interpolated volume calculation is based upon the relationship of the basal line intersection with the slice (pink line) this volume is now included in the volume results. The actual region of interest will not be shown. Slices that have been interpolated will state the volume amount with the percentage of interpolation in left hand corner of the image as shown in Figure 17.

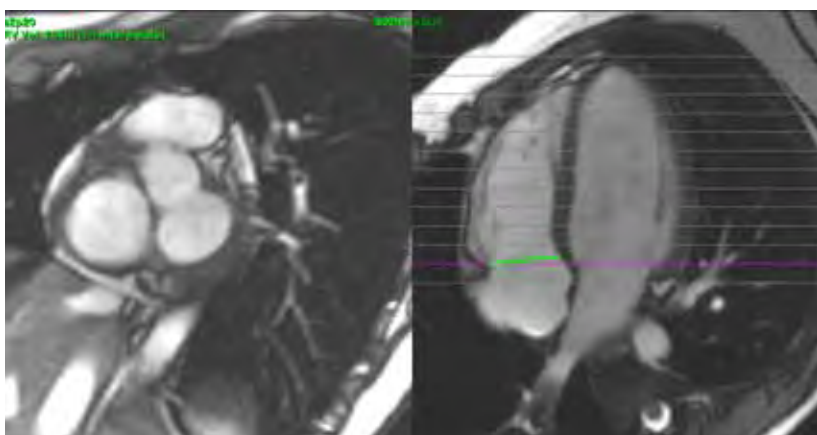FIGURE 17. Interpolated Volume Calculation



12. To reset the results back to the original manual inputs, perform a right mouse click hold directly on the basal line and select Delete; or left click and press the Delete key on the keyboard.

IMPORTANT: If the basal line is drawn before the Endocardial ROIs, the Endo/Epi ROIs shall be drawn starting from the apical slices towards the basal slices. This is important because the application sums the slice volumes to determine which side of the basal line is the RV. If Endo/Epi ROIs are drawn starting close to the basal slices or on RA slices, the software could misinterpret location of the RV. Check the volume contribution of each slice with an ROI.

Review Accuracy

1. Play the short axis series in cine mode and review the accuracy of the contours.
2. Edit any contours that are inaccurate.



CAUTION: The results are dependent upon the accurate and complete segmentation of the left ventricle. Misdiagnosis may occur if measurements are inaccurate. Measurements should only be created by a properly trained and qualified user.


Manual RV Myocardial Mass Procedure

1. Select the appropriate cardiac phase.



2. Select  for RV epicardium.




3. Trace the epicardial contour for the RV.

4. Proceed to the next slice using  or use <-- and --> or click the thumbnail.

5. Repeat steps 3 and 4 until the entire right ventricular epicardium is segmented.
 - The RV Mass result is automatically updated as the epicardial contours are defined.
 - The results in the measurement table continually update after three contours have been drawn.


Fast LV Function Analysis Procedure

This method is performed on a long-axis series.

1. Select a long-axis series.
2. Select the end-diastolic phase.
3. Select the  button from the Function mode.
4. Click the .
5. Trace the left ventricle endocardium. A center of rotation line is drawn automatically.
6. Adjust the center of rotation line so that it corresponds to the long axis of the left ventricle
7. If left ventricle mass is desired, trace the left ventricle epicardium by clicking .
8. Repeat steps 4 - 6 for end systole.

NOTE: When analyzing the LV, the correct position of the center line must run from the base to the apex.

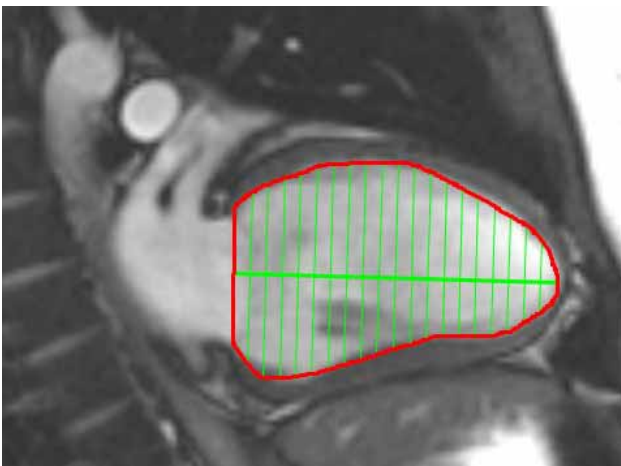FIGURE 18. Center of Rotation Line



The results are displayed on the Measurement table.

Manual LA and RA Function Analysis Procedure

1. Select the appropriate series from the Image View.

NOTE: For optimal results, it is recommended to use a 4-chamber stack for analysis. The 4-chamber view better delineates the atrial anatomy.

2. Click .

3. Select the  button

4. Locate the end-diastolic phase.

Define the Endocardium

1. Select  for LA Endocardium or  for RA Endocardium.

2. Trace the endocardial contour.

3. Proceed to the next slice using  or use <-- and --> or click thumbnail.

4. Repeat steps 2 and 3 until the entire atrium is segmented.

5. Locate the end-systolic phase.

6. Repeat steps 2 and 3 on the end-systolic phase until the entire atrium is segmented.

The values in the measurement table continually update after 3 contours have been drawn and as more volume measures are added.



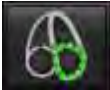
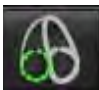
NOTE: The software automatically defines the end-diastolic phase as the phase with the largest volume, and the end-systolic phase as the phase with the smallest volume. The end-diastolic and end-systolic phase assignments are updated during segmentation.

7. If a short axis view has been used, select the basal line button and define the appropriate base.

IMPORTANT: If the basal line is drawn before the Endocardial ROIs, the Endo ROIs shall be drawn starting from the slice furthest away from the basal slice moving towards the basal slice. This is important because the application sums the slice volumes to determine which side of the basal line is the ventricle (LA/RA). If Endo ROIs are drawn starting close to the basal slices or on LV/RV slices, the software could misinterpret location of the LA/RA. Check the volume contribution of each slice with an ROI.

Fast LA or RA Function Analysis Procedure

This method is performed on a long-axis series.

1. Click .
2. Select a long-axis series.
3. Select the end-diastolic phase.
4. Select the  button.
5. Select  for LA Endocardium or  for RA Endocardium.
6. Trace the atrium endocardium. A center of rotation line is drawn automatically.
7. Adjust the center of rotation line so that it corresponds to the long axis of the atrium
8. Repeat steps 5-7 for end systole.

Atrial Dimensions and Area


1. Click .
2. Select the appropriate series.
3. To perform an atrial dimension measurement, click directly on the table in the column for either LA or RA and then deposit two points. See Figure 19.
4. To perform an atrial area measurement, click directly on the table in the column for either LA or RA and then draw an ROI. See Figure 19.


FIGURE 19. Atrial Measurement

	Measurement	LA	RA	Unit
<input checked="" type="checkbox"/>	EF			%
<input checked="" type="checkbox"/>	EDVI			ml/m ²
<input checked="" type="checkbox"/>	ESVI			ml/m ²
<input checked="" type="checkbox"/>	EDV			ml
<input checked="" type="checkbox"/>	ESV			ml
<input checked="" type="checkbox"/>	Atrial Dimension			cm
<input checked="" type="checkbox"/>	Atrial Area			cm ²

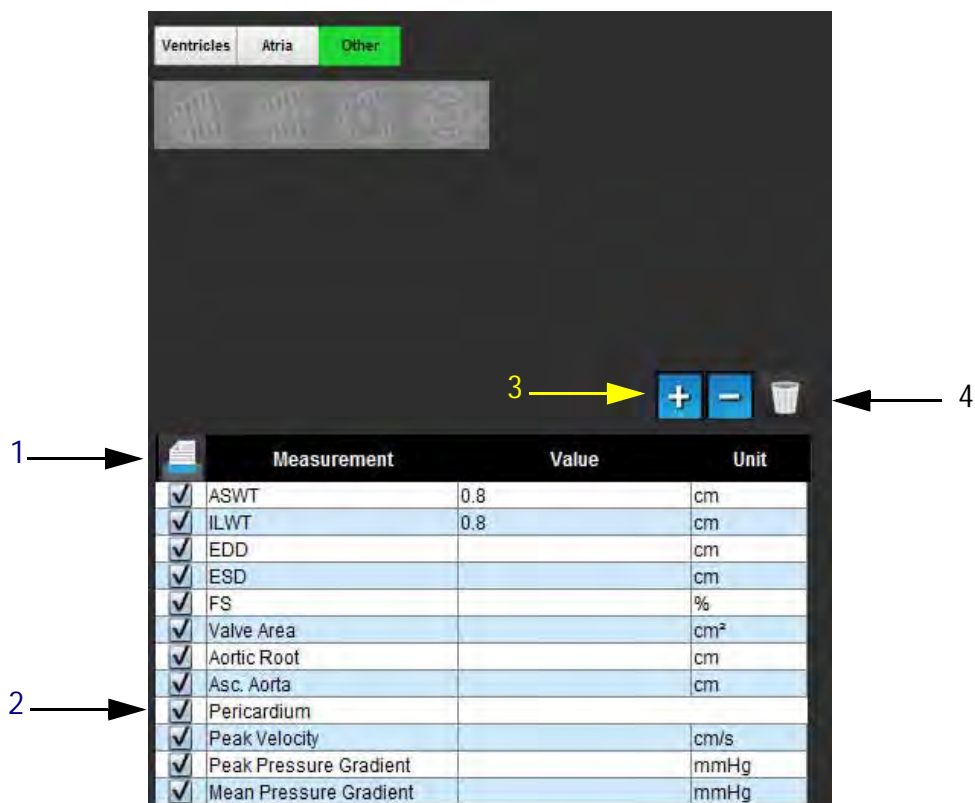Linear Measurements


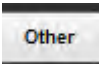
The application allows for the reporting of linear measurements. Tool tips are available by placing the cursor over the measurement listed on the table.

FIGURE 20. Linear Measurements



1. Print Option, 2. Type-in Field for Pericardium, 3. Add/Remove Custom Measurement, 4. Reset all Measurements

Linear Measurement Set-up

1. Select .
2. Select the series.
3. Click  button.
4. Locate the image with the anatomy to be measured.
5. Click the desired measurement from the Measurement table, which will turn the measurement green to indicate the selection is active.



CAUTION: Accurate placement of the line is critical to measurement results. Misdiagnosis may occur if measurements are inaccurate. Measurements should only be created by a properly trained and qualified user.

6. To edit, click on annotation and when the color changes from green to purple it is active. Place the cursor over one of the endpoints and adjust the endpoint.

The measure distance value updates accordingly in the Measurements table when you move the cursor outside of the Image Editor window.

Place the cursor over the center marker to move the entire measure distance line to another location.

NOTE: To reset the measurement, select the measure distance line and access the right mouse menu and select Delete; or use the Delete key on the keyboard.


Reset Measurements



Select  to reset all measurements.


Add Custom Measurement



1. Select .
2. Enter a unique label in the **Add Custom Measure pop-up window**.
3. Select the **measurement type** as either **Linear** or **Area**.
4. Select **OK**.

Remove Custom Measurement



1. Select the .
2. Select the custom measurement(s) to be removed from list.
3. Select **Select**.

NOTE: Custom measurements created will be present for all future analysis until removed from the listing.

Valve Plane Analysis


The valve plane analysis feature allows for the calculation of valve peak velocity, peak pressure gradient and mean pressure gradient for the valve.¹

Using the results from the LV auto segmentation, the pressure gradient is computed from the cardiac output, based on the frame-by-frame changes in left ventricular systolic volume.

Valve Plane Analysis Procedure

1. Perform LV auto segmentation on all slices in all phases (see page 53).
2. Select a series that demonstrates valve anatomy.
3. Select **Valve Area** from the measurement table (Figure 21) and perform planimetry of the valve, as shown in Figure 22.

FIGURE 21. Valve Area

	Measurement	Value	Unit
<input checked="" type="checkbox"/>	ASWT		cm
<input checked="" type="checkbox"/>	ILWT		cm
<input checked="" type="checkbox"/>	EDD		cm
<input checked="" type="checkbox"/>	ESD		cm
<input checked="" type="checkbox"/>	FS		%
<input checked="" type="checkbox"/>	Valve Area		cm ²
<input checked="" type="checkbox"/>	Aortic Root		cm
<input checked="" type="checkbox"/>	Asc. Aorta		cm


4. Upon completion of the ROI, the table will update with the results and present a graph showing the pressure gradient over time.



Click  to reset **all** measurements.

1. Wolff, Steven D., M.D., Ph.D. Noninvasive methods for determining the pressure gradient across a heart valve without using velocity data at the valve orifice. U.S. Patent 9,585,568, March 7, 2017.

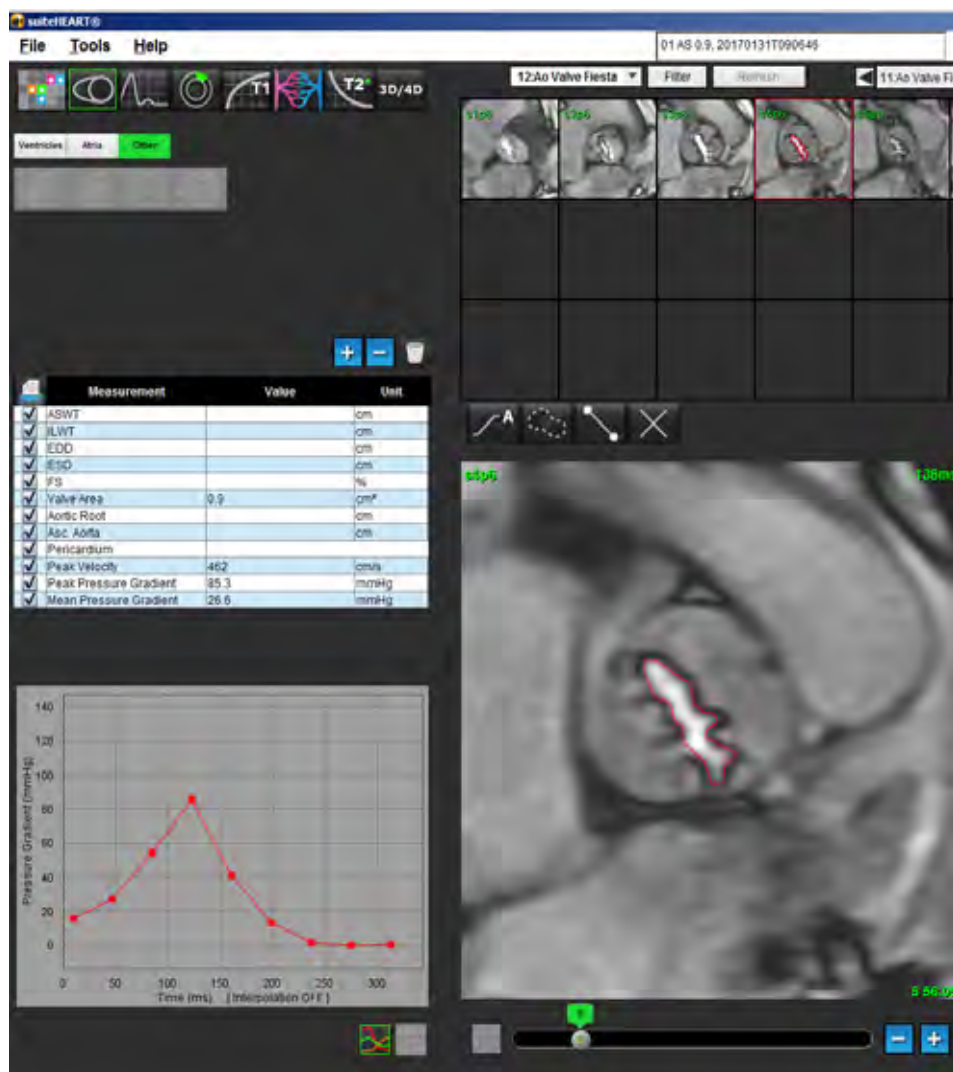FIGURE 22. Valve Plane Analysis



IMPORTANT: It is recommended that you are qualified in performing cardiac analysis, if the analysis results are to be used to reach a diagnosis.



WARNING: The application assists in the analysis of the images only and does not automatically produce quantifiable results. The use and placement of quantitative measurements is at the discretion of the user. Misdiagnosis can occur if measurements are inaccurate. Measurements should only be created by a properly trained and qualified user.

NOTE: Peak Velocity, Peak Pressure Gradient, and Mean Pressure Gradient are not valid in patients with mitral regurgitation or a shunt.

This page intentionally left blank.

Flow Analysis

The user is responsible for the accurate and complete placement (and correct assignment) of all regions of interest (ROIs), including those generated or modified by the auto segmentation algorithms. The quantitative values generated by the software depend on the accurate and complete placement (and correct assignment) of these regions of interest.

The study Preprocessing feature allows for the preprocessing of flow analysis. Refer to the suiteDXT Instructions for Use NS-03-009-0001.

The Flow Analysis tool calculates flow and velocity information at various points in the cardiac cycle from 2D Phase Contrast (PC) images with through-plane flow encoding.

There are two methods for generating flow results: Enhanced and Conventional. Enhanced auto segmentation does not require an initial ROI to be placed on the vessel of interest, whereas conventional auto segmentation does.




WARNING: The application assists in the analysis of the images only and does not automatically produce quantifiable results. The use and placement of quantitative measurements is at the discretion of the user. Misdiagnosis can occur if measurements are inaccurate. Measurements should only be created by a properly trained and qualified user.



WARNING: Using Enhanced auto segmentation requires careful review of all results. If the vessel identification is not optimal, perform conventional auto segmentation for the vessel.

Flow Window Components

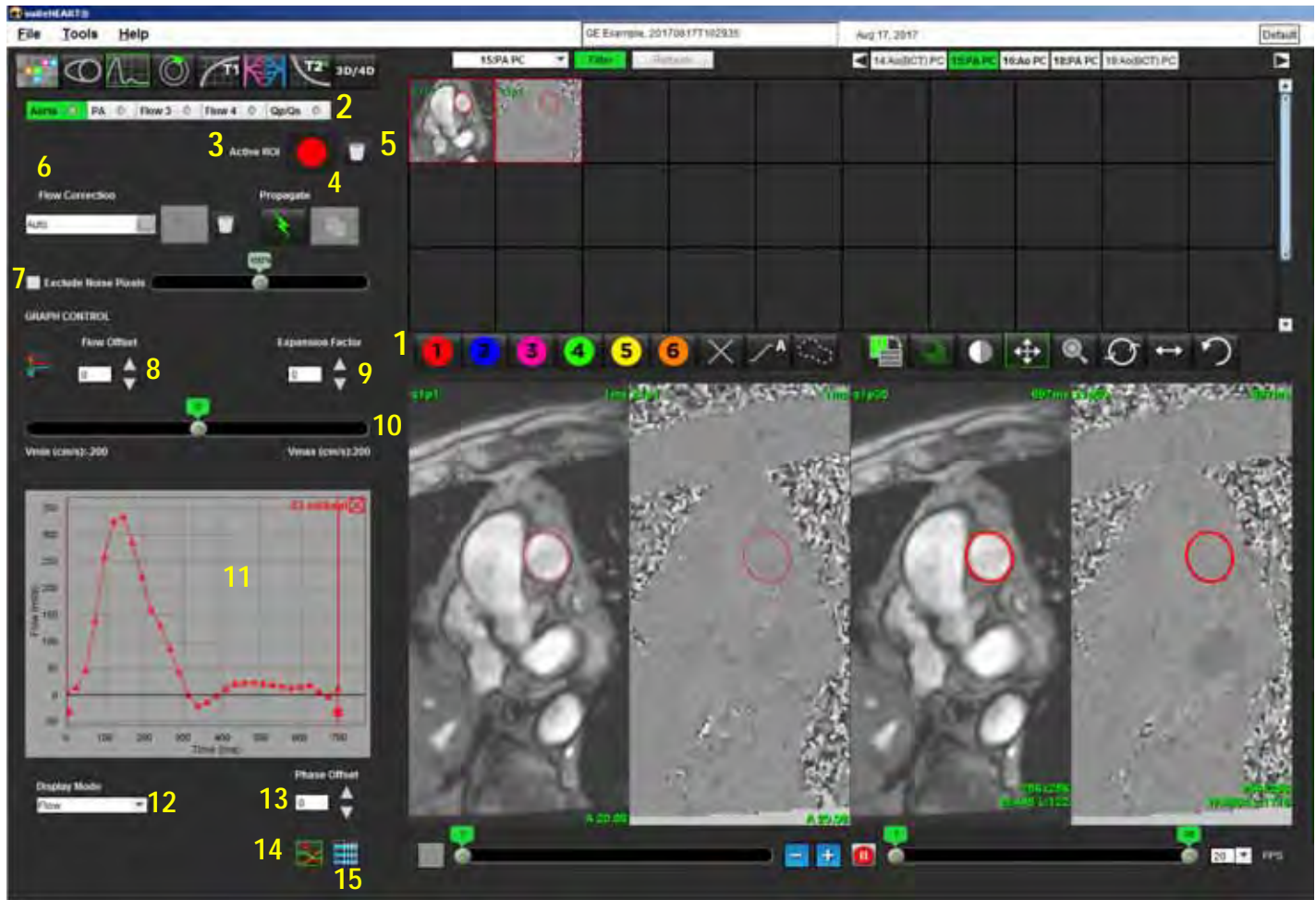FIGURE 1. Flow Window



1. Flow Analysis ROIs, 2. Flow tabs, 3. Active ROI/Enhanced Auto selection, 4. Propagate buttons, 5. ROI reset button, 6. Flow correction drop-down menu, 7. Exclude Noise Pixels, 8. Flow offset, 9. Expansion factor, 10. Venc slider bar, 11. Flow curve results, 12. Display mode, 13. Phase offset, 14. Switch to graph display, 15. Switch to summary table

NOTE: Flow Analysis displays the magnitude and phase images in a side-by-side image display. Other types of images acquired, at the same location, are not displayed and should be viewed in a separate DICOM viewer.

Flow Analysis

Generate Flow Results

Using Enhanced Auto Segmentation

Enhanced auto segmentation does not require an initial ROI to be placed on the vessel. It is important to select the appropriate flow tab that correlates to the vessel anatomy that has been acquired. Only aortic and pulmonary anatomy is supported.

NOTE: If there are more than six vessels acquired for phase contrast per tab, the Pre-Processing feature will only keep the six most recent results.

Generate an Aorta Flow Curve

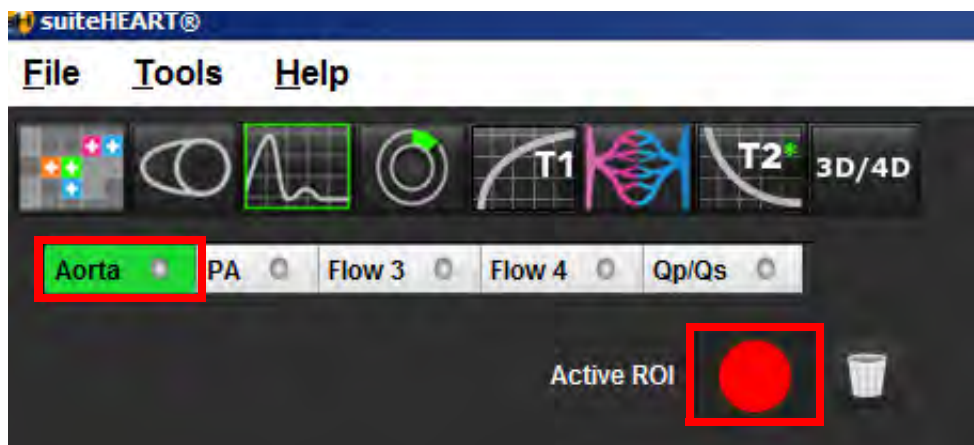
1. Select the appropriate series that demonstrates the aortic vessel, as shown in Figure 2.



FIGURE 2. Aortic Vessel



2. Select the Aorta tab and select the Active ROI color, as shown in Figure 3.

FIGURE 3. Aorta Tab



3. Select  to generate a flow curve.
4. Review the segmentation on the vessel. If the incorrect vessel is segmented perform conventional auto segmentation. Refer to "Using Conventional Auto Segmentation" on page 82.
5. To edit, click on a contour, perform the edit and click .

Generate a Pulmonic Flow Curve

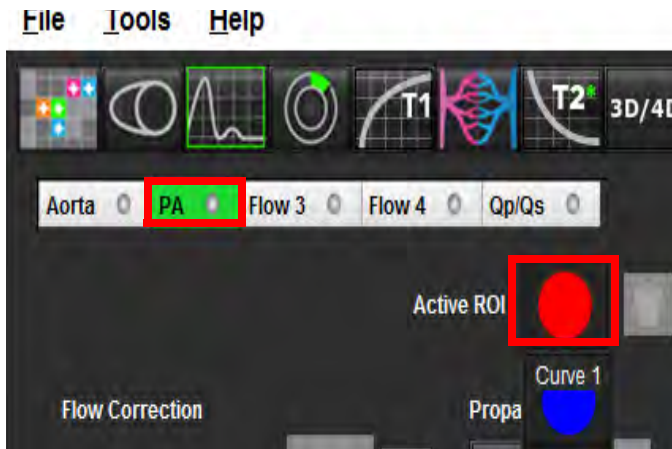
1. Select the appropriate series that demonstrates the pulmonic vessel, as shown in Figure 4.



FIGURE 4. Pulmonic Vessel



2. Select the PA tab and select the Active ROI color, as shown in Figure 5.

FIGURE 5. PA Tab




3. Select  to generate a flow curve.
4. Review the segmentation on the vessel. If the incorrect vessel is segmented perform conventional auto segmentation. Refer to "Using Conventional Auto Segmentation" on page 82.
5. To edit, click on a contour, perform the edit and click .




WARNING: Using Enhanced auto segmentation requires careful review of all results. If the vessel identification is not optimal, perform conventional auto segmentation for the vessel.


Using Conventional Auto Segmentation

1. Select **Flow Analysis Mode** .
2. Select a phase contrast series.

The magnitude image is displayed on the left, phase image is on the right.

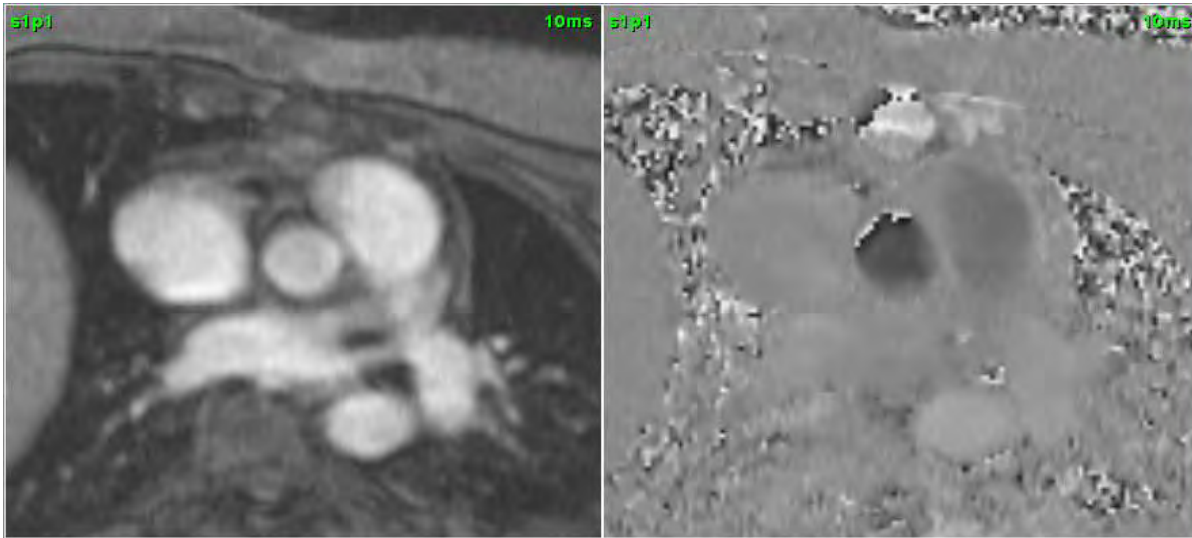FIGURE 6. Magnitude and Phase Images




3. Select a **Flow Tab**.
There are five tabs available; one labeled aorta, one pulmonary artery (PA) and two user defined tabs with default labels of Flow 3 and Flow 4. The Qp/Qs tab allows for reporting of the Qp/Qs ratio.
4. Select  curve 1.
Six ROIs are available, numbered 1 - 6. The color coding remains consistent across the analysis view, image viewports and graphs.
5. Create a contour around a vessel by depositing 4 points around the vessel of interest and double-clicking at the last point to close the ROI. Alternately, you can move the cursor out of the editor window to close the ROI.


A manual trace of the vessel can also be performed.

The contour is created on both the magnitude and phase images, as shown in Figure 7.

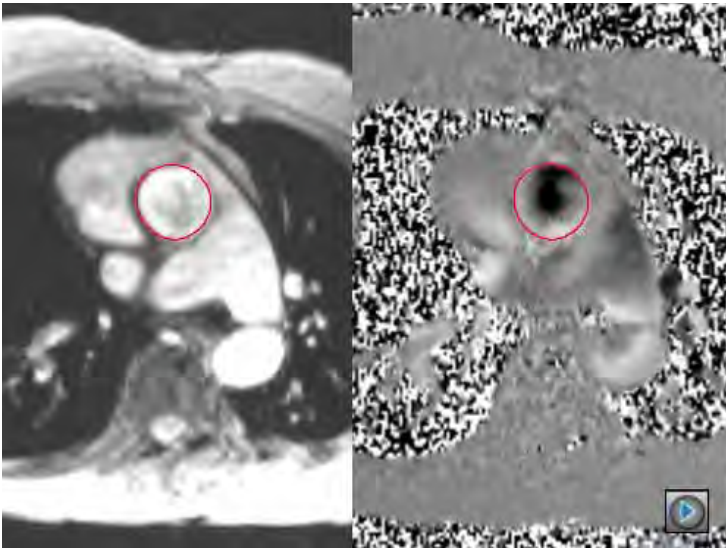
FIGURE 7. Flow ROIs



6. To perform automatic segmentation over all phases within the slice, select one of the propagation methods:

- Choose  for automatic segmentation over all phases in the slice.

or

- Choose  to propagate the same contour over all phases in the slice. This is useful when analyzing small, stationary vessels.


7. Flow results are displayed on the graph and summary tables.

- Click the check box beside the ml/beat rate to remove the associated curve from the graph. This does not change the calculations. It only removes the curve from the graph.

8. Select a Flow Correction option from the file pull-down.

- Curves with a Flow Correction applied will have solid phase data points, as shown in Figure 8. Refer to “Flow Correction Options” on page 86.

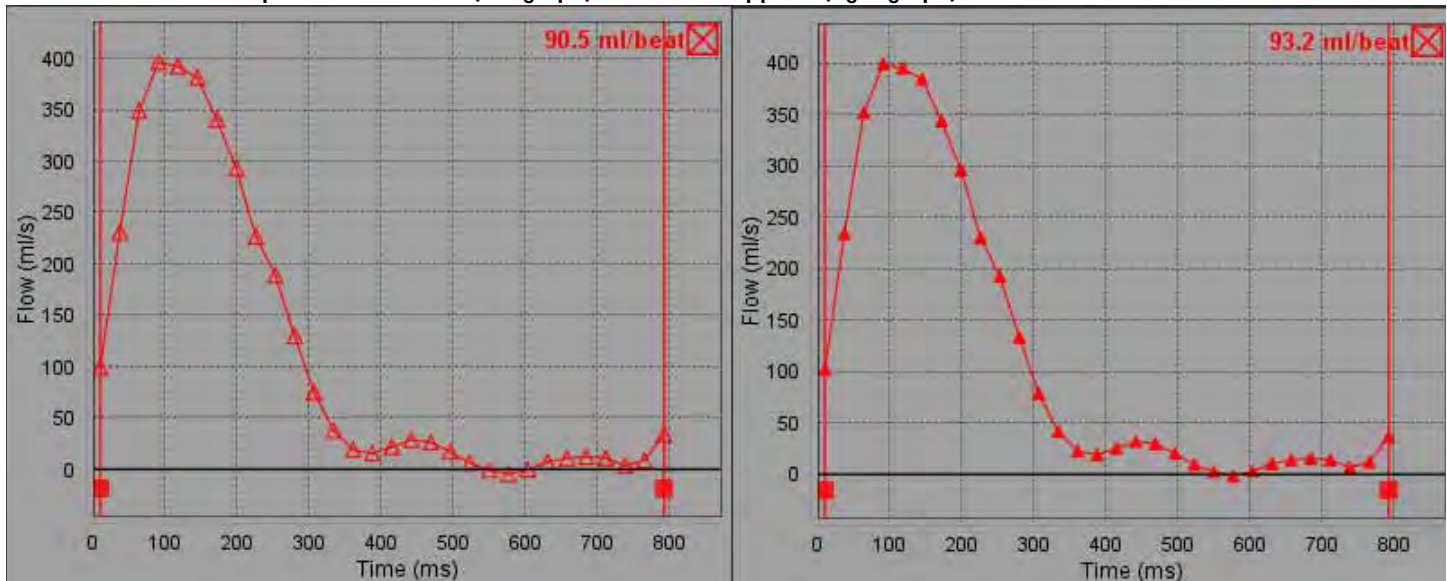FIGURE 8. Flow Graph - No Correction (left graph); Correction Applied (right graph)



9. Select  to invert the curve.

NOTE: All flow curves generated are displayed in a positive direction. Inverted curves are indicated by the active invert button.

10. Select a Phase Offset to change the ordinate of the flow curve.
11. Select any point on the graph to locate the corresponding phase image.
12. Adjust the vertical cursor for the start and end points, as necessary.
13. Review the accuracy of the contours.


Contour Editing

Contours can be edited one phase at a time or for a range of phases.

Editing a single phase

1. Select the desired phase to edit.
2. Click on the contour to activate it for editing.
The contour will turn purple indicating it can be edited.
3. Edit the contour by moving the points for point spline contours or by drawing on the image with the left mouse button depressed for free hand or computer generated contours.
4. To activate the nudge tool, left mouse click on the contour to select it. Then right mouse click and select the nudge tool from the pop-up menu, as shown Figure 9. For more information, refer to "Editing Contours" on page 45

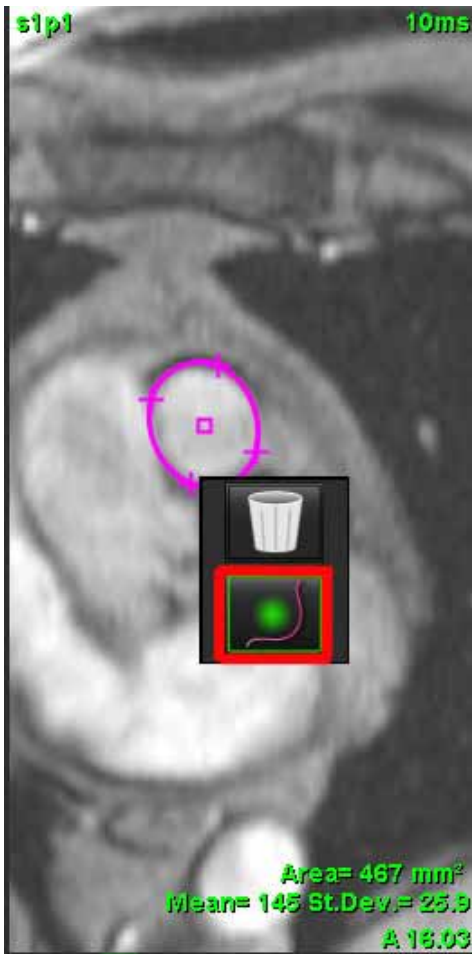FIGURE 9. Nudge Tool



Editing a range of phases

1. Select the desired slice.



2. Select the icon to display thumbnails of all the phases of a given slice.
3. Select the first phase of the range of phases to be edited.
4. Depress and hold the shift key and select the last phase of the range to be edited.
5. Edit the contour in the image editor window.
6. Deselect the contour by either clicking on the image away from the selected contour or by moving the cursor out of the editor window.

Flow Correction Options

There are three flow baseline corrections methods for 2D phase contrast. Flow curves that have a correction method applied will have solid phase data points.

NOTE: Phase contrast images that are used for analysis should not have image phase wrap. Phase wrap present in the image will cause inaccurate flow results.

Auto Phase Error Correction

The Auto baseline phase error correction corrects for phase errors that occur during image acquisition by examining the phase error in distant stationary organs (e.g., chest wall, liver, etc.) and spatially fitting the data using linear or higher order interpolation.


NOTE: If a 2D magnitude and phase series is created by using the 3D/4D viewer, the application will create an uncorrected series and a second series to which phase error correction has been applied. Do not apply Auto from the flow correction pull-down to the series labeled "Corrected."

1. Generate a flow curve using the appropriate phase contrast series.
2. Select **Auto** from the Flow Correction pull-down.

NOTE: Auto phase error correction will be automatically applied if Enable Auto Flow Correction is checked in Preferences.

3. The correction will be applied with the updated results displayed directly on the flow graph.
4. Series which fail the fitting analysis will be indicated by a warning symbol, as shown in Figure 10.

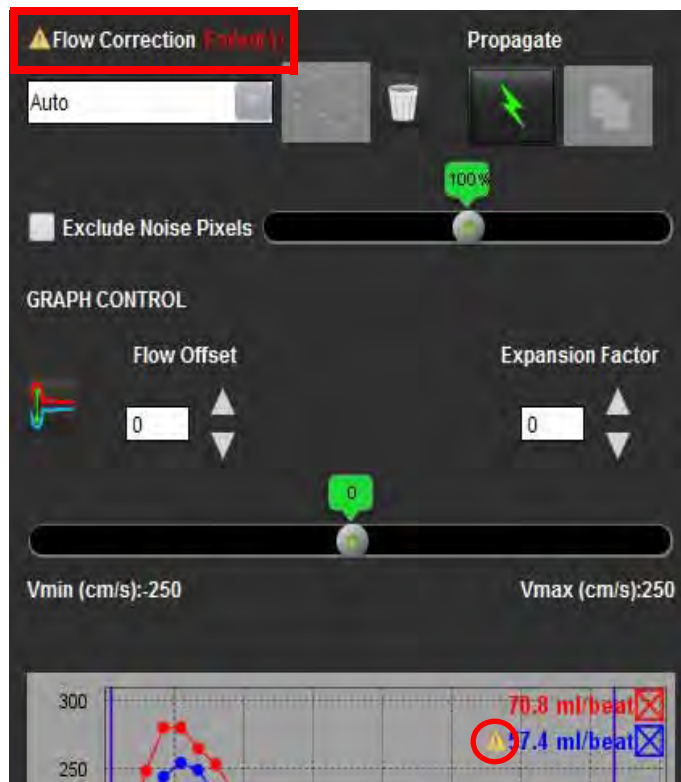FIGURE 10. Auto Flow Correction Failure



Failure Types:


- 1 - Too much noise in the image.
- 2 - Fitting error is too large.
- 3 - Input data is invalid.

NOTE: Phase wrap present in the image will cause inaccurate flow results, as shown in Figure 11. 2D Cine Phase Contrast images that are used for flow analysis should not have image phase wrap, as shown in Figure 12.

FIGURE 11. Example Images Showing Phase Wrap (White Arrows)



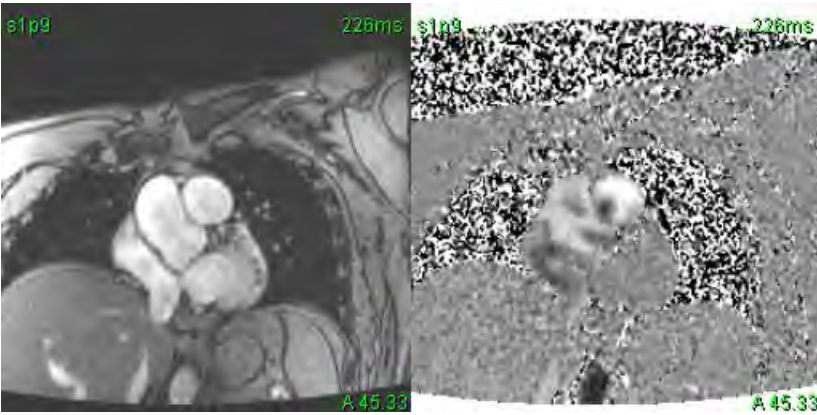FIGURE 12. Example Images with No Phase Wrap



Phantom Correction

To improve the accuracy of phase contrast results and to correct for baseline phase shift errors, a phantom acquisition can be used to calculate this error.


NOTE: The phantom correction series must have been acquired with the same scan prescription and parameters as the original phase contrast series. There must be signal from a stationary object filling the entire contour on the phantom series.

1. Generate a flow curve using the appropriate phase contrast series.
2. Select the corresponding phantom series from the Flow Correction pull-down.
3. The correction will be applied with the updated results displayed directly on the flow graph.

Background Contour Correction

This correction method can be considered for vessels that are surrounded by static tissue.

NOTE: For optimal correction the background contour must be placed in static tissue directly adjacent and surrounding the region of flow.

1. Generate a flow curve using the appropriate phase contrast series.
2. Select the **Background ROI** from the Flow Correction pull-down.
3. Click  to draw a contour.
4. The correction will be applied with the updated results displayed directly on the flow graph.

Flow Curve Options


Exclude Noise Pixels

This option identifies low intensity pixels (high fluctuation of velocities) if present within the ROI, identified by the pink overlay as shown in Figure 14 and excludes them from the flow calculation. Click the check box to apply this option. The percentage of noise pixels can be adjusted by the slider bar.

FIGURE 13. Noise Pixels



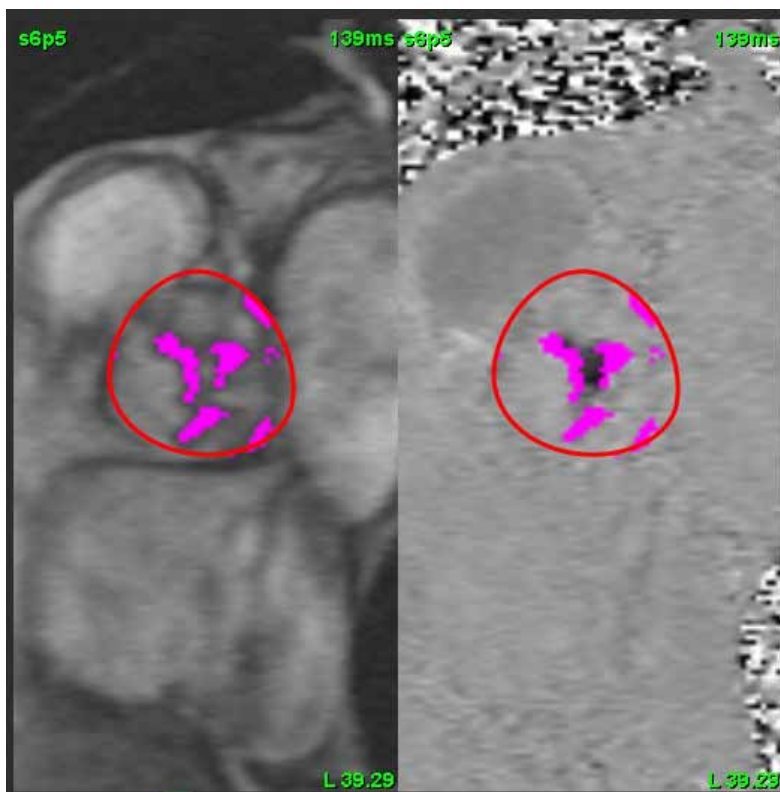FIGURE 14. Noise Pixels Identified by Pink Overlay



Flow Offset

Flow offset changes the abscissa value of the flow curve which changes the baseline values of the flow result.



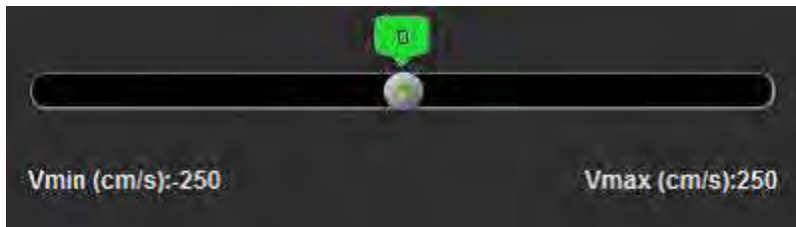
Expansion Factor

The expansion factor uniformly changes the radius of the segmented vessel by a specified pixel amount to include valid flow pixels.




Velocity Aliasing Correction

To correct for velocity aliasing, drag the slider bar control button to perform phase unwrapping. The effect of the change will be updated directly on the phase image and the results displayed directly on the flow graph.




User Defined Peak Velocity

1. Select the appropriate phase of the cardiac cycle.

2. Use  to position the cursor on the phase image.

The cursor is synchronized with both the magnitude and phase images. The velocity result appears in mm/s on the phase image next to the cursor.

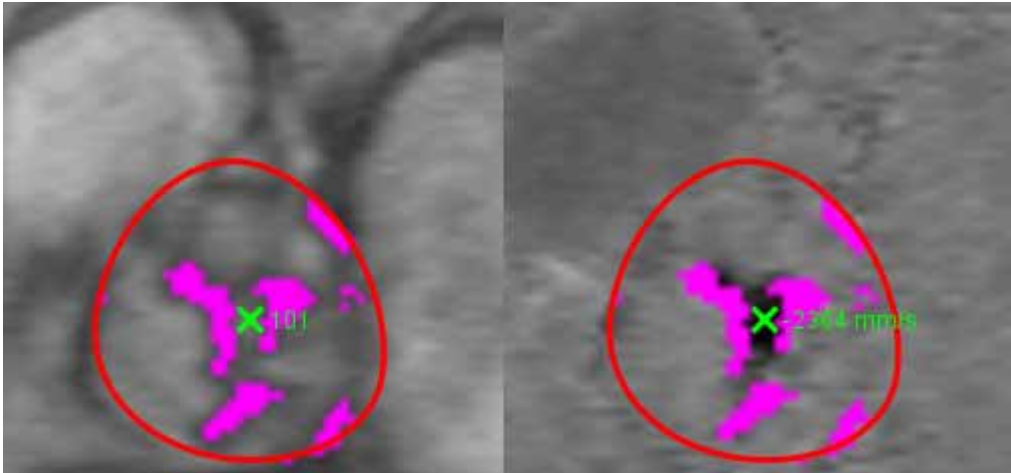
FIGURE 15. Pixel Flow Velocity



Display Modes

Select the desired **Display Mode** from the drop-down menu (Figure 16).

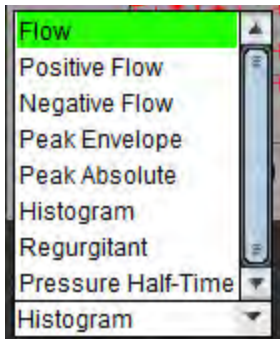FIGURE 16. Display Mode Menu



Flow Display Mode Definitions:

Flow: This graph displays a plot that represents the flow volume of each phase in the entire cardiac cycle (default). Each point on the curve represents flow for that phase.

Positive: This graph displays the sum of the positive flow area over the cardiac cycle.

Negative: This graph displays the sum of the negative flow area over the cardiac cycle.

Peak Envelope: This graph displays a plot of peak positive and negative velocities for each phase of the cardiac cycle.

Peak Absolute: This graph displays a plot of absolute peak velocity for each phase.

Histogram: This graph displays a plot of the velocity of each pixel within each region of interest for every phase of the cardiac cycle.

Regurgitant: The Regurgitant Fraction (%) is the quotient of the negative flow divided by the total positive flow.

Pressure Half-Time (PHT): The time it takes for the peak transmitral pressure gradient to decrease by half.

Histogram Mode

Select histogram mode to display a plot of velocities per pixel.


1. Generate a flow curve using the appropriate phase contrast series.
2. From the Display Mode pull-down menu, select **Histogram**.
3. Click directly on the graph to activate a cross hair cursor on the phase image, which indicates the corresponding location of that pixel.
4. Use the double arrow controls at the bottom of the graph to locate the highest or lowest velocity value, (Figure 17.)
5. Use the single arrow controls to increment discretely through the velocity values, as shown in Figure 17.

NOTE: The series locate functionality, when clicking directly on the flow curve, is disabled when in histogram mode. Switch to flow mode to enable the locate functionality.

NOTE: To ensure that the corresponding magnitude and phase image are displayed, work with one flow curve at a time, deselect the other histogram curves from the graph display.

NOTE: Studies analyzed using histogram mode with a previous version of suiteHEART® Software will need to be reanalyzed.

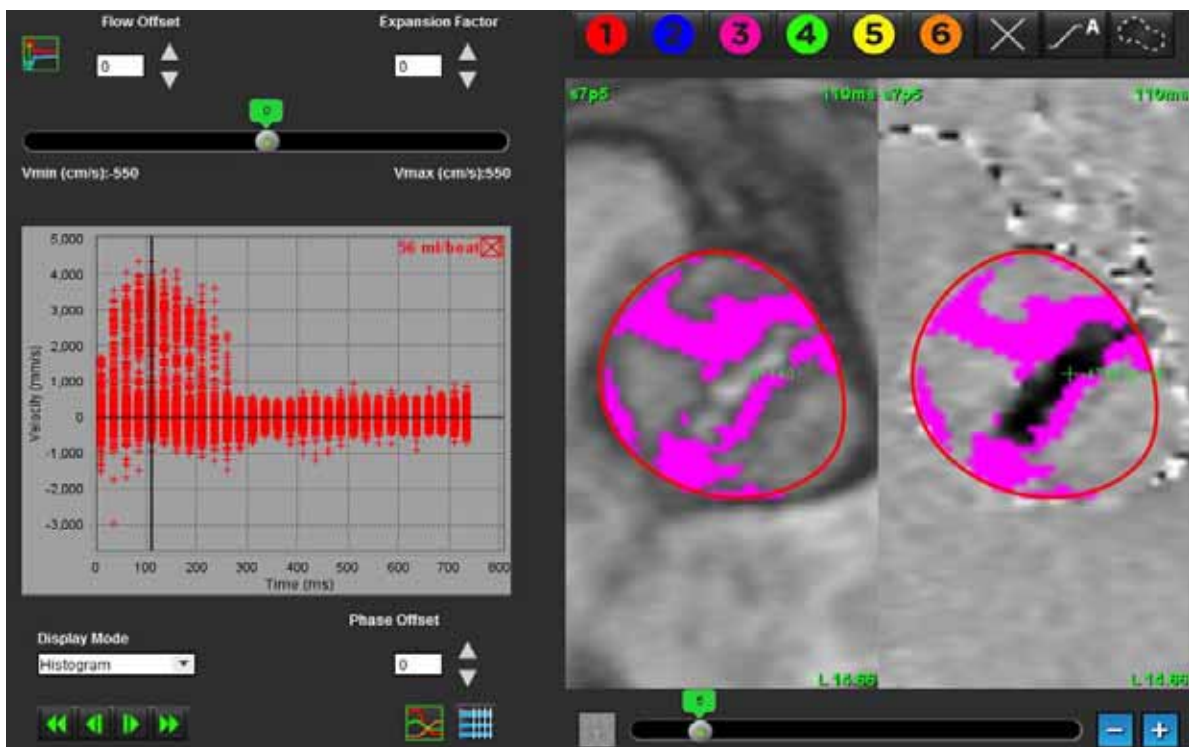FIGURE 17. Histogram Mode




Regurgitant Mode

Select Regurgitant mode to calculate the negative flow and regurgitant fraction.

1. Generate a flow curve using the appropriate phase contrast series.
2. From the Display Mode pull-down select **Regurgitant**.
3. Click on the black vertical cursors and identify the start and end of the retrograde flow as shown in Figure 18.
The results will then appear in the upper right hand corner of the flow graph display.

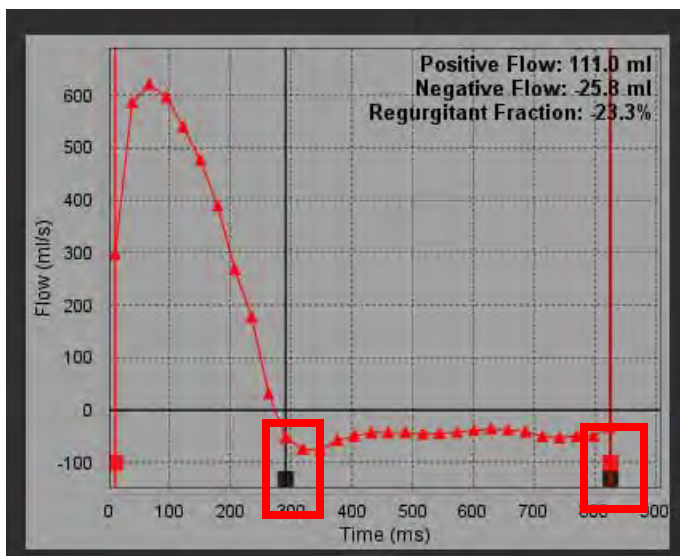FIGURE 18. Regurgitant Results




Pressure Half-Time

The Pressure Half-Time (PHT) can be obtained by measuring the deceleration slope of the E-wave on phase contrast images acquired of the mitral valve. This mode allows for the identification of slope of the graph to calculate the PHT and mitral valve area (MVA).

1. Generate a flow curve using the appropriate phase contrast series of the mitral valve.
2. For the ROI propagation use the copy paste option.
3. From the Display Mode pull-down menu, select **Pressure Half-Time**.
4. Click directly on the plot to identify the highest velocity of the deceleration portion of the curve.
5. Click an end point to calculate the slope of the curve as shown in Figure 19.
6. To reset the calculation, place the cursor over an end point, right mouse click and select the trash can.

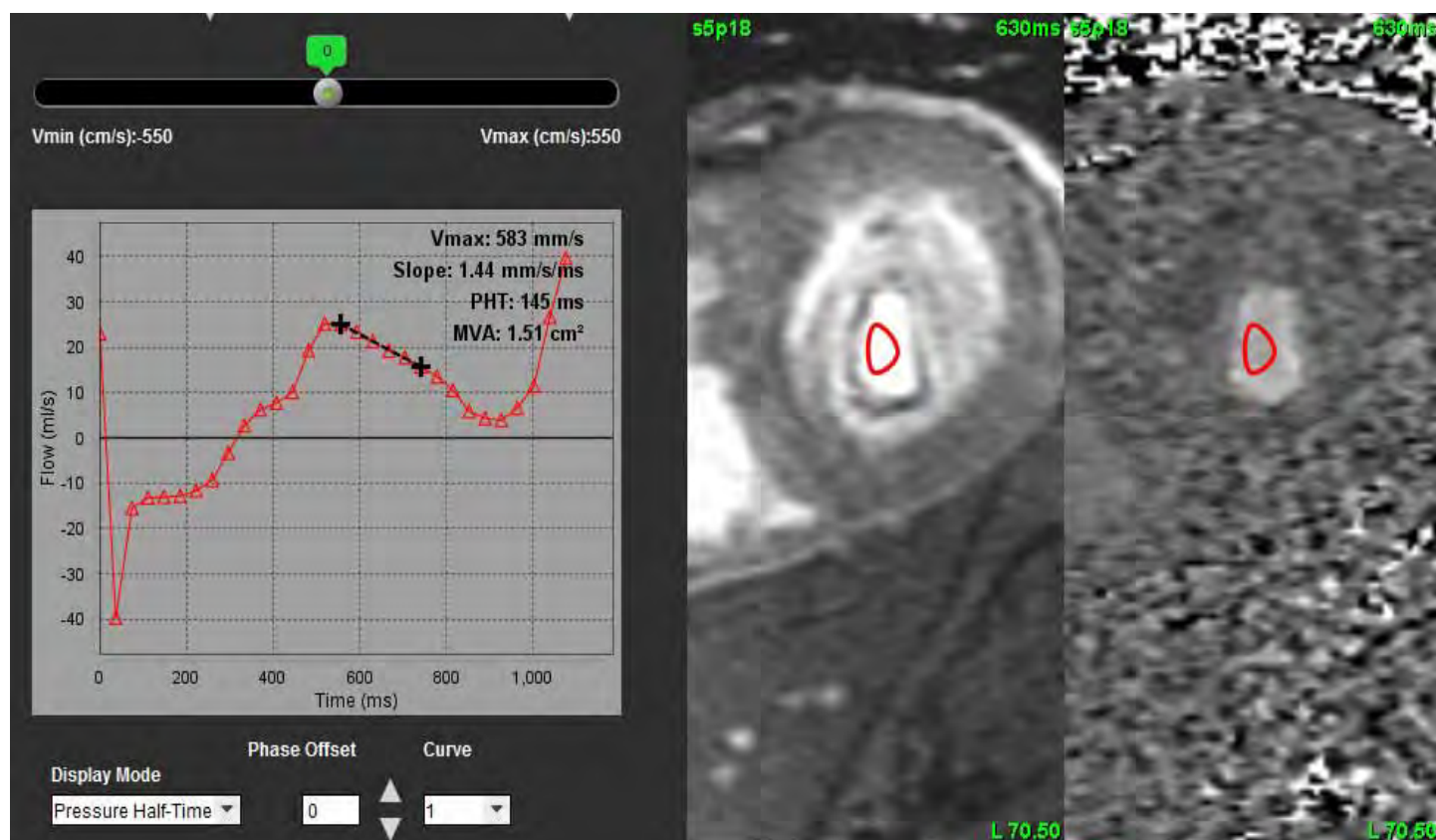FIGURE 19. Pressure Half-Time Results



NOTE: Mitral Valve area (MVA), Pressure Half-Time (PHT) results are not valid in patients with aortic insufficiency, cardiac shunt or decreased ventricular compliance.

NOTE: The series locate functionality, when clicking directly on the flow curve, is disabled when in PHT mode. Switch to flow mode to enable the locate functionality.

Reference:

<http://www.csecho.ca/wp-content/themes/twentyeleven/csecho/cardiomath/index.php?eqnHD=echo&eqnDisp=mvapht>

Review Summary Tables


FIGURE 20. Summary Table


The screenshot shows a software window titled "Summary Table". At the top is a drop-down menu labeled "All ROIs". Below it is a table with four columns: "ROI #", "Measurement", "Value", and "Unit". The table contains 12 rows of data. To the left of the table is a column of check boxes, each corresponding to a row. Below the table are two buttons: "Summary Table" (highlighted in green) and "Velocity Table". At the bottom right, there are two icons: a graph icon (highlighted with callout 3) and a table icon (highlighted with callout 4). Callout 1 points to the "All ROIs" drop-down menu, and callout 2 points to the first row of the table.

ROI #	Measurement	Value	Unit
✓ 1	Flow Rate	50.1	ml/s
✓ 1	Flow Rate	39.0	ml/beat
✓ 1	Peak Positive Velocity	248	cm/s
✓ 1	Peak Negative Velocity	-220	cm/s
✓ 1	Positive Flow Rate		ml/s
✓ 1	Positive Flow Rate		ml/beat
✓ 1	Negative Flow Rate		ml/s
✓ 1	Negative Flow Rate		ml/beat
✓ 1	Regurgitant Fraction		%
✓ 1	Cardiac Output	3.006	l/min
✓ 1	Heart Rate	77	bpm


1. ROI drop-down menu, 2. Flow Results, 3. Graph icon, 4. Summary/Velocity Table icon.

1. Select **All ROIs** from the drop-down menu to include the values of all the curves in the tables.

2. Select  to view the summary table or velocity table.

3. Click the check box to include or exclude the result from the report. 


4. Select  to toggle between summary and velocity table.

5. Select  to view the flow curves.

Qp/Qs Tab

The Qp/Qs reporting window differs from the layout of the vessel flow tabs reporting windows.

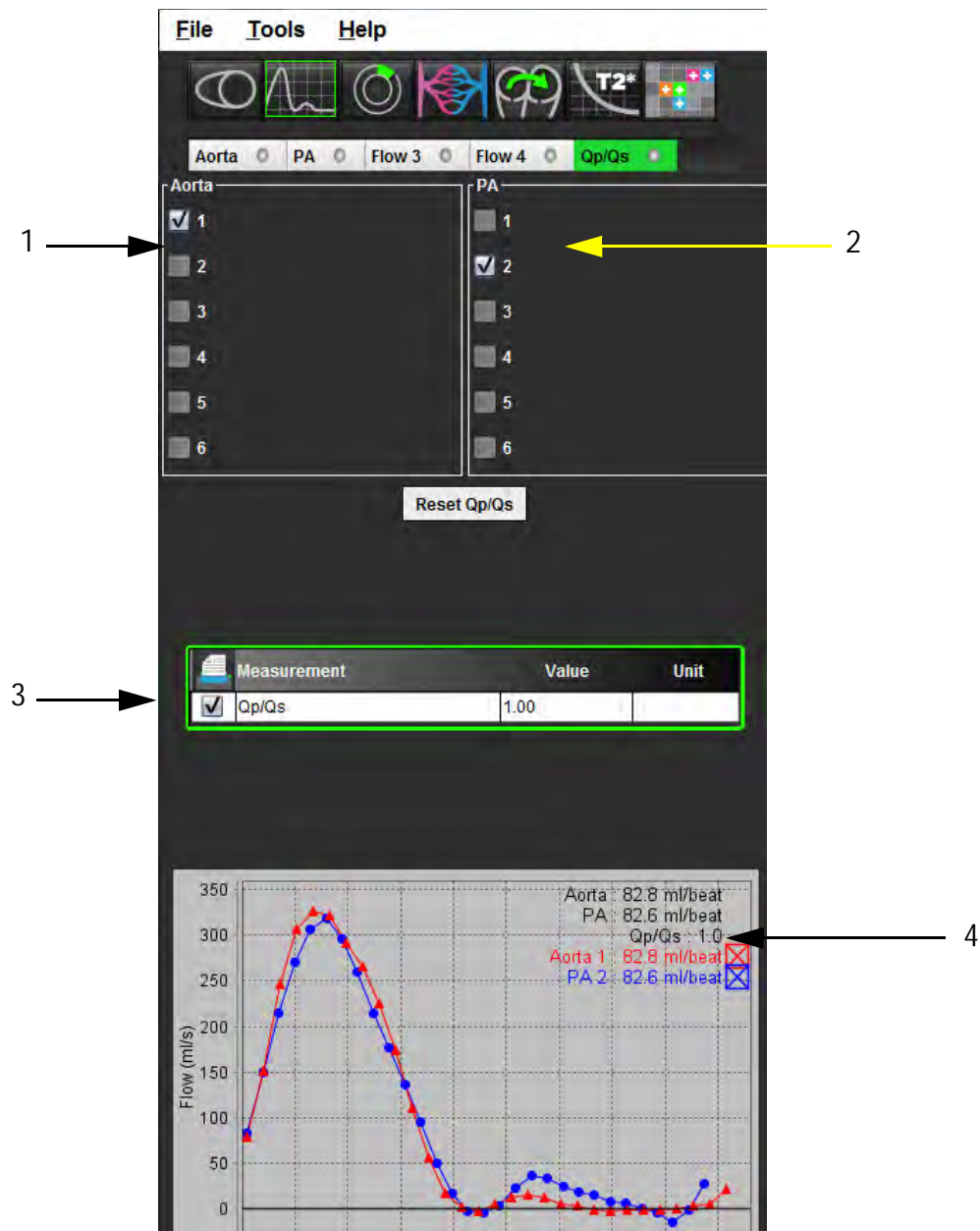FIGURE 21. Qp/Qs Reporting Window



1. Aorta curve, 2. Pulmonary artery curve, 3. Qp/Qs measurement, 4. Flow curves

Calculate Qp/Qs

NOTE: Flow curves must be generated on the appropriate series for the aorta and pulmonary artery prior to Qp/Qs calculations.

1. Select at least one aorta curve.
2. Select at least one pulmonary artery curve.

The Qp/Qs measurements are calculated automatically when both the aorta and pulmonary artery contours are selected. All contour selections affect the calculations. All the flow curves are displayed on the graph. Curves can be selected or deselected at any time.

Deselecting the display of any curve on the graph does not affect the calculations.

The system will average the values if more than one Aorta or PA is selected.

3. Select **Reset Qp/Qs** to reset the graph and all calculations on this tab, if desired.

Myocardial Evaluation

The user is responsible for the accurate and complete placement of all regions of interest (ROIs), including those generated or modified by the auto segmentation algorithms. The quantitative values generated by the software depend on the accurate and complete placement of these regions of interest and applied thresholding.


The study Preprocessing feature allows for the preprocessing of Late Enhancement. Refer to the suiteDXT Instructions for Use NS-03-009-0001.

The Myocardial Evaluation (ME) analysis tool aids in the quantitative determination of areas of different signal intensities within the myocardium.

There are three analysis tabs available:

- Late Enhancement - Determines myocardial segments of increased and low signal intensity.
- T2 - Determines myocardial segments of increased signal intensity from black-blood imaging techniques.
- Signal Differential - Displays the Salvage Mass results using both late enhancement and T2 analysis and the T2 signal intensity (SI) ratio.

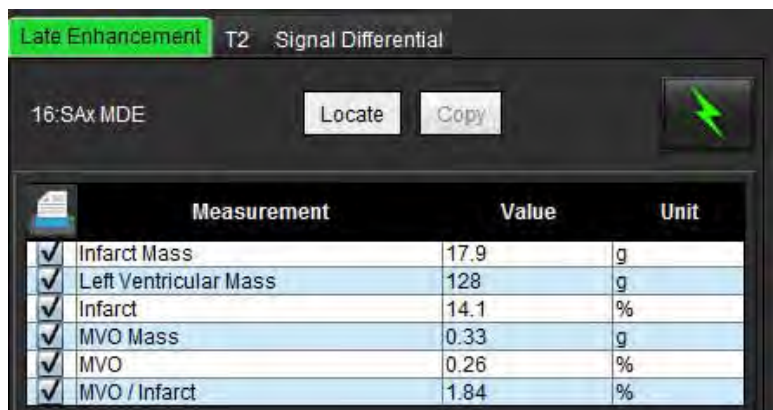FIGURE 1. Analysis Tabs



	Measurement	Value	Unit
<input checked="" type="checkbox"/>	Infarct Mass	17.9	g
<input checked="" type="checkbox"/>	Left Ventricular Mass	128	g
<input checked="" type="checkbox"/>	Infarct	14.1	%
<input checked="" type="checkbox"/>	MVO Mass	0.33	g
<input checked="" type="checkbox"/>	MVO	0.26	%
<input checked="" type="checkbox"/>	MVO / Infarct	1.84	%



WARNING: The application assists in the analysis of the images only and does not automatically produce quantifiable results. The use and placement of quantitative measurements is at the discretion of the user. Misdiagnosis can occur if measurements are inaccurate. Measurements should only be created by a properly trained and qualified user.

ME Quantitative Analysis Procedure

Late Enhancement






1. Select .
2. Select Late Enhancement tab.
3. Select the appropriate short axis series.
4. Select  to perform Auto Segmentation.
5. Review all endocardial and epicardial traces, RV insertion point and the thresholding on each slice. Edit thresholding as necessary.

FIGURE 2. ME Quantitative Analysis Reporting Window



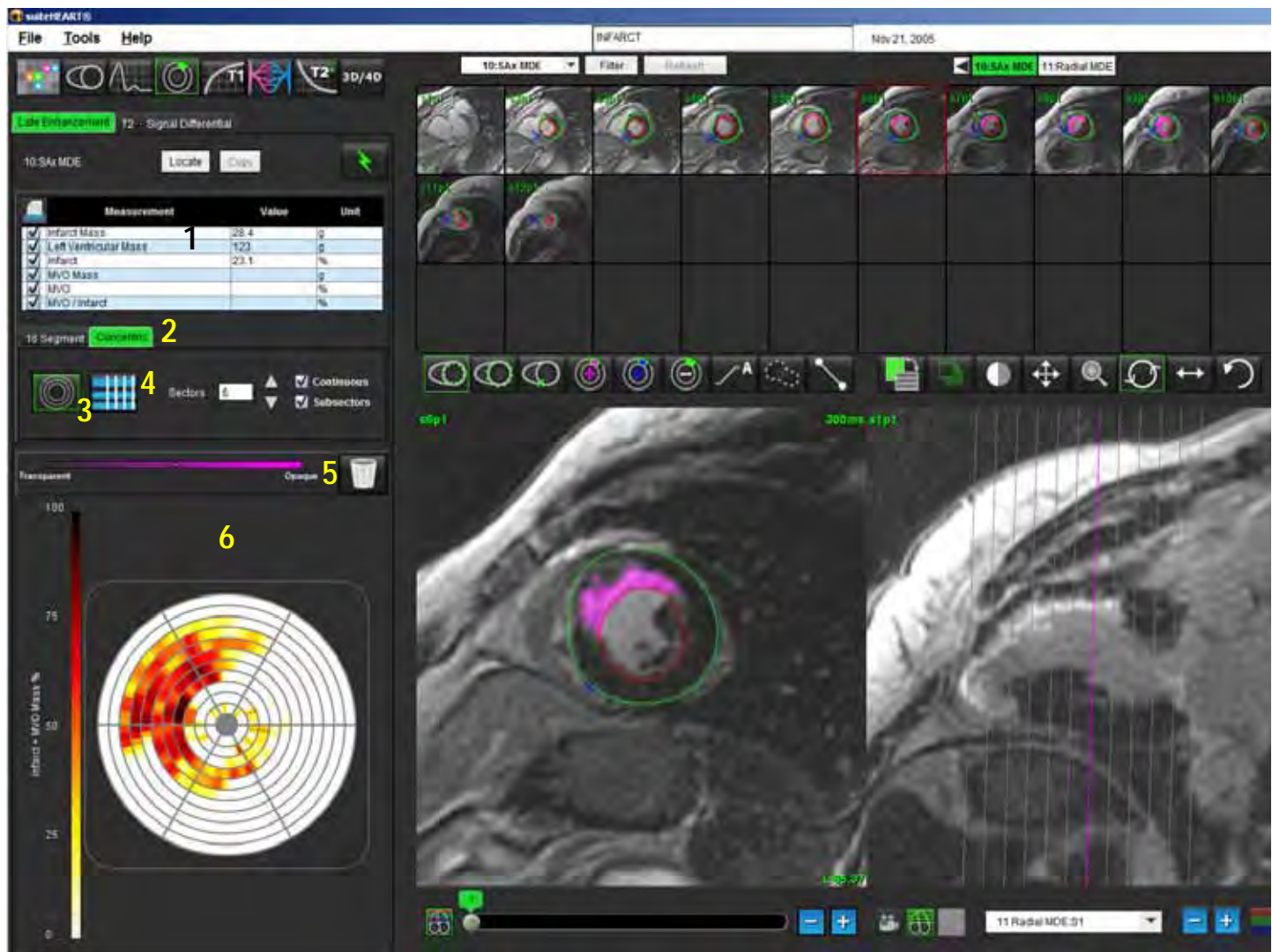1. Result table, 2. Polar Plot type, 3. Polar Plot section, 4. Table section, 5. Reset, 6. Quantitative Polar Plot


6. To perform manual segmentation, trace the LV endocardium on the most basal slice by selecting .

7. Trace LV epicardium by selecting .

8. Mark inferior RV insertion site with .

9. Move the cursor outside of the editor window to complete the ROI.


10. Repeat steps 4 - 7 until the entire ventricle is segmented.

11. Click the Slice Classification  button to open the controls for adjusting the thresholding for each slice. Pixels are identified using a Full Width Half Max (FWHM) algorithm. Confirm the base, mid and apical classification.

Threshold Editing

1. To add high signal intensity regions select .


2. To add low signal intensity regions select .

3. To delete either signal intensity regions select .

Polar Plot Display Formats

The ME analysis tool provides 2 polar plot formats: 16 Segment and Concentric

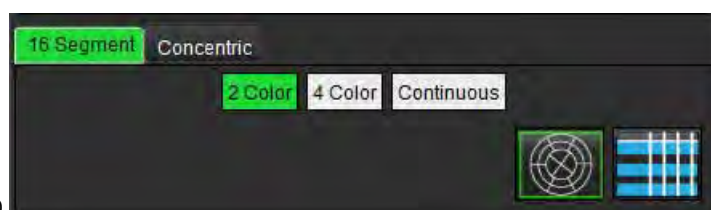
Option 1: 16-segment procedure




1. Select the **16 Segment** tab
2. Select 2 Color, 4 Color, or Continuous.

Color assignments can be defined by clicking on the color scale bar.

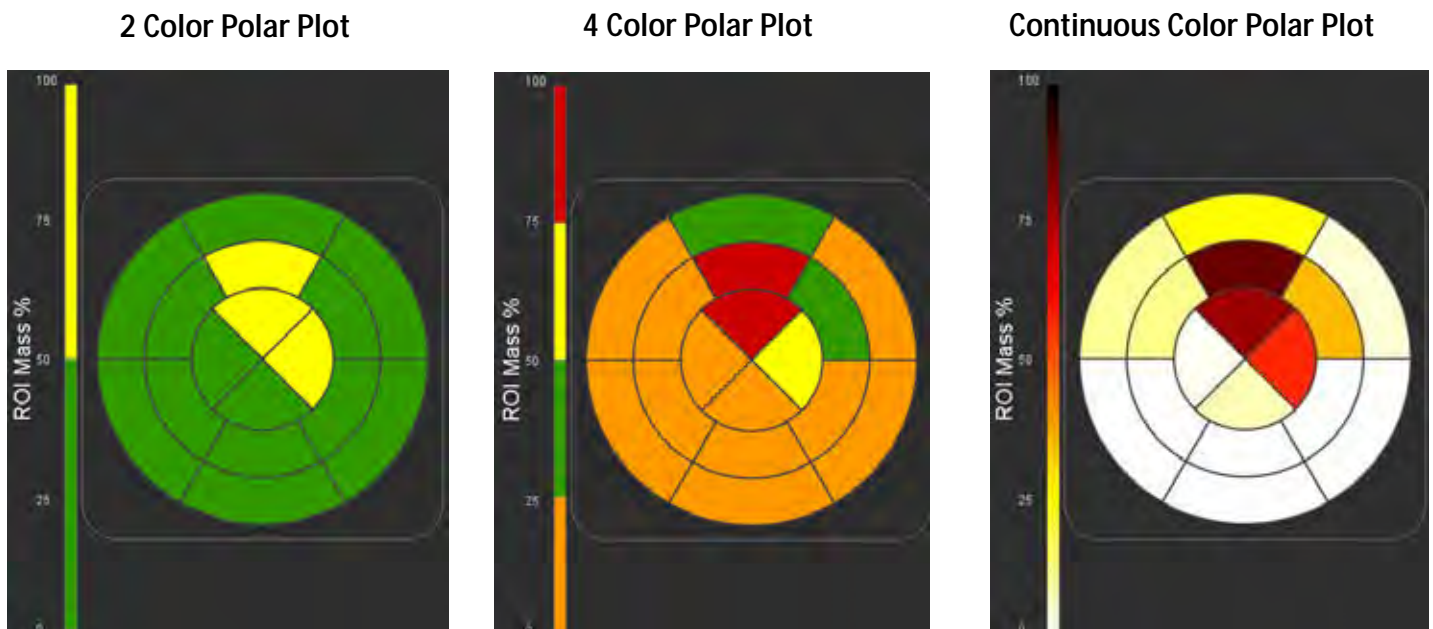FIGURE 3. Polar Plots




3. Select  to display the Polar Plot Summary Table.

Option 2: Slice by Slice format

1. Select the **Concentric** tab.

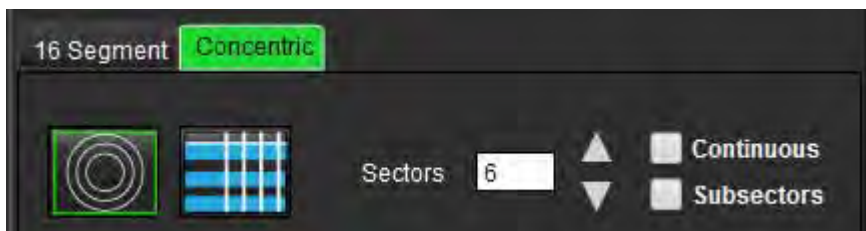FIGURE 4. Concentric Tab




The Concentric tab provides the preferences that change the format of the Polar Plot to a slice-by-slice format, where each ring represents a slice. The number of rings is determined by the number of slices analyzed.

2. Select the number of sectors.
3. Check the subsectors to show the ROI mass percent changes within the sector.
A smoothing function is applied when subsectors is selected.
4. Click the **Continuous** check box to change the Polar Plot to percent signal intensity and color code the values on a continuous spectrum from 0-100%.

Select  to reset the analysis.

NOTE: The semi-automatic thresholding for late enhancement analysis works optimally on high quality myocardial evaluation images as shown below (Image A). In images acquired without signal from the blood pool (Image B) or incorrect Inversion time, the threshold will need to be subjectively set by the user.

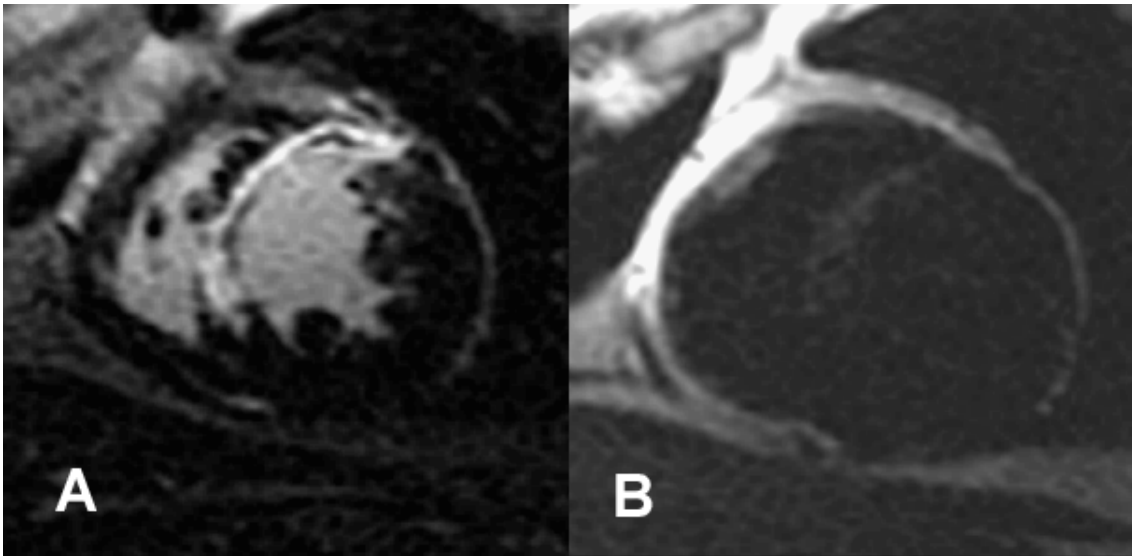FIGURE 5. Myocardial Evaluation Images



T2 Analysis

1. Select the T2 tab.
2. If the late enhancement series has been previously analyzed, the ROIs can be copied to the T2 series by clicking the copy button (see Figure 6).

NOTE: To copy ROIs it is required that the number of slices match for each series to get accurate results; if the number of slices does not match, the copy button will not be available. The DICOM import process can be used to create the appropriate series that contains the same number of slices.
Acquisition parameters, such as matrix and FOV, should be the same for each series for best results. After performing a copy, review the ROIs carefully on all of the slice locations and make appropriate edits.

FIGURE 6. Location of Copy Button

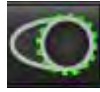


3. If there is no previous late enhancement analysis, the ROIs can be created manually.

4. Trace the LV endocardium on the most basal slice by selecting



5. Trace the LV epicardium by selecting




6. Mark the inferior RV insertion point by selecting



7. Move the cursor outside of the editor window to complete the ROI.


8. Repeat steps 4-7 until the entire ventricle is segmented.


9. To perform a 2 Standard deviation thresholding, select the Add Normal Region ROI  and place ROI in a normal segment. This ROI is copied to all of the images. Review each slice location and adjust the ROI as necessary.

NOTE: When the skeletal muscle ROI and Normal ROI are provided, the software performs the following calculation:

Normalized myocardial T2 SI = SI myocardium / SI skeletal muscle;

Threshold calculation: Threshold = 2 * STD NORMAL + AVG NORMAL

10. Click the Slice Classification  button to open the controls for adjusting the threshold results. Confirm the base, mid and apical classifications.

11. To perform T2 Signal Intensity analysis, select the Add Skeletal muscle ROI  and place an ROI in the skeletal muscle. This ROI is copied to all of the images. Review each slice location and adjust the ROI as necessary.

12. The Locate button will update the viewer with the appropriate series used for the selected analysis.

NOTE: Black-blood images may have insufficient flow suppression which could result in inaccurate signal intensity analysis and thresholding.

Threshold Editing

To add regions of high T2 signal intensity, select



To remove regions of high T2 signal intensity, select




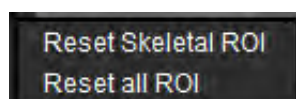
Select  to reset the analysis as shown in Figure 7.

FIGURE 7. Reset ROI Menu




Combined Analysis

Late Enhancement and T2

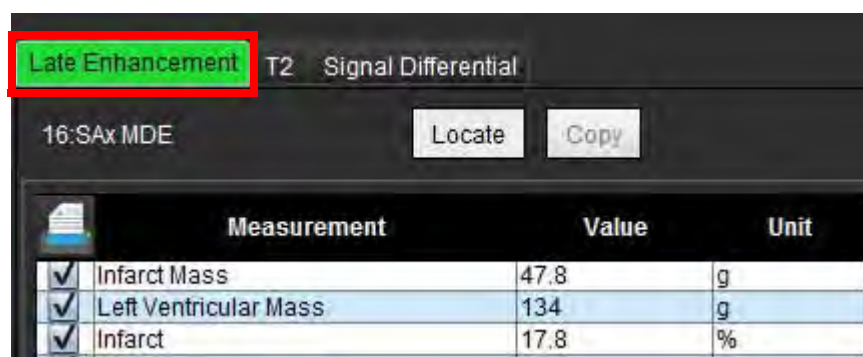
The combined analysis mode allows for side-by-side analysis with editing tools for late enhancement and T2 (Edema) images.

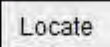
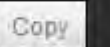

NOTE: To enable the combined analysis mode, the analysis of a short axis late enhancement series must be completed first using the Late Enhancement tab. T2 (Edema) images must be present in the same study.




1. Select .
2. Select an appropriate study with both late enhancement and T2 (edema) images. Complete the analysis procedure for late enhancement.




NOTE: Review the thresholding for each short axis slice on the Late Enhancement tab prior to selecting combined analysis mode.



Late Enhancement T2 Signal Differential			
16:SAX MDE			
 			
	Measurement	Value	Unit
<input checked="" type="checkbox"/>	Infarct Mass	47.8	g
<input checked="" type="checkbox"/>	Left Ventricular Mass	134	g
<input checked="" type="checkbox"/>	Infarct	17.8	%

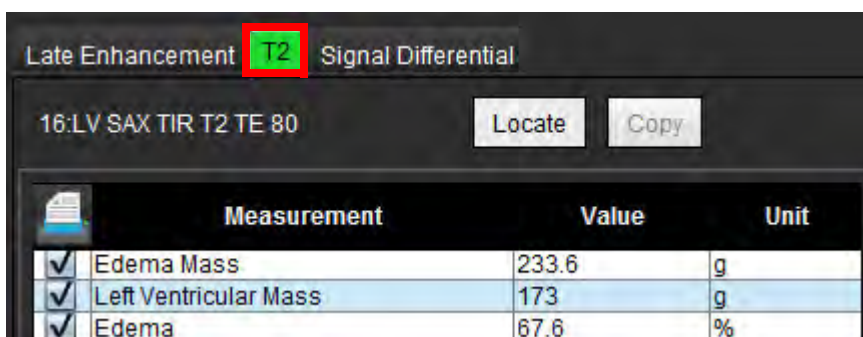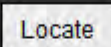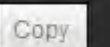
3. Select the T2 tab and complete the analysis procedure for the T2 series.



Late Enhancement T2 Signal Differential			
16:LV SAX TIR T2 TE 80			
 			
	Measurement	Value	Unit
<input checked="" type="checkbox"/>	Edema Mass	233.6	g
<input checked="" type="checkbox"/>	Left Ventricular Mass	173	g
<input checked="" type="checkbox"/>	Edema	67.6	%

4. Select  to start combined analysis, as shown in Figure 8.

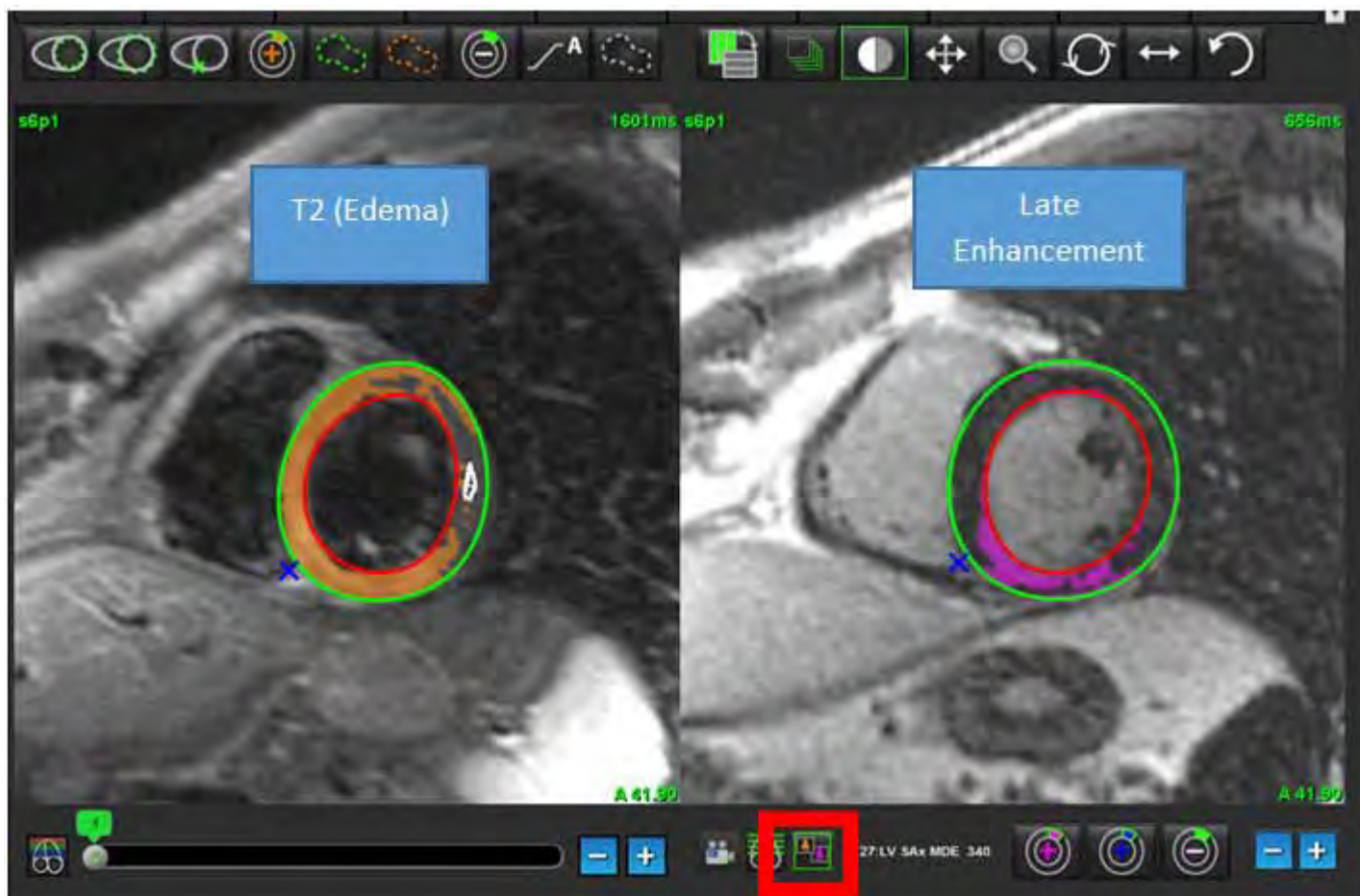FIGURE 8. Combined Analysis Mode




5. Upon selection, the previously analyzed late enhancement series will appear in the mode view window. This window then becomes an editor window for the late enhancement images.
6. For editing of the late enhancement images, use the editing tools located below the image viewport as shown in Figure 9.

NOTE: Confirm all updates to the results directly on the Late Enhancement tab.


NOTE: If the LV endocardium or LV epicardium ROIs are deleted, go back to the Late Enhancement tab to retrace.

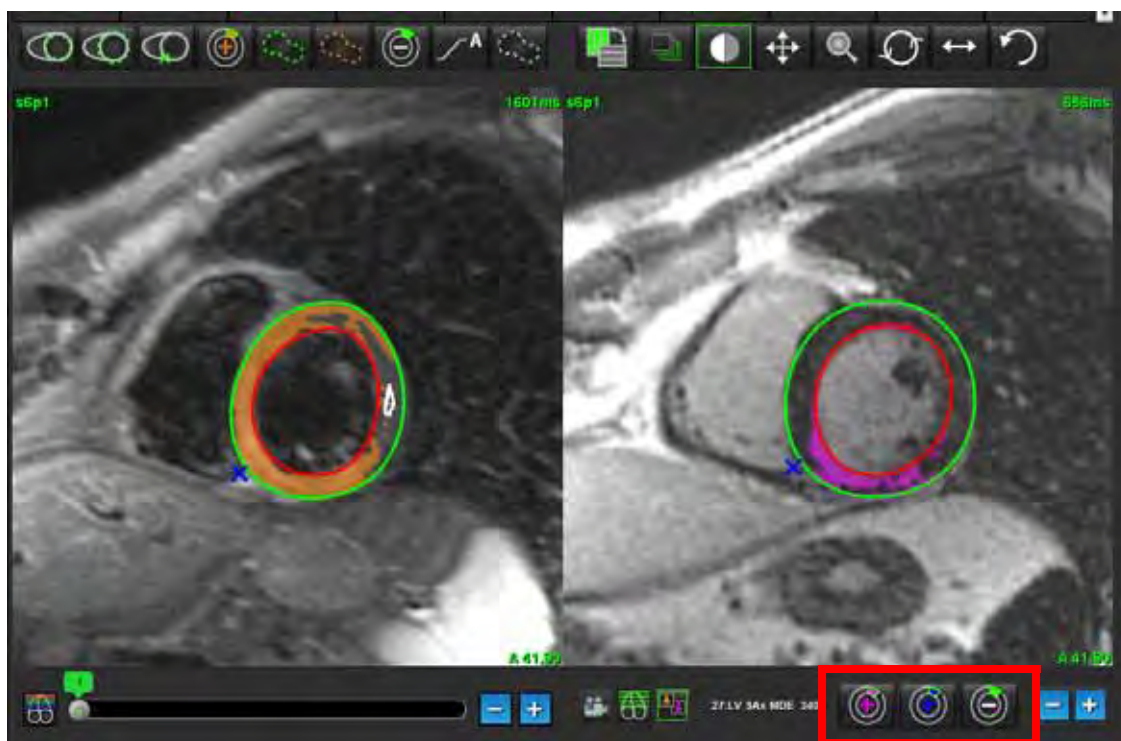FIGURE 9. Late Enhancement Editing Tools



7. For editing of the T2 (Edema) series on the left, use the editing tools located above the image viewport, as shown in Figure 10.

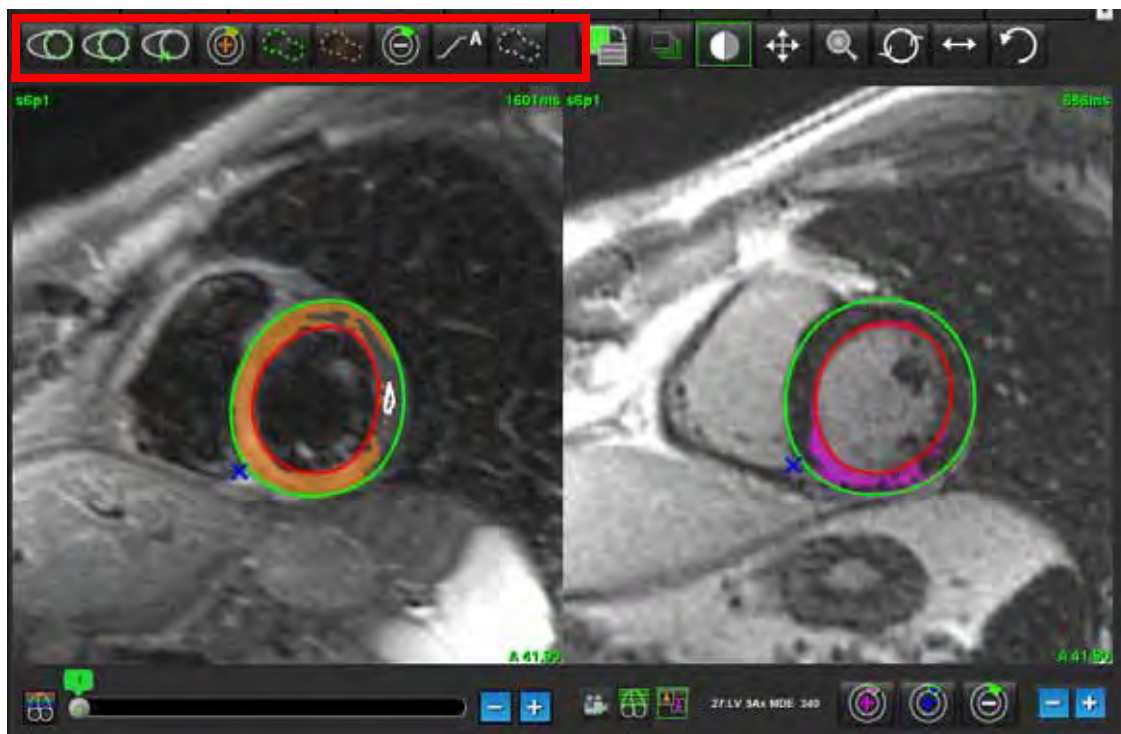
FIGURE 10. T2 (Edema) Analysis Tools



8. Use the minus and plus buttons to navigate to a different slice level for the late enhancement series, as shown in Figure 11.
- Slice location information is located in the lower righthand corner of each viewport.

NOTE: The slice location displayed for the late enhancement is determined by the slice location in the T2 (Edema) editor window. Use the minus/plus buttons to override this selection.

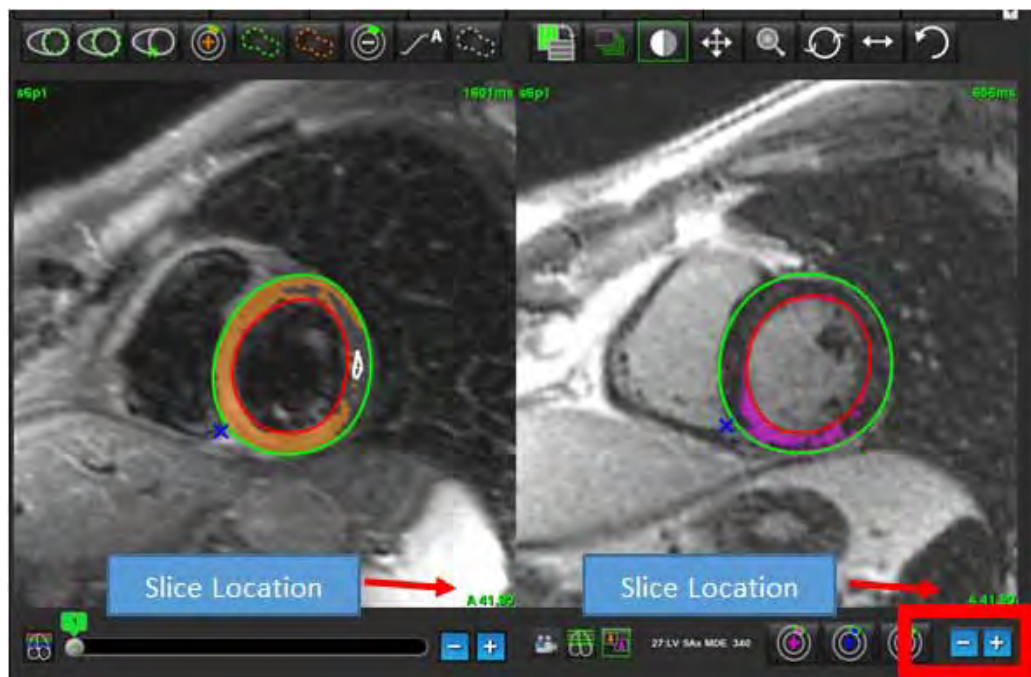
FIGURE 11. Slice Navigation Controls Late Enhancement



9. Use the minus and plus buttons located below the T2 (edema) editor viewport to navigate to a different slice level for both the late enhancement and T2 (Edema) series, as shown in Figure 12.

NOTE: In combined analysis mode the plus and minus buttons on the left link slice navigation for both viewports.

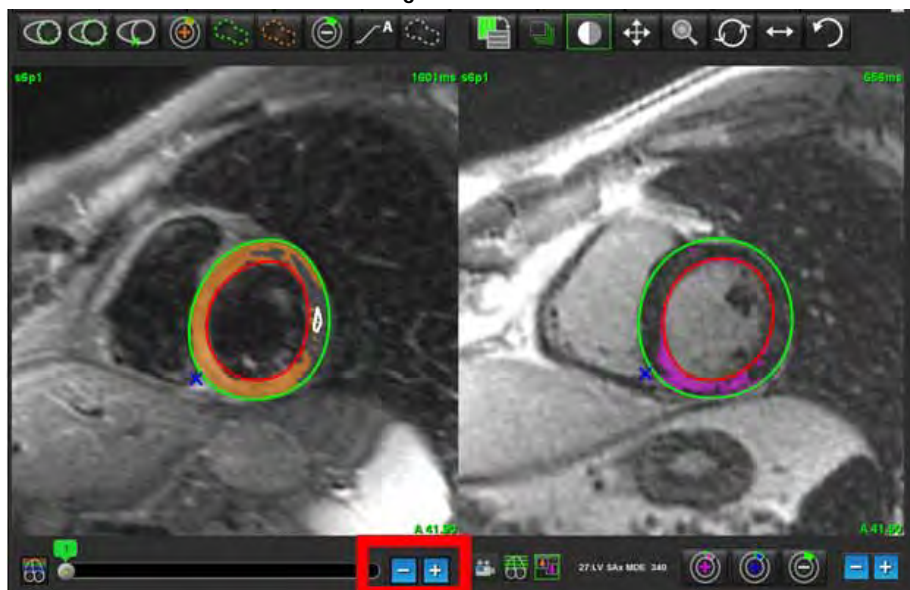FIGURE 12. Combined Slice Navigation Controls



Signal Differential Results

Select the Signal Differential Tab

NOTE: Late enhancement and T2 analysis must be completed to obtain the Salvage Mass Results. The T2 analysis must be completed with the placement of the Skeletal muscle ROI for T2 Signal Intensity (SI) analysis.

NOTE: If the T2 (Edema) result is less than the Late Enhancement result (Infarct + MVO) the Salvage Mass result will be blank.

FIGURE 13. Signal Differential Tab

Late Enhancement T2 Signal Differential			
			
Measurement		Value	Unit
<input checked="" type="checkbox"/>	Salvage Mass		g
Slice	T2 SI Ratio	Myo SI	SM SI
1	—	—	—
2	—	—	—
3	1.2	62	51
4	1.4	84	58
5	1.7	95	56
6	1.7	102	60
7	1.6	98	62
8	1.7	107	63
9	2.0	119	59
10	2.5	140	56
11	2.5	140	56
12	2.5	140	56
13	2.5	140	56
14	2.5	140	56

This page intentionally left blank.

T1 Mapping Analysis

This feature allows for the signal quantification of the longitudinal spin-lattice relaxation time (T1). The application supports T1 analysis for both Native (non-enhanced) and Post enhancement images and the calculation of the extracellular volume fraction (ECV).

Required Images: Inversion or saturation recovery images with varying inversion times (TI). Series that have motion corrected applied are recommended for analysis. Representative slice locations for the left ventricular base, mid and apex are recommended.



WARNING: The application assists in the analysis of the images only and does not automatically produce quantifiable results. The use and placement of quantitative measurements is at the discretion of the user. Misdiagnosis can occur if measurements are inaccurate. Measurements should only be created by a properly trained and qualified user.

Launch T1 Mapping Analysis




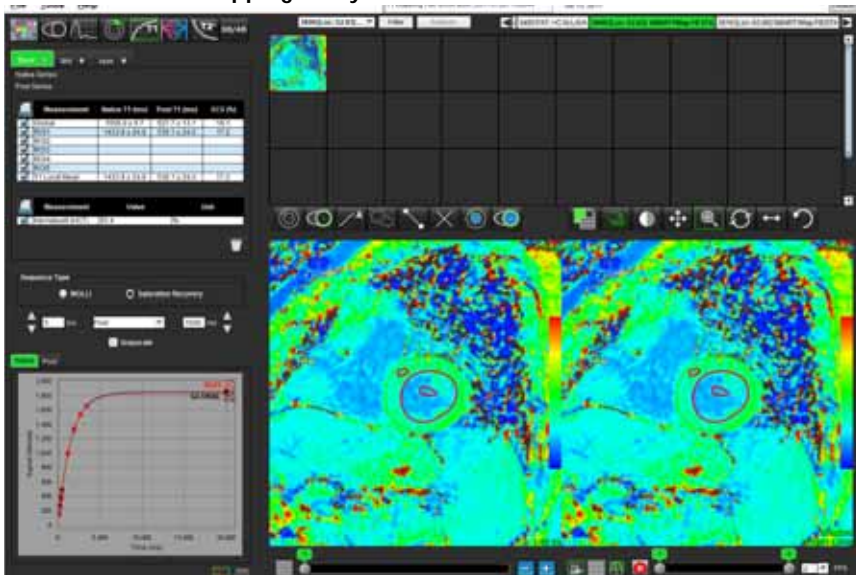
1. Select .
2. Select the appropriate T1 Mapping series.
3. Select the appropriate tab: Base, Mid or Apex.
4. Select the appropriate short axis slice from the series listing or thumbnail panel.
5. Select MOLLI for the analysis of inversion recovery images or Saturation Recovery.

FIGURE 1. T1 Mapping Analysis



Native T1 Analysis Procedure



1. To perform regional T1 analysis select the Native Local ROI tool and create a region of interest within the myocardium.
2. The Native Local T1 result for the region in milliseconds (ms) will be displayed in the result table along with the T1 curve. If there is more than 1 Native Local ROI placed on the image the T1 Local Mean result will update.
3. Up to 5 Native Local ROI measurements can be created on an image per tab.



4. To perform a Global T1 analysis select the LV Native endocardium and trace the endocardial boarder.



5. Select the LV Native epicardium and trace the epicardial boarder. The Global T1 result will be displayed in the result table.

Post Enhancement T1 Analysis Procedure



1. To perform regional T1 analysis use the Post Local ROI tool and create a region of interest within the myocardium.
2. The Post Local T1 result for the region in milliseconds (ms) will be displayed in the result table and T1 curve displayed in the result table. If there is more than 1 Post Local ROI placed on the image the T1 Post Local Mean result will update.
3. Up to 5 Post Local ROI measurements can be created on an image per tab.



4. To perform a Global T1 analysis select the Post LV endocardium and trace the endocardial boarder.




5. Select the Post LV epicardium and trace the endocardial boarder. The Global T1 result will be displayed in the result table.

Display a Color Parametric Map

NOTE: A Local or a Global ROI is required to display a color parametric map option.

1. Select the appropriate option from the file pull-down menu as shown in Figure 2.
2. Click Grayscale to display parametric map options in grayscale.

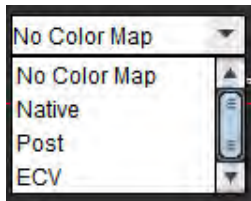
FIGURE 2. Color Map Options



ECV Quantification

1. Complete either a Local or Global T1 analysis for both Native and Post-enhancement images.



2. Select  for Native images and place a ROI in the left ventricular cavity.



3. Select  for Post-enhancement images and place a ROI in the left ventricular cavity.

4. Enter the Hematocrit (HCT) value.

5. The ECV result (%) will be displayed in the result table.

Reference: Wong. et al., "Association Between Extracellular Matrix Expansion Quantified by Cardiovascular Magnetic Resonance and Short-Term Mortality." *Circulation* (2012):126:1206-1216.

Create a 16-Segment Polar Map

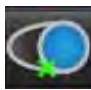
NOTE: ECV Polar Plot requires that the ECV analysis is completed.

1. Complete the Global T1 analysis for Base, Mid and Apex tabs.



2. Select  for Native images and place the RV insertion point.



3. Select  for Post-enhancement images and place the RV insertion point.



4. Select the 16 Segment Polar Plot .



5. Select Graphs  to return to the T1 curves.

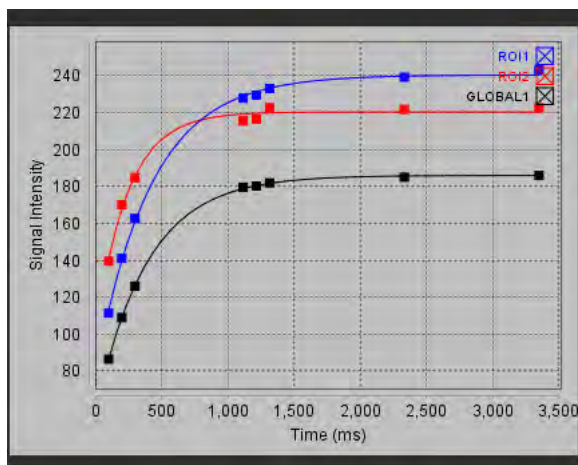
Reset the Analysis



Click on the trashcan icon and select: Local->All, Native or Post, Global->All, Native or Post, Blood Pool->All, Native or Post, Clear ALL ROIs.

Review the T1 Curves

1. The curve fitting results show the signal behavior from the image data. In cases of image artifacts due to misregistration, breathing artifacts or arrhythmias the curve fit may not be optimal.
2. A signal intensity point can be eliminated from the calculation by clicking directly on the point on the graph and selecting the contour on the image which turns purple.
3. Select delete from the right mouse (click and hold) or select the keyboard delete.



WARNING: Review the results of the T1 curve fit. A properly trained and qualified user should perform the analysis.

Result	Equation Reference	Fit Type
T1 Look-Locker (MOLLI)	$y=A-B \exp(-t/T1^*)$	Nonlinear curve fitting using a Levenberg-Marquardt algorithm*

Reference: *Messroghli D. R. et al., "Modified Look-Locker Inversion Recovery (MOLLI) for High Resolution T1 Mapping of the Heart." Magnetic Resonance in Medicine (2004) 52: 141-146.

T2 Mapping Analysis

This feature allows for the signal quantification of the T2 relaxation time. T2 mapping is a tissue characterization technique.

Required Images: T2 preparation sequence with a steady-state free precession readout with varying echo times (TE). Representative slice locations for the left ventricular base, mid and apex and recommended.

The calculation for the T2 decay curve is: $y = a \exp(-TE/T2^*) + c$

There are 2 fitting correction methods a 2-parameter fit where the background noise c is calculated using a histogram based algorithm and subtracted from the signal intensity after which a non-linear fit is performed. The 3-parameter fit uses a non-linear approach.



WARNING: The application assists in the analysis of the images only and does not automatically produce quantifiable results. The use and placement of quantitative measurements is at the discretion of the user. Misdiagnosis can occur if measurements are inaccurate. Measurements should only be created by a properly trained and qualified user.

Launch T2 Mapping Analysis




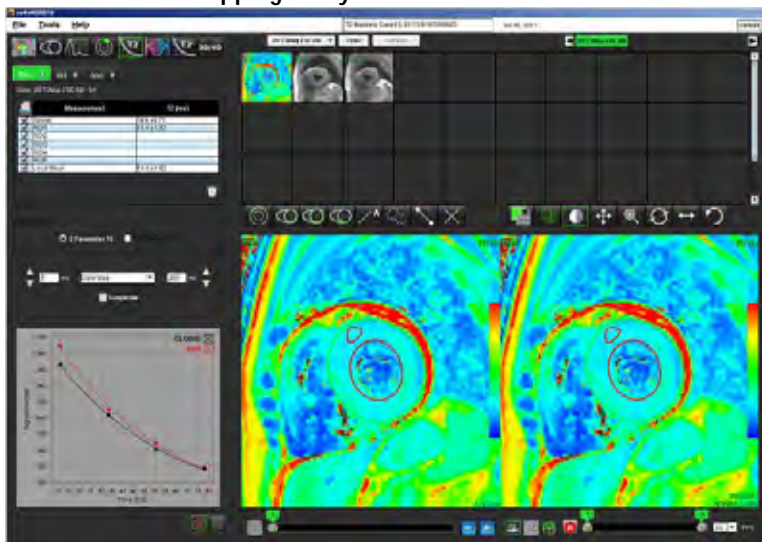



1. Select .
2. Select the appropriate T2 Mapping series.
3. Select the appropriate tab either Base, Mid or Apex.
4. Select the appropriate short axis slice from the series listing or thumbnail panel.
5. For the fitting correction select either a 2-Parameter or a 3-Parameter Fit.

FIGURE 1. T2 Mapping Analysis




T2 Analysis Procedure

1. To perform regional T2 analysis select the Local ROI tool  and create a region of interest within the myocardium.
2. The Local T2 result for the region in milliseconds (ms) will be displayed in the result table along with the T2 curve. If there is more than 1 Local ROI placed on the image the T2 Local Mean result will update.
3. Up to 5 local ROI measurements can be created on an image per tab.
4. To perform a global T2 analysis select the LV endocardium  and trace the endocardial boarder.
5. Select the LV epicardium  and trace the epicardial boarder. The Global T2 result will be displayed in the result table.

Display a Color Parametric Map

Select the appropriate option from the file pull-down menu as shown in Figure 2.

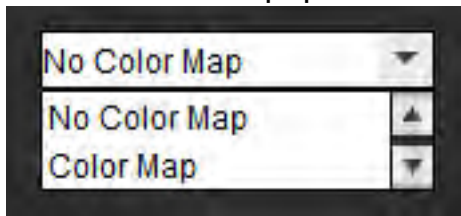FIGURE 2. Color Map Options



Create a 16-Segment Polar Map

1. Complete the Global T2 analysis for Base, Mid and Apex tabs.

2. Select  for the RV insertion point.

3. Select the 16 Segment Polar Plot .

4. Select Graphs  to return to the T2 curves.

Reset the Analysis



Click on the trashcan icon and select: Clear All, Clear Local ROIs, Clear Global ROIs.

Review the T2 Curves

1. The curve fitting results show the signal behavior from the image data. In cases of image artifacts due to misregistration, breathing artifacts or arrhythmias the curve fit may not be optimal.
2. A signal intensity point can be eliminated from the calculation by clicking directly on the point on the graph and selecting the contour on the image which turns purple.
3. Select delete from the right mouse (click and hold) or select the keyboard delete.



This page intentionally left blank.

Time Course Analysis

The Time Course analysis tool allows the user to review and analyze myocardial time course images.

NOTE: Time Course results may not be accurate on images that are not surface coil intensity corrected.



WARNING: The application assists in the analysis of the images only and does not automatically produce quantifiable results. The use and placement of quantitative measurements is at the discretion of the user. Misdiagnosis can occur if measurements are inaccurate. Measurements should only be created by a properly trained and qualified user.

Launch Time Course

Time Course Quantitative Procedure




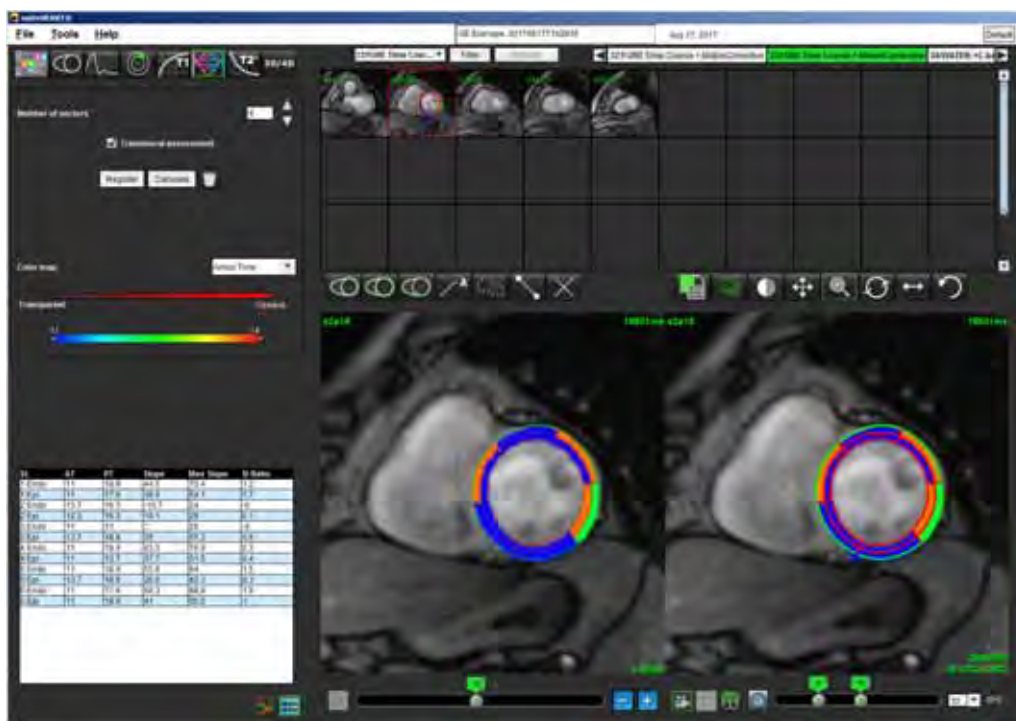
1. Select .
2. Select a myocardial time course series.



FIGURE 1. Time Course Analysis Window




Define Endocardial and Epicardial Contours

1. Select start and end phases using the cine window controls.

IMPORTANT: Be sure the phase selected on which to draw ROIs is included within the start and end range of phases.

2. Select  to draw the endocardial contour.
3. Select  to draw the epicardial contour.
4. Move the cursor out of the editing window.
5. Click **Register** to perform auto registration on all the images defined within the start and end phases.

Review the Image Registration

1. Click the  button.
2. Review the image registration in the thumbnails and adjust contours as needed. Press **Shift** + middle-click or select the pan tool to pan the images to correct registration. The scope of this pan/manual registration can be controlled by using the Scope selection.

Generate Curve and Color Map Overlay

1. Select the **Number of Sectors** from a range of 1 - 16.

The sectors are numbered counter clockwise from the RV insertion point.

2. Click on the **transmural assessment check box** to create radial regions to distinguish between sub-epicardial and sub-endocardial values if desired.


3. Define the RV insertion point by selecting  to deposit the cursor to mark the image.

4. Click **Calculate**.

A graph is generated that averages the signal intensity values versus time within each sector.

To change the results, select from the color map drop down menu.

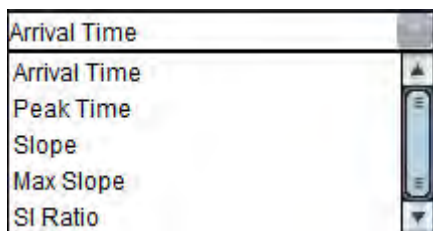FIGURE 2. Color Map Pull-down Menu




Review Results

1. Select **Chart icon** to review the Signal Intensity versus Time graph .

A color code of the various sectors is provided below the graph. Placing the cursor directly on a colored segment on the image will highlight the corresponding graph for that segment.

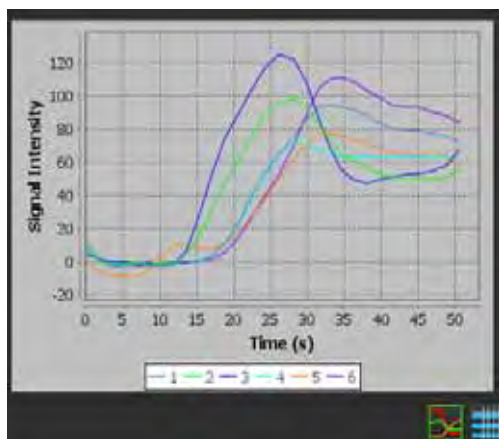
FIGURE 3. Signal Intensity/Time Graph



2. Select **Table icon** to review the summary result table .

FIGURE 4. Summary Result Table

SL	AT	PT	Slope	Max Slope	SI Ratio
1	15.4	33.1	9.4	15.2	2.5
2	11.9	28.1	10.9	15.2	3.1
3	11.9	28.1	13.5	23.1	4.6
4	15.4	29.9	9.1	14.3	3
5	17.1	33.1	8.5	12.1	2.4
6	17.1	34.6	11.1	18.3	3.8

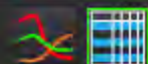


3. Placing the cursor directly on a colored segment on the image will highlight the corresponding result in green for that segment in the result table.

Graph and Contour Reset

Reset of the graph and contours can be performed at any point by clicking on .

Time Course Curve

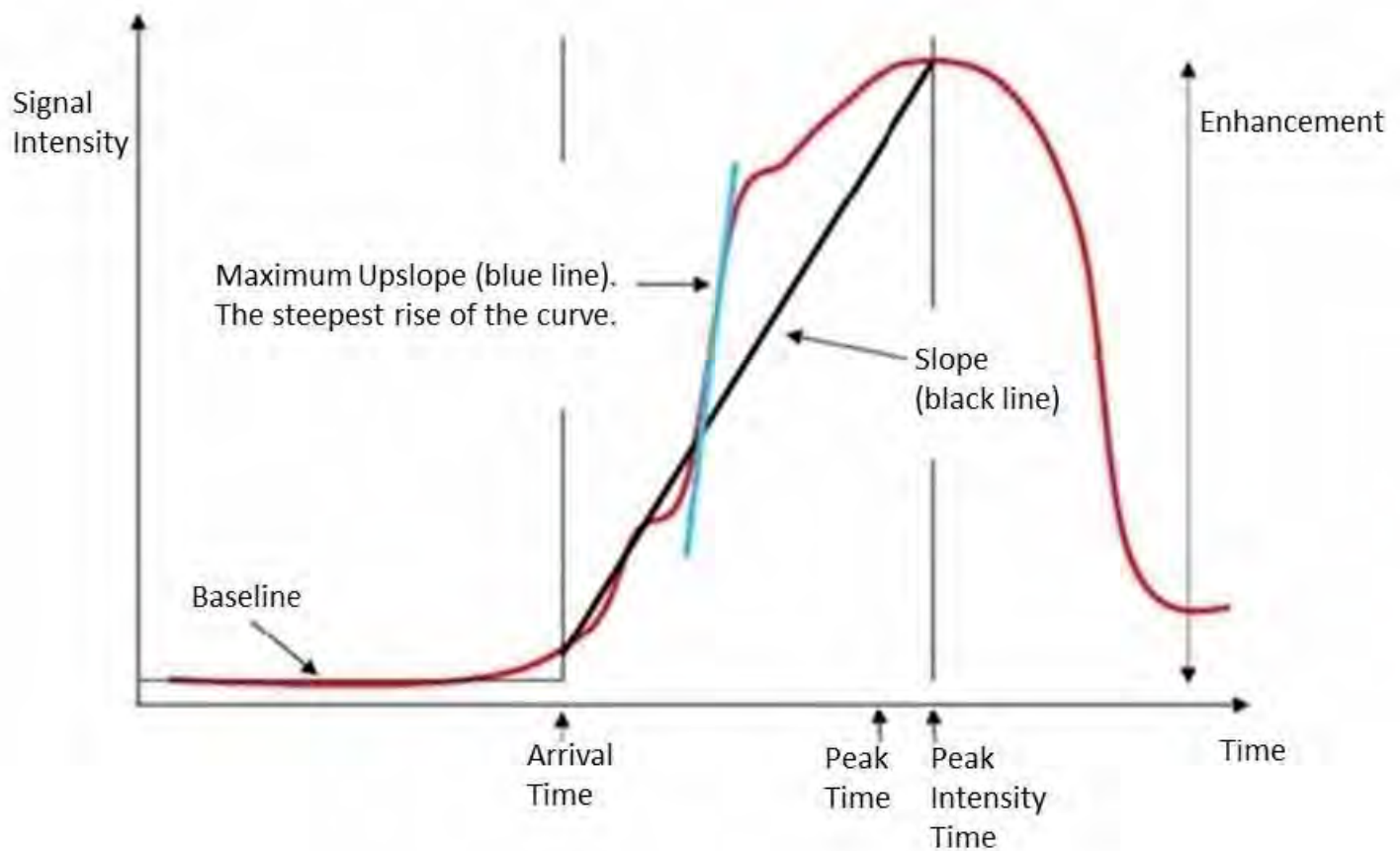The Time Course Curve results are defined as follows:



Where:

slope	is the gradient difference of the arrival time and the peak time,
maximum upslope	is the highest gradient (rise) of the curve representing intensity over time,
signal intensity ratio	is calculated as follows: $\frac{\text{Intensity enhancement (peak signal - baseline signal)}}{\text{Baseline}}$
peak time	is time (in seconds) of which the signal intensity reaches 90% of the peak signal intensity
arrival time	time (in seconds) of which the signal intensity reaches 10% of the peak signal intensity

This page intentionally left blank.

Patent Foramen Ovale (PFO) Analysis

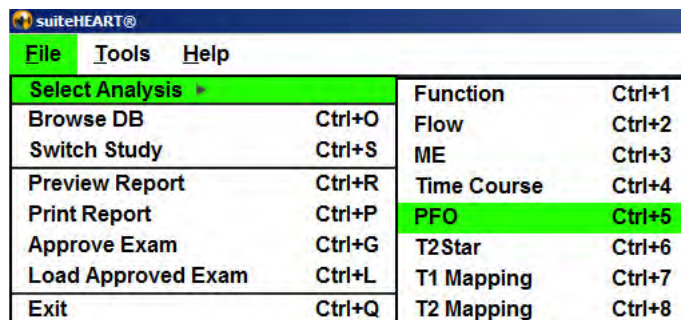
The PFO analysis tool allows the generation of signal versus time curves to demonstrate an early peak for the detection of a PFO.



WARNING: The application assists in the analysis of the images only and does not automatically produce quantifiable results. The use and placement of quantitative measurements is at the discretion of the user. Misdiagnosis can occur if measurements are inaccurate. Measurements should only be created by a properly trained and qualified user.


Launch PFO

1. Select File > Select Analysis > PFO.



2. Select a realtime series.

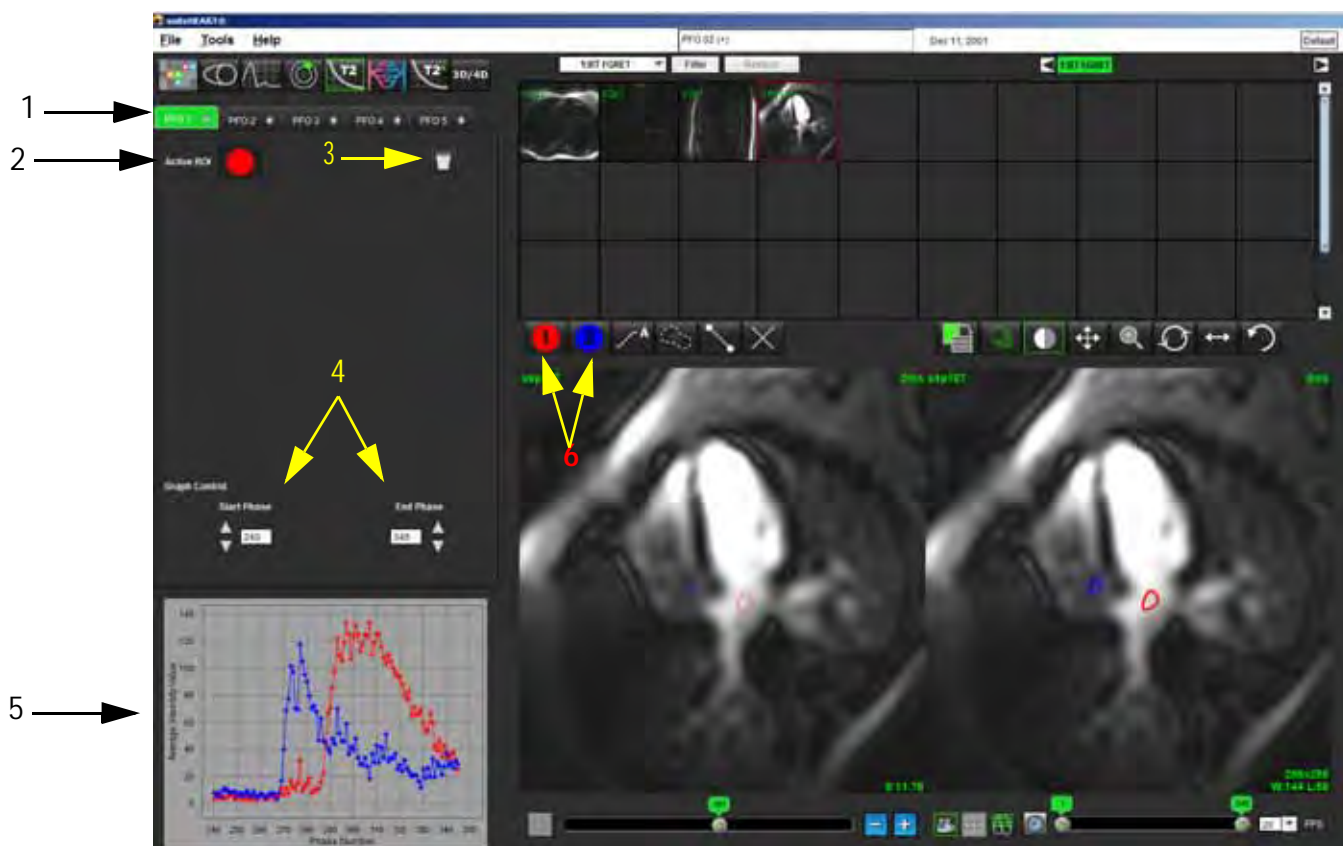FIGURE 1. PFO Analysis Window




1. PFO editable tabs, 2. Active ROIs, 3. Reset, 4. Start and End phase, 5. Signal Intensity vs Phase curve, 6. PFO Analysis icons

Select Atrial Anatomy

1. Select a PFO slice where the anatomy of the left atrium (LA) and right atrium (RA) can be appreciated.

Generate Left Atrial (LA) Intensity Curve

1. Draw the curve by selecting .
 2. Trace a contour on the LA in the Image Editor window.
 3. Move the cursor out of the Image Editor window.
 4. Generate LA intensity curve.
- The signal intensity curve for the LA is automatically generated.

Generate Right Atrial (RA) Intensity Curve


1. Generate the RA intensity curve following the same steps previously listed to generate the LA intensity curve while

using .

The curves are overlaid and displayed in the curve results display window.

NOTE: If a ROI has been placed on phase 1, for example, and the start phase is changed, the user drawn ROI will still be present on the original image where the ROIs were placed.

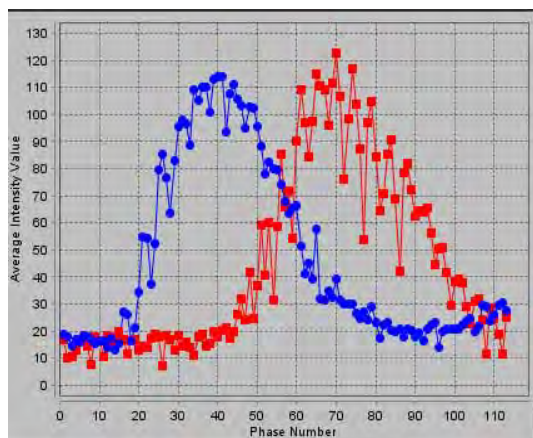FIGURE 2. PFO Curve Results



Review Curve Data and Select Phase Range

1. Review the curves in the report window and adjust the **Start Phase** and **End Phase**.
2. Use the up and down arrows to select the **Start Phase** and **End Phase** to set the phase range for curve display.

Adjusting the start and end phases affect the display of the PFO curves.

Clicking on a point on the graph updates the phase displayed in the Image Editor window.

FIGURE 3. Start and End Phase Selection Screen



NOTE: If there are two acquisitions in the same series, you can set the Start and End Phases for the first acquisition, draw the LA and RA ROIs (resulting in automatic generation of curves), and then repeat the process on another PFO tab for the second set of images. All PFO tab labels are editable.

Editing Contours

Editing multiple phases at a single slice location:

1. Select the slice location



2. Select
3. Select the first phase of the range of phases to be edited.
4. Depress and hold the shift key and select the last phase of the range to be edited.

The selected thumbnails will appear highlighted with a red border.

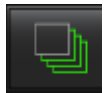
5. Edit the contour in the image editor window.
6. Deselect the contour by either clicking on the image away from the selected contour or move the cursor out of the editor window.

ROI editing can be controlled by setting the scope.

Select the proper scope function from the Image View.



Scope All – Applies ROI edits to all phases.



Scope Current to End – Applies ROI edits from the current phase to the end.



Scope Current Only – Applies ROI edits to the current phase only.

Reset Graph and Contours

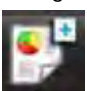
1. Optional: Click  to make the appropriate selection from the Reset PFO menu.

FIGURE 4. Reset PFO Curve Menu



Review Final Curve Results

A graph is generated from the contours showing pixel intensity versus time. Right mouse click on the

graph  to send to the report.

T2Star

The T2Star analysis tool calculates the T2* values of tissue from a multi-echo fast gradient echo sequence.

The T2* curve is a graph of the signal intensity versus echo time using an exponential decay curve formula. The T2* fitting algorithm is based on Levenberg-Marquardt non-linear least square algorithm.

The calculation for the T2* decay curve is: $y = a * \exp(-TE/T2^*) + c$

Where:


y	is the signal intensity at time TE
a	is the transverse magnetization at time 0 (zero)
TE	is the echo time
T2*	is the decay constant, and
c	is the background noise



WARNING: The application assists in the analysis of the images only and does not automatically produce quantifiable results. The use and placement of quantitative measurements is at the discretion of the user. Misdiagnosis can occur if measurements are inaccurate. Measurements should only be created by a properly trained and qualified user.

Heart Analysis Procedure

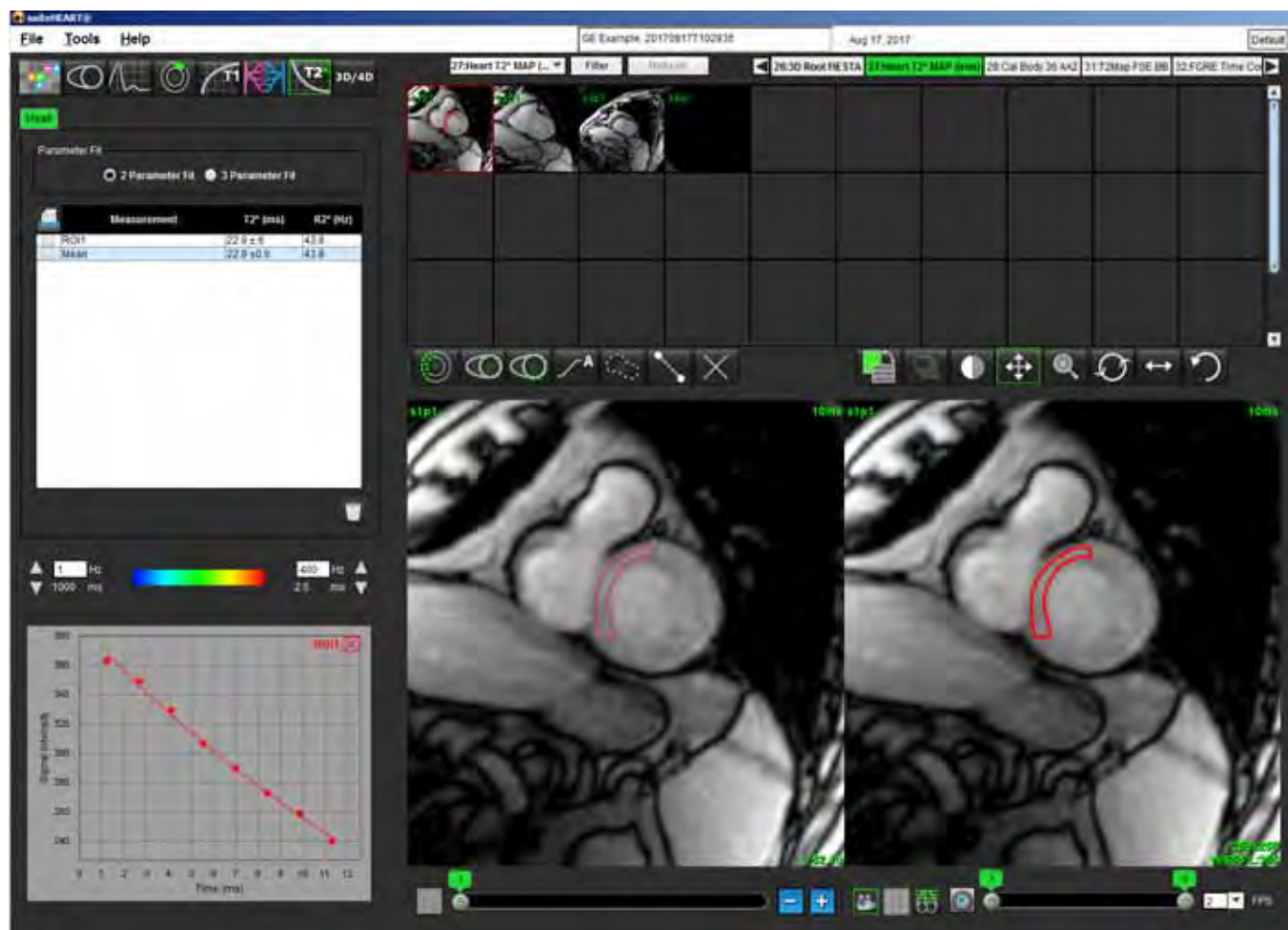FIGURE 1. T2Star Analysis View





1. Select .
2. Select the appropriate series.
3. Select the short axis slice from the thumbnail panel.



4. Draw a contour that encompasses the interventricular septum using .
The T2* and R2* are calculated and are displayed in the result table.

Create Myocardial Colormap


1. Draw a contour the endocardial border using .

2. Draw a contour of the epicardial border using .

The T2*/R2* colormap is overlaid on the image.

3. The R2* colormap value can be changed.


NOTE: The default range for 1.5T images is 5ms - 500ms for T2*. The default range for 3.0T images is 2.5ms - 1000ms for T2*.

4. Click  to toggle between a grey scale and a color map.

5. Click and drag up or down on arrows to adjust the dynamic color range for the color map.

The color overlay on the Image Editor changes dynamically.


The Hz and ms values also change dynamically.

6. The T2* and R2* values can be determined by selecting the  and placing it over the color map overlay on the image.

Fitting Parameters

Select either the **2 Parameter** or **3 Parameter Fit** for the T2* decay curve.

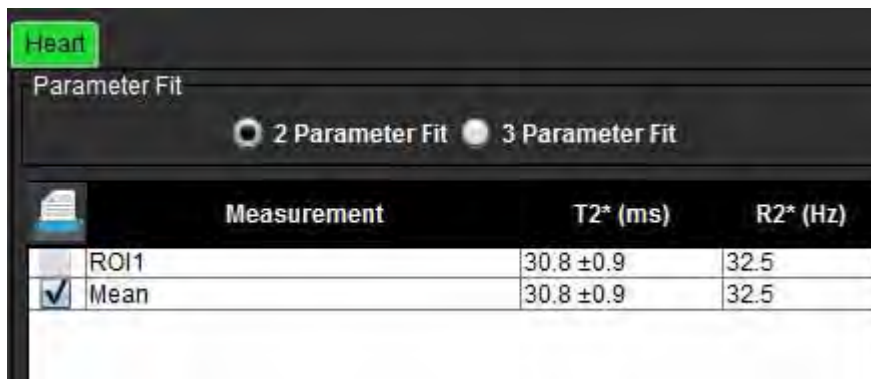FIGURE 2. Parameter Fit



Heart			
Parameter Fit			
<input type="radio"/> 2 Parameter Fit <input checked="" type="radio"/> 3 Parameter Fit			
	Measurement	T2* (ms)	R2* (Hz)
	ROI1	30.8 ± 0.9	32.5
<input checked="" type="checkbox"/>	Mean	30.8 ± 0.9	32.5

The 2-parameter fit is widely accepted based on peer review literature [1]. In this model, the background noise, c , is calculated using a histogram-based algorithm and subtracted from the signal intensity, after which a non-linear fit is performed.

The 3-parameter fit is also available as referenced in peer review literature [2]. This model is a non-linear approach that works directly from the original input signal.

For both models, the initial T2Star value is estimated using a trial linear fitting.

1. D.J Pennell, et al. "Cardiovascular T2-star (T2Star) magnetic resonance for the early diagnosis of myocardial iron overload," Eur Heart J 2001; 22: 2171-2179.
2. Ghugre NR, et al. "Improved R2* Measurements in Myocardial Iron Overload," Journal of Magnetic Resonance Imaging 2006; 23: 9-16.

Review the T2Star Results

1. Review the contour position on all of the images.
2. The table lists the individual T2*/R2* measurements and also calculates a mean value.


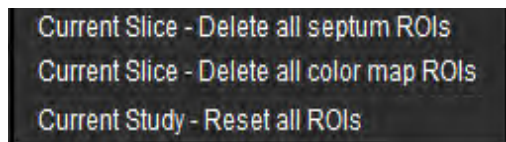
Click  to reset analysis as shown in Figure 3.

FIGURE 3. T2/R2 ROI Delete Choices



3. The option to print the results of each contour can be made by selecting the Printer setting for each value in the measurement table.


Review and Update the T2Star Curve

The T2* curve is a graph of the signal intensity versus echo time using an exponential decay formula. T2*/R2* values may be inaccurate if all images do not have adequate signal to noise ratio (ie. signal plateau to near zero).

To delete a single contour from an image

1. Select the contour and it turns purple.
2. Select Delete from the right-mouse menu or use the Delete key on the keyboard.
 - The contour is deleted and the associated point is removed from the graph.

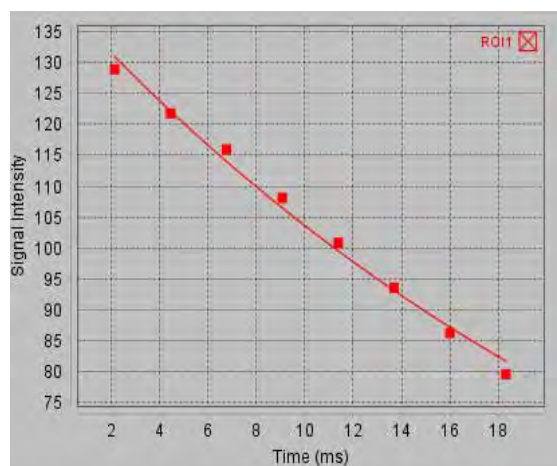FIGURE 4. T2Star Curve



WARNING: Review the results of the T2Star curve fit. A properly trained and qualified user should perform analysis and diagnosis.

R2*/T2* Conversions

Table 1:

Result	Unit	Conversion
R2*	Hz	$R2^* = 1000/T2^*$
T2*	ms	$T2^* = 1000/R2^*$

The factor of 1000 is used as the T2 and T2* are reported in units of milliseconds (ms) and R2 and R2* are Hertz (or s⁻¹).

This page intentionally left blank.

3D/4D Viewer

3D/4D viewer allows for the visualization of 3D and 4D Flow MRI images. It provides tools for reformatting and supports the creation of 2D phase contrast and 2D function images from 4D Flow MRI images that can be analyzed conventionally using suiteHEART® Software.

NOTE: A 3D series with isometric voxels and overlapping slices improves the quality of the reformatted images.

NOTE: The 4D Flow viewer is not available in a suiteHEART® Software multiuser environment (Citrix).

NOTE: The 3D/4D analysis shall display a 4D series only if 4D is licensed.



CAUTION: 3D or image reformats only provide additional supplemental information in the formulating of a diagnosis and should always be used in conjunction with conventional imaging techniques.



WARNING: Always correlate any 3D reformats with the original acquisition data.



WARNING: Window width and level (WW/WL) settings can affect the appearance of different pathologies and the ability to discern other anatomical structures. Incorrect WW/WL settings may cause the imaging data to not display. Different WW/WL settings may be needed to review all imaging data.

3D/4D Viewer Components

FIGURE 1. View Control Tools and Viewports

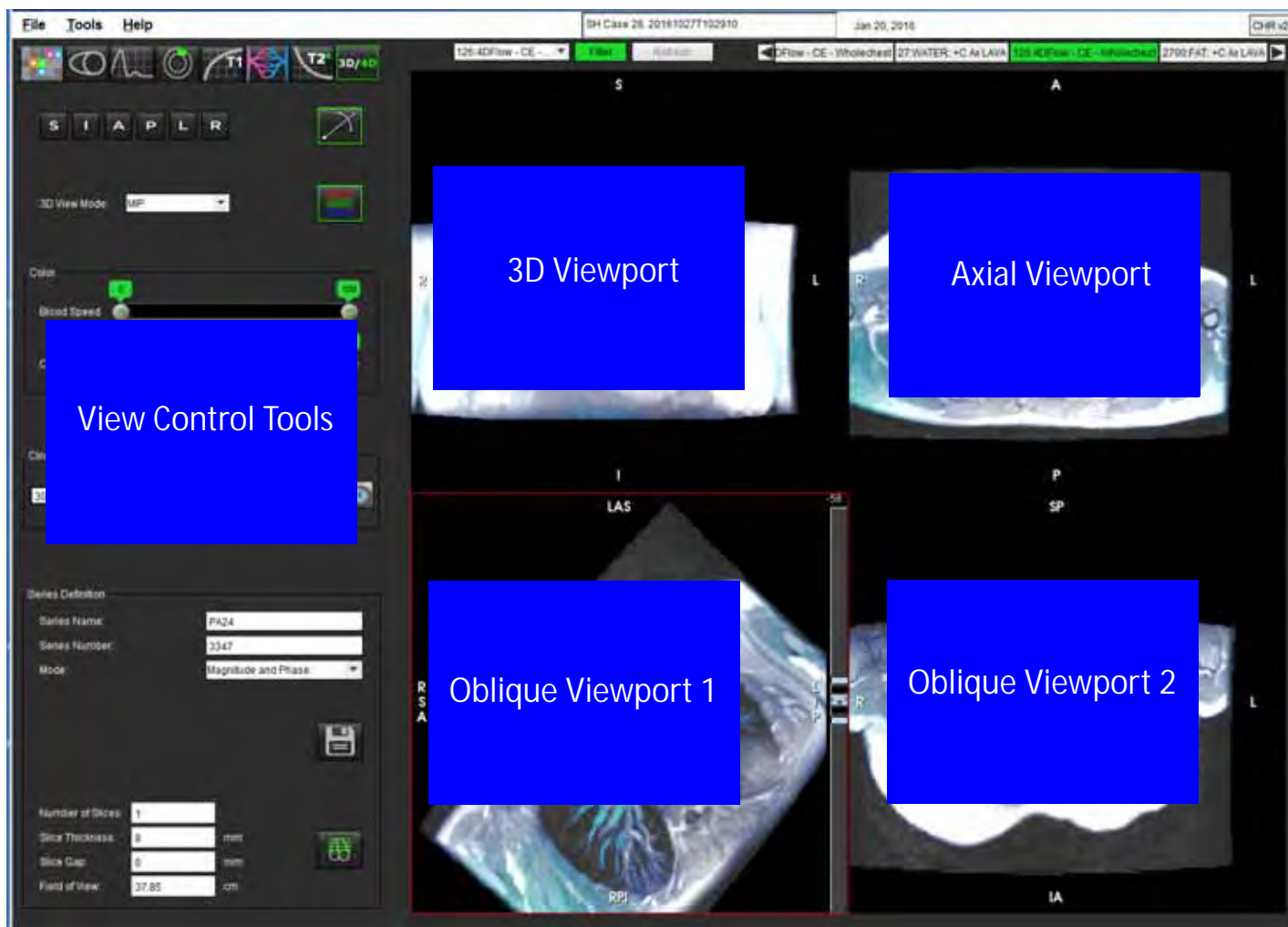


Table 1: View Control Tools








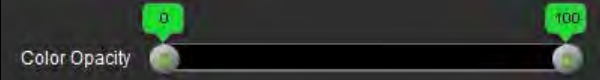
Tool	Description
	Crosshair Cursor - synchronizes navigation between all viewports.
	Orientation Buttons - change the image plane in the 3D and oblique viewports. S = Superior I = Inferior A = Anterior P = Posterior L = Left R = Right
	Oblique Mode - displays the plane of the oblique reformat and the perpendicular intersection to display desired anatomy.
	3D View Mode - provides image render modes in the 3D viewport Surface MIP = Maximum intensity projection (Default) MINIP = Minimum intensity projection
	Color Overlay - toggles the color overlay on/off. Available for 4D flow images only.
	Blood Speed - adjusts the assignment of the velocity encoding. Available for 4D Flow images only.
	Color Opacity - controls the color opacity on the image to improve the visualization underlying anatomy. Available for 4D flow images only.
	Cine - controls frames per second and defines start and end frame of the cine movie. Available for 3D time-resolved magnitude and 4D flow images only.

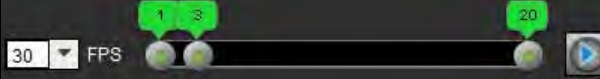Table 1: View Control Tools













Tool	Description
	<p>Series Definition - creates 2D conventional functional or a flow image series for analysis or post-processed MIP images. Use to enter the number of slices, slice thickness, gap and field of view.</p> <p>Magnitude Only - creates a single-slice or multi-slice multiphase magnitude series from the original images for use in function analysis.</p> <p>Magnitude and Phase - creates a single-slice or multi-slice multiphase magnitude with phase series from the original images for use in flow analysis. This option is only available when a 4D Flow series has been selected. (A duplicate series that is auto phase corrected is also created.)</p> <p>Post-processed - creates maximum intensity projection images from 3D images. When a 4D flow data is present single-slice or multi-slice multiphase series with color overlay will be created on the images for review purposes.</p>
	<p>Save- saves all image series types created by the series definition to the local database.</p>
	<p>Rx Planning - defines the desired scan plane axis created by series definition.</p>
	<p>Paging and Thickening - changes the thickness of the MIP image and pages through the image set.</p> <p>1= click and drag either side buttons to change the thickness of the MIP image 2= click and drag the slider to page through the image set.</p> <p>Controls are found on the right hand side of the selected viewport.</p>
	<p>3D Rotate - tilts or rotates the images in the 3D viewport and/or oblique viewports 1 and 2. Left mouse click and drag directly in the viewport to tilt or rotate.</p>
	<p>Flow Direction - displays the perpendicular plane in oblique viewports 1 and 2. Left mouse click directly on the anatomy of interest to use this feature. Available for 4D flow images only.</p>

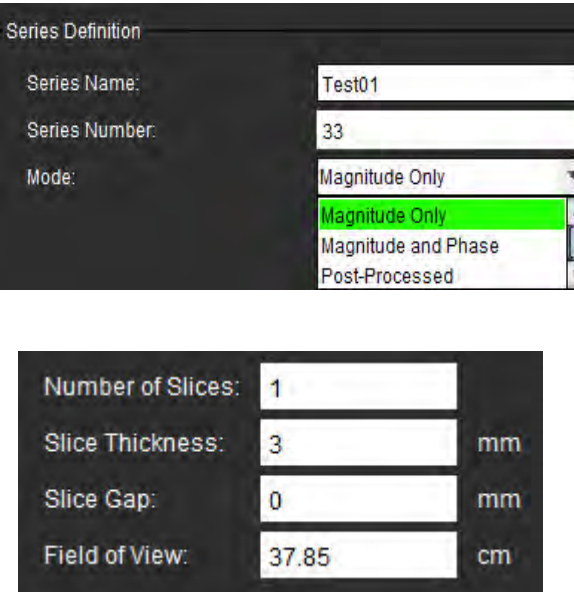Table 1: View Control Tools

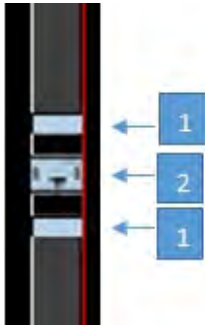Tool	Description
	Window/Level - available in all viewports.
	Pan - available in all viewports.
	Zoom - available in all viewports.
	Rotate - available for the 3D viewport, viewport 1 and viewport 2.
	Reset - available in all viewports.
	Scan Parameters - available in all viewports.

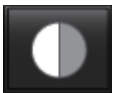3D/4D Viewer Layout and Series Creation Outputs

Depending on the type of image series that are selected for reformatting, the image creation type is summarized in the table below.

Table 2: 3D/4D Layouts and Output

3D/4D Viewer Layout	3D Image Series Outputs	4D Flow Image Series Outputs
3D view (upper left viewport)	Post-Processed	Post-Processed
Axial (upper right viewport)	Magnitude Only Post-Processed (MIP)	Magnitude Only*, Magnitude and Phase*, and Post-Processed (color overlay)*
Oblique 1 (lower left viewport)	Magnitude Only Post-Processed (MIP)	Magnitude Only*, Magnitude and Phase*, and Post-Processed (color overlay)*
Oblique 2(lower right viewport)	Magnitude Only Post-Processed (MIP)	Magnitude Only*, Magnitude and Phase*, and Post-Processed (color overlay)*
*This series type can be used for conventional analysis in suiteHEART® Software		
For each magnitude and phase series, a duplicate series that is auto phase corrected will be created.		

Sample Workflow: Create MIP Images from a 3D Image Series

1. Select the appropriate study and launch suiteHEART® Software.



3. Select the appropriate 3D series from the series navigation pull-down. The image type selected will be indicated on the button, as shown in Figure 2.

FIGURE 2. Series Navigation




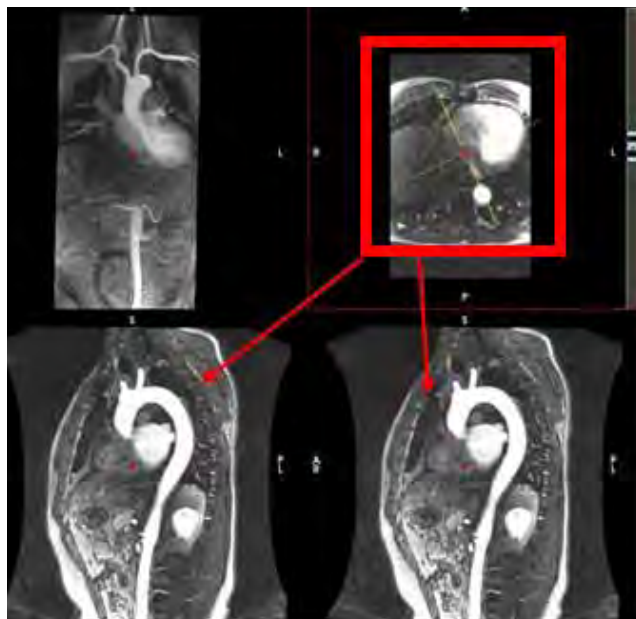
4. Select the  and click on the desired viewport. The active viewport will be highlighted in red. The yellow reformat lines will appear as shown in Figure 3.

FIGURE 3. Oblique Mode



5. Click on the solid yellow line, left mouse click and drag and tilt the line to display the desired anatomy.
 - a.) Click on the desired viewport for saving.
 - b.) Adjust the MIP thickness using the controls on the right hand side of the viewport.
 - c.) Complete the series definition entries, as shown in Figure 4.
 - d.) Click the save button to save the MIP image to the local database.

FIGURE 4. Series Definition

Series Definition

Series Name:

PA24

Series Number:

3347

Mode:

Post-Processed

Number of Slices:

1

Slice Thickness:

50

mm

Slice Gap:

0

mm

Field of View:

37.85

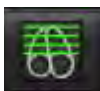
cm

Save

Preview

1. Select Post-Processed

2. Select Save




6. Create a stack of MIP images by selecting

NOTE: The maximum number of post-processed MIP images that can be created is 512.

7. Click on the viewport to be used as the reference image and define a stack of batch images, as shown in Figure 5.
 - a.) Extend the range of the slice coverage.
 - b.) Adjust the angle and arrows indicate slice direction.
 - c.) Move the Rx.

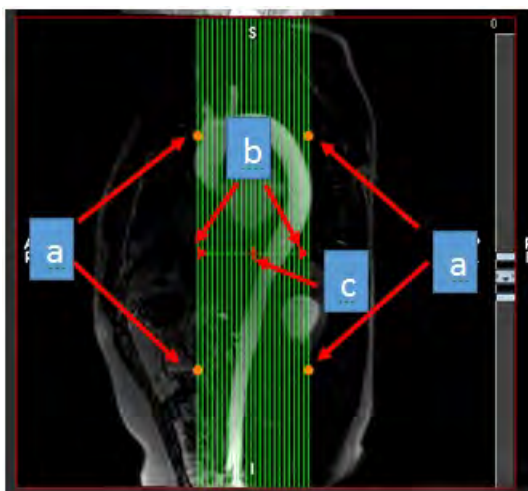FIGURE 5. Rx Planning



8. Enter the series definition options and click
9. To view the created series, switch to function analysis mode, select review mode and click refresh.

Example Workflow: Create 2D Series for Conventional Analysis

The creation of conventional 2D phase contrast and 2D functional images requires a 4D Flow series that has both time-resolved magnitude and flow conventions of R/L, A/P and S/I.

Series created as magnitude alone or magnitude and phase from 4D flow images are a valid 2D conventional series that can be used in function or flow analysis.


Series that are created as post-processed from 4D Flow will have a color flow overlay. If the appropriate scan plane is created the series can be used for function analysis.

1. Select the appropriate study and launch suiteHEART® Software.



2. Select
3. Select the appropriate 4D series from the series navigation pull-down, as shown in Figure 6. The image type selected will be indicated on the button, as shown in Figure 6.

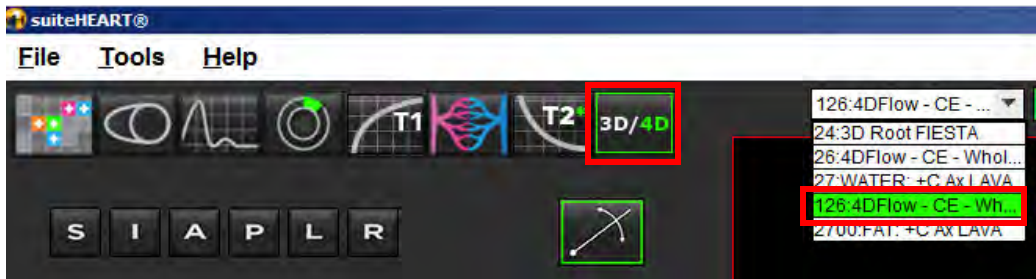FIGURE 6. Series Navigation




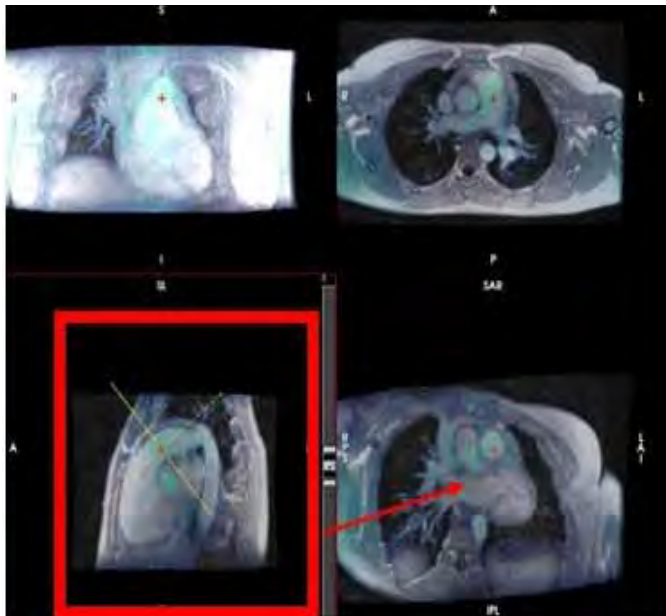
4. Select the  and click on the desired viewport. The active viewport will be highlighted in red. The yellow reformat lines will appear as shown in Figure 7.

FIGURE 7. Oblique Mode Reformat 4D



5. Click on the solid yellow line, left mouse click and drag and tilt the line to display the desired anatomy.
 - a.) Click on the desired viewport for saving and select Magnitude and Phase mode to create a 2D phase contrast series or select Magnitude to create a functional series.
 - b.) Adjust the slice thickness using the controls on the right hand side of the viewport.
 - c.) Complete the series definition entries, as shown in Figure 8, and click the save button to save the series to the local database.

FIGURE 8. Series Definition and Save

Series Definition

Series Name: PA24

Series Number: 3347

Mode: Magnitude and Phase

Number of Slices: 1

Slice Thickness: 8 mm

Slice Gap: 0 mm

Field of View: 37.85 cm

Save

3D/4D

1. Select Magnitude and Phase

2. Select Save




6. To create a stack of multi-slice multiphase images select

NOTE: The maximum number of multiphase images that can be created is 32.


NOTE: If a 2D magnitude and phase series is created by using the 3D/4D viewer, the application will create a duplicate auto phase corrected series. The series will be labeled “corrected,” as shown in Figure 9.

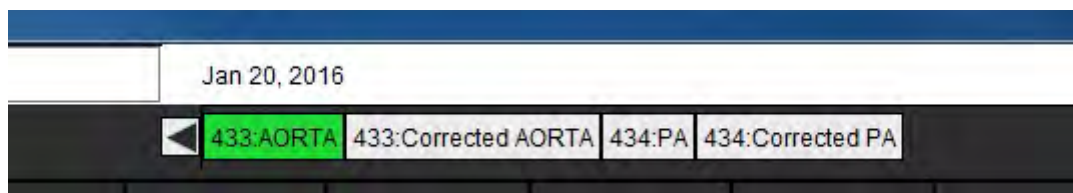FIGURE 9. Duplicate Auto Phase Corrected Series Example



- Click on the viewport to be used as the reference image and define a stack of batch images, as shown in Figure 10.

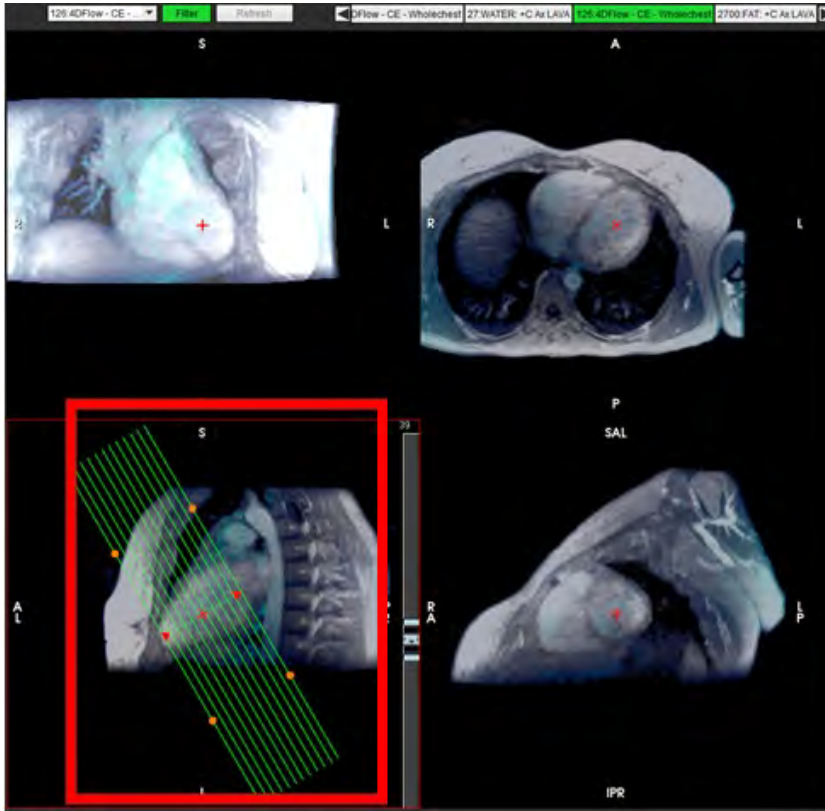FIGURE 10. Rx Planning



- Select the Series Definition options and click  to save the image stack to the local database.
- To analyze the created series, switch to the appropriate analysis mode and click refresh.

This page intentionally left blank.

Structured Reporting

Define Report Content

The measurements and graphs that populate reports are taken from the results of the analysis modes. Each individual analysis result may be selected for inclusion in the report.


Predefined clinical impressions and techniques streamline custom reports. Refer to the Impression Tab section for procedural details on how to create clinical impressions and techniques. Report Preferences allows entering site information that will appear as titles and headers on the patient report.

Structured Report View

The Structured Report View is designed to aid in generating clinical reports. There are four tabs:

- History
- Impression
- Images
- Polar Plots



Each printable field is associated with a check box toggle button . Click the check box to include or exclude the field from the report.

NOTE: The print selection is saved with the template. If print selections are changed, a message box will appear when closing the study to confirm whether or not to save the changes to the template.

History Tab

The **History tab** contains patient information from the DICOM header. Editing information highlights the field in yellow.

NOTE: Edited patient information affects the report only. The DICOM header is left intact.

FIGURE 1. History Tab

1 →

2 →

3 →

4 →

Name	Value
Study Date	Jan 17, 2007
Institution	
Referred By	
Copies To	
Description	
Name	suiteHEART Example Case 01
ID	AW1903342710.717.1400755457
Age	38
Sex	Female
Height(in)	66
Weight(lb)	195
BSA	1.98 [DuBois and DuBois]

☒ HISTORY

NOTES

1. DICOM Header Information, 2. BSA Selection, 3. Patient History, 4. Notes

The BSA calculation type can be selected by performing a right-mouse click on the inverted yellow triangle.

BSA Calculation Method	Formula
DuBois and DuBois	$BSA (m^2) = 0.20247 \times Height(m)^{0.725} \times Weight(kg)^{0.425}$
Mosteller	$BSA (m^2) = \sqrt{[Height(cm) \times Weight(kg)] / 3600}$ $BSA (m^2) = \sqrt{[Height(in) \times Weight(lbs)] / 3131}$
Gehan and George	$BSA (m^2) = 0.0235 \times Height(cm)^{0.42246} \times Weight(kg)^{0.51456}$
Haycock	$BSA (m^2) = 0.024265 \times Height(cm)^{0.3964} \times Weight(kg)^{0.5378}$
Boyd	$BSA (m^2) = 0.0003207 \times Height(cm)^{0.3} \times Weight(grams)^{(0.7285 - (0.0188 \times \log(grams)))}$

Reference: <http://halls.md/formula-body-surface-area-bsa/>

History and Notes Text Boxes

Enter any information relevant to the patient history in the History field or select the appropriate macro. The Notes panel displays the notes entered by the user during analysis, but will not be available for inclusion on the report.

Impression Tab

FIGURE 2. Impressions Tab

1 →

2 →

1. Impression, 2. Technique

Impression

Enter impression information by typing in the text box and/or click an impression macro button.

Predefined impression macros are located on buttons below the Impression panel.

NOTE: All appropriate analysis needs to be performed prior to generating result calculations with macros.

Technique

Enter technique information by typing in the text box and/or clicking a technique macro button.

Predefined technique macros are located on buttons below the Technique panel.

NOTE: All appropriate analysis needs to be performed prior to generating result calculations with macros.

Images Tab


FIGURE 3. Images Tab




1. Graphs/Tables, 2. Images for report

Review Graphs and Summary Tables for Report

The Graphs View panel contains all the graphs and summary table results that are included in the report during analysis.




1. Step through each graph and summary table using the  icons.
2. Click in the white text box to add a graph or summary table caption for the printed report.


3. When  is enabled, the graph or table will be included in the report.


4. Click on  to delete a graph or table.


Review the Images

The Image panel contains all the images that were sent to the Report during analysis.

1. Step through each image using the  buttons.
2. Click in the white text box to add an image caption for the printed report.
3. Select the image size by choosing the small format  or large format  buttons.
4. Images in the image panel can be re-ordered by clicking and dragging the image into a different viewport.
5. Perform a right-mouse click directly on an image to access the image manipulation tools.

6. To locate the series from which the image originated, right-mouse click directly on the image and select the  locate button.

7. When  is enabled, the image will be included in the Report.


8. Delete an image by selecting .

NOTE: If a study is opened which has been analyzed from a previous software version (2.1.0 or below), images previously added to the Report View cannot be manipulated using the image manipulation tools. Any new images added can be manipulated as expected.


Polar Plots Tab


This table allows for the identification of functional, myocardial evaluation, and time course abnormalities qualitatively in a polar plot format. To change the color coding of the segments, right mouse click on the segment color legends to open the color palette.

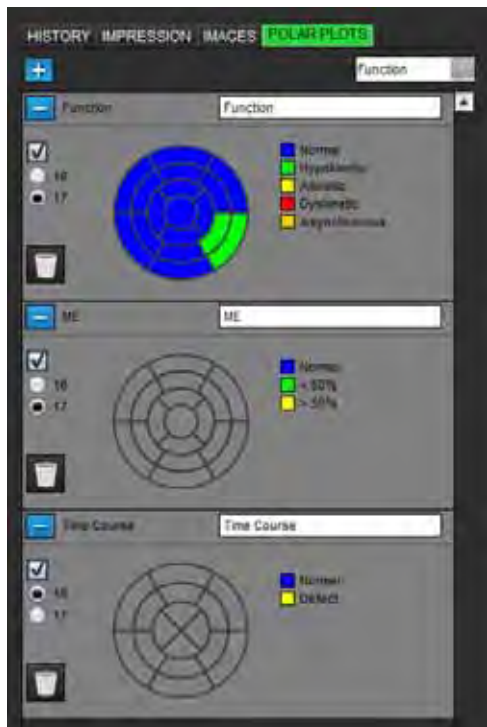FIGURE 4. Polar Plot Tab



Add Polar Plots to Report

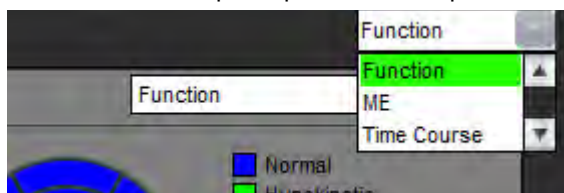To add additional polar plots to the report, click on the  and select the polar plot type from the file pull-down menu




Selecting Colors per Segment

Click on the color box next to the desired terminology to describe the abnormality. The cursor changes to a paint brush. Then click on the segment directly on the polar plot to set the color.

Selecting Colors for All Segments

Right mouse click outside of the polar plot outline in the corners and make the desired selection from the list.

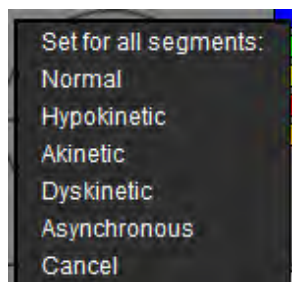FIGURE 5. Selection for Function




Selecting 16 or 17 Segment Plots

Select the appropriate radio button located on the left of the polar plot.

Editing the Title of the Polar Plot

Type title of each polar plot can be edited by clicking in the type-in field.

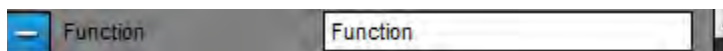FIGURE 6. Edit Polar Plot Title Type-in Field



Removing a Polar Plot


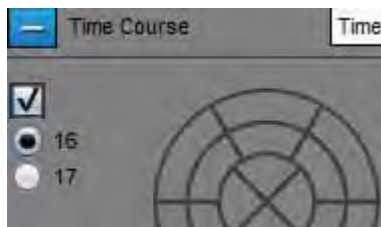


Each plot can be removed from the tab by clicking the  button. To exclude the polar plot from the report, deselect the check box.

FIGURE 7. Exclude Polar Plot from Report



Select  to reset the polar plot back to default.


Preview the Report

1. Select **File > Preview Report** or select **Preview Report** from the Review window.
2. Review the report to ensure that all the desired analysis results and structured information is included.
3. Select  to save the report to the local hard drive.

The Save pop-up window provides the tools to define the destination, name and report format options of the report.

IMPORTANT: Values shown in red are out of range, which will not be obvious if printing the report on a black and white printer.

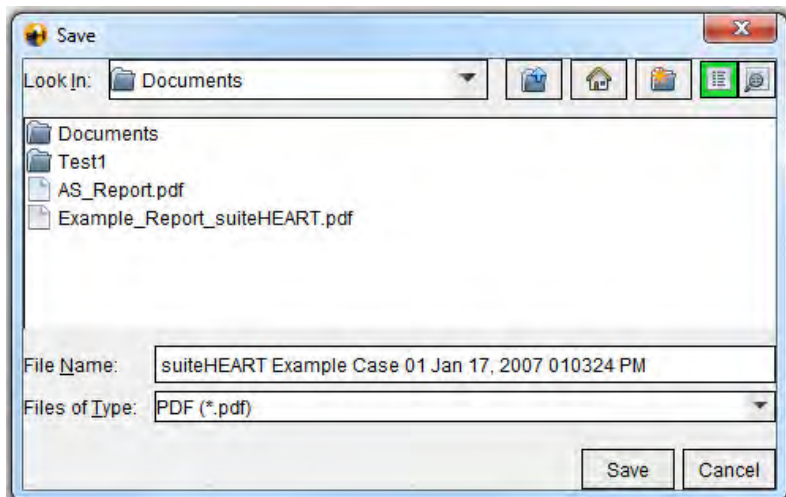FIGURE 8. Save Window



4. Select **Print** to print the report.



WARNING: The report should be inspected prior to approval and distribution to ensure that the content matches the analysis. Delayed or misdiagnosis may occur should the report contents be erroneous. Analysis and interpretation should be done by properly trained and qualified users.

Approve the Exam


The application has a feature that approves and locks reports. The approved report is saved and can be viewed but it cannot be changed.

NOTE: Prerequisites: The user must be an authorized report signatory. See “Authorized Report Approvers” on page 33.

NOTE: The “Approved Exam” button and menu are not enabled until an action has been performed on an image.

1. Select **Approve Exam** or select **File > Approve Exam**.

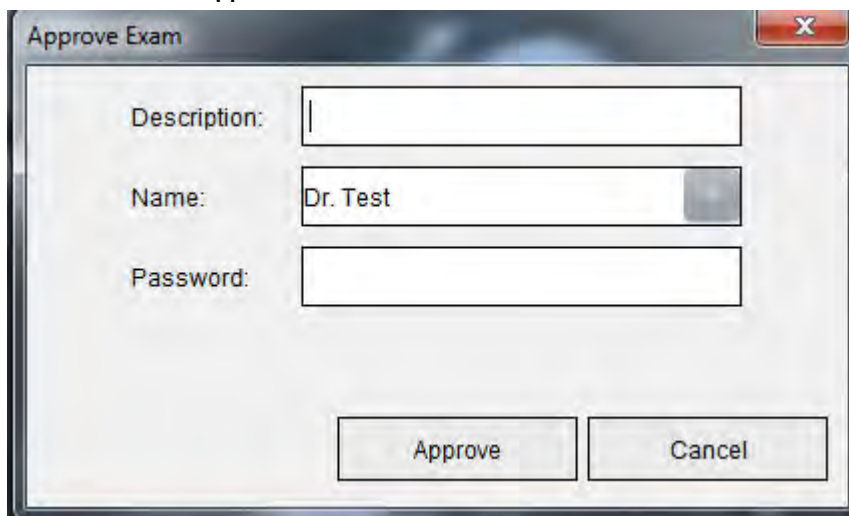FIGURE 9. Approve Exam Window

The image shows a software dialog box titled "Approve Exam" with a standard Windows-style title bar (minimize, maximize, close buttons). Inside the dialog, there are three input fields: "Description:" with an empty text box, "Name:" with a pull-down menu showing "Dr. Test", and "Password:" with an empty text box. At the bottom of the dialog, there are two buttons: "Approve" and "Cancel".

2. Enter a signature description if desired.
3. Select your user name from the Name pull-down menu.
4. Type in your password.
5. Click **Approve** to confirm and close the window. Click **Cancel** to close the window without completing the sign-off procedure.

Using the description provided, a series is created.

NOTE: When an approved exam has been performed the report will have the date and time stamp.

Export Options

The exporting feature is designed to create backups of the exams and the analysis results for future review. The exporting feature allows you to create uncompressed AVI movies, compressed QuickTime movies, and JPEG, TIFF and PNG files of the images. These files can be written to the file system.

Select **Tools > Export > Export Report**.

A secondary capture (SCPT) is created and is saved in the series listing.

Select **Tools > Export > Export Cine DICOM**.

A secondary capture (SCPT) is created and is saved in the series listing.


Select **Tools > Export > Export Cine Files**.

The Save Cine popup window is displayed.

Select **Tools > Export > Export to Matlab**

Exports a Mat-file in binary form.

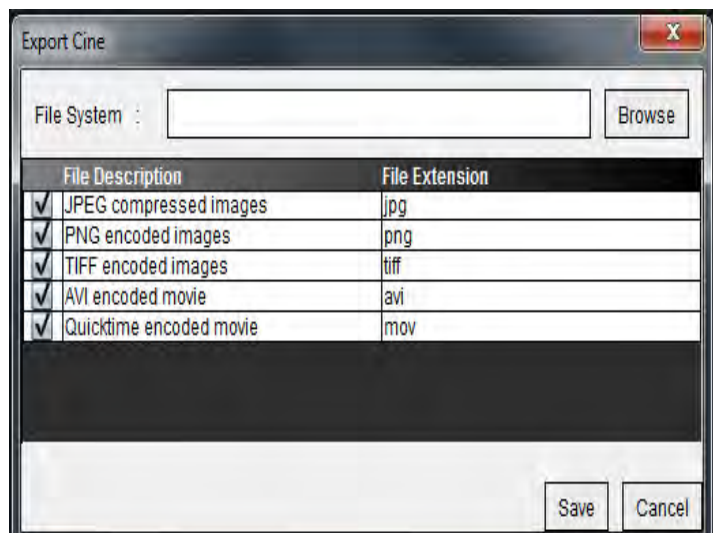FIGURE 10. Save Cine Window



1. Select the file types to be exported.
2. Browse to the location in which to save the file(s).
3. Click **Save** to start the exporting process and close the window. The currently viewed series is the only file exported.

NOTE: When exporting data to AVI or MOV files, suiteHEART® Software sets the maximum frames-per-second rate to 20 frames-per-second regardless of the settings used for viewing within the application.


NOTE: If exporting a custom series with both multiphase and single-phase images as a .avi or .mov file, ensure that a viewport containing a multiphase image is selected prior to exporting.

Review an Approved Exam

1. Select **File > Load Approved Exam**.

This displays the Select Approved Exam window. All the approved exams related to the exam are displayed in the list.

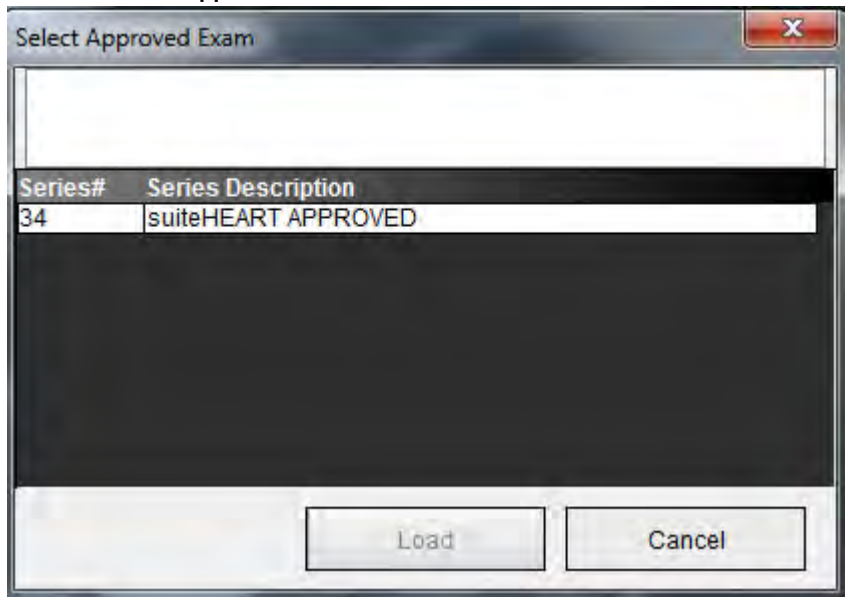FIGURE 11. Approved Exam Selection Window



2. Select the series from the list.
3. Click **Load** to load and display the approved exam and it's accompanying analysis.
 - An approved exam can be viewed only.
 - A new exam can be generated from an approved exam by editing an approved report and saving those changes to a new exam. The new exam is saved as a secondary capture series.

NOTE: Loading an approved exam and analysis will overwrite the information in the current analysis session.


NOTE: When restoring exams that have been analyzed with prior versions of suiteHEART® Software, and if a "Load Approved Exam" has been performed, the report will not have the name of the approver or the date and time stamp. ***It is recommended to review all analysis and confirm all results prior to reissuing the report.***

This page intentionally left blank.

Report Database

The Report Database Tool allows you to perform a search on the contents of previously approved reports. A report is only entered into the report database after it has been approved.

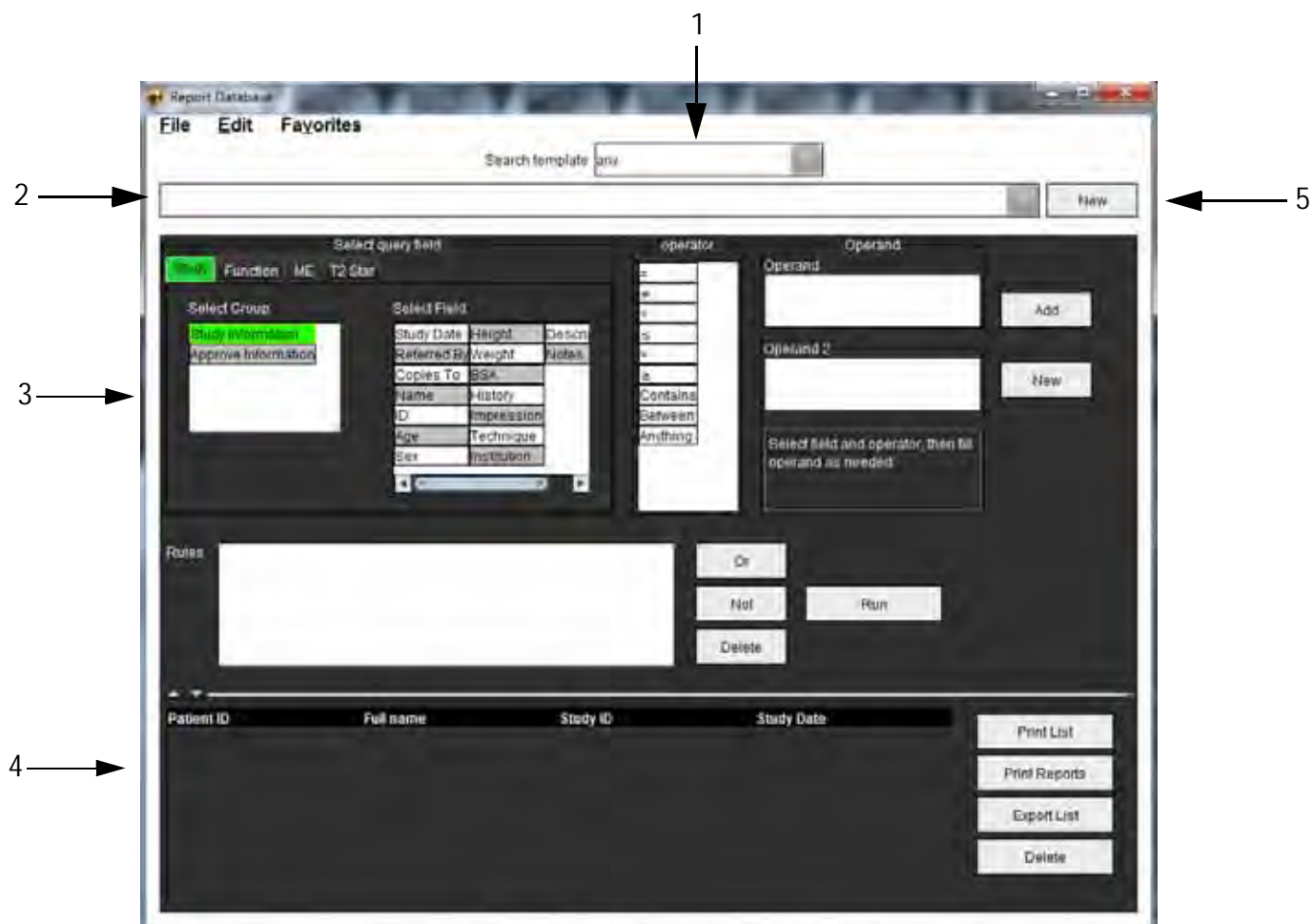FIGURE 1. Report Database Window



1. Search template drop-down menu, 2. History bar, 3. Create query fields, 4. Query results, 5. New button

Report Database Tool Procedure


Open the Database Tools Window

1. Select **Tools > Report Database**.

Select Search Criteria

1. Select the correct template for the search from the Search template drop-down menu.
2. Select the search query from the History drop-down menu. The current query bar displays your selected values.

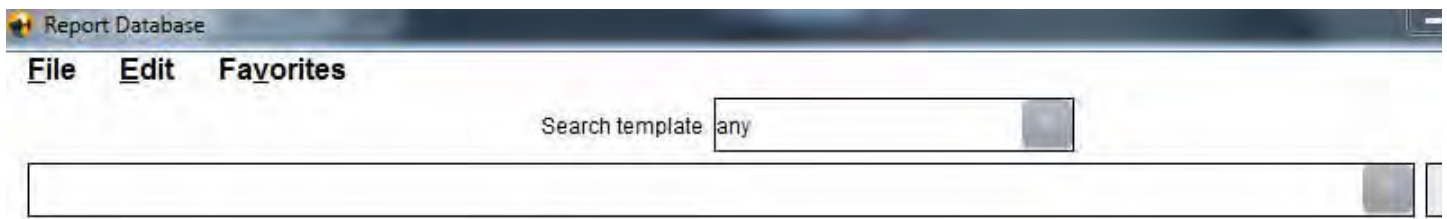FIGURE 2. Search Template Menu




NOTE: If the desired query does not exist already, create a new query.

Create a Query

1. Select **New** to the right of the History bar.
The create query panels are displayed in the Report Database window.

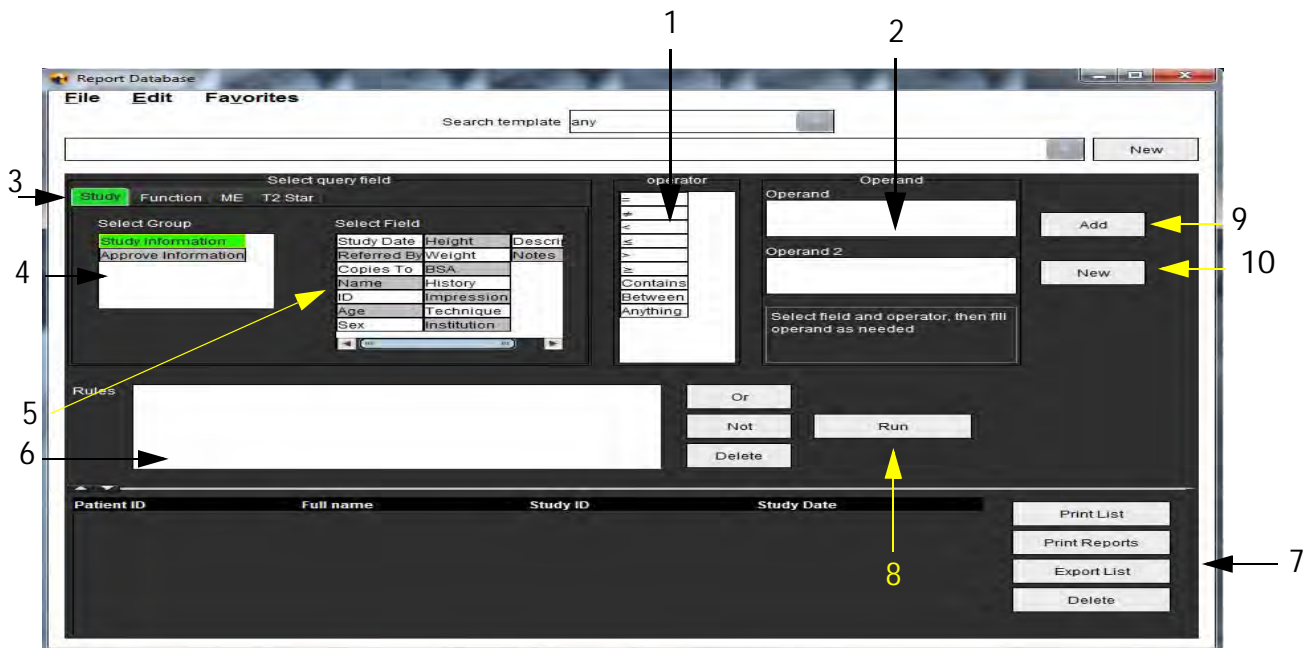FIGURE 3. Create Query Panel



1. Query Operators, 2. Query Operands, 3. Query category tabs, 4. Query group, 5. Query fields, 6. Query rules, 7. Query options, 8. Run button, 9. Add button, 10. New button

2. Select the query category tab from Study, Function, ME and T2 Star. The query groups and fields update accordingly.
3. Select the query group.
4. Select the query field.

NOTE: The Report Database cannot perform a search on custom measurements.

5. Select the operator to define the query search parameters.
6. Enter the operand(s) to provide values to the search parameters.
7. Select **Add** to display the query values in the **Rules** panel. Multiple queries can be executed during a single search operation. Repeat steps 1 through 7 for each additional rule.

The **Not** button will negate a query value.

The **Or** button will concatenate multiple queries while satisfying the search with only one of the queries. The **Or** function applies to the query rule above the selection.


The **Delete** button provides a means to select and delete a query rule.

Activate the Search

1. Select **Run** to search the database.

The search results are displayed in the Query result area. The query values that satisfy the search are displayed in the right most column of the result window.

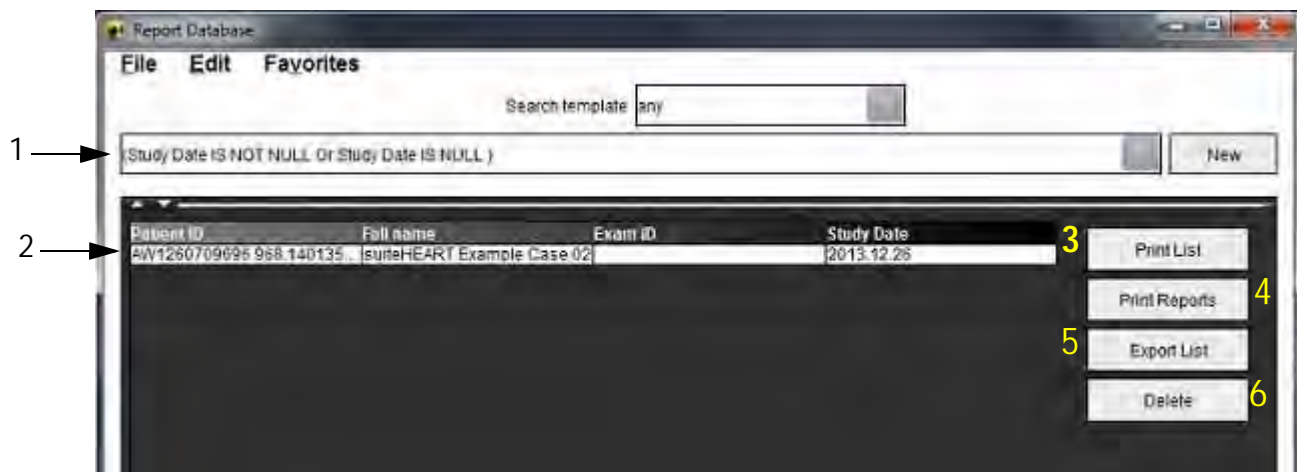FIGURE 4. Query Result Window



1. History bar, 2. Query results, 3. Print list button, 4. Print reports button, 5. Export list button, 6. Delete button

NOTE: New query results are only created based on a unique combination of exam ID, exam date, authorized signature and report template. If a duplicate of these fields is recognized, the old report is replaced by the new report.

View the Results

- To view a report, double click an entry in the Query result area.

A new window opens displaying the selected report. If more than one report is available, use **Next Report** and


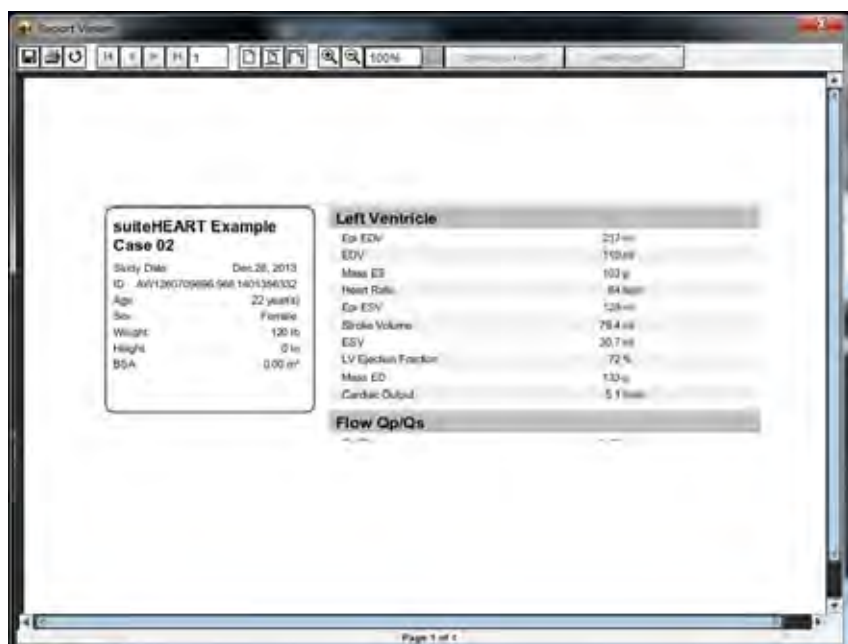
Previous Report to step through the reports. Click the close window marker  to close the Report Review window.

FIGURE 5. Report Window



2. Apply the Edit menu selection options to modify the result selections:
 - Edit > Select All** selects all search results.
 - Edit > Clear Selection** deselects all search results.
 - Edit > Invert Selection** toggles the selection state of each result.
 - Edit > Clear History** deletes record of previous queries.
3. Select **Print List** to send the query list to the printer.
4. Select **Print Reports** to send the selected reports to the printer.
5. Select **Export List** to save the list as an html file.
6. Select **Delete** to remove the selected report(s) from the report database

Save a Query

1. Select **Favorites > Add to Favorites**.
2. In the Add To Favorites text box, type in a label for the query and click **OK**.

FIGURE 6. Favorites Menu

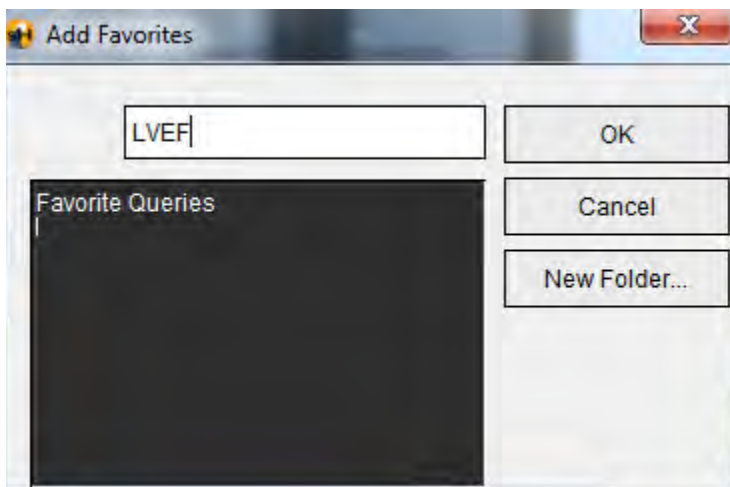



FIGURE 7. Favorites Pull-down



Delete a Favorite

1. Select **Favorites > Manage Favorites** from the Report Database window.

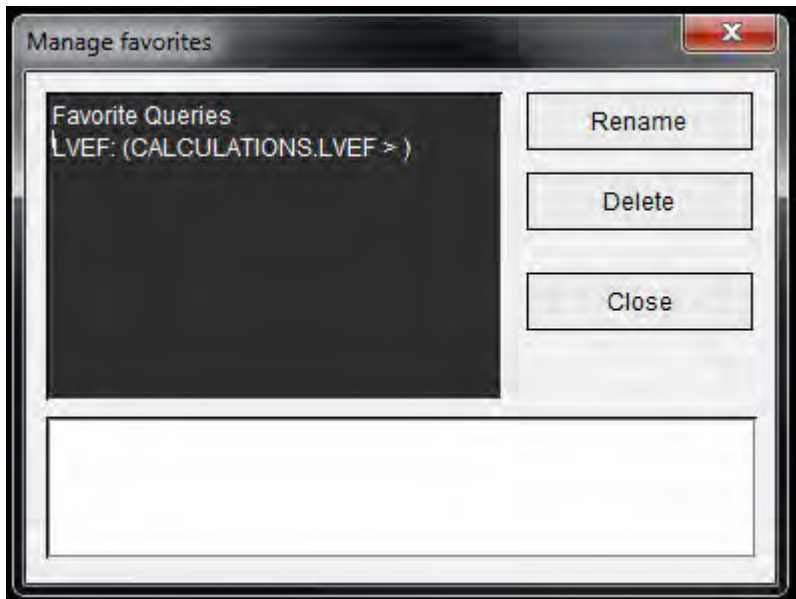FIGURE 8. Manage Favorites Window




2. Select the favorite item.
The entire query formula is displayed in the Result window.
3. Click **Delete**.
A confirmation popup will verify your delete selection. Select **Yes**.
4. Select **Close**.

Export Search Results to an HTML File

1. Select **Export List** on the right hand side of the Report Database window.

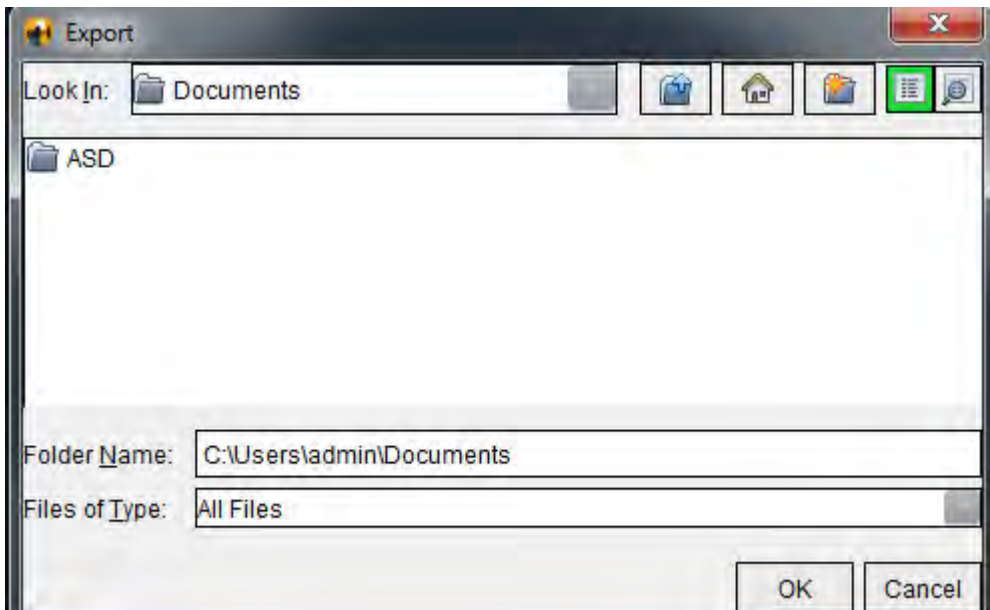FIGURE 9. Export Window



2. Select the directory to which to export the list.
3. Select **OK**.
 - A popup window inquires whether the reports should be included.
 - The listing and reports are exported to an HTML file.

Export the Database

As the database becomes larger it is advisable to archive the data.


1. Select **File > Export** from the Report Database menu bar.
2. Select the directory to which to export the list.
3. Select **OK**. The database is exported to the external storage device.

Import a Database

The database can be imported from another PC to which it was exported.

1. Select **File > Import**.

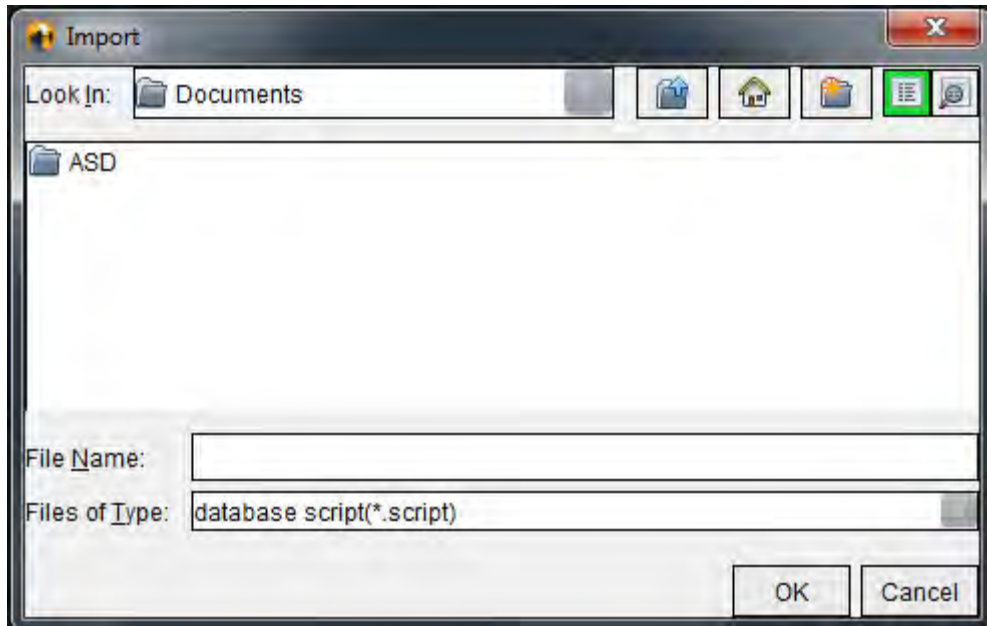FIGURE 10. Import Window




2. Select the directory from which to import the database.
3. The imported database is merged with the existing database.

Tablet Mode

suiteHEART® Software is supported on tablets with a 64-bit processor running on Windows 10 Professional or Windows 10 Enterprise operating system. Review the following section for using suiteHEART® Software on a tablet device.

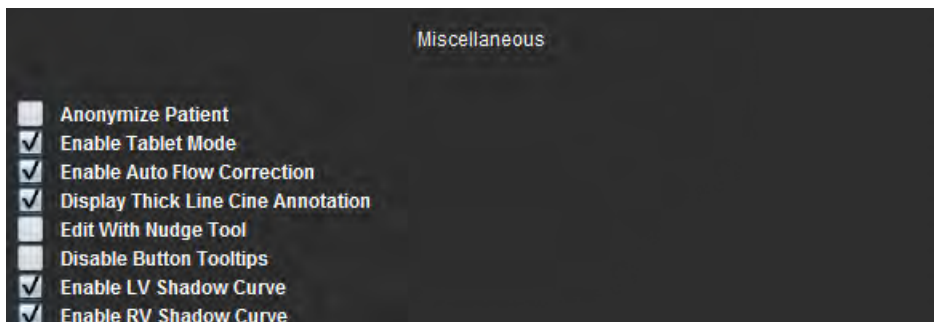Enable Tablet Mode



1. From the Image View menu bar, select **Tools > Preferences > Edit Preferences**.
2. Select the **Global Settings** tab and place the cursor in the **Miscellaneous** panel.
3. Check the **Enable Tablet Mode** check box.
4. On the Report View a keyboard icon will be active, as shown in Figure 1.

FIGURE 1. Keyboard Toggle



5. When a text field is used, a virtual keyboard will appear.
The virtual keyboard can be moved on the interface.

6. When a selection is made on a non-text area, the keyboard will close.




7. To manually activate the virtual keyboard, click . To close, click .

Image Manipulation Tools

To use the image manipulation tools on a tablet, press using a stylus or, if a mouse is connected, left mouse click and drag on the tool.

To reorder images on the images tab, right mouse click, select the drop hand icon .

Appendix

Technical Reference

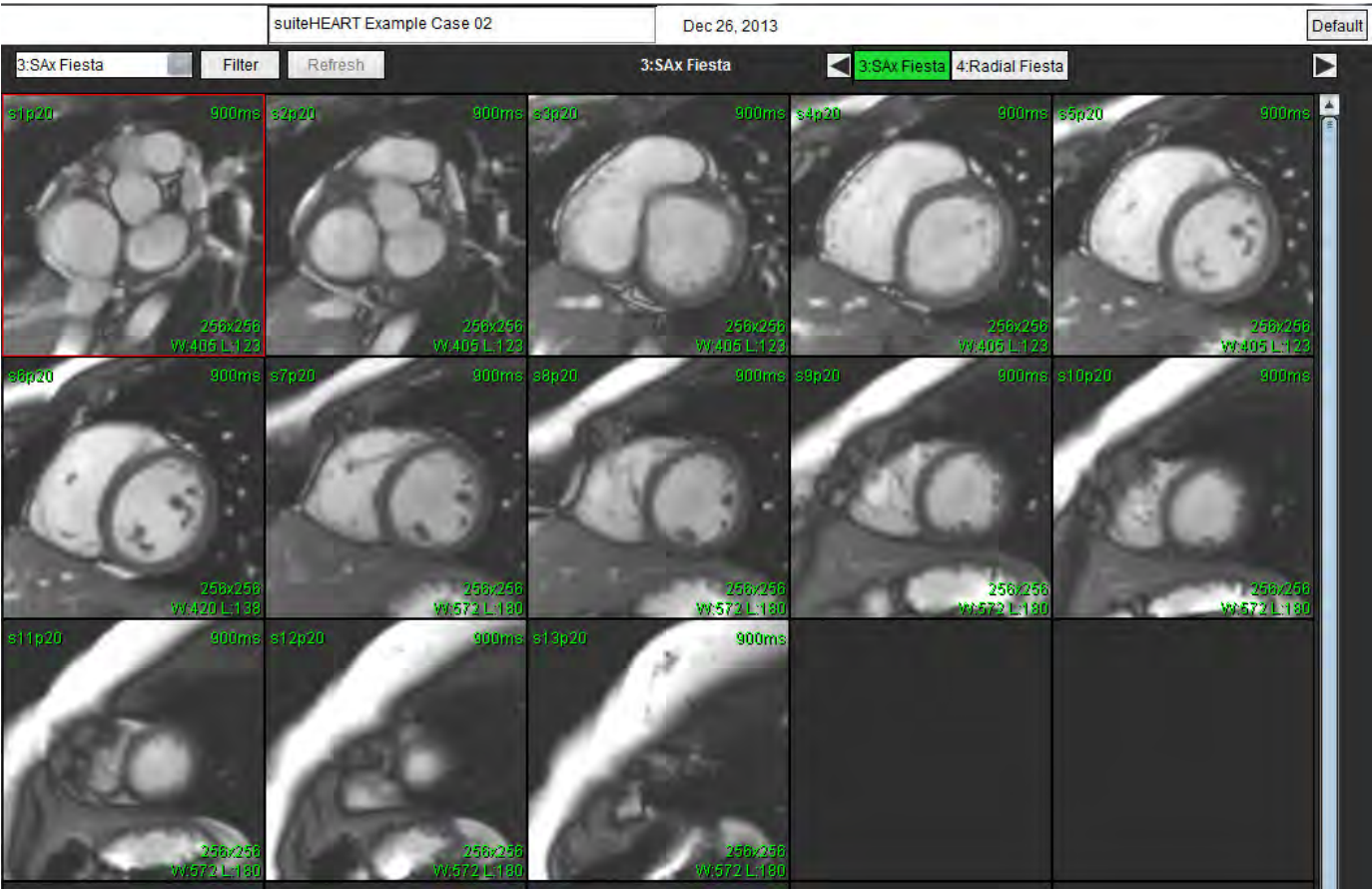
Appendix A - Reference Articles

Preference ranges, as described on page 39 of this manual, may be established from the following peer review literature references:


1. Maceira A.M. et al, "Normalized Left Ventricular Systolic and Diastolic Function by Steady State Free Precession Cardiovascular Magnetic Resonance." *Journal of Cardiovascular Magnetic Resonance* (2006) 8, 417-426.
2. Lorenz C. et al. "Normal Human Right and Left Ventricular Mass, Systolic Function, and Gender differences by Cine Magnetic Resonance Imaging." *Journal of Cardiovascular Magnetic Resonance* 1(1), 7-21, 1999.
3. Sechtem, U. et al. "Regional left ventricular wall thickening by magnetic resonance imaging: evaluation in normal persons and patients with global and regional dysfunction." *Am. J. Cardiol.* 1987 Jan 1;59(1):145-51.
4. Storey P, et al. "R2* Imaging of Transfusional Iron Burden at 3T and Comparison with 1.5T," *Journal of Magnetic Resonance Imaging* 25:540–547 (2007)
5. D.J Pennell, et al. "Cardiovascular T2-star (T2Star) magnetic resonance for the early diagnosis of myocardial iron overload", *Eur Heart J* 2001; 22: 2171-2179.

Appendix B - Functional Analysis Scan Plane Example

For accurate function results, analysis should be performed on a short axis view as shown in the first figure below.



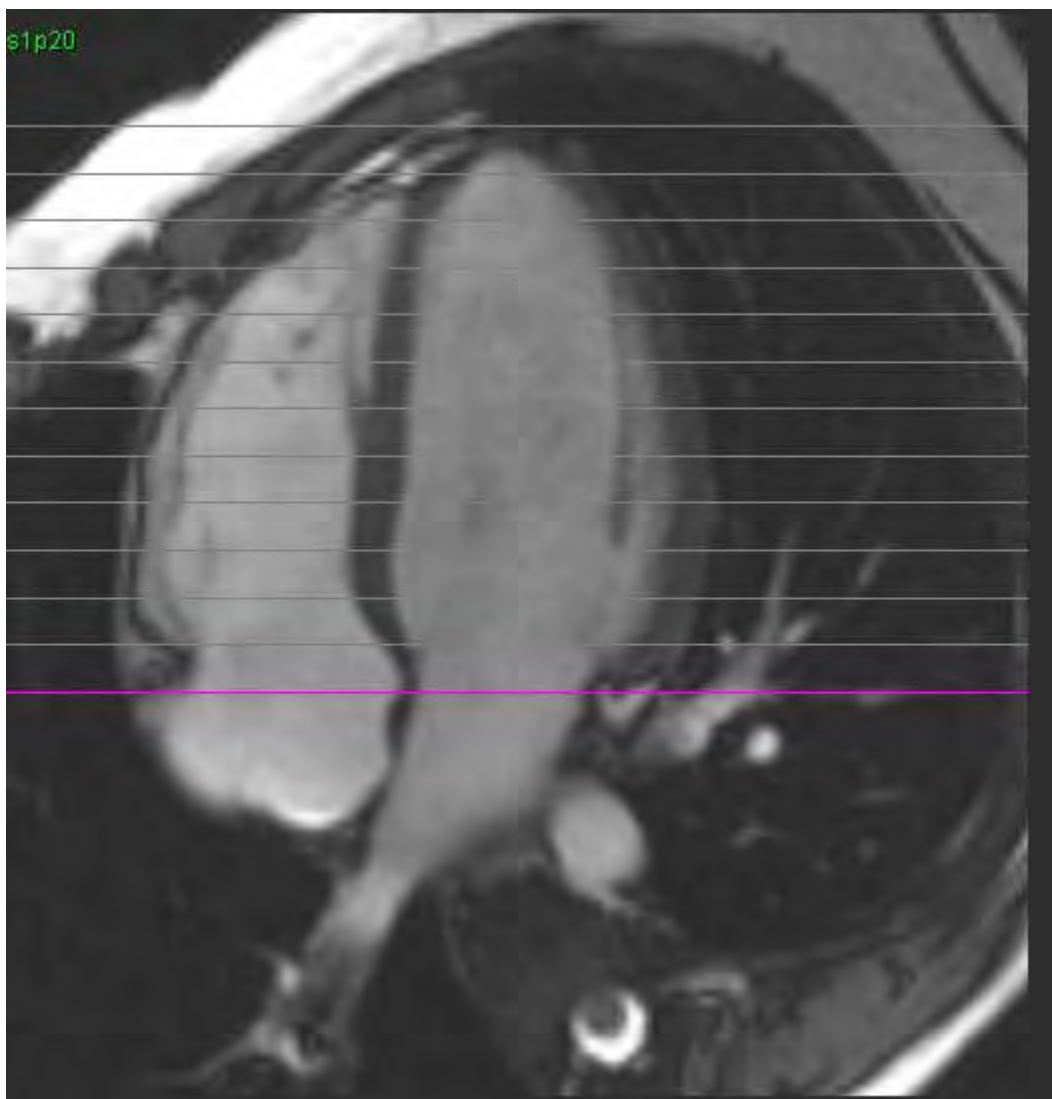Correct scan plane prescription for the acquisition of the short axis view. Slices should be prescribed perpendicular to the long axis of the left ventricle with at least 2 slices above the base and 1 slice after the apex included in the series.



Appendix C - Supported Manufacturers

suiteHEART® Software has been evaluated with cardiac MRI images from the following manufacturers listed in the table below.

Manufacturer	Scanner Type	Analysis Mode
GE Healthcare	Discovery MR750 Discovery MR750w Optima MR360 Optima MR450w Optima MR450 Signa HD Signa HDx Signa HDxt	All Analysis Modes
Philips Healthcare	Achieva Ingenia Intera Intera Achieva	Function, Myocardial Evaluation, Time Course, Flow
SIEMENS	Aera Avanto Espree Skyra Sonata Symphony Verio	Function, Myocardial Evaluation, Time Course, Flow, T2 Mapping, T1 Mapping

Index

Numerics

3D/4D

Series Creation Outputs 138

Viewer Components 134

Viewer Layout 138

3D/4D Viewer 133

A

Analysis View 16

Analysis View Functions 17

Approve Exam, Structured Reporting 152

Atria 70

Auto Save Preferences 36

B

Browse DB 21

C

Chamber Volume Table 60

Cine Mode 11

Compare Mode 26

Contour Deletion 47

Contour Edit

Conventional Editing 45

Delete a Contour 47

Nudge Tool 46

Options 45

Cross Reference Mode 12

Custom Measurement

Add 73

Remove 73

D

Database, Browse 21
Delete a Contour 47
Delete Favorite, Report Database 162

E

Edit
 Preference Ranges 39
 Tab Labels 19
Editing Contours 45
Equipment Hazards 3
Export
 Preferences 42
Export Search Results to HTML
 Report Database 163

F

File Menu Options 9
Flow Analysis 77
 Display Modes 91
 Flow Correction Options 86
 Peak Velocity 90
 Review Summary 94
 Window Components 78
Function Analysis 49
 Custom Measurement, Add 73
 Custom Measurement, Remove 73
 Linear Measurement Set-up 72
 Manual LV Function Analysis Procedure 63
 Reset Measurements 73
 Ventricular Function Analysis Results 59

H

Help Menu Options 10
History Tab 145

I

Image Management Tools 23

 Compare Mode 26

Image Manipulation Tools 13

Image View Controls 10

Images Tab 148

Import

 Database 164

 Preferences 42

Impression Tab 147

Impressions

 Macro, Add 40

Indications for Use 1

Intended Use 2

L

LA

 Function Analysis, Manual 70

 Functional Analysis, Fast 71

Launch the Application 5

Linear Measurement

 Set-up 72

LV

 Function Analysis, Fast 69

 Function Analysis, Manual 63

 Function Analysis, Results 59

M

Macro

 Delete 41

 Enter Text 41

 Execute 41

 Impressions, Add 40

 Preferences 40

Matrix Mode 11

Measurements

 Custom, Add 73

 Custom, Remove 73

- Linear 72
- Reset 73
- Miscellaneous Panel 35
- Myocardial Colormap 129
- Myocardial Evaluation 97
 - Polar Plot Formats 99
 - Quantitative Analysis Procedure 98

N

- Nudge Tool 46

P

- Patent Foramen Ovale (PFO) Analysis 123
- Peak Velocity, Flow Analysis 90
- PFO
 - Analysis Window 124
 - Atrial Anatomy 124
 - Contours 126
 - Curve Data 125
 - Curve Results 126
 - LA Intensity Curve 124
 - Phase Range Selection 125
 - RA Intensity Curve 124
- Polar Plots Tab 149
- Preferences
 - Auto Save 36
 - Defining 31
 - Edit 31
 - Export 36, 42
 - Import 42
 - Macro 40
 - Report 31
 - Template 37

Q

- Qp/Qs
 - Calculate 96
 - Tab 95

Quick Keys 14
Quitting the Application 5

R

RA

- Fast Function Analysis 71
- Function Analysis, Manual 70

Regional Analysis 61

Report

- Approvers 33
- Approvers, Manage 33
- Preferences Procedure 32

Report Database 157

- Activate Search 159
- Delete Favorite 162
- Export Search to HTML 163
- Import Database 164
- Query 158
- Save Query 161
- Search Criteria 158
- Tools Procedure 158
- Window 157

Report Preview, Structured Reporting 152

Reporting Tabs 20

Reset Measurements 73

RV

- Manual Function Analysis 66

S

Safety Notices 3

Save Query, Report Database 161

Search, Report Database 159

Series Navigation 8

Structured Reporting 145

- Approve Exam 152
- Approved Exam 155
- Export 154
- Graphs 149
- History Tab 145

- Images Tab 148
- Impressions 147
- Polar Plots Tab 149
- Preview Report 152
- Summary Table 149
- Tabs 145
- Techniques 147

T

- T1 Mapping 109
- T2 Mapping 113
- T2Star 127
 - Analysis Procedure 128
 - Myocardial Colormap, Create 129
 - Parameter Fit 129
 - Results 130
 - Review & Update Curve 130
- Tab
 - Edit Labels 19
 - History 145
 - Images 148
 - Impression 147
 - Polar Plots 149
 - Qp/Qs 95
 - Reporting 20
 - Structured Reporting, Images 148
 - Structured Reporting, Impression 147
- Technique, Structured Reporting 147
- Template
 - Create 38
 - Delete 39
 - Duplicate 38
 - Preferences 37
- Time Course
 - Analysis 117
 - Color Map 119
 - Curve Generation 119
 - Define Contours 118
 - Image Registration 118
 - Quantitative Procedure 117
 - Results 119
- Tools Menu 9

U

User Interface

- Image Manipulation 13

User Interface

- Analysis Modes 8

- Analysis Panel 16

- Cine 11

- Cross Reference Mode 12

- Editor Window 9

- File Menu 9

- Help Menu 10

- Image Viewer Controls 10

- Matrix Mode 11

- Mode View 9

- Overview 7

- Reporting Tabs 20

- Series Navigation 8

- Tools Menu 9

V

Valve Plane Analysis 74

Ventricles 49

Viewer 23

