[suiteHEART® Software cMRI Analysis Software Release Notes](#)

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NeoSoft, LLC

NEOSOFT

NS-03-040-0027 Rev. 1
Copyright 2020 NeoSoft, LLC
All rights reserved

2797

Manufacturer

NeoSoft, LLC
N27 W23910A Paul Road
Pewaukee, WI 53072 USA

Sales: orders@neosoftmedical.com
Service: service@neosoftmedical.com

The Instructions for Use for this device are provided electronically in Portable Document Format, (.pdf). A pdf viewer is required to view the Instructions for Use. A paper copy of the Instructions for Use can be provided upon request, at no cost, within 7 calendar days, by emailing service@neosoftmedical.com.

Instructions for Use may be accessed in the following ways:

1. After launching the application, click “Help” or “About” from the main screen. Select the “Instructions for Use” option. The Instructions for Use will open in a pdf viewer.
2. If the original installation package received from NeoSoft is available, open the zip file and navigate to the “Documentation” folder, then the “Instructions for Use” folder and double click on the Instructions for Use.pdf in your language, denoted by EN - English, FR - French, DE - German, EL - Greek, IT - Italian, LT - Lithuanian, ES - Spanish, SV - Swedish, TR - Turkish, RO - Romanian, NL - Dutch.
3. Navigate to the folder where the application is installed. Locate the “Documentation” folder, open the “Instructions for Use” folder and double click on the Instructions for Use.pdf in your language, denoted by EN - English, FR - French, DE - German, EL - Greek, IT - Italian, LT - Lithuanian, ES - Spanish, SV - Swedish, TR - Turkish, RO - Romanian, NL - Dutch.
4. Electronic copies of Instructions for Use are also available at www.neosoftllc.com/neosoft/product_manuals/.

Table of Contents

Software Revision History	3
Software Upgrade Notes.....	4
Software Updates.....	4
New Quick Key	4
Viewer.....	4
Series Navigation	4
DICOM Series Creation.....	4
Viewer and Virtual Fellow™: Find Feature*	4
Virtual Fellow™	5
Function Analysis	5
Result Table.....	5
ROI Editing Tools.....	5
Flow Analysis.....	5
Time Course	6
3D/4D Viewer.....	6
Preferences	6
Release Notes.....	6
Function Analysis	6
Time Course	6
Preprocessing.....	6
Virtual Fellow™	6
T2 Mapping	7
General	7

Software Revision History

suiteHEART® Software Revision History

Software Version	Release Date	Description
1.1.1	March 2013	Initial Release
1.1.2	February 2014	Implemented software licensing options
1.1.3	April 2014	Investigational Device Release for Canada
1.1.4	May 2014	Fixed Export Cine files not retaining image manipulation settings. Fixed images not present after performing a DICOM import Corrected infarct overlay paint issue with ME Analysis Adjusted cross reference criteria Corrected slice order issue for DICOM created series Corrected Custom Series not present after reloading of an approved exam Corrected issue with suiteHEART not launching from a non-admin user account
2.0.0	October 2014	Translated application to German and French Added support for Enhanced MR Image Storage Added Valve Plane Analysis calculations Added support for Microsoft Windows 8.1 Redesigned User Interface Look and Feel Dropped support for Microsoft Windows XP
2.1.0	January 2015	Minor updates and defect fixes Added Medical Device Directive information
3.0.0	October 2015	Added Edema and Salvage mass results Added Dyssynchrony Added T1 Mapping (Research Only) Minor updates and defect fixes
3.0.1	March 2016	Added Multiuser capabilities Minor updates and defect fixes
4.0.0	October 2016	Added 3D/4D Viewer Added Nudging Tool Added Compare Mode Added Auto Phase Error Correction
4.0.1	January 2017	Minor updates and defect fixes
4.0.2	May 2017	Added tablet mode. Removed support for Microsoft Windows 8.1 Improved Auto Segmentation workflow.
4.0.3	July 2017	Added ED/ES Only mode to Auto Segmentation. Improved Auto Segmentation algorithms. Minor updates and defect fixes.
4.0.4	November 2017	Added the Viewer. Updated T1 Mapping. Added T2 Mapping. Added the Siemens Auto Series creator.
4.0.6	May 2018	Upgrade CUDA to version 9.1 Add Strain Analysis (Research Only) Add DENSE Strain Analysis (Research Only) Combine Manual/Auto Analysis modes Upgrade to JAVA 9

Software Version	Release Date	Description
4.0.7	November 2018	3D/4D Viewer Improvements Improved editing of contours Improved papillary segmentation DENSE performance improvements Automate basal line Added Undo functionality
5.0.0	July 2019	Contains updates relating to: Preprocessing, Virtual Fellow™, Series Auto Combine, Function Analysis, T2 Mapping, and general information.
5.0.1	February 2020	Contains updates relating to: Viewer, Virtual Fellow™, Function Analysis, Flow Analysis, Time Course, 3D/4D Viewer and Preferences.

Software Upgrade Notes

suiteHEART® supports software upgrades from all previous versions to the current version, 5.0.1. Software upgrade support is provided by NeoSoft. Please contact NeoSoft Support at service@neosoftmedical.com to schedule your software upgrade. Considerations for upgrade options will be discussed and implemented on your system.

Software Updates

New Quick Key

- Ctrl+L - Toggles annotation line thickness.

Viewer

Series Navigation

Navigate to the next series on the list by pressing the right arrow key on the keyboard and the left arrow for the previous series.

When navigating to a multi-phase series they are displayed in an auto layout, whereas a single-phase series will be displayed in a 1x1 layout.

DICOM Series Creation

Series types for DICOM creation now include T1 mapping, T2 Mapping and T2 Star.

Viewer and Virtual Fellow™: Find Feature*

1. Select the icon to use the Find Feature cross reference tool. The purple cursor is the primary cursor that can be positioned on the image.
2. To automatically view all close slice locations, use the <Ctrl> key. With the primary cursor activated and positioned press the <Ctrl> key and release the purple cursor with the left mouse. The main view will then be populated with only those slices where the secondary green cursor was calculated as close to the primary purple cursor.

Note: The green secondary annotations appear on viewports containing NON-PARALLEL images and at points that are calculated to be within 10mm 3D distance of the primary cursor.

Note: The green secondary annotations appear on viewports containing PARALLEL images and at points that are calculated to be within 5mm 3D distance of the primary purple cursor.

*US Provisional Patent Application No. 62/923,061

Title: Method and System for Identifying and Displaying Medical Images

Inventor(s): Wolff et al.

Virtual Fellow™

- User selection of long axis cross reference views can now be performed.

Function Analysis

Result Table

- LV Mass result can be selected from the yellow arrow on the result table. The default result is ED.

ROI Editing Tools

For Atrial LA endo, RA Endo for both manual and fast analysis; short axis LV Endo, LV Epi, RV endo and RV epi; Long axis LV Endo, LV Epi ROIs are selected for editing additional quick keys include:

- Ctrl +C - Copy ROI
- Ctrl+V - Paste ROI
- Ctrl +S - Smooth ROI
- Left and Right Arrow Keys - Shift ROI horizontally
- Up and Down Arrow Keys – Shift ROI vertically

Flow Analysis

- Regurgitant mode option to only calculate the net negative flow (below x-axis).
- Additional Editing tools are now available on the interface when the ROI is selected in edit mode.
- Ctrl +C - Copy ROI
- Ctrl+V - Paste ROI
- Ctrl +S - Smooth ROI
- Left and Right Arrow Keys - Shift ROI horizontally
- Up and Down Arrow Keys – Shift ROI vertically

Tool	Description
	Copy edit to end of phases
	Copy ROI from previous phase
	Copy ROI to next phase
	Expand ROI size
	Decrease ROI size

Time Course

- Multi-slice analysis for both stress and rest series. (Refer to the suiteHEART® Software Instructions for Use NS-03-039-0003.)
- Image registration is not supported, it is recommended to use motion corrected images for analysis that are generated by the MRI scanner.

3D/4D Viewer

- Inline flow analysis is now supported for 4D flow. (Refer to the suiteHEART® Software Instructions for Use NS-03-039-0003.)
- Inline flow results are based upon the acquired slice thickness.
- If the image plane has been changed such as performing a pan the ROI cannot be edited. To enable editing, perform a right mouse click and select *Locate* or click a point on the flow curve. This will ensure that the editing plane is in the same plane as when the ROI was first placed.

Preferences

- The Global settings tab preferences layout has been updated. For changes in preferences to take effect close the current exam and then close and re-launch suiteDXT.

Release Notes

Function Analysis

- The Mass ED, Mass EDI, Mass ES and Mass ESI results have been removed; Macros created in preferences with these results will need to be updated.
- The Epi EDV and Epi ESV results are no longer available and have been removed from the results table. Marcos created with these results will need to be removed.

Time Course

Time Course is not backwards compatible to any previous release.

Preprocessing

- Auto Detect is not supported for Canon Medical studies.

Virtual Fellow™

- If the review or results views are open Ctrl-T will not function.
- If typing in the height or weight on the History Tab the function indexed results will not update, switch to Function analysis to update the results.
- If viewing a current and prior study and if measurement tools are used, measurements placed on the prior study will not be saved.

- Long axis Time Course images can be selected for the long axis cross reference viewports. If the long axis is in the same series as the short axis create a DICOM series which just contains the long axis view.

T2 Mapping

- Siemens T2 mapping studies previously analyzed with version 4.0.4, the T2 results will need to be updated by directly clicking on a ROI or by deleting the ROIs and performing the analysis.

General

- The quick key for Load Approve Exam Ctrl+L is no longer available.
- When using suiteHEART® Software at a screen resolution higher than 1920x1080 the matrix remains at the same size as it appears at 1920x1080.
- When networking a study to suiteDXT, do not perform any analysis until the study has been transferred completely.

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