[suiteDXT Release Notes](#)

[suiteDXT Versionshinweise](#)

[suiteDXT Notes de publication](#)

[suiteDXT Note di rilascio](#)

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NeoSoft, LLC

NEOSOFT

NS-03-040-0028 Rev. 2
Copyright 2020 NeoSoft, LLC
All rights reserved

suiteDXT

Release Notes

NeoSoft, LLC

The logo for NeoSoft, featuring the word "NEO" in a bold, blue, sans-serif font, followed by "SOFT" in a lighter, grey, sans-serif font.

NS-03-040-0028 Rev. 2
Copyright 2020 NeoSoft,
LLC All rights reserved

CE

Manufacturer

NeoSoft, LLC
N27 W23910A Paul Road
Pewaukee, WI 53072 USA

Sales: orders@neosoftmedical.com

Service: service@neosoftmedical.com

The Instructions for Use for this device are provided electronically in Portable Document Format, (.pdf). A pdf viewer is required to view the Instructions for Use. A paper copy of the Instructions for Use can be provided upon request, at no cost, within 7 calendar days, by emailing service@neosoftmedical.com.

Instructions for Use may be accessed in the following ways:

1. After launching the application, click “Help” or “About” from the main screen. Select the “Instructions for Use” option. The Instructions for Use will open in a pdf viewer.
2. If the original installation package received from NeoSoft is available, open the zip file and navigate to the “Documentation” folder, then the “Instructions for Use” folder and double click on the Instructions for Use.pdf in your language, denoted by EN - English, FR - French, DE - German, EL - Greek, IT - Italian, LT - Lithuanian, ES - Spanish, SV - Swedish, TR - Turkish, RO - Romanian, NL - Dutch.
3. Navigate to the folder where the application is installed. Locate the “Documentation” folder, open the “Instructions for Use” folder and double click on the Instructions for Use.pdf in your language, denoted by EN - English, FR - French, DE - German, EL - Greek, IT - Italian, LT - Lithuanian, ES - Spanish, SV - Swedish, TR - Turkish, RO - Romanian, NL - Dutch.
4. Electronic copies of Instructions for Use are also available at www.neosoftllc.com/neosoft/product_manuals/.

Software Revision History

suiteDXT Software Revision History

Software Version	Release Date	Description
1.0.0	June 2013	Initial Release.
1.0.1	February 2014	Implemented software licensing options.
2.0.0	October 2014	Removed Microsoft SQL dependency. Updated UI to match suiteHEART. Added support for study export to zipfile. Added support for compressed DICOM images. Translated application to German and French. Added support for Microsoft Windows 8.1. Dropped support for Microsoft Windows XP.
2.1.0	January 2015	Minor updates and defect fixes. Added Medical Device Directive information.
3.0.0	October 2015	Minor updates and defect fixes.
3.0.1	March 2016	Added Multiuser capabilities. Minor updates and defect fixes.
4.0.0	October 2016	Added support for suiteDXT to suiteDXT DICOM network transfer. Improved DICOM transfer performance.
4.0.2	May 2017	Added Pre-Processing. Removed support for Microsoft Windows 8.1.
4.0.4	November 2017	Added support for configuring multiple series in preprocessing.
4.0.5	November 2017	Minor updates and defect fixes.
4.0.6	May 2018	Updated support for preprocessing configuration. Improved DICOM import performance.
4.0.7	November 2018	Added access controls\permission
5.0.0	July 2019	Contains updates relating to: Preprocessing.
5.0.1	February 2020	Contains updates relating to: Exporting and Importing of Studies

suiteDXT supports software upgrades from all previous versions to the current version, 5.0.1. Software upgrade support is provided by NeoSoft. Please contact NeoSoft Support at service@neosoftmedical.com to schedule your software upgrade. Considerations for upgrade options will be discussed and implemented on your system.

Software Updates

Batch Exporting of Studies

- Studies can be exported as a .zip file by selecting a group of studies and performing a right mouse click and selecting ***Export Study***.
- If the exporting is canceled the incomplete .zip file will not be saved.
- A check will be performed of the free space available prior to exporting.

Importing of Studies

- The automatic identification of shared network drives is now supported.

Software Release Notes

Study Import

- During an import do not click the Refresh button.

Preprocessing

- Preprocessing will not overwrite a previously analyzed series.
- If there are more than six phase contrast series detected for the Flow analysis tabs the preprocessing will be performed on the last six phase contrast series detected.

Performance

To improve loading performance time, add the suiteDXT\Image directory (ex. C:\ProgramData\NeoSoft, LLC\suiteDXT\Images) and the suiteDXT\SQLData directory (ex. C:\ProgramData\NeoSoft, LLC\suiteDXT\SQLData) to the anti-virus exclusion list.

Windows 10 Enterprise

If suiteDXT becomes unresponsive after resizing when installed on Windows 10 Enterprise, disable Snap in Windows Multitasking Settings and then restart suiteDXT.

Multi-User Environment

When a study is launched in suiteHEART® Software and then suiteDXT is set to full screen, the minimize, restore and close functions in the upper right corner may become unresponsive. If this happens, select suiteHEART® Software from the toolbar at the bottom of the screen and minimize the application to regain the functions in suiteDXT. To avoid the issue either auto hide the toolbar or do not make suiteDXT full screen.

In the rare event that a pop-up is unresponsive, press the Enter key on the keyboard to reset the pop-up functionality.

suiteDXT

Versionshinweise

NeoSoft, LLC

NEOSOFT

NS-03-040-0028 Rev. 2
Copyright 2020 NeoSoft, LLC
Alle Rechte vorbehalten

Hersteller

NeoSoft, LLC
N27 W23910A Paul Road
Pewaukee, WI 53072, USA

Vertrieb: orders@neosoftmedical.com

Service: service@neosoftmedical.com

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