[suiteDXT Release Notes](#)

[suiteDXT Versionshinweise](#)

[suiteDXT Notes de publication](#)

[suiteDXT Note di rilascio](#)

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NEOSOFT

NeoSoft, LLC

NS-03-041-0004 Rev. 2
Copyright 2021 NeoSoft,
LLC All rights reserved

suiteDXT

Release Notes

NeoSoft, LLC

NEOSOFT

NS-03-041-0004 Rev. 2
Copyright 2021 NeoSoft,
LLC All rights reserved

Manufacturer

NeoSoft, LLC
N27 W23910A Paul Road
Pewaukee, WI 53072 USA

Sales: orders@neosoftmedical.com
Service: service@neosoftmedical.com

The Instructions for Use for this device are provided electronically in Portable Document Format, (.pdf). A pdf viewer is required to view the Instructions for Use. A paper copy of the Instructions for Use can be provided upon request, at no cost, within 7 calendar days, by emailing service@neosoftmedical.com.

Instructions for Use may be accessed in the following ways:

1. After launching the application, click “Help” or “About” from the main screen. Select the “Instructions for Use” option. The Instructions for Use will open in a pdf viewer.
2. If the original installation package received from NeoSoft is available, open the zip file and navigate to the “Documentation” folder, then the “Instructions for Use” folder and double click on the Instructions for Use.pdf in your language, denoted by EN - English, FR - French, DE - German, EL - Greek, IT - Italian, LT - Lithuanian, ES - Spanish, SV - Swedish, TR - Turkish, RO - Romanian, NL – Dutch, ZH_CN - Simplified Chinese, PT-PT - Portuguese, HU - Hungarian.
3. Navigate to the folder where the application is installed. Locate the “Documentation” folder, open the “Instructions for Use” folder and double click on the Instructions for Use.pdf in your language, denoted by EN - English, FR - French, DE - German, EL - Greek, IT - Italian, LT - Lithuanian, ES - Spanish, SV - Swedish, TR - Turkish, RO - Romanian, NL – Dutch, ZH_CN - Simplified Chinese, PT-PT - Portuguese, HU - Hungarian.
4. Electronic copies of Instructions for Use are also available at www.neosoftllc.com/neosoft/product_manuals/ for at least 15 years from last date of manufacture.

Medical device directive

This product conforms with the requirements of council directive 93/42/EEC concerning medical devices when it bears the following CE mark of Conformity:

European Representatives:

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

Canada:

Health Canada device license number: 99490

Malaysian Medical Device Registration Number: GB10979720-50888

Malaysia Authorized Representative:

Emergo Malaysia Sdn. Bhd.
Level 16, 1 Sentral Jalan Stesen Sentral 5
KL Sentral, 50470 Kuala Lumpur MALAYSIA

CAUTION: *United States federal law restricts this device to sale by, or on the order of, a physician.*

Software Revision History

suiteDXT Software Revision History

Software Version	Release Date	Description
1.0.0	June 2013	Initial Release.
1.0.1	February 2014	Implemented software licensing options.
2.0.0	October 2014	Removed Microsoft SQL dependency. Updated UI to match suiteHEART. Added support for study export to zipfile. Added support for compressed DICOM images. Translated application to German and French. Added support for Microsoft Windows 8.1. Dropped support for Microsoft Windows XP.
2.1.0	January 2015	Minor updates and defect fixes. Added Medical Device Directive information.
3.0.0	October 2015	Minor updates and defect fixes.
3.0.1	March 2016	Added Multiuser capabilities. Minor updates and defect fixes.
4.0.0	October 2016	Added support for suiteDXT to suiteDXT DICOM network transfer. Improved DICOM transfer performance.
4.0.2	May 2017	Added Pre-Processing. Removed support for Microsoft Windows 8.1.
4.0.4	November 2017	Added support for configuring multiple series in preprocessing.
4.0.5	November 2017	Minor updates and defect fixes.
4.0.6	May 2018	Updated support for preprocessing configuration. Improved DICOM import performance.
4.0.7	November 2018	Added access controls\permission
5.0.0	July 2019	Contains updates relating to: Preprocessing.
5.0.1	February 2020	Contains updates relating to: Exporting and Importing of Studies
5.0.2	March 2021	Added Auto Delete functionality, folder creation, enhanced Anonymization options, minor updates and defect fixes. This release is the initial software release for China.


suiteDXT supports software upgrades from all previous versions to the current version, 5.0.2. Software upgrade support is provided by NeoSoft. Please contact NeoSoft Support at service@neosoftmedical.com to schedule your software upgrade. Considerations for upgrade options will be discussed and implemented on your system.

Software Updates

Anonymize a Study

- Additional options are available for anonymization which include the option for the Institution name and the Study date.
- To not include NeoSoft secondary capture series, deselect the option **Include NeoSoft Secondary Capture Series**.

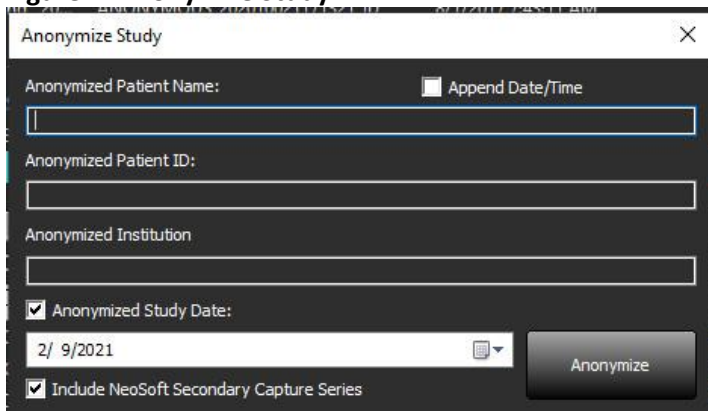Figure 1: Anonymize Study

The screenshot shows a dialog box titled "Anonymize Study". It contains the following fields and options:

- Anonymized Patient Name:** A text input field with a checkbox labeled "Append Date/Time" to its right.
- Anonymized Patient ID:** A text input field.
- Anonymized Institution:** A text input field.
- Anonymized Study Date:** A date picker showing "2/ 9/2021".
- Include NeoSoft Secondary Capture Series:** A checked checkbox.
- Anonymize:** A button at the bottom right.

Software Release Notes

Study Import

- During an import do not click the Refresh button.

Preprocessing

- Preprocessing will not overwrite a previously analyzed series.
- If there are more than six phase contrast series detected for the Flow analysis tabs the preprocessing will be performed on the last six phase contrast series detected.
- If the Preprocessing configuration is not displayed when both 32 and 64 bit Java is installed, uninstall the 32-bit version of Java.

Windows 10 Enterprise

If suiteDXT becomes unresponsive after resizing when installed on Windows 10 Enterprise, disable Snap in Windows Multitasking Settings and then restart suiteDXT.

Multi-User Environment

When a study is launched in suiteHEART[®] Software and then suiteDXT is set to full screen, the minimize, restore and close functions in the upper right corner may become unresponsive. If this happens, select suiteHEART[®] Software from the toolbar at the bottom of the screen and minimize the application to regain the functions in suiteDXT. To avoid the issue either auto hide the toolbar or do not make suiteDXT full screen.

In the rare event that a pop-up is unresponsive, press the Enter key on the keyboard to reset the pop-up functionality.

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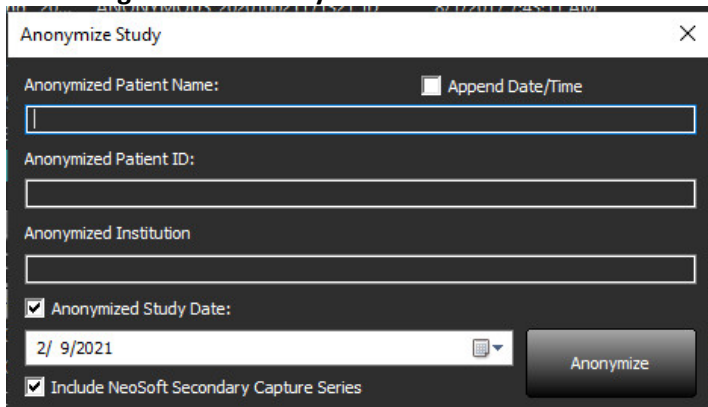
Abbildung 1: Studie anonymisieren

The screenshot shows a dialog box titled "Anonymize Study" with a close button (X) in the top right corner. The dialog contains several input fields and checkboxes:

- Anonymized Patient Name:** A text input field with a blue border. To its right is a checkbox labeled "Append Date/Time".
- Anonymized Patient ID:** A text input field.
- Anonymized Institution:** A text input field.
- Anonymized Study Date:** A checked checkbox followed by a date input field showing "2/ 9/2021" and a calendar icon.
- Include NeoSoft Secondary Capture Series:** A checked checkbox.
- Anonymize:** A button located at the bottom right of the dialog.

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- Anonymized Patient Name:** A text input field with a checkbox labeled "Append Date/Time" to its right.
- Anonymized Patient ID:** A text input field.
- Anonymized Institution:** A text input field.
- Anonymized Study Date:** A date picker showing "2/ 9/2021" with a checked checkbox to its left.
- Include NeoSoft Secondary Capture Series:** A checked checkbox at the bottom left.
- Anonymize:** A button at the bottom right.

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- Anonymized Patient Name:** A text input field with a blue border. To its right is a checkbox labeled "Append Date/Time".
- Anonymized Patient ID:** A text input field.
- Anonymized Institution:** A text input field.
- Anonymized Study Date:** A date picker showing "2/ 9/2021". It has a checked checkbox to its left.
- Include NeoSoft Secondary Capture Series:** A checked checkbox at the bottom left.
- Anonymize:** A button at the bottom right.

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