suiteHEART® Software cMRI Analysis Software Release Notes
suiteHEART®-Software cMRI-Analyse-Software Versionshinweise
Logiciel suiteHEART® Logiciel d'analyse cMRI Notes de publication
Software suiteHEART® Software per analisi cMRI Note di rilascio
Λογισμικό suiteHEART® cMRI, λογισμικό ανάλυσης Σημειώσεις έκδοσης
„suiteHEART®“ programinė įranga cMRI analizės programinė įranga Išleidimo pastabos
Software suiteHEART® Software de análisis cMRI Notas de publicación
suiteHEART® cMRI Analysprogramvara Versionsinformation
suiteHEART® Software cMRI Analiz Yazılımı Sürüm Notları
Software-ul suiteHEART® Software de analiză cMRI Note privind versiunea
suiteHEART® Software cMRI-analysesoftware Opmerkingen bij deze uitgave
suiteHEART® Software de análise cMRI Notas de lançamento
suiteHEART® szoftver cMRI elemzőszoftver Kibocsátási megjegyzések
Phần mềm suiteHEART® Phần mềm phân tích cMRI Thông tin phiên bản
suiteHEART® Software Software de análise cMRI Notas de versões
suiteHEART® ソフトウェア cMRI 解析ソフトウェア リリースノート
suiteHEART® 软件 cMRI 分析软件 发布说明
Tarkvara suiteHEART® cMRT analüüsi tarkvara Väljalaskemärkmed

NEOSOFT

NeoSoft, LLC

NS-03-043-0011 Rev. 2
Copyright 2023 NeoSoft, LLC
All rights reserved

suiteHEART[®] Software

cMRI Analysis Software

Release Notes

NeoSoft, LLC

NEOSOFT

NS-03-043-0011-EN Rev. 2
Copyright 2023 NeoSoft, LLC
All rights reserved

Manufacturer

NeoSoft, LLC
N27 W23910A Paul Road
Pewaukee, WI 53072 USA

Phone: 262-522-6120
website: www.neosoftllc.com

Sales: orders@neosoftmedical.com
Service: service@neosoftmedical.com

To view compliance information (Authorized Representative, Importer, Registration information) after launching the application, click “Help” or “About” from the main screen. Select the “Regulatory Information” option. The document will open in a pdf viewer.

The Instructions for Use for this device are provided electronically in Portable Document Format, (.pdf). A pdf viewer is required to view the Instructions for Use. A paper copy of the Instructions for Use can be provided upon request, at no cost, within 7 calendar days, by emailing service@neosoftmedical.com.

Instructions for Use may be accessed in the following ways:

1. After launching the application, click “Help” or “About” from the main screen. Select the “Instructions for Use” option. The Instructions for Use will open in a pdf viewer.
2. If the original installation package received from NeoSoft is available, open the zip file and navigate to the “Documentation” folder, then the “Instructions for Use” folder and double click on the Instructions for Use.pdf in your language, denoted by EN - English, FR - French, DE - German, EL - Greek, IT - Italian, LT - Lithuanian, ES - Spanish, SV - Swedish, TR - Turkish, RO - Romanian, NL - Dutch, PT-PT - European Portuguese, HU - Hungarian, JA - Japanese, VI - Vietnamese, ZH-CN - Simplified Chinese, PT-BR - Brazilian Portuguese, ET - Estonian.
3. Navigate to the folder where the application is installed. Locate the “Documentation” folder, open the “Instructions for Use” folder and double click on the Instructions for Use.pdf in your language, denoted by EN - English, FR - French, DE - German, EL - Greek, IT - Italian, LT - Lithuanian, ES - Spanish, SV - Swedish, TR - Turkish, RO - Romanian, NL - Dutch, PT-PT - European Portuguese, HU - Hungarian, JA - Japanese, VI - Vietnamese, ZH-CN - Simplified Chinese, PT-BR - Brazilian Portuguese, ET - Estonian.
4. Electronic copies of Instructions for Use are also available at www.neosoftllc.com/neosoft/product_manuals/ for at least 15 years from last date of manufacture.
5. suiteHEART / suiteDXT Software Group Basic UDI DI – 00850001088039.

suiteHEART® Software Revision History

Software Version	Release Date	Description
1.1.1	March 2013	Initial Release
1.1.2	February 2014	Implemented software licensing options
1.1.3	April 2014	Investigational Device Release for Canada
1.1.4	May 2014	Fixed Export Cine files not retaining image manipulation settings. Fixed images not present after performing a DICOM import Corrected infarct overlay paint issue with ME Analysis Adjusted cross reference criteria Corrected slice order issue for DICOM created series Corrected Custom Series not present after reloading of an approved exam Corrected issue with suiteHEART not launching from a non-admin user account
2.0.0	October 2014	Translated application to German and French Added support for Enhanced MR Image Storage Added Valve Plane Analysis calculations Added support for Microsoft Windows 8.1 Redesigned User Interface Look and Feel Dropped support for Microsoft Windows XP
2.1.0	January 2015	Minor updates and defect fixes Added Medical Device Directive information
3.0.0	October 2015	Added Edema and Salvage mass results Added Dyssynchrony Added T1 Mapping (Research Only) Minor updates and defect fixes
3.0.1	March 2016	Added Multiuser capabilities Minor updates and defect fixes
4.0.0	October 2016	Added 3D/4D Viewer Added Nudging Tool Added Compare Mode Added Auto Phase Error Correction
4.0.1	January 2017	Minor updates and defect fixes
4.0.2	May 2017	Added tablet mode. Removed support for Microsoft Windows 8.1 Improved Auto Segmentation workflow
4.0.3	July 2017	Added ED/ES Only mode to Auto Segmentation. Improved Auto Segmentation algorithms. Minor updates and defect fixes
4.0.4	November 2017	Added the Viewer Updated T1 Mapping Added T2 Mapping Added the Siemens Auto Series creator
4.0.6	May 2018	Upgrade CUDA to version 9.1 Add Strain Analysis (Research Only) Add DENSE Strain Analysis (Research Only) Combine Manual/Auto Analysis modes Upgrade to JAVA 9

Software Version	Release Date	Description
4.0.7	November 2018	3D/4D Viewer Improvements Improved editing of contours Improved papillary segmentation DENSE performance improvements Automate basal line Added Undo functionality
5.0.0	July 2019	Contains updates relating to: Preprocessing, Virtual Fellow™, Series Auto Combine, Function Analysis, T2 Mapping, and general information
5.0.1	February 2020	Contains updates relating to: Viewer, Virtual Fellow™, Function Analysis, Flow Analysis, Time Course, 3D/4D Viewer and Preferences
5.0.2	March 2021	Added enhancements to T1 and T2 Mapping. ROI editing tool enhancements. Added additional Flow vessel categories. Enhancements to system preferences. Minor updates and defect fixes. This release is the initial software release for China.
5.0.2	June 2021	Added MD symbol, EU importer reference, incident reporting information
5.0.3	September 2021	Enhancements including: Auto Update, Preprocessing for T1 and T2 Mapping, Function editing, system preferences, exporting to XLS and XML. Minor updates and defect fixes. This is the initial software release for Japanese and Vietnamese.
5.0.4	November 2021	Minor updates and defect fixes
5.1.0	December 2022	Added support for Microsoft Windows 11 and removed support for Microsoft Windows 7. Added Menu Driven Reporting interface, as well as other feature updates. This is the initial software release for Brazilian Portuguese.
5.1.1	October 2023	Added interfaces to Epic and CMR Coop. Updated 4D Flow analysis functionality. Added Viewer Image/series filter functionality. suiteHEART version 5.1.1 will be the last version of software supported on Server 2012.

Software Updates

General

Improved visualization of the RV insertion point for all analysis.

Integration with EPIC and CMR Coop.

Improved window/level optimization for studies that do not have Virtual Fellow[®] applied.

Viewer

Image/series filter feature supported, refer to the suiteHEART Instructions for Use.

Viewport mouse scroll wheel slice navigation is supported.

Directly double clicking on a viewport will open the image in a 1 X 1 viewport. Double clicking again on the viewport will return to all images.

Virtual Fellow[®]

Viewport mouse scroll wheel slice navigation is supported.

Function Analysis

Clicking and dragging directly on the ED or ES notations on the ventricular volume curve changes the phase assignment. Visually confirm ED and ES assignments on the matrix.

Three states of ROI editing supported: Limit ROI/No Limit/Overlap. Selection can be set in preferences.

Improved matrix indicator visualization of the phase assignments for the right atrium.

Myocardial Evaluation

Reset option available for FWHM algorithm for late enhancement analysis.

The enhancement results are now dynamically updated with thresholding adjusting.

T1/T2 Mapping

GE generated T1 and T2 Maps series can be manually segmented.

Preprocessing for GE generated T2 Map series is not supported.

Myocardial Perfusion

RR Interval plot can now be reviewed if the Acquisition DICOM tag is present.

Averaged values according to coronary distribution are now displayed.

3D/4D Flow Viewer

Vector visualization is now supported for 4D Flow.

Strain Analysis*

Preprocessing is now supported for strain analysis, available if requested from NeoSoft.

***Strain Analysis is only for research purposes. Claims have not been evaluated by the FDA. Use pursuant to company instruction and research agreement. NeoSoft, LLC proprietary and confidential.**

Release Notes

Flow Analysis

When reviewing flow analysis from prior studies confirm the vessel category for each flow result.

Myocardial Evaluation

When performing T2 analysis for edema do not manually delete a single normal ROI; delete all normal ROIs.

T1/T2 Mapping

In T1 Mapping, when Fire or Royal is selected for the scale and a polar plot preference is checked and exam is closed and reopened, the polar plot scale displayed in the report will be set to Rainbow, however the polar plot will be displayed in the scale that was selected.

The polar plots displayed in the UI are displayed with the correct scale.

If the polar plot is sent to the report via the right click menu or the report is previewed under the T1 analysis mode the correct scale is displayed.

If the short axis slices are not exactly parallel the auto-combine will fail. Use the viewer to create a series for analysis.

If the ROI editing tool is set to Overlap in preferences and the ROIs are overlapped the resulting values will not update.

For T1 Mapping editing a blood pool ROI using the original DICOM images and dragging the ROI to the edge of the Endo ROI will remove the blood pool ROI from all times points. The blood pool T1 value will still be present on the table. Select the trashcan and select Blood Pool to reset.

When selecting the ECV color overlay, if the positioning of the anatomy in the FOV has changed from native to post, the ROIs will be misaligned. Results are not affected.

3D/4D Flow Viewer

If a study has been launched and 4D flow results have been generated using ROI 1 for a vessel category, and if additional 2D phase contrast series are then added and being networked with an auto update performed, the 2D flow series will not be preprocessed.

For 4D Flow the cine option will be enabled after image loading has been completed.

If using single oblique or double oblique mode when using the locate feature, the oblique lines may not represent the location of the anatomy in the viewport. Reselect the oblique tool to reset.

Preferences

In the multi-user environment, changes to preferences should only be performed by one user.

Templates

The results for cardiac output (CO) for the LV and RV are reported in l/min. When entering parameters for the gender specific equation for Z-scores, then a parameter for both male and female must be converted to l/min. When entering parameters from reference papers the units used must be the same.

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