suiteHEART® Software Addendum to Instructions for Use
suiteHEART® Software Addendum zur Bedienungsanleitung
Logiciel suiteHEART® Addendum aux instructions d'utilisation
Software suiteHEART® Documento aggiuntivo alle istruzioni per l'uso
Λογισμικό suiteHEART® Συμπλήρωμα για τις Οδηγίες χρήσης
„suiteHEART®“ programinē ģranga Priedas ģrangos naudojimo instrukcijoms
Software suiteHEART® Anexo de las Instrucciones de uso
suiteHEART® Tillägg till Bruksanvisning
suiteHEART® Yazılımı Kullanım Talimatlarına Ek
Software-ul suiteHEART® Completare la Instrucțiuni de utilizare
suiteHEART® Software Addendum bij de gebruikshandleiding
suiteHEART® Software Adenda às instruções de utilização
suiteHEART® szoftver A használati útmutató kiegészítése
Phần mềm suiteHEART® Phụ lục Hướng dẫn sử dụng
Software suiteHEART® Adendo às Instruções de Uso
suiteHEART® ソフトウェア 使用説明書補遺
suiteHEART® 软件 使用说明书附录

NeoSoft, LLC

NEOSOFT

NS-03-043-0006 Rev. 3
Copyright 2024 NeoSoft, LLC
All rights reserved

suiteHEART® Software

Addendum to Instructions for Use

NeoSoft, LLC

The logo for NeoSoft, featuring the word "NEO" in a bold, cyan-colored sans-serif font, followed by "SOFT" in a grey-colored sans-serif font. The letters are closely spaced and have a slight shadow effect.

NS-03-043-0006-EN Rev. 3
Copyright 2024 NeoSoft, LLC
All rights reserved

Revision History

Rev	Date	Description of Change	Safety Related Update (Yes/No)
1	22JULY2022	Updated for 5.1.0 product release. Brazilian Portuguese added. This IFU replaces the previous language/revision/part numbers: suiteHEART® Software Addendum to Instructions for Use, NS-03-039-0014 Rev. 3 which included EN, DE, FR, IT, EL, LT, ES, SV, TR, RO, NL, PT-PT, HU, and VI. NS-03-042-0013, Rev. 1 JA NS-03-041-0020, Rev. 2 ZH-CN	No
2	21APRIL2023	Updated for the 5.1.1 product release. Moved regulatory information to Regulatory Addendum document.	No
3	20DECEMBER2024	Updated for the 5.1.2 product release.	No

Manufacturer

NeoSoft, LLC
N27 W23910A Paul Road
Pewaukee, WI 53072 USA

Phone: 262-522-6120
website: www.neosoftllc.com

Sales: sales@neosoftmedical.com
Service: service@neosoftmedical.com


To view compliance information (Authorized Representative, Importer, Registration information) after launching the application, click “Help” or “About” from the main screen. Select the “Regulatory Information” option. The document will open in a pdf viewer.

System Requirements

suiteHEART® Software Minimum System Requirements (Multiuser Environment)
Supported Operating Systems: <ul style="list-style-type: none">• Windows Server 2016 Standard• Windows Server 2019 Standard• Windows Server 2022 Standard• Windows Server 2025 Standard
Citrix Virtual Apps and Desktops, VMWare Horizon, or Remote Desktop access
Video card (NVIDIA); 24GB RAM (minimum), minimum of 5000 CUDA Cores, support for OpenGL 4.0 and support for 1920 x 1080 resolution or higher
Monitor shall support 1920 x 1080 resolution or higher (Landscape); 2160 x 3840 resolution or higher (Portrait)
Anti-Virus/Malware software installed
Minimum 128GB memory (160GB required for 4D Flow Analysis)
Available hard drive space minimum 1TB
PDF viewer, Adobe Reader 11.0 or higher
An open network port for DICOM transfer over network
Static IP address (recommended)
Amazon Corretto
Solid-State hard drive(s) (SAS) Locally attached storage
Intel® Core™ processor(s) (8 cores)
Citrix client displays must not exceed scaling/zoom levels greater than 125%

Installation Examples

This diagram shows the physical topography of NeoSoft's suiteHEART® / suiteDXT installation. The software can be installed on a single workstation, (labeled Application Server below), that interfaces via DICOM with an existing PACS server and/or scanners. One or more users may log into the single workstation to use the software. Alternatively, the software may be integrated with a customer's existing Citrix infrastructure by publishing the software installed on an application server, then sharing the applications via a secondary Citrix server, (labeled Citrix Virtual Apps & Desktops below). In both cases, the same software is installed.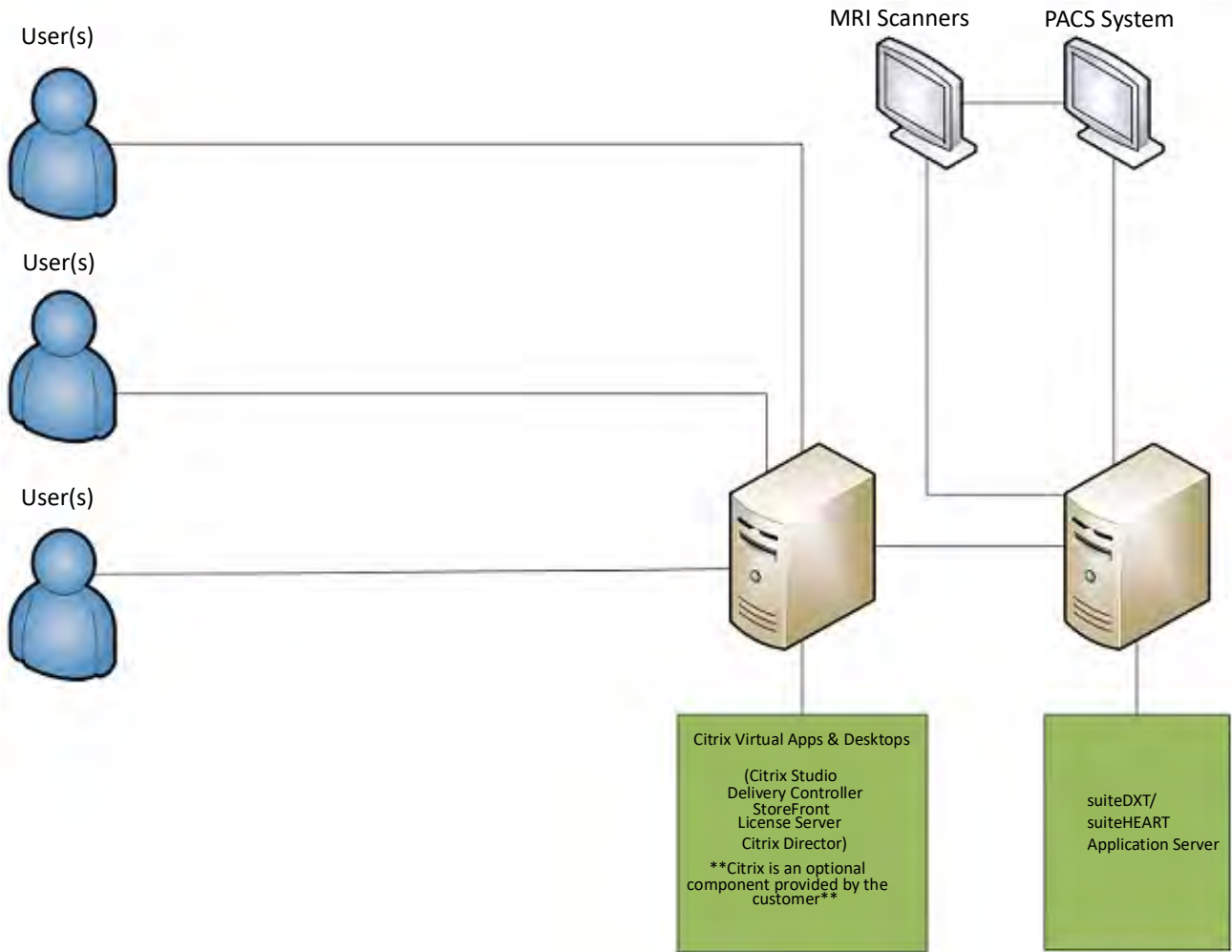

Addendum

This addendum applies to suiteHEART® Software Instructions for Use and suiteDXT Instructions for Use.

suiteHEART® Software supports remote connectivity capability, as well as running in a multiuser environment. The remote connectivity capability allows suiteHEART® Software users remote access from computers not loaded with suiteHEART® Software. The remote user will have the ability to access the suiteHEART® Software system and have full functionality. Remote connectivity is supported for only a single concurrent remote connection, whereas the multiuser environment supports up to five concurrent users.

Remote Connectivity - Single User

suiteHEART® remote connectivity needs to be installed/configured/supported by your Information Technology personnel. Remote connectivity has been verified by NeoSoft using the following platforms:

- TeamViewer
- Windows Remote Desktop (Microsoft Windows 10/11 Professional/Enterprise)

NOTE: When using Windows Remote Desktop, screen resolution on the remote computer must be set to 1920x1080 or higher video display resolution.

The performance of the suiteHEART® Software via the remote connection cannot be guaranteed. Performance is dependent upon factors that are outside the scope of the suiteHEART® Software application. These factors include:

- Remote computer performance
- Internet download/upload speeds (Mbps)
- Network bandwidth speed (wired ethernet or wireless connection)

Multiuser Environment

The suiteDXT/suiteHEART® Software application allows multiple concurrent users when installed in Citrix Virtual Apps and Desktops or VMWare Horizon.

Using suiteDXT Software in the Multiuser Environment

For complete installation instructions review the Installation Manual.

Configuration

Local Storage (i.e. Restart Service, Browse, Synchronize DB and Permissions) configuration changes affect all system users and therefore should be managed by the system administrator for your facility.

Preprocessing


Preprocessing and Virtual Fellow™ configuration changes affect all system users and therefore should be managed by the System Administrator for your facility.

Launching a Study from suiteDXT

If a study has been opened in suiteHEART® Software by one user, that study is locked and a warning message will appear when a second user attempts to open the same study. The study can only be opened after it's closed by the first user unless Read Only mode is enabled. In this mode, subsequent launches will be opened but in Read Only format. (Fig 1)

NOTE: The suiteHEART® Software will close after 60 minutes of inactivity and all analysis performed on open studies will be saved. The automatic closing of the suiteHEART® Software application will unlock open studies. It is recommended to always close the study after the analysis session is completed.

FIGURE 1. Launch Error

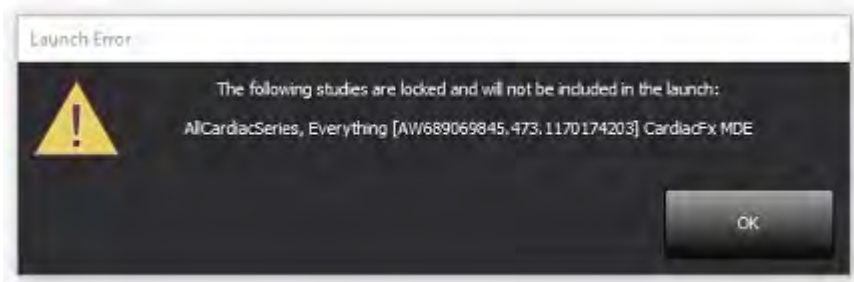Query/Retrieve from suiteDXT

During Query/Retrieve the Network status indicator, in the lower right of the main DXT screen, will not be active (Fig. 2). The status of a Query/Retrieve can be viewed on the Query/Retrieve Window.

Any new study that has been networked or imported into suiteDXT may not appear automatically in the directory listing. Click on the **Refresh** button, on the main DXT screen, to refresh the study and series listings (Fig. 2).

FIGURE 2. Query/Retrieve

The screenshot shows the suiteDXT application window. At the top, there is a 'Launch Application' button and a 'Study Filter' dropdown set to 'All'. A 'Refresh' button is next to the filter. To the right, a 'Configuration' button is highlighted with a red box. Below this is a table of studies with columns for Patient Name, Patient ID, Study Description, Accession, Date/Time, and Study ID. The row for 'Case 26' is highlighted in blue. Below the studies table is a table of series with columns for Series #, Images, Description, Start Time, Manufacturer, and Transferred To. The 'No DICOM Network Activity' indicator is highlighted with a red box at the bottom right. The disk usage is shown as 15.87% (C:\).

Patient Name	Patient ID	Study Description	Accession	Date/Time	Study ID
Case 22	ANONYMOUS_202...	Cardiac		9/7/2017 10:49 AM	ANONYM
Case 26	ANONYMOUS_202...	Cardiac		7/3/2018 2:20 PM	ANONYM
Case 36	ANONYMOUS_201...	Cardiac		11/22/2013 6:17 PM	ANONYM
Case 4 - BP 2015-03-02		Cine RM cuore		3/2/2015 3:22 PM	
Case 51, 20180921T150428	ANONYMOUS_201...	Cardiac		10/20/2017 11:39 AM	ANONYM
Case 52, 20180921T142741	ANONYMOUS_201...	Cardiac		11/16/2017 10:52 AM	ANONYM
Case 53, 20180921T151350	ANONYMOUS_201...	Cardiac		9/14/2017 11:14 AM	ANONYM
Case 55, 20180923T135256	ANONYMOUS_201...	CARDIAC Pericardium		1/5/2006 4:35 PM	ANONYM
Case 58, 20181108T123852	ANONYMOUS_201...	Cardiac		9/25/2018 2:10 PM	ANONYM
Case 61 Current	25769-1	Cardiac		1/11/2018 8:14 AM	ANONYM
Case 61 Prior	25769-1	Cardiac		9/25/2017 8:23 AM	ANONYM
Case 62 Prior	8143-1	Valve Study		10/30/2013 11:32 AM	ANONYM
Case 66	ANONYMOUS_201...	CAI Fx		1/6/2006 3:21 PM	ANONYM
Case 8 - BV 2016-04-03		Cine RM cuore			
CASE01	AW1760033629.875...	02		1/29/2002 4:46 PM	988

Series #	Images	Description	Start Ti...	Manufacturer	Transferred To
3615	1	CardiacVX DATA APPROVED vx		GE MEDICAL SYSTEMS	
3614	1	CardiacVX DATA APPROVED vx		GE MEDICAL SYSTEMS	
3613	1	CardiacVX DATA APPROVED vx		GE MEDICAL SYSTEMS	
3610	9	CardiacVX DATA		GE MEDICAL SYSTEMS	
3600	14	MAG:SSh PS MDE SPGR 4RR FB Anhy	3:14 PM	GE MEDICAL SYSTEMS	
3500	3	MAG:PS:2D PSMDE	3:12 PM	GE MEDICAL SYSTEMS	
3400	3	MAG:PS FSMDE FGRE 2RR	3:10 PM	GE MEDICAL SYSTEMS	

Deletion of a Study or Series from suiteDXT

Deletion of a study and its associated series that is opened and locked by another user is prohibited. A warning message (Fig 3) will appear if this is attempted. Deletion can only be performed when the study is closed by the other user.

NOTE: The suiteHEART® Software will close after 60 minutes of inactivity and all analysis performed on open studies will be saved. The automatic closing of the suiteHEART® Software application will unlock open studies. It is recommended to always close the study after the analysis session is completed.

FIGURE 3. Warning Message

The screenshot shows a warning dialog box titled "Item In Use". It contains an information icon on the left. The text inside the dialog reads: "Selected item for deletion is currently in use: AllCardiacSeries, Everything [AW689069845.473.1170.174203] CardiacFx MDE. Only select studies currently not in use." There is an "OK" button at the bottom right of the dialog.

Using suiteHEART® Software in the Multiuser Environment

For complete installation instructions review the Installation Manual.

Setting up Preferences

Editing preferences should be managed by the system administrator for your facility.

Image Management Tools

When exporting file types such as Matlab, excel, JPEG, etcetera, verify the selection of the desired destination directory is on your local machine.

Report Database

Search queries created in the Report Database can be overwritten if other users are simultaneously creating search queries.

A fully qualified share folder (i.e. //192.168.3.82/test) must be used when defining a (SECONDARY_REPORT_FOLDER=) to be used to send approved exams from suiteHEART® Software in a multiuser environment.

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Series #	Images	Description	Start Ti...	Manufacturer	Transferred To
3615	1	CardiacVX DATA APPROVED vx		GE MEDICAL SYSTEMS	
3614	1	CardiacVX DATA APPROVED vx		GE MEDICAL SYSTEMS	
3613	1	CardiacVX DATA APPROVED vx		GE MEDICAL SYSTEMS	
3610	9	CardiacVX DATA		GE MEDICAL SYSTEMS	
3600	14	MAG-SSH PS MDE SPGR 4RR FB Arrhy	3:14 PM	GE MEDICAL SYSTEMS	
3500	3	MAG-PS2D PSMDE	3:12 PM	GE MEDICAL SYSTEMS	
3400	3	MAG-PS FPMDE FGRE 2RR	3:10 PM	GE MEDICAL SYSTEMS	

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Series #	Images	Description	Start Ti...	Manufacturer	Transferred To
3615	1	CardiacVX DATA APPROVED vx		GE MEDICAL SYSTEMS	
3614	1	CardiacVX DATA APPROVED vx		GE MEDICAL SYSTEMS	
3613	1	CardiacVX DATA APPROVED vx		GE MEDICAL SYSTEMS	
3610	9	CardiacVX DATA		GE MEDICAL SYSTEMS	
3600	14	MAG:Ssh PS MDE SPGR 4RR FB Arrhy	3:14 PM	GE MEDICAL SYSTEMS	
3500	3	MAG:PS:2D PSMDE	3:12 PM	GE MEDICAL SYSTEMS	
3400	3	MAG:PS FSMDE FGRE 2RR	3:10 PM	GE MEDICAL SYSTEMS	

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Series #	Images	Description	Start Ti...	Manufacturer	Transferred To
3615	1	CardiacVX DATA APPROVED vv		GE MEDICAL SYSTEMS	
3614	1	CardiacVX DATA APPROVED vx		GE MEDICAL SYSTEMS	
3613	1	CardiacVX DATA APPROVED vx		GE MEDICAL SYSTEMS	
3610	9	CardiacVX DATA		GE MEDICAL SYSTEMS	
3600	14	MAG:Ssh PS MDE SPGR 4RR FB Arrhy	3:14 PM	GE MEDICAL SYSTEMS	
3500	3	MAG:PS:2D PSMDE	3:12 PM	GE MEDICAL SYSTEMS	
3400	3	MAG:PS FSMDE FGRE 2RR	3:10 PM	GE MEDICAL SYSTEMS	

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Series #	Images	Description	Start Ti...	Manufacturer	Transferred To
3615	1	CardiacVX DATA APPROVED vv		GE MEDICAL SYSTEMS	
3614	1	CardiacVX DATA APPROVED vx		GE MEDICAL SYSTEMS	
3613	1	CardiacVX DATA APPROVED vx		GE MEDICAL SYSTEMS	
3610	9	CardiacVX DATA		GE MEDICAL SYSTEMS	
3600	14	MAG:SSh PS MDE SPGR 4RR FB Arrhy	3:14 PM	GE MEDICAL SYSTEMS	
3500	3	MAG:PS:2D PSMDE	3:12 PM	GE MEDICAL SYSTEMS	
3400	3	MAG:PS FSMDE FGRE 2RR	3:10 PM	GE MEDICAL SYSTEMS	

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