suiteHEART® Software Installation Manual
suiteHEART® Software Installationshandbuch
Logiciel suiteHEART® Manuel d'installation
Software suiteHEART® Manuale d'installazione
Λογισμικό suiteHEART® Εγχειρίδιο εγκατάστασης
„suiteHEART®“ programinė įranga Diegimo vadovas
suiteHEART® Software Manual de instalación
suiteHEART® Installationshandbok
suiteHEART® Yazılımı Kurulum El Kitabı
Software-ul suiteHEART® Manual de instalare
suiteHEART® Software Installatiehandleiding
suiteHEART® Software Manual de instalação
suiteHEART® szoftver Telepítési útmutató
Phần mềm suiteHEART® Hướng dẫn cài đặt
Software suiteHEART® Manual de Instalação
suiteHEART® ソフトウェアインストールマニュアル
suiteHEART® 软件 安装手册

NeoSoft, LLC

NEOSOFT

NS-03-043-0005 Rev. 3
Copyright 2024 NeoSoft, LLC
All rights reserved

suiteHEART[®] Software

Installation Manual

NeoSoft, LLC

NEOSOFT

NS-03-043-0005 Rev 3
Copyright 2024 NeoSoft, LLC
All rights reserved

Revision History

Rev	Date	Description of Change	Safety Related Update (Yes/No)
1	18JULY2022	Updated for 5.1.0 product release. Brazilian Portuguese added. This IFU replaces the previous language/revision/part numbers: suiteHEART® Software Installation Manual, NS-03-039-0013, Rev. 5 which included EN, DE, FR, IT, EL, LT, ES, SV, TR, RO, NL, PT-PT, HU, and VI NS-03-042-0012, Rev. 1 JA NS-03-041-0019, Rev. 2 ZH-CN	No
2	26APRIL2023	Updated for the 5.1.1 product release. Moved regulatory information to Regulatory Addendum document.	No
3	20DECEMBER2024	Moved information on “consumed exam” to the suiteHEART IFU. Updated Safety information.	Yes

Manufacturer

NeoSoft, LLC
N27 W23910A Paul Road
Pewaukee, WI 53072 USA

Phone: 262-522-6120
website: www.neosoftllc.com

Sales: sales@neosoftmedical.com
Service: service@neosoftmedical.com

To view compliance information (Authorized Representative, Importer, Registration information) after launching the application, click “Help” or “About” from the main screen. Select the “Regulatory Information” option. The document will open in a pdf viewer.

Table of Contents

Safety	1
<i>Introduction</i>	1
<i>Indications for Use</i>	2
<i>Intended Use</i>	2
<i>Supported DICOM Image Formats</i>	2
<i>Safety Notices</i>	3
<i>Equipment Hazards</i>	3
Prerequisites	4
suiteDXT Installation & Configuration	5
Installation	5
Configuration	5
Local Storage Configuration	8
<i>Auto Delete</i>	9
<i>Auto Unlock</i>	9
Licensing	9
<i>Configuring Remote License Server</i>	10
<i>Request a License</i>	10
<i>Apply the License</i>	10
suiteHEART® Software Installation	11
Software Installation	11
<i>Documentation Installation</i>	11
Anti-Virus Exclusions	11
Application Inactivity Setting	12
Licensing Schemes	12
<i>Remote Connectivity</i>	13
Install Verification	13

Safety

Introduction

To assure efficient and safe use it is essential to read this safety section and all associated topics before attempting to use the software. It is important for you to read and understand the contents of this manual before attempting to use this product. You should periodically review the procedures and safety precautions.

The software is intended for use by trained and qualified personnel only.

suiteDXT / suiteHEART® software has an expected useful service life of 7 years from its original release date.

CAUTION: Federal Law restricts this device to sale, distribution and use by or on the order of a physician.

The terms danger, warning, and caution are used throughout this manual to point out hazards and to designate a degree or level of seriousness. Hazard is defined as a source of potential injury to a person. Familiarize yourself with the terminology descriptions listed in the following table:

Table 1: Safety Terminology

Graphic	Definition
 DANGER :	Danger is used to identify conditions or actions for which a specific hazard is known to exist which <u>will</u> cause severe personal injury, death, or substantial property damage if the instructions are ignored.
 WARNING:	Warning is used to identify conditions or actions for which a specific hazard is known to exist.
 CAUTION:	Caution is used to identify conditions or actions for which a potential hazard is known to exist.

Indications for Use

suiteHEART® Software is an analytical software tool, which provides reproducible tools for the review and reporting of medical images. suiteHEART® Software can import medical images from a MR system and display them in a viewing area on the computer screen. The viewing area allows access to multiple studies and series of multi-slice, multi-phase images. Multi-phase sequences of images can be displayed in cine mode to facilitate visualization.

A report input interface is also available. Measurement tools on the report interface make it possible to quickly and reliably fill out a complete clinical report of an imaging exam. Available tools include: point, distance, area, and volume measurement tools such as ejection fraction, cardiac output, end-diastolic volume, end-systolic volume, and volume flow measurements.

Semi-automatic tools are available for left ventricular contour detection, valve plane detection, vessel contour detection for flow analysis, signal intensity analysis for myocardium and infarct sizing measurement, and T2 star analysis.

The results of the measurement tools are interpreted by the physician and can be communicated to referring physicians.

When interpreted by a trained physician these tools may be useful in supporting the determination of a diagnosis.

suiteDXT is intended to allow users the ability to import, export, anonymize DICOM images, and launch related imaging applications. This product has no diagnostic medical function or purpose.

Intended Use

suiteHEART® Software is intended to assist trained clinical personnel in the qualification and quantification of cardiac function. The software provides the tools to adjust the parameters of the DICOM images and provides presentation states where the user can appreciate various MRI acquired images of the heart and vasculature over time. Additionally, the software provides tools for measuring linear distances, areas, and volumes that can be used to quantify cardiac function. Finally, the software provides the tools for volumetric flow measurements and the ability to calculate flow values.

Supported DICOM Image Formats

suiteHEART® Software supports the following DICOM format; MR and Enhanced MR. Refer to the suiteHEART® Software DICOM Conformance Statement manual for further detail on supported formats.

CAUTION: Data stored as a DICOM image that has been imported by an external PACS may not be compatible viewing for suiteHEART® Software.

Safety Notices

WARNING: The application assists in the analysis of the images only and does not automatically produce quantifiable results. The use and placement of quantitative measurements is at the discretion of the user. Misdiagnosis can occur if measurements are inaccurate. Measurements should only be created by a properly trained and qualified user.

WARNING: Artifacts on an image can be misinterpreted, leading to inaccurate results. Do not use images containing artifacts for diagnosis. Analysis should only be performed by a properly trained and qualified user.

WARNING: Diagnosis for the wrong patient could occur if images do not contain patient name or ID. Do not use images that do not contain patient name and ID for diagnosis. Visually confirm the patient information prior to analysis.

CAUTION: Using images upon which an image filter has been applied could result in inaccurate results. The user must exercise discretion before analyzing pixel intensity corrected images. The software will display a warning message if loading images that have been filtered.

Equipment Hazards

CAUTION: Using equipment that is damaged or has been compromised can put the patient at risk by delaying diagnosis. Make sure that equipment is in proper working order.

CAUTION: Applications may run on equipment that includes one or more hard disk drives, which may hold medical data related to patients. In some countries, such equipment may be subject to regulations concerning the processing of personal data and free circulation of such data. Release of personal data may result in legal action depending on the applicable regulatory body. It is strongly recommended that access to patient files be protected. The user is responsible for understanding the laws regulating patient information.

Prerequisites

Prior to installation of the software, the following prerequisites must be met:

Minimum System Requirements
Supported Operating Systems: <ul style="list-style-type: none">• Windows 11 Professional or Enterprise• Windows 10 Professional or Enterprise (64 bit version)
Video card (NVIDIA); 6GB RAM, minimum of 1664 CUDA Cores, support for OpenGL 4.0, support 1920 x 1080 resolution or higher <ul style="list-style-type: none">• Recommended 8GB and 2500 CUDA Cores
Monitor shall support 1920 x 1080 resolution or higher (Landscape); 2160 x 3840 resolution or higher (Portrait)
Anti-Virus/Anti-Malware software installed
16GB memory (32GB required for 4D Flow analysis)
Available hard drive space minimum 100GB
PDF viewer, Adobe Reader 11.0 or higher
An open network port for DICOM transfer over network
Static IP address (recommended)
Amazon Corretto
Solid-State hard drive (SAS) Locally attached storage
Intel® Core™ i7 processor
27" Monitor (dual monitors recommended)

NOTE: In order to support Auto analysis and 3D/4D in suiteHEART in a Virtual Machine environment, the appropriate NVIDIA video card with compute capabilities and openGL support must be passed through.

NOTE: suiteHEART® Software does not support scaling/zoom levels greater than 125%.

NOTE: You must have administrative privileges in order to install the software, restart the DICOM service and move the Local DICOM Storage.

NOTE: For support of application frame rates of 30FPS at display resolutions in the range of 2K/4K, the NVIDIA video card hardware must support a minimum of 1920 CUDA Cores.

NOTE: In the event that the host system loses or is reassigned its IP address, remote devices will need to be manually reconfigured to request and accept network traffic.

NOTE: The following ports are the defaults used for the software:

- Public Port 5678 - NeoSoft License service
- Public Port 104 - Local DICOM connection
- Public Port 2762 - Encrypted Local DICOM connection
- Private Port 9001 - Report database
- Private Ports 1099, 1100, 1101, 1102, 1103, 1104, 1105, 1106, 1107, 1108 suiteHEART to suiteHEART communications

Public means that the port is open on all network interfaces.

Private means that the port is only open on localhost (127.0.0.1).

suiteDXT Installation & Configuration

Installation

1. Copy the NeoSoft provided MySQL package and suiteDXT installer to the same Windows folder.
2. Double click suiteDXT-windows-installer.
3. If prompted, select "Yes" on the User Account Control pop-up.
4. Select the desired language, then click "OK."

NOTE: If another version of the software is already installed on the system, a pop-up will ask permission to continue. Click "Yes" to continue with the installation.


5. Select "Next" on the suiteDXT setup screen.
6. Check the box for "I accept the agreement" and select "Next."
7. Uncheck the boxes for the software that is not being installed and select "Next."
8. Accept or change the installation directory and select "Next."
9. Select "Next" in ready to install setup screen.
10. Select "Finish" in the setup screen.

Configuration

You will need to run through the configuration steps when suiteDXT is newly installed or when/if the static IP address is renewed.

1. Launch suiteDXT via the desktop shortcut or navigate through the start menu.
If prompted, select "Yes" on the user account pop-up.
2. Select the Configuration button, outlined in the figure below.

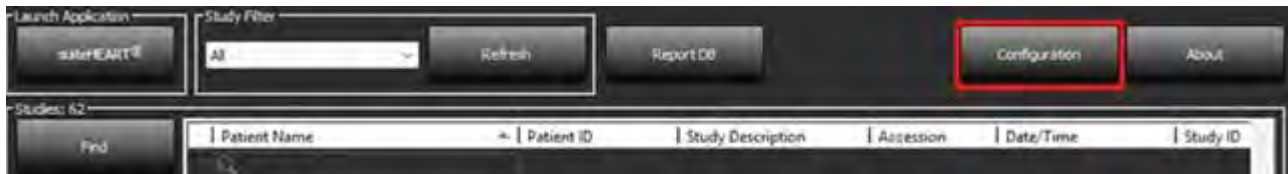
FIGURE 1. suiteDXT Main Screen

3. In the Configuration dialog, check the Port or SSL Port checkbox then enter information for the local host:
 - AE Title (case sensitive) is the name used to identify the host by other hosts.
 - Port (104).


NOTE: The information entered for Local DICOM must be entered exactly the same on any remote device for suiteDXT to retrieve or accept images from the remote host. See Figure 2.

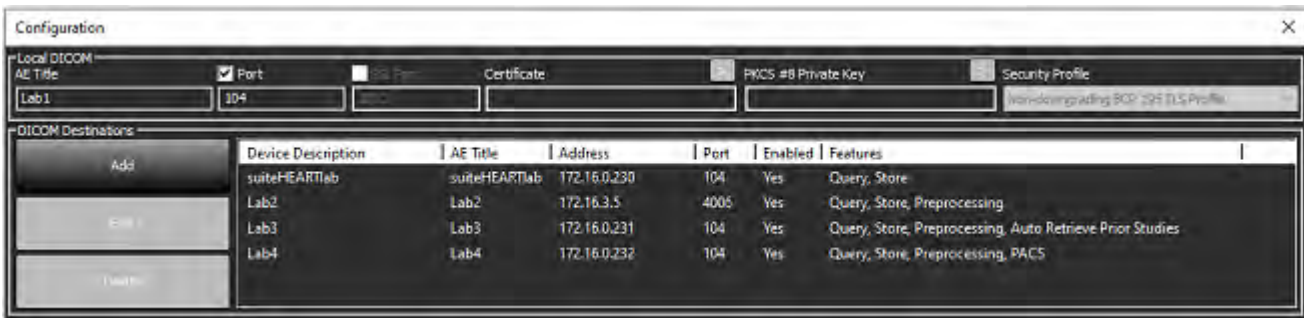FIGURE 2. Configuration Dialog

- If available, perform a DICOM echo from any remote device to this host to ensure DICOM connectivity.
- Select the “Add” button to input all DICOM destinations (remote hosts).
 - Description
 - AE Title
 - Address
 - Port
 - Select/deselect the checkboxes as necessary for: Enabled; Store to device, Query device and Preprocessing.
 - Enable SSL, Auto Retrieve Prior Studies, PACS, define Max Priors, and Prior Study Description. (Figure 3).

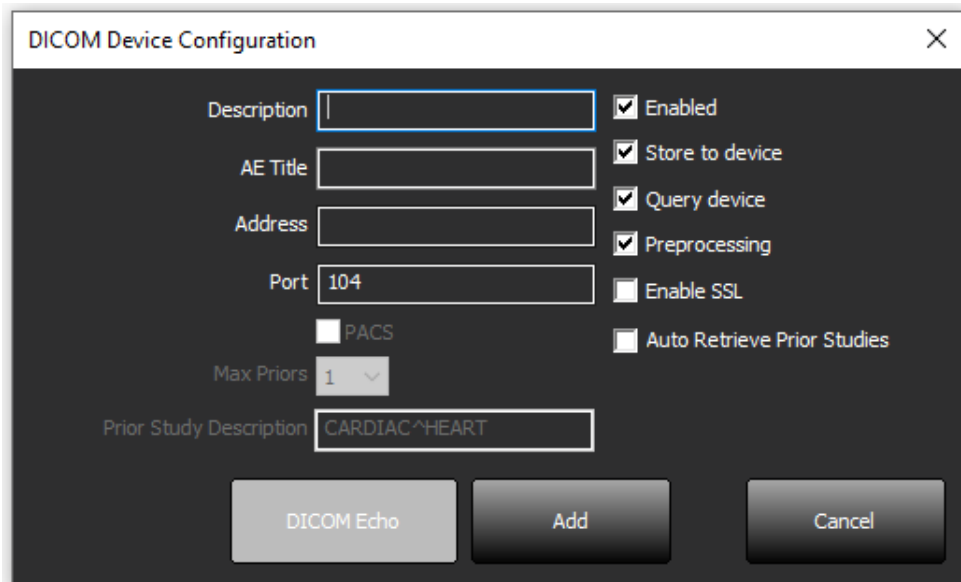FIGURE 3. DICOM Configuration Window

- Select the “DICOM Echo” button.
 - If you don’t see the DICOM Success Message window (Figure 4) contact your network administrator.


FIGURE 4. DICOM Success Message

NOTE: DICOM Destinations can be modified and a DICOM echo performed via the Edit button and destinations can be deleted via the Delete button.

Local Storage Configuration

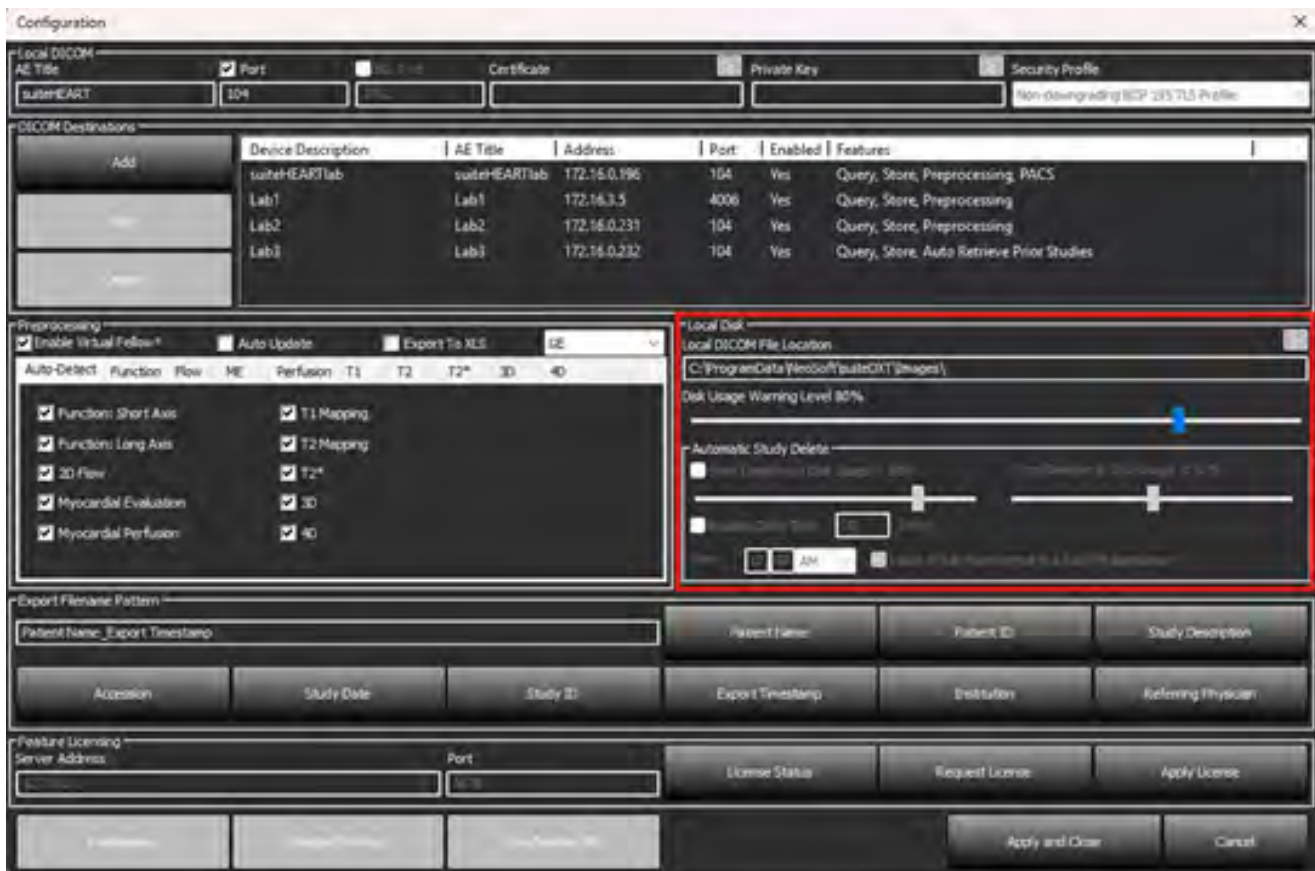
FIGURE 5. Local Storage Configuration

The Disk Usage Warning Level can be modified to alert you when disk space is low. When disk usage exceeds the limit set, the indicator on the main screen (Figure 6) will change from blue to yellow.

To restore the study listing, click the Synchronize DB button. This refreshes the data currently stored on the local DICOM file location.

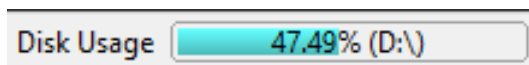FIGURE 6. Disk Usage Indicator on Main Screen

The Local DICOM File Location Selection allows you to change the directory where DICOM data is stored.

Auto Delete

Auto Delete can be configured to automatically delete the oldest studies in the system that match the defined criteria. The Auto Delete configuration should be maintained by the system administrator for your facility.

CAUTION: Studies that have been removed using the Auto Delete function cannot be retrieved.

Auto Unlock

Auto Unlock is configured to automatically unlock studies. The Auto Unlock configuration should be maintained by the System Administrator for your facility.

Licensing

Use the Licensing section to request and apply the license for all software distributed by NeoSoft, LLC. (Figure 7)


FIGURE 7. Request and Apply License

The screenshot shows the 'Configuration' window with the following sections:

- Local DICOM:** AE Title: suiteHEART, Port: 104, Certificate: [blank], Private Key: [blank], Security Profile: Non-downgrading BCP 195 TLS Profile.
- DICOM Destinations:** A table with columns: Device Description, AE Title, Address, Port, Enabled, Features.

Device Description	AE Title	Address	Port	Enabled	Features
suiteHEARTlab	suiteHEARTlab	172.16.0.196	104	Yes	Query, Store, Preprocessing, PACS
Lab1	Lab1	172.16.3.5	4006	Yes	Query, Store, Preprocessing
Lab2	Lab2	172.16.0.231	104	Yes	Query, Store, Preprocessing
Lab3	Lab3	172.16.0.232	104	Yes	Query, Store, Auto Retrieve Prior Studies
- Preprocessing:** Includes checkboxes for 'Enable Virtual Fellow', 'Auto Update', and 'Export To XLS'. A table for 'Auto-Detect' has columns: Function, Flow, ME, Perfusion, T1, T2, T2*, 3D, 4D. Checked items include: Function: Short Axis, Function: Long Axis, 2D Flow, Myocardial Evaluation, Myocardial Perfusion, T1 Mapping, T2 Mapping, T2*, 3D, 4D.
- Local Disk:** Local DICOM File Location: C:\ProgramData\NeoSoft\suiteDXT\Images\; Disk Usage Warning Level: 80%.
- Automatic Study Delete:** Includes checkboxes for 'Disk | Decision | Disk Usage > 80%' and 'Study User Time'.
- Export Filename Pattern:** Patient Name_Export Timestamp. Buttons for: Patient Name, Patient ID, Study Description, Accession, Study Date, Study ID, Export Timestamp, Institution, Referring Physician.
- Feature Licensing:** Server Address: 127.0.0.1, Port: 5000. License Status: [blank]. **Request License** and **Apply License** buttons are highlighted with a red box.
- Buttons:** Previous, Next, Cancel, Apply and Close.

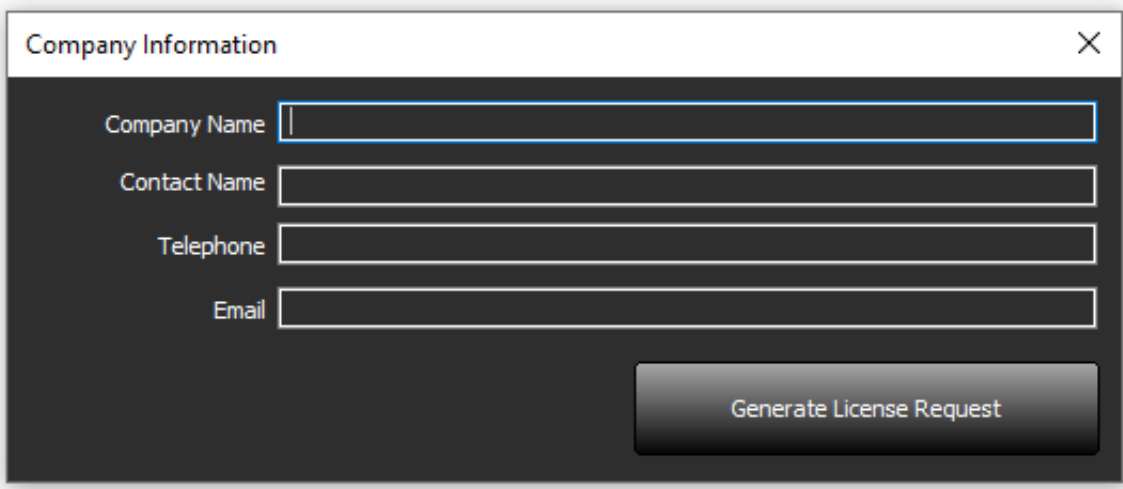FIGURE 8. Licensing

The image shows a software dialog box titled "Company Information" with a close button (X) in the top right corner. The dialog contains four text input fields stacked vertically, labeled "Company Name", "Contact Name", "Telephone", and "Email". At the bottom right of the dialog is a button labeled "Generate License Request".

NOTE: Access to email will be needed to request a license.

Configuring Remote License Server

During Install or upgrade, NeoSoft can configure suiteHEART to connect to a remote license server by modifying the Server Address and Port information within the Licensing section.

Request a License

1. Enter complete company and contact information.

NOTE: Selecting the Generate License Request button will create a machine specific file that locks the software for which you are requesting a license, to the hardware (host) on which the License Server is installed.

2. Select the Generate License Request button.
3. Navigate to a memorable directory (e.g. the desktop) in which to save the license request file.
4. Email the license request file to licensing@neosoftmedical.com

Apply the License

1. From NeoSoft's response email, copy the license to an easily accessible location (e.g. the desktop).
2. Open suiteDXT, select the Configuration button and click on Apply License (Figure 8).
3. Navigate to the directory to which you saved the license.
4. Select Open.

suiteHEART[®] Software Installation

Software Installation

1. Double-click on the suiteHEART[®] Software windows application installer.

NOTE: Installing the software on a machine could result in a Open File - Security Warning indicating that the publisher could not be verified. Click Yes when asked if you are sure you want to run the software.

2. Select the appropriate language for your installation and click **OK**.

NOTE: If another version of the software is already installed on the system, a pop-up will ask permission to uninstall. Click “Yes” to continue with the installation.

3. In the “Welcome to the suiteHEART[®] Setup Wizard” window, click **Next**.
4. On the License Agreement Window, select “I accept the agreement” and click **Next**.
5. Browse to the directory in which to install the software or use the default directory (recommended). Click **Next**.
6. Click **Next** on the Ready to Install screen.
A progress bar will display.
7. Click **Finish** to complete the installation.

Documentation Installation

Prior to installing the documentation, the suiteDXT and suiteHEART[®] applications must be installed.

1. Double-click on the Documentation installer.

NOTE: Installing NeoSoft documentation on a machine could result in an Open File - Security warning indicating that the publisher could not be verified. Click **Yes** when asked if you are sure you want to run the software.

2. In the “Welcome to the NeoSoft Documentation Setup Wizard” window, click **Next**.
3. In the “Select Components” window, select the documentation to install (checking all available options is recommended), and click **Next**.
4. Click **Next** on the Ready to Install screen.
A progress bar will display.
5. Click **Finish** to complete the installation.

Anti-Virus Exclusions

To increase the performance of the application, the following Anti-Virus and Windows Defender exclusions should be made:

- C:\Program Files\NeoSoft, LLC or C:\Program Files\NeoSoft
- C:\ProgramData\NeoSoft, LLC or C:\ProgramData\NeoSoft

NOTE: The install locations noted above assume a default installation.

Application Inactivity Setting

The suiteHEART® Software application will automatically close, save all analysis for any open study and remove the associated lock file when the application has been inactive for 60 minutes. To set this setting configure it in suiteHEART® Software, launched in Admin mode, using Tools > Preferences > Edit System.

Licensing Schemes

suiteHEART® Software has 3 different licensing schemes. They are Unlimited, Annual Subscription, and Per-Case Pack Limited.

- Unlimited: No restrictions on time or analysis.
- Annual Subscription: Restricted on time; must renew license annually. Unlimited analysis and viewing during license duration.
- Per-Case Pack Limited: Restricted on number of cases analyzed. The software will use the unique study instance id to identify the number of cases analyzed against the Per-Case Pack that has been purchased.

Once 100% of the Per-Case Pack has been used the software will indicate this condition by a pop-message prior to starting a new exam or an existing exam.

suiteHEART® Software is licensed and sold using a user/seat license model.

Per-Case Packs can be purchased in increments of 25, 50 and 100. Website: <http://neosoftllc.com/>
T: (262) 522-6120 email: orders@neosoftmedical.com

Exams that have been counted once towards the Per-Case Pack limit can be re-opened and reviewed within the software and will not be counted against the Per-Case Pack limit.

Exams that are opened only for image viewing or if image manipulation tools are performed (Pan, Zoom, Window level, Rotate) will not be counted against the Per-Case Pack limit.

An exam is considered “consumed” or “counted” against the Per-Case Pack limit when any of the following actions are performed:

- a.) Starting of any analysis mode by placing any ROI on an image.
- b.) Creating a custom series.
- c.) Report sign off.
- d.) Export Cine DICOM.
- e.) Export Report.
- f.) DICOM series creation.
- g.) Preprocessed study.
- h.) Virtual Fellow® preprocessing.
- i.) Any Auto Combine / Split Mode.

suiteHEART® Software has an expected useful service life of 7 years from its original release date.

Remote Connectivity

Remote connectivity allows suiteHEART® Software users remote access from computers not loaded with suiteHEART® Software. The remote user will have the ability to access the suiteHEART® Software system and have full functionality.

suiteHEART® remote connectivity needs to be installed/configured/supported by your Information Technology personnel.

Remote connectivity has been verified using the following platforms:

- TeamViewer
- Windows Remote Desktop (Microsoft Windows 10/11 Professional/Enterprise)

NOTE: When using Windows Remote Desktop, screen resolution on the remote computer must be set to 1920x1080 (Landscape) or higher video display resolution.

NOTE: When remotely accessing suiteHEART® Software using Windows Remote Desktop, the suiteHEART® 3D/4D viewer is not operational. The workaround is to use one of the other qualified remote access software packages (TeamViewer).

The performance of the suiteHEART® Software via the remote connection cannot be guaranteed. Performance is dependent upon factors that are outside the scope of the suiteHEART® Software application. These factors include:


- Remote computer performance
- Internet download/upload speeds (Mbps)
- Network bandwidth speed (wired ethernet or wireless connection)

For more information on remote connectivity, refer to the suiteHEART® Remote Connectivity Addendum to Instructions for Use manual, NS-03-043-0006.

Install Verification


After a successful installation of suiteDXT this screen is displayed.

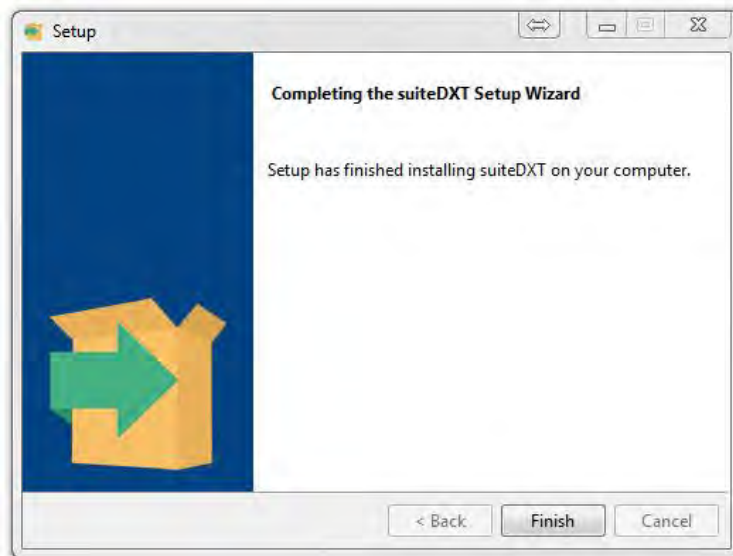FIGURE 1. suiteDXT Setup Wizard

Following the successful installation of suiteHEART® Software this screen displays.

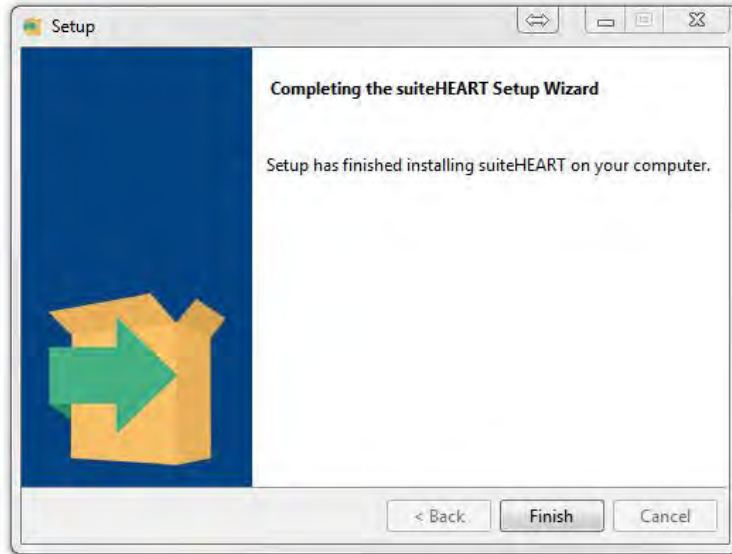FIGURE 2. suiteHEART® Software SetupWizard


To further verify the successful installation of both software packages:

1. Double click the suiteDXT icon to verify suiteDXT is launched.
2. Highlight a study in suiteDXT and open it with suiteHEART to verify that the study is launched and that various analyses can be executed.
3. Highlight a series in a demo study and choose Delete to verify the logged in user has delete privileges.

After a successful installation of the NeoSoft documentation this screen is displayed.

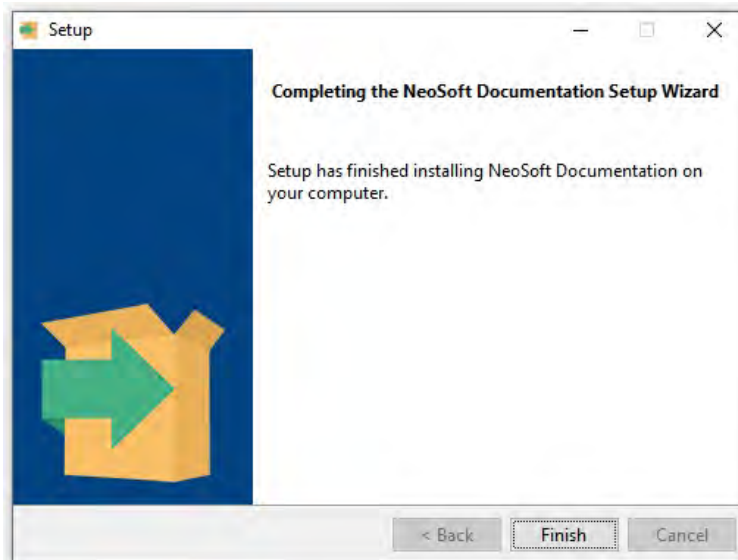FIGURE 3. NeoSoft Documentation Setup Wizard

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
- Local DICOM:** AE Title: suiteHEART, Port: 104, Certificate: 2782, Private Key: [empty], Security Profile: Non-downgrading BCP 195 TLS Profile.
- DICOM Destinations:** A table with columns: Device Description, AE Title, Address, Port, Enabled, Features.
- Preprocessing:** Includes checkboxes for 'Enable Virtual Fellow', 'Auto Update', and 'Export To XLS'. A table lists features like 'Function: Short Axis', 'Function: Long Axis', '2D Flow', 'Myocardial Evaluation', 'Myocardial Perfusion', 'T1 Mapping', 'T2 Mapping', 'T2*', '3D', and '4D'.
- Local Disk:** Local DICOM File Location: C:\ProgramData\NeoSoft\suiteDXT\Images\, Disk Usage Warning Level: 80%.
- Export Filename Pattern:** Patient Name_Export Timestamp.
- Feature Licensing:** Server Address: 127.0.0.1, Port: 5000. Buttons for 'License Status', 'Request License', and 'Apply License' are visible, with 'Request License' and 'Apply License' highlighted in red.

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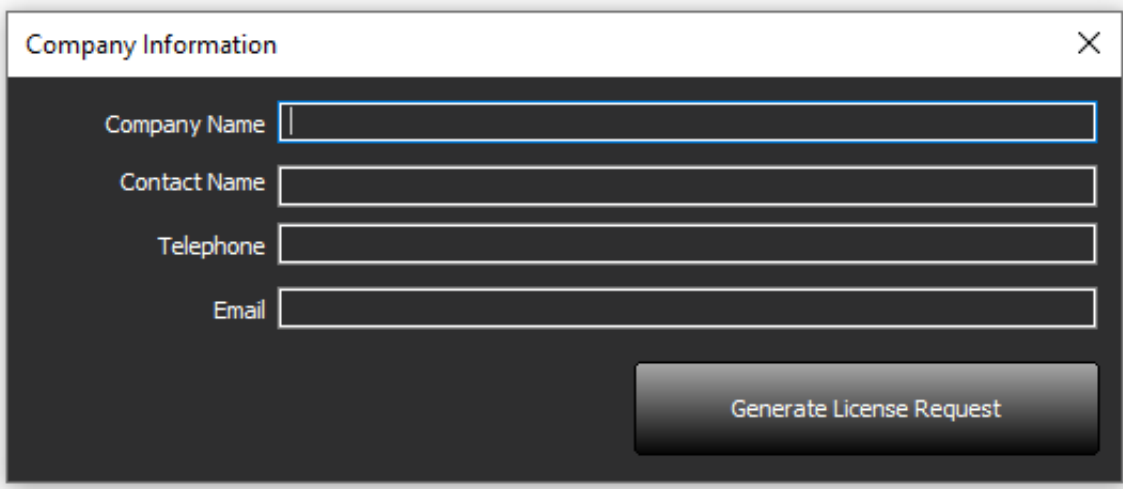The image shows a software window titled "Company Information" with a close button (X) in the top right corner. The window contains four text input fields: "Company Name", "Contact Name", "Telephone", and "Email". Below these fields is a large, dark button labeled "Generate License Request".

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
- Local DICOM:** AE Title: suiteHEART, Port: 104, Certificate: 2782, Private Key: (empty), Security Profile: Non-downgrading BCP 195 TLS Profile.
- DICOM Destinations:** A table with columns: Device Description, AE Title, Address, Port, Enabled, Features.

Device Description	AE Title	Address	Port	Enabled	Features
suiteHEARTlab	suiteHEARTlab	172.16.0.196	104	Yes	Query, Store, Preprocessing, PACS
Lab1	Lab1	172.16.3.5	4006	Yes	Query, Store, Preprocessing
Lab2	Lab2	172.16.0.231	104	Yes	Query, Store, Preprocessing
Lab3	Lab3	172.16.0.232	104	Yes	Query, Store, Auto Retrieve Prior Studies
- Preprocessing:** Enable Virtual Fellow (checked), Auto Update (checked), Export To XLS (checked), GE (dropdown).

Auto-Detect	Function	Flow	ME	Perfusion	T1	T2	T2*	3D	4D
<input checked="" type="checkbox"/>	Function: Short Axis	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	T1 Mapping	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
<input checked="" type="checkbox"/>	Function: Long Axis	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	T2 Mapping	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
<input checked="" type="checkbox"/>	2D Flow	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	T2*	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
<input checked="" type="checkbox"/>	Myocardial Evaluation	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	3D	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
<input checked="" type="checkbox"/>	Myocardial Perfusion	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	4D	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
- Local Disk:** Local DICOM File Location: C:\ProgramData\NeoSoft\suiteDXT\Images\, Disk Usage Warning Level: 80%.
- Export Filename Pattern:** Patient Name_Export Timestamp, Patient Name, Patient ID, Study Description, Accession, Study Date, Study ID, Export Timestamp, Institution, Referring Physician.
- Feature Licensing:** Server Address: 127.0.0.1, Port: 2782, License Status, Request License, Apply License.

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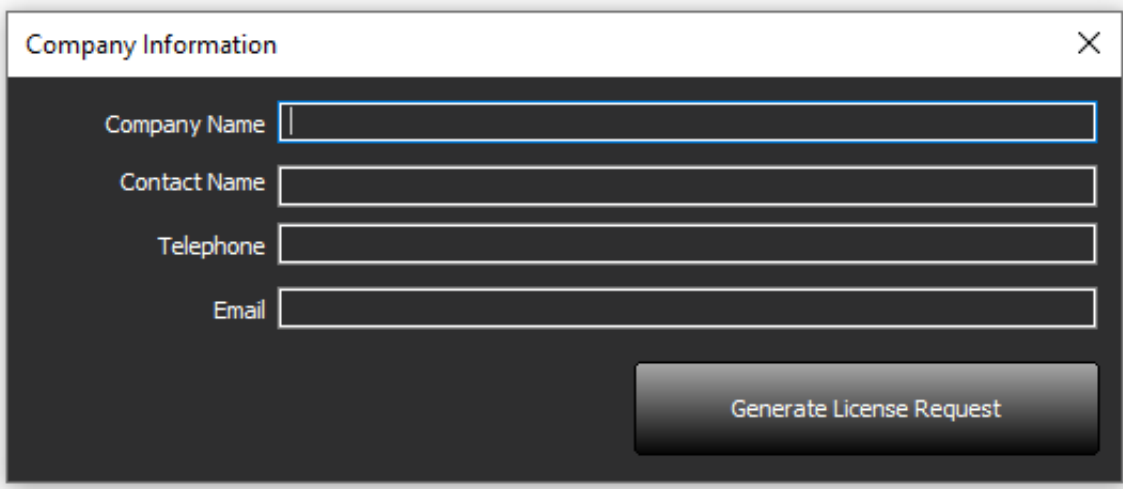The image shows a software dialog box titled "Company Information" with a close button (X) in the top right corner. Inside the dialog, there are four text input fields labeled "Company Name", "Contact Name", "Telephone", and "Email". At the bottom right of the dialog is a button labeled "Generate License Request".

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
- Local DICOM:** AE Title: suiteHEART, Port: 104, Certificate: 2782, Private Key: [empty], Security Profile: Non-downgrading BCP 195 TLS Profile.
- DICOM Destinations:** A table with columns: Device Description, AE Title, Address, Port, Enabled, Features.
- Preprocessing:** Includes checkboxes for 'Enable Virtual Fellow', 'Auto Update', and 'Export To XLS'. A table lists features like 'Function: Short Axis', 'Function: Long Axis', '2D Flow', 'Myocardial Evaluation', 'Myocardial Perfusion', 'T1 Mapping', 'T2 Mapping', 'T2*', '3D', and '4D'.
- Local Disk:** Local DICOM File Location: C:\ProgramData\NeoSoft\suiteDXT\Images\, Disk Usage Warning Level: 80%.
- Export Filename Pattern:** Patient Name_Export Timestamp. Includes buttons for Patient Name, Patient ID, Study Description, Accession, Study Date, Study ID, Export Timestamp, Institution, and Referring Physician.
- Feature Licensing:** Server Address: 127.0.0.1, Port: 5000. Includes buttons for License Status, Request License, and Apply License (highlighted with a red box).
- Buttons:** Previous, Next, Cancel, Apply and Close.

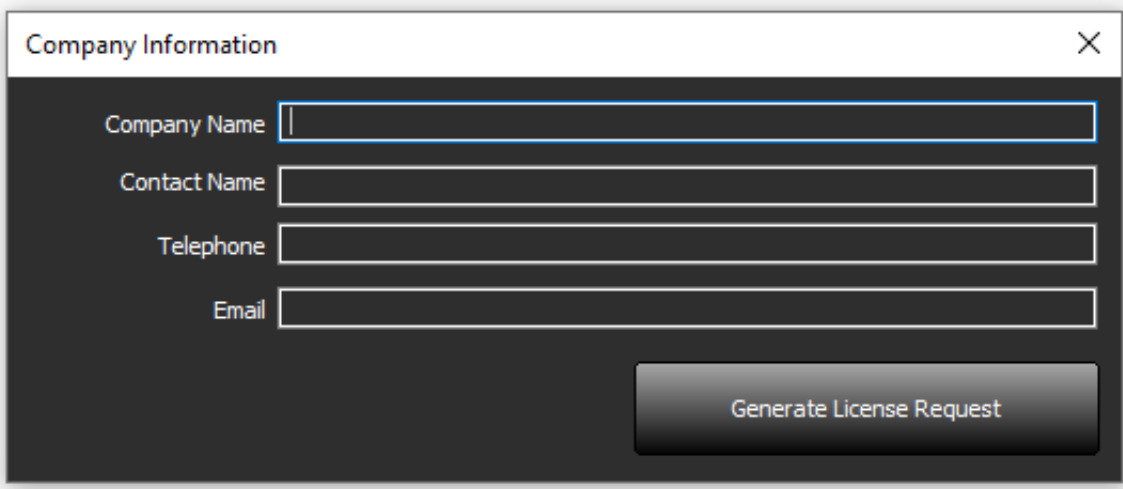FIGURA 8. Licencia

The image shows a software dialog box titled "Company Information" with a close button (X) in the top right corner. Inside the dialog, there are four text input fields stacked vertically, labeled "Company Name", "Contact Name", "Telephone", and "Email". At the bottom right of the dialog is a button labeled "Generate License Request".

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
The screenshot shows the 'Configuration' window for suiteHEART. The 'Feature Licensing' section at the bottom includes fields for 'Server Address' (127.0.0.1) and 'Port' (2208). Below these fields are buttons for 'License Status', 'Request License', and 'Apply License'. The 'Request License' and 'Apply License' buttons are highlighted with a red box. Other sections include 'Local DICOM' settings, 'DICOM Destinations' table, 'Preprocessing' options, and 'Export Filename Pattern'.

Device Description	AE Title	Address	Port	Enabled	Features
suiteHEARTlab	suiteHEARTlab	172.16.0.196	104	Yes	Query, Store, Preprocessing, PACS
Lab1	Lab1	172.16.3.5	4006	Yes	Query, Store, Preprocessing
Lab2	Lab2	172.16.0.231	104	Yes	Query, Store, Preprocessing
Lab3	Lab3	172.16.0.232	104	Yes	Query, Store, Auto Retrieve Prior Studies

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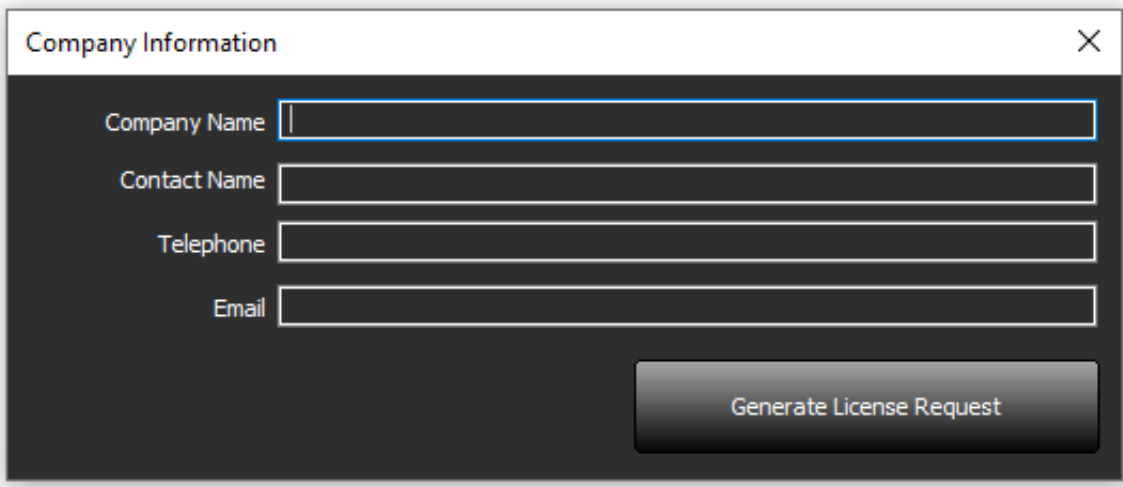
The screenshot shows the 'Configuration' window of the NeoSoft software. The 'Feature Licensing' section at the bottom is highlighted with a red box. It contains the following fields and buttons:

- Server Address: 127.0.0.1
- Port: 2008
- License Status: (empty)
- Request License: (button)
- Apply License: (button)

Other sections visible in the window include:

- Local DICOM: AE Title (suiteHEART), Port (104), Certificate (2782), Private Key, Security Profile (Non-downgrading BCP 195 TLS Profile).
- DICOM Destinations: Table with columns Device Description, AE Title, Address, Port, Enabled, Features.
- Preprocessing: Enable Virtual Fellow, Auto Update, Export To XLS, GE, Auto-Detect, Function, Flow, ME, Perfusion, T1, T2, T2*, 3D, 4D.
- Local Disk: Local DICOM File Location (C:\ProgramData\NeoSoft\suiteDXT\Images), Disk Usage Warning Level 80%, Automatic Study Delete (Disable Decision of Disk Usage > 80%, Enable User Input).
- Export Filename Pattern: Patient Name_Export Timestamp, Patient Name, Patient ID, Study Description, Accession, Study Date, Study ID, Export Timestamp, Institution, Referring Physician.

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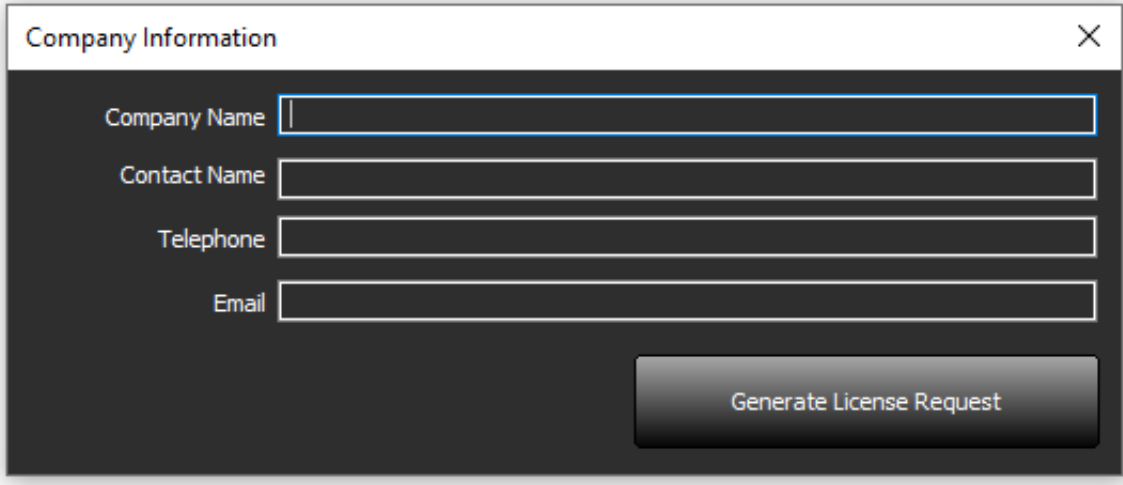The image shows a 'Company Information' dialog box with a dark background and a light border. The title bar at the top left says 'Company Information' and has a close button (X) on the right. Below the title bar, there are four input fields stacked vertically, each with a label to its left: 'Company Name', 'Contact Name', 'Telephone', and 'Email'. At the bottom right of the dialog box, there is a button labeled 'Generate License Request'.

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
- Local DICOM:** AE Title: suiteHEART, Port: 104, Certificate: 2782, Private Key: [empty], Security Profile: Non-downgrading BCP 195 TLS Profile.
- DICOM Destinations:** A table with columns: Device Description, AE Title, Address, Port, Enabled, Features.
- Preprocessing:** Includes checkboxes for 'Enable Virtual Fellow', 'Auto Update', and 'Export To XLS'. A table lists features like 'Function: Short Axis', 'Function: Long Axis', '2D Flow', 'Myocardial Evaluation', 'Myocardial Perfusion', 'T1 Mapping', 'T2 Mapping', 'T2*', '3D', and '4D'.
- Local Disk:** Local DICOM File Location: C:\ProgramData\NeoSoft\suiteDXT\Images\, Disk Usage Warning Level 80%.
- Export Filename Pattern:** Patient Name_Export Timestamp. Includes buttons for Patient Name, Patient ID, Study Description, Accession, Study Date, Study ID, Export Timestamp, Institution, Referring Physician.
- Feature Licensing:** Server Address: 127.0.0.1, Port: 5000. Includes License Status, Request License, and Apply License buttons (highlighted in red).
- Buttons:** Previous, Next, Cancel, Apply and Close.

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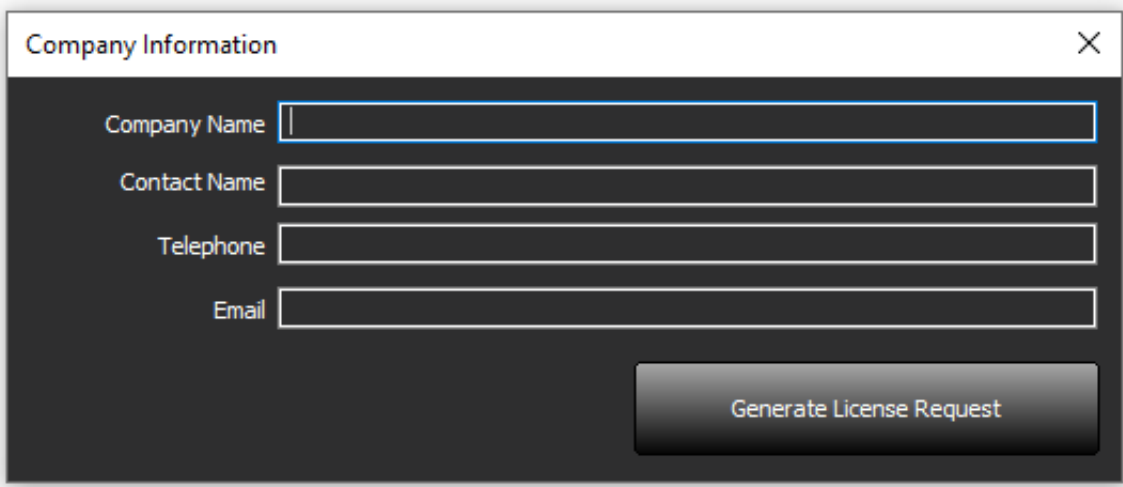
The screenshot shows the 'Configuration' window of the suiteHEART software. The 'Feature Licensing' section at the bottom is highlighted with a red box. It contains the following fields and buttons:

- Server Address: 127.0.0.1
- Port: 5000
- License Status: (button)
- Request License: (button, highlighted in red)
- Apply License: (button, highlighted in red)

Other visible sections include:

- Local DICOM: AE Title (suiteHEART), Port (104), Certificate (2762), Private Key, Security Profile (Non-downgrading BCP 195 TLS Profile).
- DICOM Destinations: A table listing destinations like suiteHEARTlab, Lab1, Lab2, and Lab3 with their respective addresses and ports.
- Preprocessing: Checkboxes for 'Enable Virtual Fellow', 'Auto Update', and 'Export To XLS'. A table of features like 'Function: Short Axis', 'T1 Mapping', etc., with checkboxes.
- Local Disk: Local DICOM File Location (C:\ProgramData\NeoSoft\suiteDXT\Images), Disk Usage Warning Level (80%), and Automatic Study Delete options.
- Export Filename Pattern: Patient Name_Export Timestamp, with buttons for Patient Name, Patient ID, Study Description, Accession, Study Date, Study ID, Export Timestamp, Institution, and Referring Physician.

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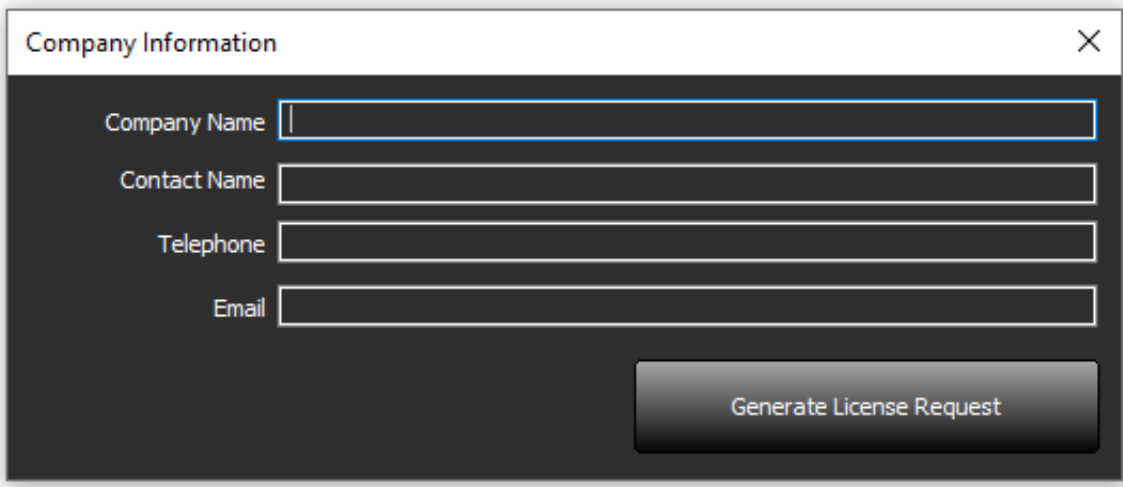
- Local DICOM:** Includes fields for AE Title (suiteHEART), Port (104), Certificate (2782), Private Key, and Security Profile (Non-downgrading BCP 195 TLS Profile).
- DICOM Destinations:** A table listing destinations with columns: Device Description, AE Title, Address, Port, Enabled, and Features.
- Preprocessing:** Includes checkboxes for 'Enable Virtual Fellow', 'Auto Update', and 'Export To XLS'. It also has a table for 'Auto-Detect' with columns: Function, Flow, ME, Perfusion, T1, T2, T2*, 3D, 4D.
- Local Disk:** Includes 'Local DICOM File Location' (C:\ProgramData\NeoSoft\suiteDXT\Images), 'Disk Usage Warning Level 80%', and 'Automatic Study Delete' options.
- Export Filename Pattern:** Includes a text field 'Patient Name_Export Timestamp' and buttons for 'Patient Name', 'Patient ID', 'Study Description', 'Accession', 'Study Date', 'Study ID', 'Export Timestamp', 'Institution', and 'Referring Physician'.
- Feature Licensing:** Includes 'Server Address' (127.0.0.1) and 'Port' (8080). The 'Request License' and 'Apply License' buttons are highlighted with a red box.

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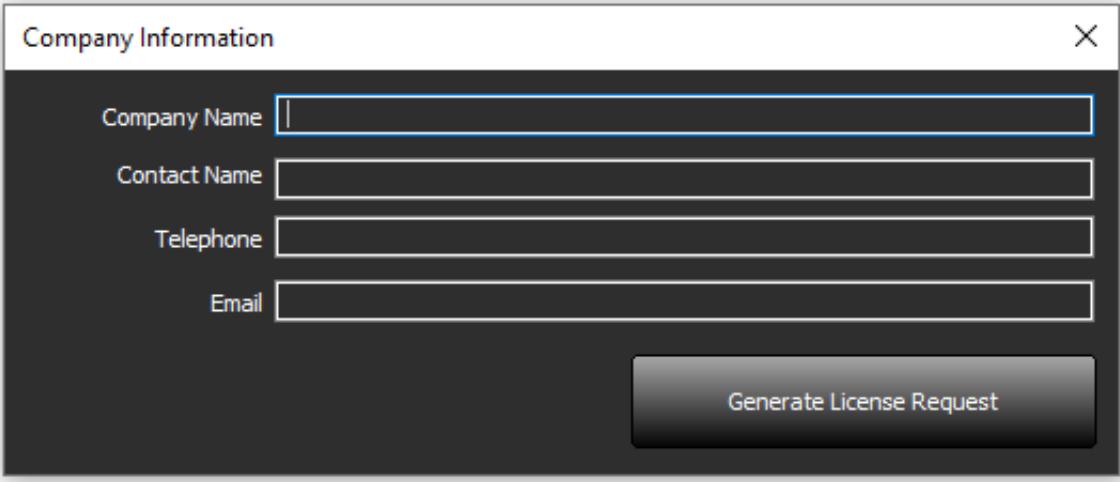The image shows a dialog box titled "Company Information" with a close button (X) in the top right corner. Inside the dialog, there are four text input fields stacked vertically, labeled "Company Name", "Contact Name", "Telephone", and "Email". The "Company Name" field is currently selected with a blue border. At the bottom right of the dialog, there is a button labeled "Generate License Request".

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